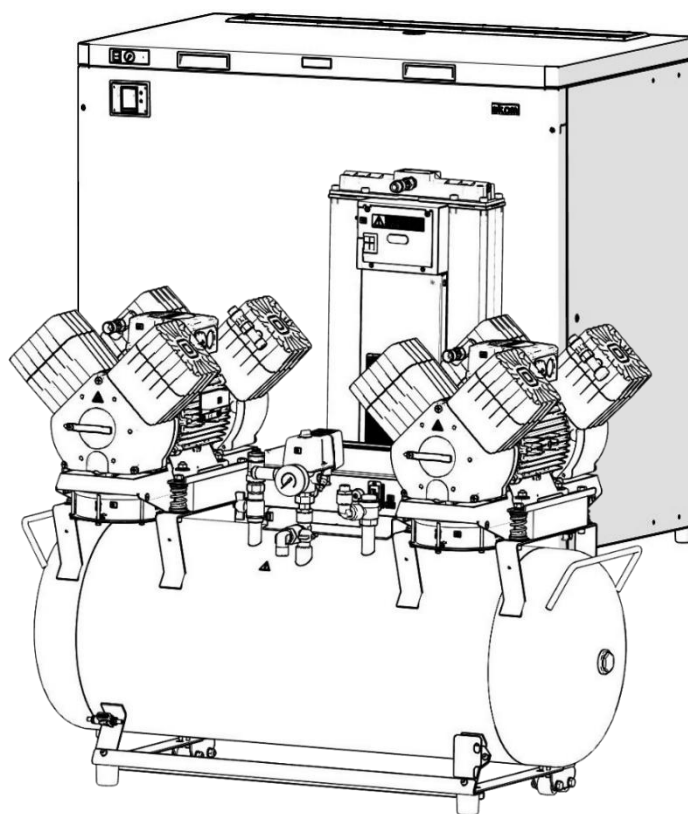
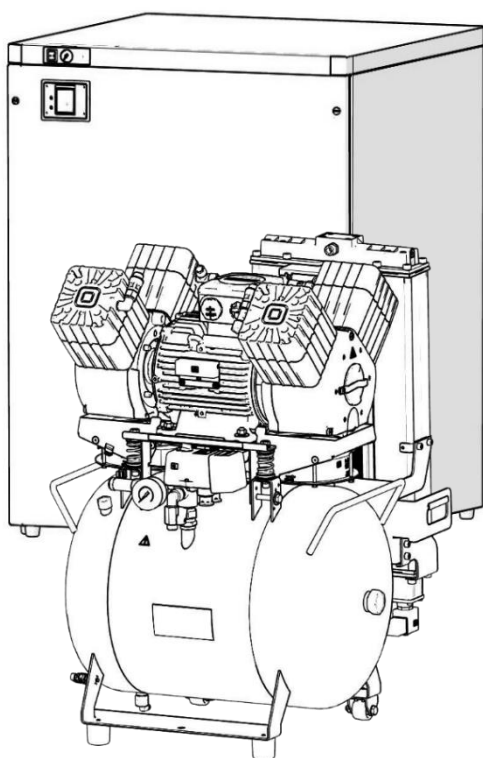
DK50 4VR/50/M

DK50 2X4VR/110/M

- EN USER MANUAL
- DE BENUTZERHANDBUCH
- FR MANUEL DE L'UTILISATEUR
- RU РУКОВОДСТВО ПОЛЬЗОВАТЕЛЯ
- PL INSTRUKCJA OBSŁUGI
- SK NÁVOD NA POUŽITIE
- CS NÁVOD K POUŽITÍ

CE

COMPRESSOR
KOMPRESSOR
COMPRESSEUR
КОМПРЕССОР
SPRĘŻARKA
KOMPRESOR

DK50 4VR/50/M
DK50 2x4VR/110/M

EKOM spol. s r. o.
Priemyselná 5031/18
SK-921 01 Piešťany
Slovak Republic
tel.: +421 33 7967255
fax: +421 33 7967223

www.ekom.sk
email: ekom@ekom.sk

DATE OF LAST REVISION
DATUM DER LETZTEN ÜBERARBEITUNG
DATE DE LA DERNIÈRE RÉVISION
ДАТА ПОСЛЕДНЕЙ РЕДАКЦИИ
DATA OSTATNIEJ AKTUALIZACJI
DÁTUM POSLEDNEJ REVÍZIE
DATUM POSLEDNÍ REVIZE

10/2024

NP-DK50 4VR 50, DK50
2x4VR 110-AD-10_10-2024
112000571-000

EN	<u>CONTENTS..... 5</u>
DE	<u>INHALT 77</u>
FR	<u>TABLES DES MATIÈRES 150</u>
RU	<u>ОГЛАВЛЕНИЕ 223</u>
PL	<u>SPIS TREŚCI 296</u>
SK	<u>OBSAH 369</u>
CS	<u>OBSAH 442</u>

CONTENTS

IMPORTANT INFORMATION	6
1. CONFORMITY WITH THE REQUIREMENTS OF EUROPEAN UNION DIRECTIVES	6
2. INTENDED USE	6
3. CONTRAINDICATIONS AND SIDE-EFFECTS	6
4. SYMBOLS.....	6
5. WARNINGS	7
6. STORAGE AND TRANSPORT CONDITIONS	9
PRODUCT DESCRIPTION	10
7. VARIANTS	10
8. ACCESSORIES	11
9. PRODUCT FUNCTION	12
TECHNICAL DATA	19
INSTALLATION	31
10. INSTALLATION CONDITIONS	31
11. COMPRESSOR ASSEMBLY	31
12. PNEUMATIC CONNECTION	36
13. ELECTRICAL CONNECTION	38
14. PLACEMENT OF THE COMPRESSOR IN THE CABINET	39
15. COMMISSIONING	46
16. PNEUMATIC AND ELECTRICAL DIAGRAMS	47
OPERATION	54
17. SWITCHING ON THE COMPRESSOR.....	55
18. SWITCHING OFF THE COMPRESSOR	55
AD DRYER	56
19. PRINCIPLE OF OPERATION	56
20. USER INTERFACE / SETTINGS	57
21. MAIN SCREENS.....	59
22. INDICATION OF MAINTENANCE AND ALARM STATES.....	61
PRODUCT MAINTENANCE	63
23. PRODUCT MAINTENANCE.....	63
24. LONG-TERM SHUTDOWN.....	73
25. DISPOSAL OF DEVICE	73
TROUBLESHOOTING	74
26. REPAIR SERVICE	76
ANNEX	515
26. INSTALLATION RECORD	515

IMPORTANT INFORMATION

1. CONFORMITY WITH THE REQUIREMENTS OF EUROPEAN UNION DIRECTIVES

This product conforms to the requirements of the Regulation (EU) on medical devices (MDR 2017/745) and is safe for the intended use if all safety instructions are followed.

2. INTENDED USE

The compressor is used as a source of clean, oil-free compressed air to power active medical devices where the parameters and properties of the compressed air are suitable for the specific application.

Compressed air supplied by the compressor is unsuitable for use with artificial lung ventilation devices.

Any other use of the product beyond its intended use is considered an incorrect use. The manufacturer is not liable for any damages or injuries resulting from the incorrect use.

3. CONTRAINDICATIONS AND SIDE-EFFECTS

There are no contraindications or side-effects known.

4. SYMBOLS

The following symbols and marks are used in the User manual, on the device and its packaging:

General warning

Warning - risk of electric shock

Warning - compressor is controlled automatically

Warning - hot surface

General caution

Read the operating instructions

Refer to instruction manual

CE – marking

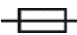
Medical device

Serial number

Article number

	Unique Device Identifier
	Swiss Authorised Representative
	Swiss Importer
	Protecting earthing
	Fuse
	Package handling label – fragile
	Package handling label – this side up
	Package handling label – keep dry
	Package handling label – temperature limits
	Package handling label – limited stacking
	Package label – recyclable material
	Manufacturer

5. WARNINGS

The product is designed and manufactured to be safe for the user and the surrounding environment when used in the defined manner. Keep the following warnings in mind.

5.1. General warnings

READ THE USER MANUAL CAREFULLY BEFORE USING THE DEVICE AND KEEP IT FOR FUTURE USE!

- The user manual supports the correct installation, operation and maintenance of the product. Careful review of this manual will provide the information necessary for the correct operation of the product for its intended use.
- Original packaging should be retained for the return of the device. Only the original packaging ensures protection of the device during transport. If it is necessary to return the product during the warranty period, the manufacturer is not liable for damages caused by improper packaging.
- This warranty does not cover damages originating from the use of accessories other than those specified or recommended by the manufacturer.
- The manufacturer only guarantees the safety, reliability and function of the device if:
 - installation, new settings, changes, expansion, and repairs are performed by the manufacturer or an organisation authorised by the manufacturer

- the product is used pursuant to the user manual
- The user manual corresponds to the configuration of the product and its compliance with the applicable safety and technical standards at the time of printing. The manufacturer reserves all rights for the protection of its configuration, methods and names.
- Translation of the user manual is performed in accordance with the best available knowledge. The Slovak version is to be used in the event of any uncertainties.
- This user manual is the original instructions. Translation is performed in accordance with the best available knowledge.

5.2. General safety warnings

The manufacturer designed and manufactured the product to minimise all risks when used correctly for the intended use. The manufacturer considers it its obligation to lay down the following general safety precautions.

- Use and operation of the product must comply with all laws and local regulations valid in the place of use. The operator and user are responsible for following all the appropriate regulations in the interests of performing work safely.
- Only the use of original parts guarantees the safety of operating personnel and the flawless operation of the product itself. Only the accessories and parts mentioned in the technical documentation or expressly approved by the manufacturer should be used.
- The operator must ensure that the device is functioning correctly and safely before every use.
- The user must be familiar with the operation of the device.
- Do not use the product in environments with a risk of explosion.
- The user must inform the supplier immediately if any problem directly related to the operation of the device occurs.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

5.3. Safety warnings on protection from electric current

- The device must only be connected to a properly installed, earthed socket.
- Before the product is plugged in, ensure that the mains voltage and frequency stated on the product are in compliance with the values of the mains.
- Prior to putting the device into operation check for any damage to the connected pneumatic lines and electrical wiring. Replace damaged pneumatic lines and electrical wirings immediately.
- Immediately disconnect the product from the mains (remove power cord from the socket) in hazardous situations or in the case of a technical malfunction.
- During all repairs and maintenance, ensure that:
 - the mains plug is removed from the power socket
 - pressure is vented from the air tank and pipes
- The product shall only be installed by a qualified technician.

6. STORAGE AND TRANSPORT CONDITIONS

The compressor is shipped from the manufacturer in transport packaging. This protects the product from damage during transport.

Potential for damage to pneumatic components.

The compressor must be transported only when all air has been vented. Before moving or transporting the compressor, release all the air pressure from the tank and pressure hoses and drain condensate from the tank and from the condensate separator on the dryer.

Keep the original factory packaging in case the device needs to be returned Use the original factory packaging during transport as it provides optimum protection for the product. If it is necessary to return the product during the warranty period, the manufacturer is not liable for damages caused by improper packaging.

The compressor is shipped in a vertical position and must be secured using transport straps.

Protect the compressor from humid and dirty environments and extreme temperatures during transport and storage. Do not store near any volatile chemical substances.

If not, please dispose of the original packaging material in an environmentally-friendly way. The packaging cardboard can be recycled with old paper.

Storing or shipping the equipment in any conditions other than those specified below is prohibited.

6.1. Ambient conditions

Products may only be stored and transported in vehicles that are free of any traces of volatile chemicals under the following climactic conditions:

Temperature

–25°C to +55°C


Relative humidity

max. 90% (non-condensing)


PRODUCT DESCRIPTION**7. VARIANTS**

The compressor is manufactured according to its intended application in the following variants:


DK50 4VR/50/M	Compressor with adsorption dryer.
DK50 4VR/50 S/M	Compressor in cabinet with adsorption dryer
Cabinet S50R	It decreases noise level of the compressor
DK50 2x4VR/110/M	Compressor with adsorption dryer
DK50 2x4VR/110S/M	Compressor in cabinet with adsorption dryer
Cabinet S110R	It decreases noise level of the compressor.

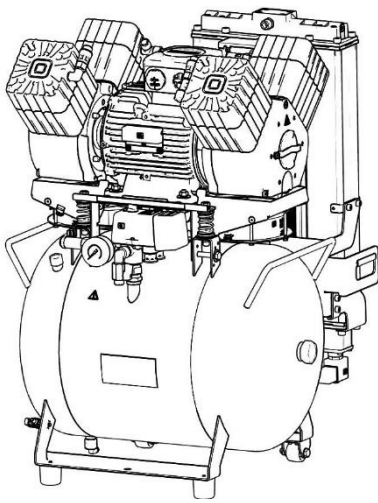
DK50 4VR/50/M

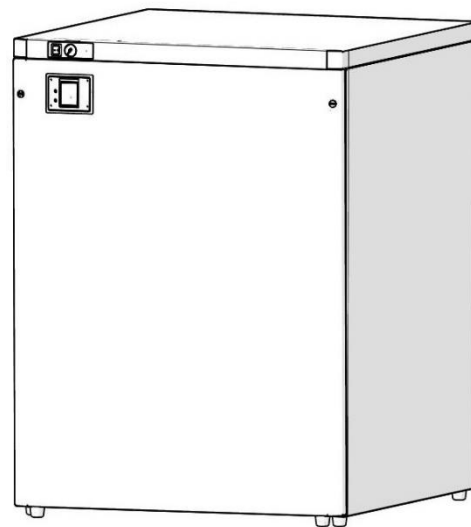S50R
DK50 4VR/50S/M

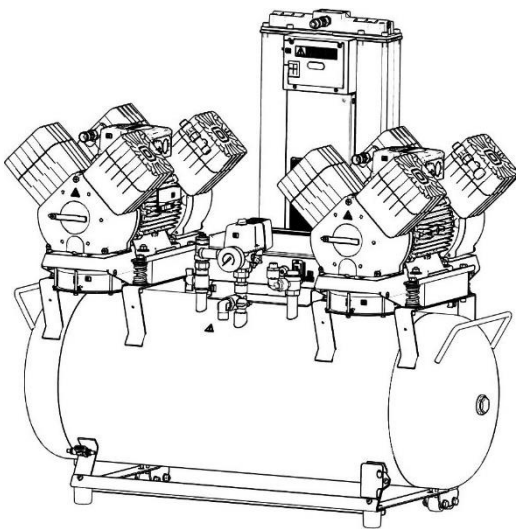DK50 2x4VR/110/M

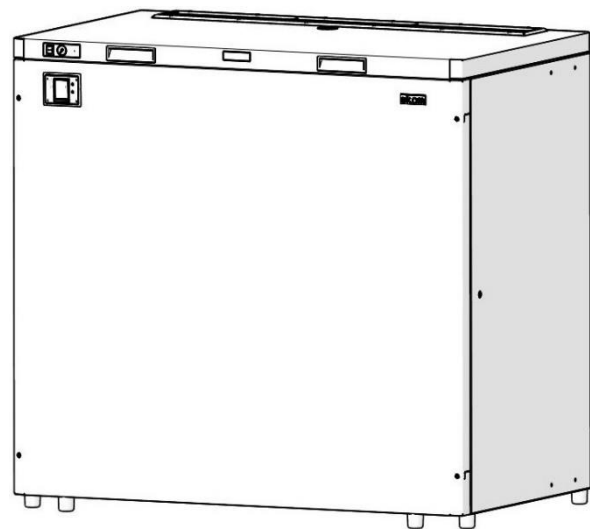S110R
DK50 2x4VR/110S/M

8. ACCESSORIES

Accessories that are not included in the standard order must be ordered separately.

Set of compressed air outlet filters

The compressor may be equipped with a set of filters if specified. The filter set may be equipped with an air pressure regulator.

Where a different level of air filtration is required, this requirement must be agreed upon with the supplier and specified in the order.

Type	Use	Level of filtration (µm)	Article number
FS 30F		1	604014119-014
FS 30M	DK50 4VR/50/M	1+0.1	604014119-015
FS 30S		1+0.01	604014119-017
FS 30A		1+0.01+A**	604014119-018*
FS 40F		1	604014119-000
FS 40M	DK50	1+0.1	604014119-004
FS 40S	2x4VR/110/M	1+0.01	604014119-024
FS 40AH		1+AC+HC (0.01)	604014119-005

**) A –submicrofilter with activated carbon

*) - set **FS 30A**, in the standard configuration, this cannot be combined with a pressure regulator on products DK50 4VR/50/M; please contact your supplier if you are interested in such set (with a regulator)

Filter set regulator assembly

The compressor may be equipped with a pressure regulator of the compressed air outlet if specified. The regulator must be selected according to the application to the filter set, or separately The regulator shall ensure constant pressure at the outlet.

Type	Use	Article number
Regulator complete (to the filter assembly)	DK50 4VR/50/M	604014130-000
REG12 (separate)		447000001-088
Regulator complete	DK50 2x4VR/110/M	604014125-000
REG15		447000001-243

Filter set brackets

A suitable bracket must be ordered for every filter set.

Type	Use	Article number
Compressor-mounted bracket	DK50 4VR/50/M	603014177-000
Compressor-mounted bracket	DK50 2x4VR/110/M	604014131-000
Wall-mounted bracket		603014120-000

3KOM modules for controlling multiple compressors in Master-Slave functionality

A 3KOM control module may be ordered to manage the operation of 2 or 3 identical compressors with the same pressure range in Master-Slave functionality. In use, the controller in the module takes over the compressor switching function. The control module may be ordered in the configurations shown in the following table.

Compressor model	Control schematic	Article number	System air outlet
DK50 4VR/50/M	1+1	603002152-000	G3/4"
DK50 2x4VR/110/M	1+1+1		

Equipotential bonding socket

The socket allows an equipotential bonding

Type	Use	Name	Article number
POAG-KBT6-EC	DK50 4VR/50S/M	Mains plug connector	033400075-000
FLEXI-S/POAG-HK6	DK50 2x4VR/110S/M	Conductor (1m)	034110083-000

9. PRODUCT FUNCTION

9.1. Compressor with adsorption dryer

(Fig. 1, Fig. 2) The compressor air pump (1) draws in air through the inlet filter (8) and compresses it through a non-return valve (3) and into a manifold, from which it is routed to the adsorption dryer (9) through a connecting hose. From the inlet to the dryer module, the air is first cooled in the integrated cooler (17) and then moves through the condensate separator (24), entering the active chamber with adsorbent (19), where the air is dried. A portion of the air is fed into the second, regeneration chamber, where this air is used to remove moisture from the adsorbent, after. The active chamber switches on a cyclical basis. The dry air then passes through the non-return valve (3) into the air tank (2).

* For DK50 4VR/50/M compressors, the manifold with the non-return valve is not used and the air is routed directly to the air dryer.

9.2. Compressor cabinet

(Fig. 4) The soundproof cabinet provides compact covering of the compressor, dampens noise efficiently, providing sufficient exchange of cooling air. With its design It can be placed in a dentist's office as a part of furniture. The fan (10) under the compressor pump provides cooling of the compressor. It is in operation at the same time with the compressor motor or after switching on the temperature switch at temperature above 40°C. After cooling the cabinet area approximately under 32°C, the fan turns off automatically. The dryer display (33) is located on the front panel of the cabinet.

Risk of compressor overheating.

Make sure that there are no obstacles at the cooling air inlet into the cabinet (around the bottom part of the cabinet) and at the hot air outlet on the top back side of the cabinet.

If the compressor is placed on a soft floor, e.g. carpet, create space between the base and the floor or the cabinet and the floor, e.g. underlay the footings with hard pads to ensure sufficient cooling of the compressor.

Descriptions for figures 1 - 3

1	Air pump	22	Regeneration solenoid valve
2	Air tank	23	Dryer fan
3	Non-return valve	24	Condensate separator
4	Pressure switch	25	Condensate drain solenoid vane
5	Safety valve	26	*Dryer display
6	Pressure gauge	27	Compressed air outlet
7	Drain valve	28	Cabinet fan
8	Inlet filter	29	Cabinet pressure gauge
9	Adsorption dryer	30	Switch
10	Compressor fan	31	Connector of cabinet
11	Condensate collection vessel	32	Pressure gauge hose
12	Magnetic holder	33	Cabinet display
13	Compressed air outlet	34	Cabinet gas springs
14	Electrical box / switchboard	35	Pulling system
15	***Dew point sensor	36	Power cord
16	Outlet module	37	**Bushing
17	Cooling module	38	Outlet from condensate separator and regeneration valves
18	Electrical panel	39	Wheels
19	Dryer chamber	40	Wall stop
20	Inlet valve module	41	Connecting reinforcement
21	Compressed air inlet	42	Handle

- * Position (26) applies to the DK50 4VR/50/M, DK50 2x4VR/110/M model without the cabinet; position (26) in the case of the DK50 4VR/50 S/M, DK50 2x4VR/110 S/M with the cabinet is a simple cover.
- ** For the cabinet model, the cable connected to the display (33) in the front panel of the cabinet is routed through the universal bushing (37).
- *** Applies to model with dew point sensor.

Fig. 1: DK50 4VR/50/M – Compressor with adsorption dryer

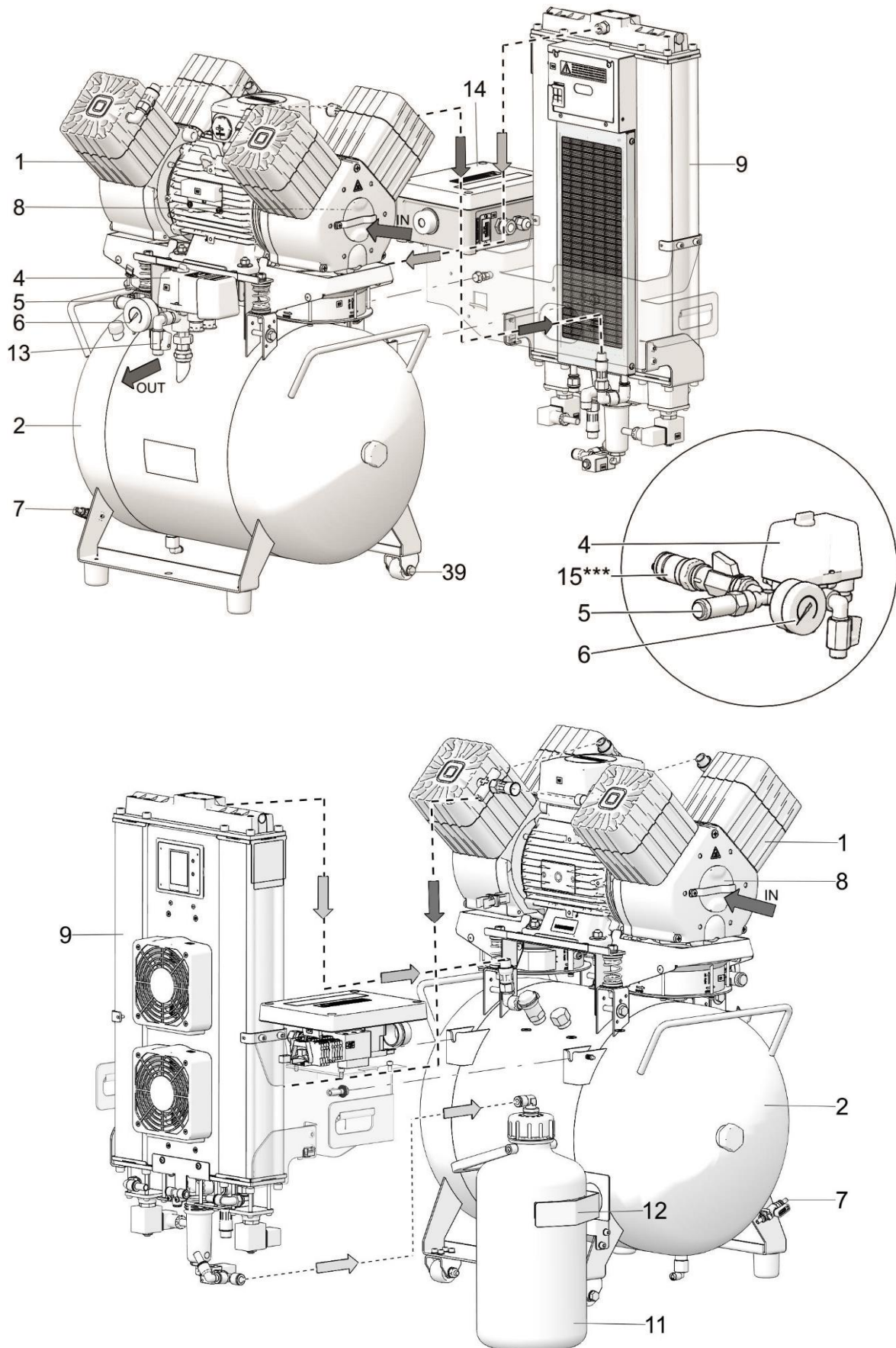
Fig. 2: DK50 2x4VR/110/M –Compressor with adsorption dryer



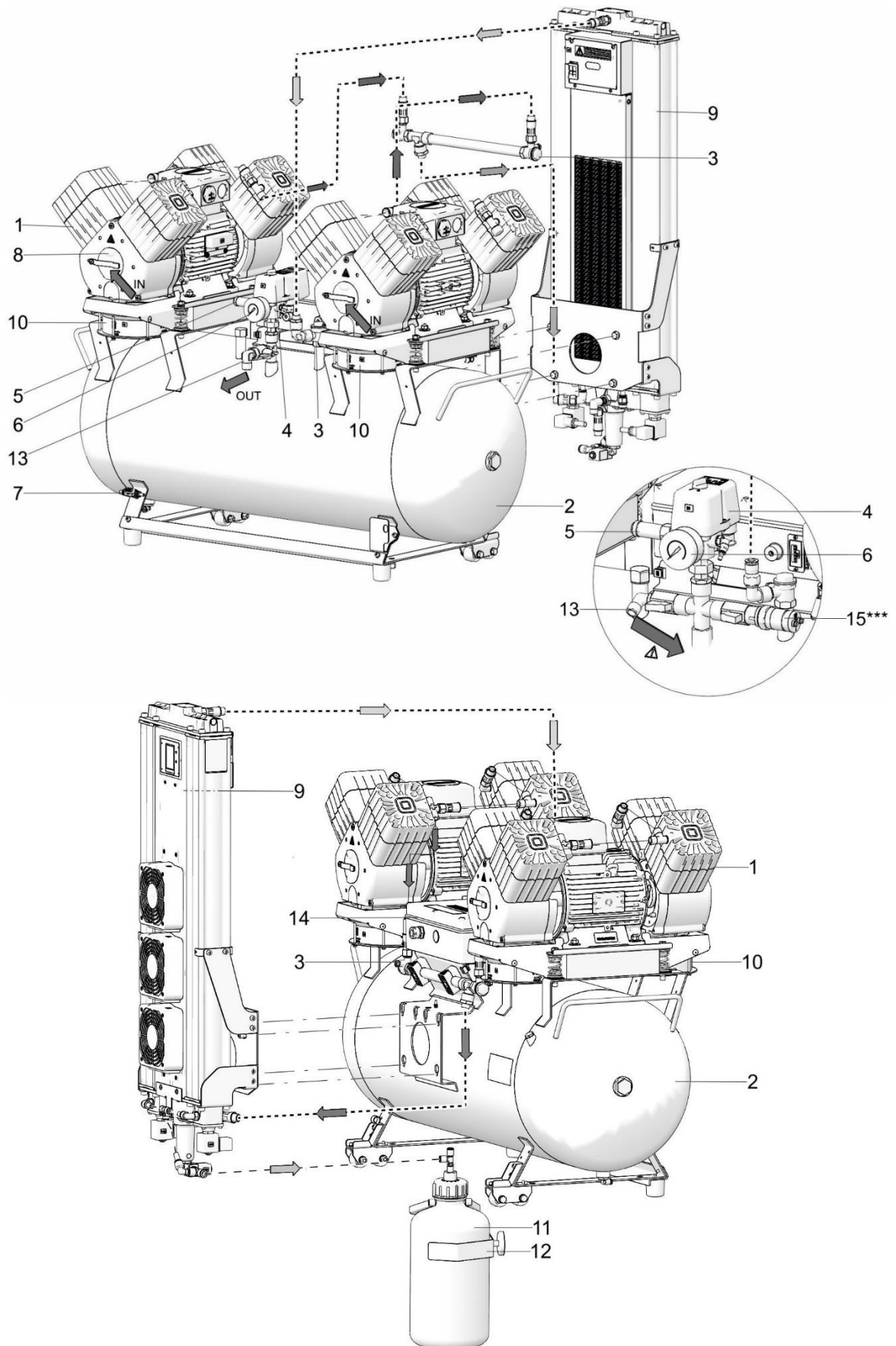
Fig. 3: Adsorption dryer

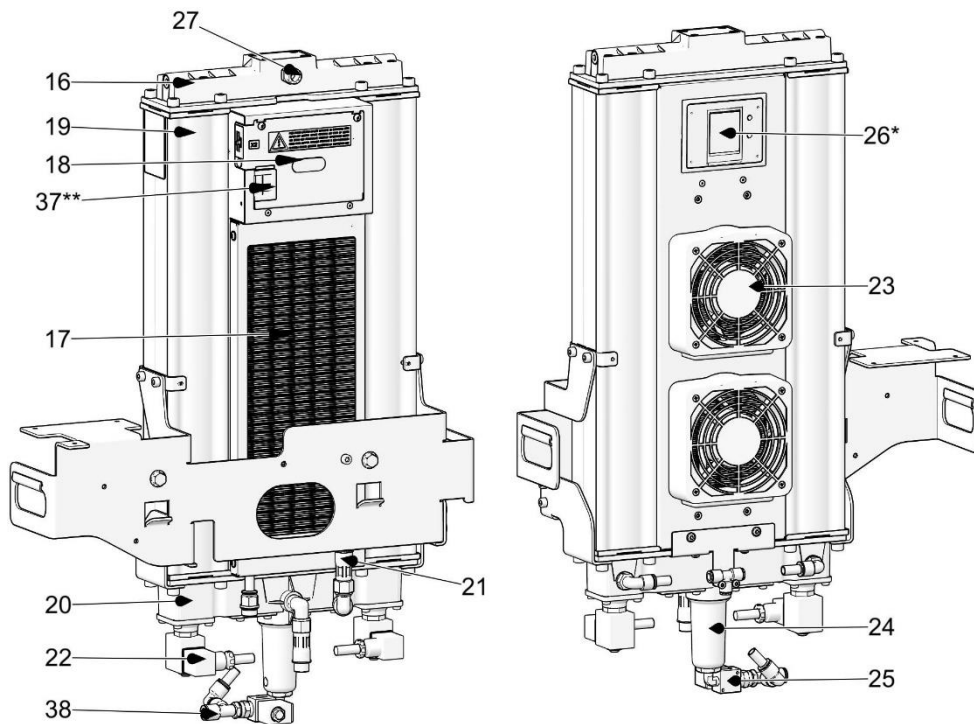ADM 280/50


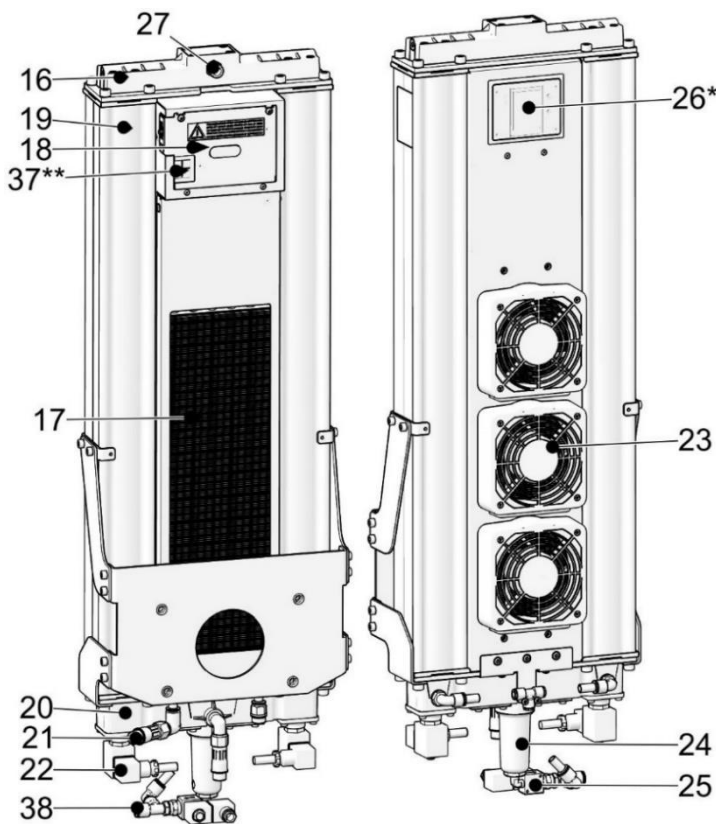
ADM 500/110


* Position (26) applies to the DK50 4VR/50/M, DK50 2x4VR/110/M model without the cabinet; position (26) in the case of the DK50 4VR/50 S/M, DK50 2x4VR/110 S/M with the cabinet is a simple cover.

** For the cabinet model, the cable connected to the display (33) in the front panel of the cabinet is routed through the universal bushing (37).

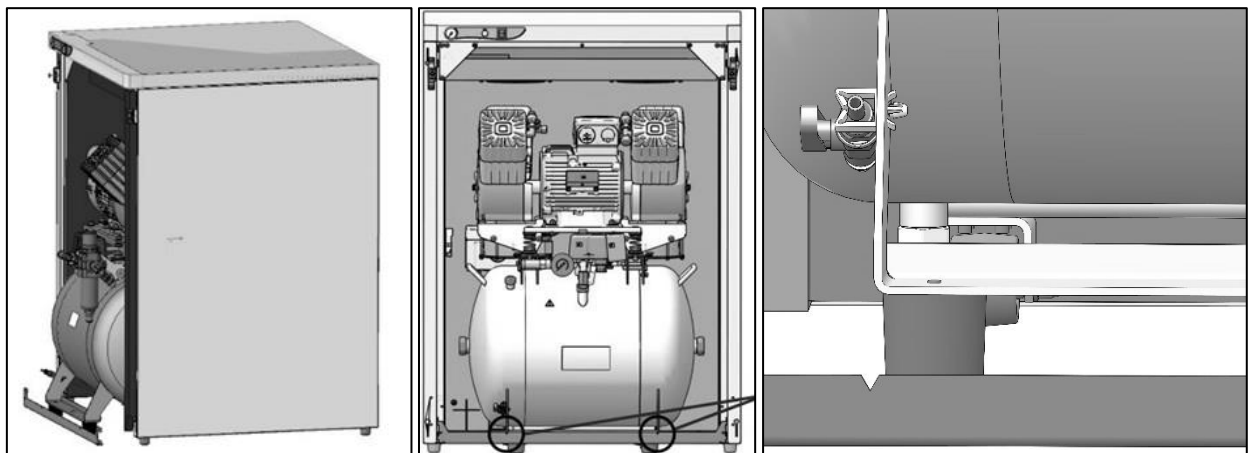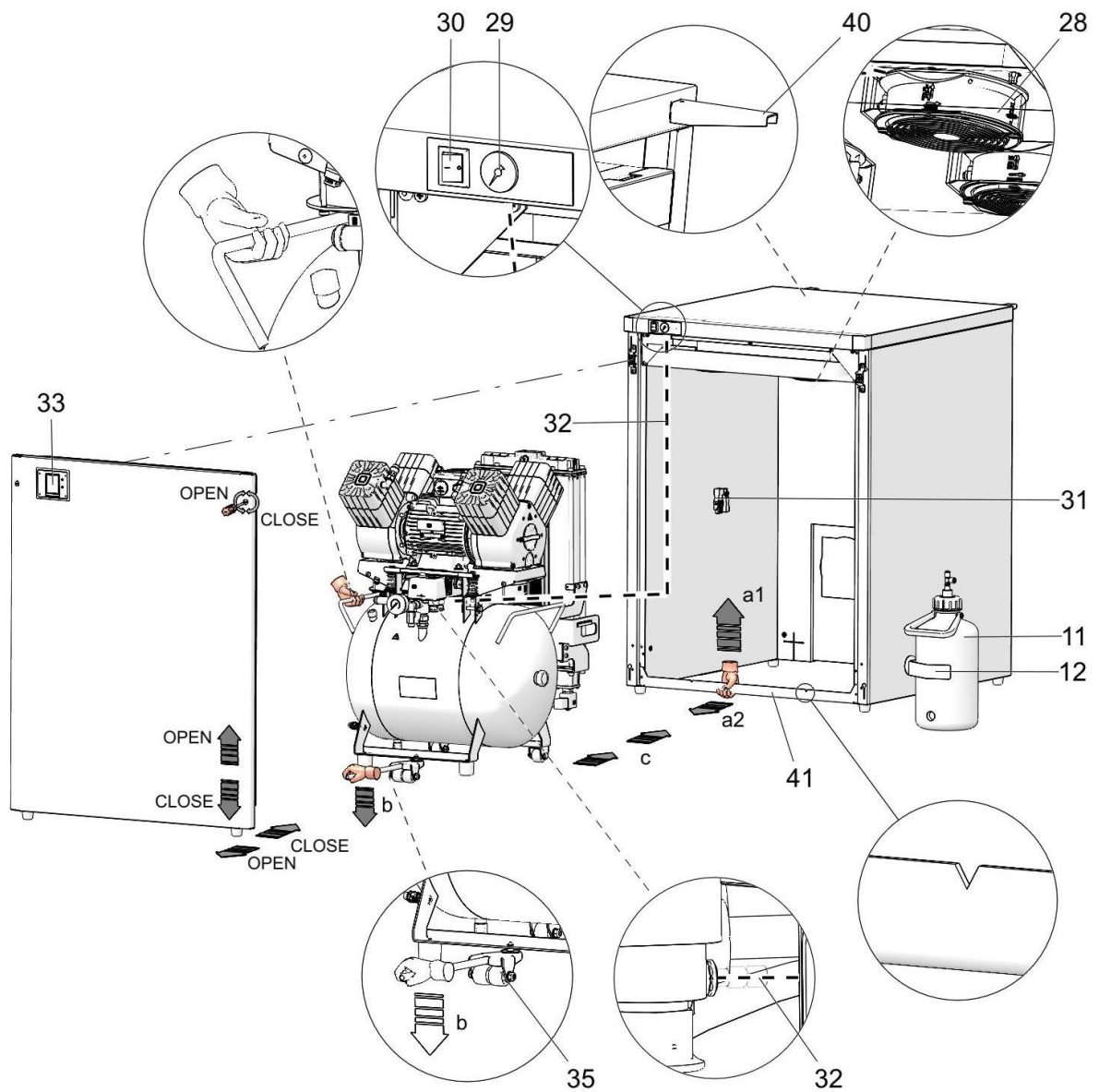Fig. 4: Cabinet

A - DK50 4VR/50S/M

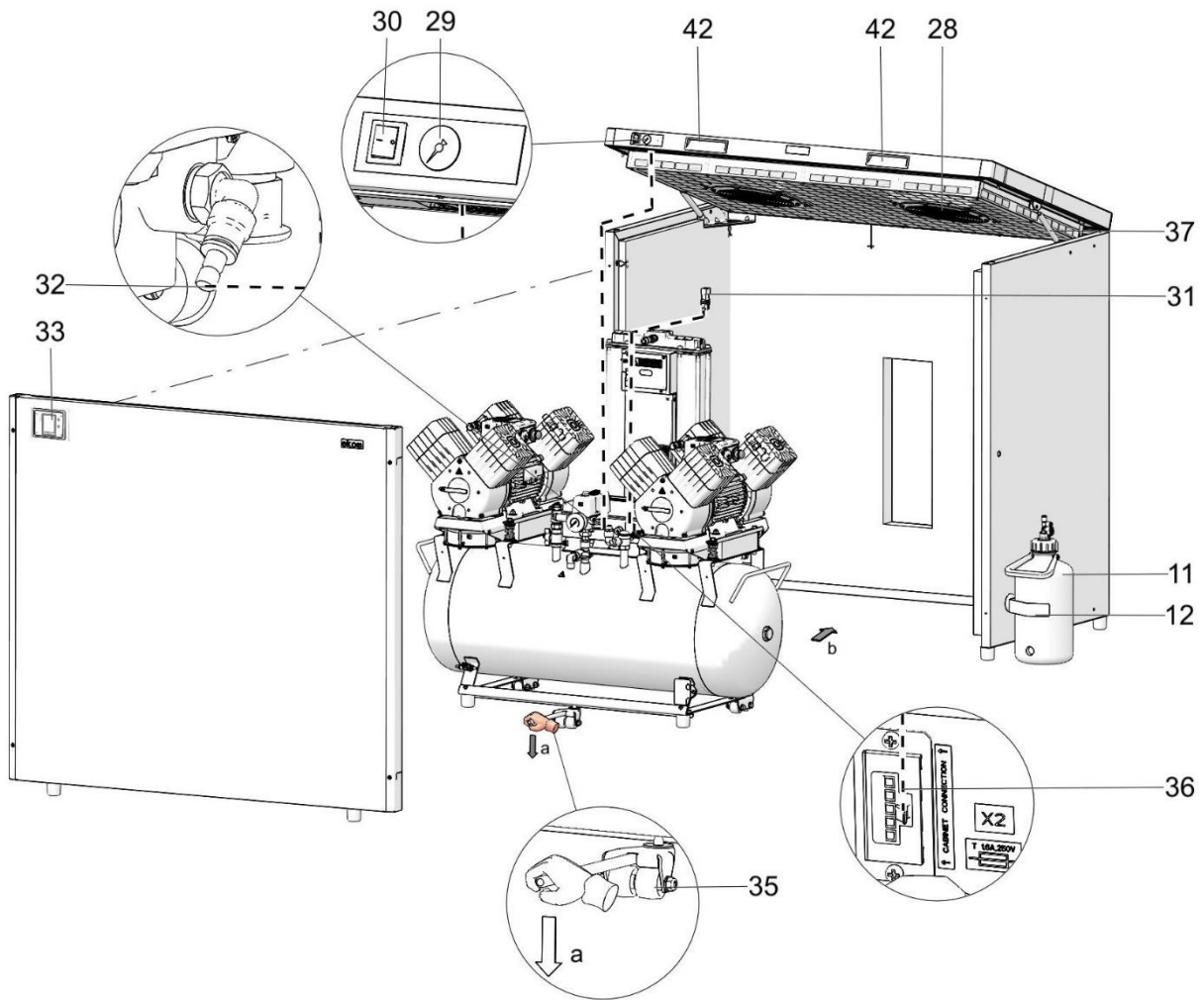B - DK50 2x4VR/110S/M

TECHNICAL DATA

Compressors are designed for operation in dry, ventilated and dust-free indoor rooms under the following climactic conditions:

Temperature +5°C to +40°C

Relative humidity max. 70%

Working pressure 6 – 8 bar		DK50 4VR/50/M	DK50 4VR/50 S/M
Rated voltage, Frequency ^{a)}	V, Hz	3x400, 50	3x400, 50
Capacity at 6 bar (FAD)	l/min	235	235
Working pressure ^{b)}	bar	6.0 – 8.0	6.0 – 8.0
Rated current	A	5.4	5.8
Motor power	kW	2.2	2.2
Air tank volume	l	50	50
Air quality - filtration	µm	-	-
Maximum operating pressure of safety valve	bar	9.0	9.0
Noise level at 5 bar (L _{pA})	dB	≤ 73.5	≤ 56.0
Operating mode	%	S1-100	S1-100
PDP drying performance at 7 bar ^{d)}	°C	≤ -20	≤ -20
Dryer low pressure alarm	bar	< 3	< 3
Dryer high pressure alarm	bar	> 9	> 9
Dryer high dew point alarm	°C	> -20	> -20
Time to fill air tank from 0 to 7 bar	s	85	85
Dimensions (net) w x d x h	mm	580x590x815	750x780x1015
Net weight ^{c)}	kg	95	151
Classification under EN 60601-1		Class I.	

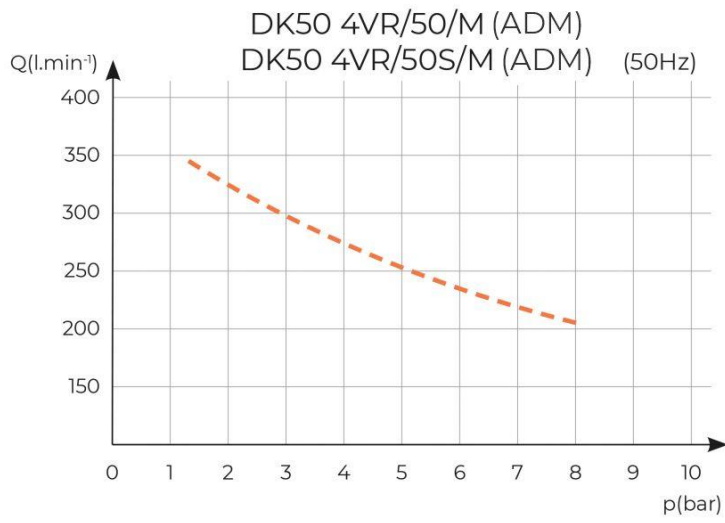
^{a)} Specify the compressor version when ordering

^{b)} Consult any other range of pressure with the supplier

^{c)} Weight is indicative and only applies to the product without accessories

^{d)} Applies to ambient temperatures of <30°C PDP – pressure dew point


Dependence of compressor output on working pressure

Working pressure 6 – 8 bar		DK50 4VR/50/M	DK50 4VR/50 S/M
Rated voltage, Frequency ^{a)}	V, Hz	3x400, 50/60	3x400, 50
Capacity at 6 bar (FAD)	l/min	225	225
Working pressure ^{b)}	bar	6.0 – 8.0	6.0 – 8.0
Rated current	A	5.4	5.8
Motor power	kW	2.2	2.2
Air tank volume	l	50	50
Air quality - filtration	µm	-	-
Maximum operating pressure of safety valve	bar	9.0	9.0
Noise level at 5 bar (L _{pA})	dB	≤ 73.5	≤ 56.0
Operating mode	%	S1-100	S1-100
PDP drying performance at 7 bar ^{d)}	°C	≤ -40	≤ -40
Dryer low pressure alarm	bar	< 3	< 3
Dryer high pressure alarm	bar	> 9	> 9
Dryer high dew point alarm	°C	> -40	> -40
Time to fill air tank from 0 to 7 bar	s	90	90
Dimensions (net) w x d x h	mm	580x590x815	750x780x1015
Net weight ^{c)}	kg	95	151
Classification under EN 60601-1		Class I.	

- ^{a)} Specify the compressor version when ordering
- ^{b)} Consult any other range of pressure with the supplier
- ^{c)} Weight is indicative and only applies to the product without accessories
- ^{d)} Applies to ambient temperatures of <30°C PDP – pressure dew point

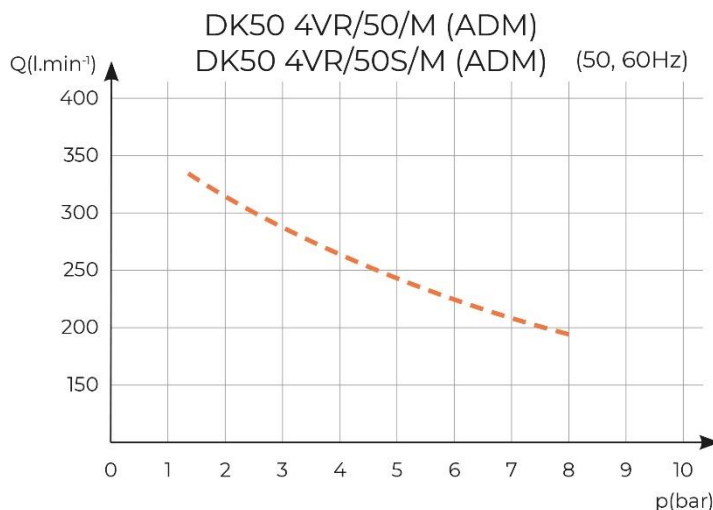Dependence of compressor output on working pressure

Working pressure 8 – 10 bar		DK50 4VR/50/M	DK50 4VR/50 S/M
Rated voltage, Frequency ^{a)}	V, Hz	3x400, 50	3x400, 50
Capacity at 8 bar (FAD)	l/min	195	195
Working pressure ^{b)}	bar	8.0 – 10.0	8.0 – 10.0
Rated current	A	5.4	5.9
Motor power	kW	2.2	2.2
Air tank volume	l	50	50
Air quality - filtration	µm	-	-
Maximum operating pressure of safety valve	bar	12.0	12.0
Noise level at 5 bar (L _{pA})	dB	≤ 73.5	≤ 56.0
Operating mode	%	S1-100	S1-100
PDP drying performance at 7 bar ^{d)}	°C	≤ -20	≤ -20
Dryer low pressure alarm	bar	< 3	< 3
Dryer high pressure alarm	bar	> 11	> 11
Dryer high dew point alarm	°C	> -20	> -20
Time to fill air tank from 0 to 7 bar	s	105	105
Dimensions (net) w x d x h	mm	580x590x815	750x780x1015
Net weight ^{c)}	kg	95	151
Classification under EN 60601-1		Class I.	

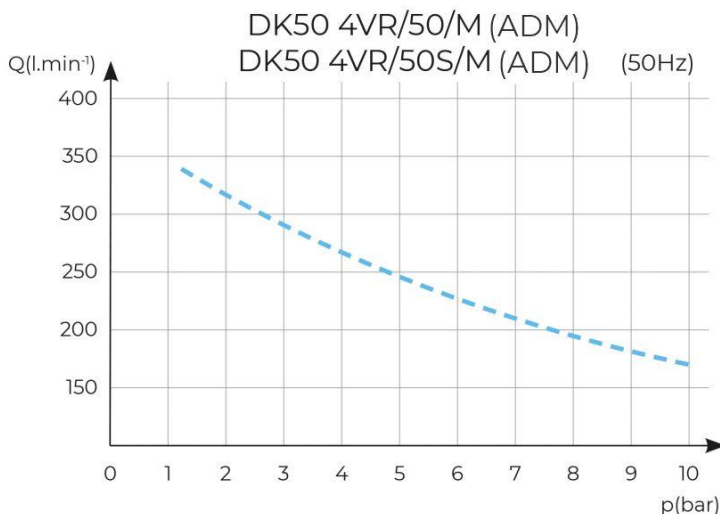
^{a)} Specify the compressor version when ordering

^{b)} Consult any other range of pressure with the supplier

^{c)} Weight is indicative and only applies to the product without accessories

^{d)} Applies to ambient temperatures of <30°C PDP – pressure dew point


Dependence of compressor output on working pressure

Working pressure 8 – 10 bar		DK50 4VR/50/M	DK50 4VR/50 S/M
Rated voltage, Frequency ^{a)}	V, Hz	3x400, 50	3x400, 50
Capacity at 8 bar (FAD)	l/min	190	190
Working pressure ^{b)}	bar	8.0 – 10.0	8.0 – 10.0
Rated current	A	5.4	5.9
Motor power	kW	2.2	2.2
Air tank volume	l	50	50
Air quality - filtration	µm	-	-
Maximum operating pressure of safety valve	bar	12.0	12.0
Noise level at 5 bar (L _{pA})	dB	≤ 73.5	≤ 56.0
Operating mode	%	S1-100	S1-100
PDP drying performance at 7 bar ^{d)}	°C	≤ -40	≤ -40
Dryer low pressure alarm	bar	< 3	< 3
Dryer high pressure alarm	bar	> 11	> 11
Dryer high dew point alarm	°C	> -40	> -40
Time to fill air tank from 0 to 7 bar	s	105	105
Dimensions (net) w x d x h	mm	580x590x815	750x780x1015
Net weight ^{c)}	kg	95	151
Classification under EN 60601-1		Class I.	

- ^{a)} Specify the compressor version when ordering
- ^{b)} Consult any other range of pressure with the supplier
- ^{c)} Weight is indicative and only applies to the product without accessories
- ^{d)} Applies to ambient temperatures of <30°C PDP – pressure dew point

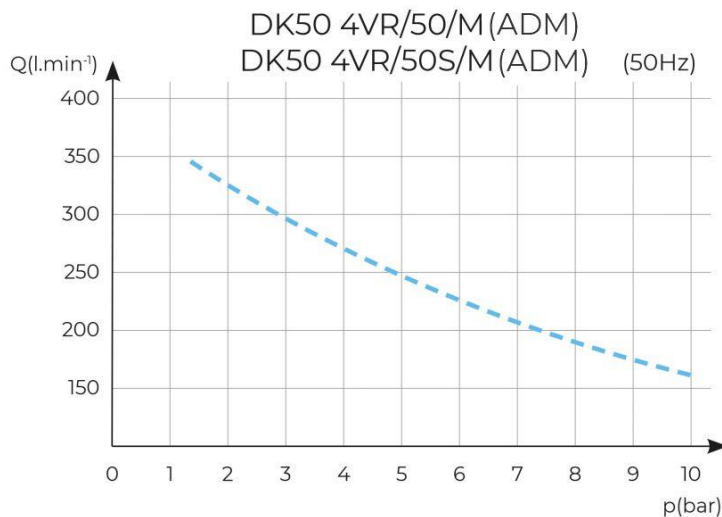Dependence of compressor output on working pressure

Working pressure 6 – 8 bar		DK50 2x4VR/110/M	DK50 2x4VR/110S/M
Rated voltage, Frequency ^{a)}	V, Hz	3x400V, 50Hz	3x400V, 50Hz
Capacity at 6 bar (FAD)	l/min	470	470
Working pressure ^{b)}	bar	6.0 – 8.0	6.0 – 8.0
Rated current	A	10.8	11.2
Motor power	kW	2x2.2	2x2.2
Air tank volume	l	110	110
Air quality - filtration	µm	-	-
Maximum operating pressure of safety valve	bar	9.0	9.0
Noise level at 5 bar (L _{pA})	dB	≤ 77.5	≤ 62.0
Operating mode	%	S1-100	S1-100
PDP drying performance at 7 bar ^{d)}	°C	≤ -20	≤ -20
Dryer low pressure alarm	bar	< 3	< 3
Dryer high pressure alarm	bar	> 9	> 9
Dryer high dew point alarm	°C	> -20	> -20
Time to fill air tank from 0 to 7 bar	s	95	95
Dimensions (net) w x d x h	mm	1085x675x1050	1260x830x1205
Net weight ^{c)}	kg	186	286
Classification under EN 60601-1		Class I.	

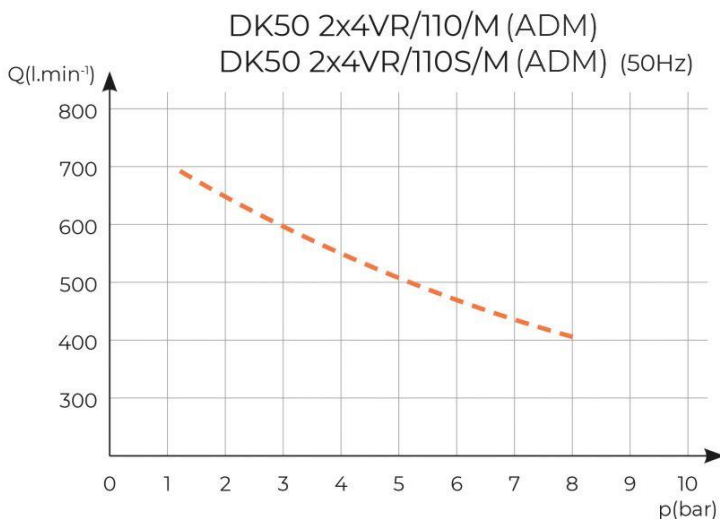
^{a)} Specify the compressor version when ordering

^{b)} Consult any other range of pressure with the supplier

^{c)} Weight is indicative and only applies to the product without accessories

^{d)} Applies to ambient temperatures of <30°C PDP – pressure dew point


Dependence of compressor output on working pressure

Working pressure 6 – 8 bar		DK50 2x4VR/110/M	DK50 2x4VR/110S/M
Rated voltage, Frequency ^{a)}	V, Hz	3x400V, 50Hz	3x400V, 50Hz
Capacity at 6 bar (FAD)	l/min	450	450
Working pressure ^{b)}	bar	6.0 – 8.0	6.0 – 8.0
Rated current	A	10.8	11.2
Motor power	kW	2x2.2	2x2.2
Air tank volume	l	110	110
Air quality - filtration	µm	-	-
Maximum operating pressure of safety valve	bar	9.0	9.0
Noise level at 5 bar (L _{pA})	dB	≤ 77.5	≤ 62.0
Operating mode	%	S1-100	S1-100
PDP drying performance at 7 bar ^{d)}	°C	≤ -40	≤ -40
Dryer low pressure alarm	bar	< 3	< 3
Dryer high pressure alarm	bar	> 9	> 9
Dryer high dew point alarm	°C	> -40	> -40
Time to fill air tank from 0 to 7 bar	s	100	100
Dimensions (net) w x d x h	mm	1085x675x1050	1260x830x1205
Net weight ^{c)}	kg	186	286
Classification under EN 60601-1		Class I.	

- ^{a)} Specify the compressor version when ordering
- ^{b)} Consult any other range of pressure with the supplier
- ^{c)} Weight is indicative and only applies to the product without accessories
- ^{d)} Applies to ambient temperatures of <30°C PDP – pressure dew point

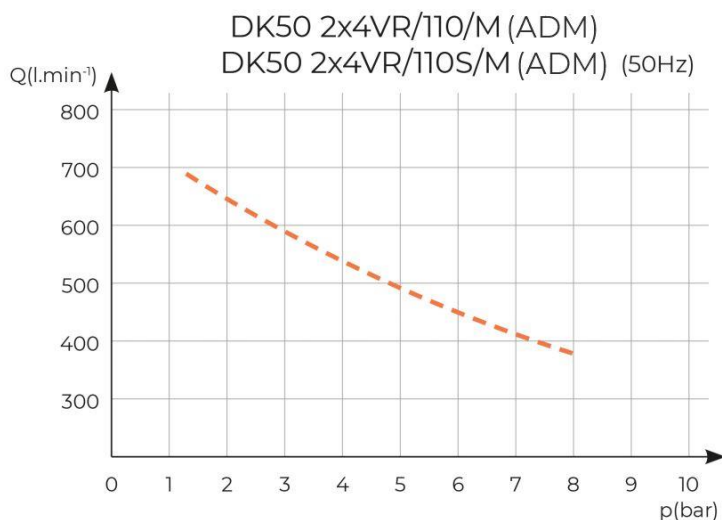Dependence of compressor output on working pressure

Working pressure 8 – 10 bar		DK50 2x4VR/110/M	DK50 2x4VR/110S/M
Rated voltage, Frequency ^{a)}	V, Hz	3x400V, 50Hz	3x400V, 50Hz
Capacity at 8 bar (FAD) -20°C	l/min	395	395
Working pressure ^{b)}	bar	8.0 – 10.0	8.0 – 10.0
Rated current	A	10.3	10.8
Motor power	kW	2x2.2	2x2.2
Air tank volume	l	110	110
Air quality - filtration	µm	-	-
Maximum operating pressure of safety valve	bar	12.0	12.0
Noise level at 5 bar (L _{pA})	dB	≤ 77.5	≤ 62.0
Operating mode	%	S1-100	S1-100
PDP drying performance at 7 bar ^{d)}	°C	≤ -20	≤ -20
Dryer low pressure alarm	bar	< 3	< 3
Dryer high pressure alarm	bar	> 11	> 11
Dryer high dew point alarm	°C	> -20	> -20
Time to fill air tank from 0 to 7 bar	s	115	115
Dimensions (net) w x d x h	mm	1085x675x1050	1260x830x1205
Net weight ^{c)}	kg	186	286
Classification under EN 60601-1		Class I.	

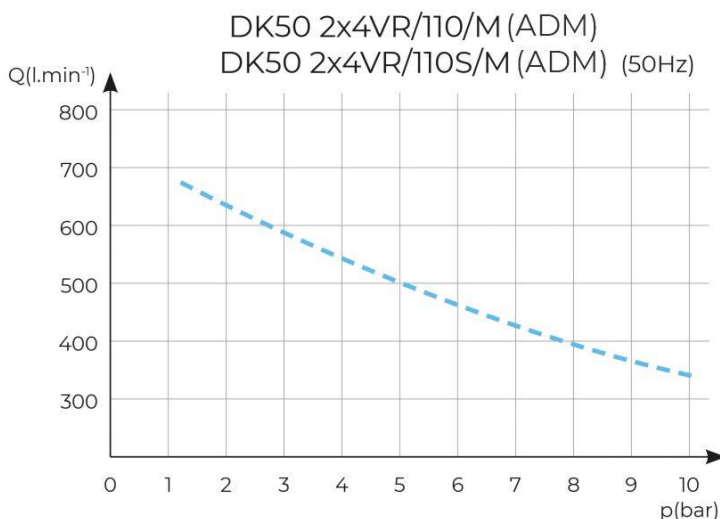
^{a)} Specify the compressor version when ordering

^{b)} Consult any other range of pressure with the supplier

^{c)} Weight is indicative and only applies to the product without accessories

^{d)} Applies to ambient temperatures of <30°C PDP – pressure dew point


Dependence of compressor output on working pressure

Working pressure 8 – 10 bar		DK50 2x4VR/110/M	DK50 2x4VR/110S/M
Rated voltage, Frequency ^{a)}	V, Hz	3x400V, 50Hz	3x400V, 50Hz
Capacity at 8 bar (FAD)	l/min	380	380
Working pressure ^{b)}	bar	8.0 – 10.0	8.0 – 10.0
Rated current	A	10.3	10.8
Motor power	kW	2x2.2	2x2.2
Air tank volume	l	110	110
Air quality - filtration	µm	-	-
Maximum operating pressure of safety valve	bar	12.0	12.0
Noise level at 5 bar (L _{pA})	dB	≤ 77.5	≤ 62.0
Operating mode	%	S1-100	S1-100
PDP drying performance at 7 bar ^{d)}	°C	≤ -40	≤ -40
Dryer low pressure alarm	bar	< 3	< 3
Dryer high pressure alarm	bar	> 11	> 11
Dryer high dew point alarm	°C	> -40	> -40
Time to fill air tank from 0 to 7 bar	s	120	120
Dimensions (net) w x d x h	mm	1085x675x1050	1260x830x1205
Net weight ^{c)}	kg	186	286
Classification under EN 60601-1		Class I.	

- ^{a)} Specify the compressor version when ordering
- ^{b)} Consult any other range of pressure with the supplier
- ^{c)} Weight is indicative and only applies to the product without accessories
- ^{d)} Applies to ambient temperatures of <30°C PDP – pressure dew point

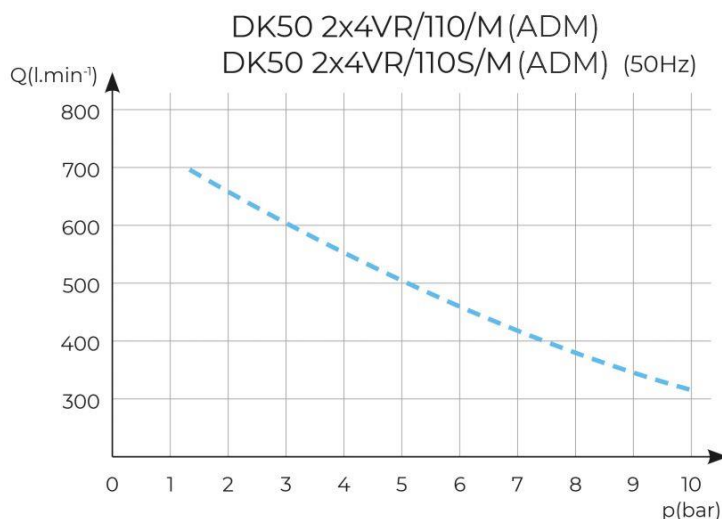Dependence of compressor output on working pressure

FAD correction of capacity for altitude

Capacity given in the form of FAD („Free Air Delivery“) applies to the following conditions:

Altitude	0 m.n.m.	Temperature	20°C
Atmospheric pressure	101325 Pa	Relative humidity	0%

To calculate FAD compressor capacity in dependence on altitude, it is necessary to apply correction factor according to the following table:

Altitude [m.n.m.]	0 -1500	1501 - 2500	2501 - 3500	3501 - 4500
FAD correction factor	1	0.80	0.71	0.60

Electromagnetic compatibility declaration

Medical equipment needs special precautions regarding electromagnetic compatibility (EMC) and needs to be installed and put into service according to the EMC information provided below.

Guidance and manufacturer's declaration - electromagnetic emissions		
Pursuant to IEC 60601-1-2:2014/AMD1:2020 - Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances		
The equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the equipment should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The equipment uses RF energy only for its internal functions. Therefore, the RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The equipment is suitable for use in all establishments, including domestic establishments and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	The equipment is not likely to cause any flicker, as the current flow is approx. constant after the start up.	

Guidance and manufacturer's declaration - electromagnetic immunity			
Pursuant to IEC 60601-1-2:2014/AMD1:2020 - Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances			
The equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the equipment should assure that it is used in such an environment.			
Immunity test	IEC 60601-1-2 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±15 kV air	±8 kV contact ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV 100 kHz repetition frequency Applied to mains connection	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV differential mode ±2 kV common mode	±1 kV L-N ±2 kV L-PE; N-PE Applied to mains connection	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruption, and voltage variations on power supply input lines IEC 60601-4-11	$U_T=0\%$, 0,5 cycle (at 0, 45, 90, 135, 180, 225, 270 and 315°) $U_T=0\%$, 1 cycle $U_T=70\%$ 25/30 cycles (at 0°) $U_T=0\%$, 250/300 cycles	$U_T=>95\%$, 0,5 cycle (at 0, 45, 90, 135, 180, 225, 270 and 315°) $U_T=>95\%$, 1 cycle $U_T=70\%$ (30% dipp U_T), 25(50Hz)/30(60Hz) cycles (at 0°) $U_T=>95\%$, 250(50Hz)/300(60Hz) cycle	Mains power quality should be that of a typical commercial or hospital environment. The device stops and restarts automatically at each dip. In this case does not occur unacceptable pressure drop.
Power frequency (50/60 Hz) IEC 61000-4-8	N/A	N/A	The equipment does not contain magnetically sensitive components or circuits.
Radiated fields in close proximity per IEC 61000-4-39	N/A	N/A	The equipment does not contain magnetically sensitive components or circuits.
NOTE: U_T is the A.C. mains voltage prior to application of the test level.			

Guidance and manufacturer's declaration - electromagnetic immunity			
Pursuant to IEC 60601-1-2:2014/AMD1:2020 - Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances			
The equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the equipment should assure that it is used in such an environment.			
Immunity test	IEC 60601-1-2 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80MHz	3 Vrms	<p>Portable and mobile RF communications equipment should be used no closer to any part of the equipment, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> $d=1,2\sqrt{P}$ $d=1,2\sqrt{P}, 80 \text{ MHz to } 800 \text{ MHz}$ $d=2,3\sqrt{P}, 800 \text{ MHz to } 2,7 \text{ GHz}$ <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a, should be less than the compliance level in each frequency range ^b. Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2,7 GHz	3 V/m	
Proximity fields from RF wireless communications equipment IEC 61000-4-3	9 to 28 V/m 15 specific frequencies (380 to 5800 MHz)	9 to 28 V/m 15 specific frequencies (380 to 5800 MHz)	
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.			
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the equipment is used exceeds the applicable RF compliance level above, the equipment should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the equipment.			
^b Over the frequency range 150 kHz to 80 kHz, field strengths should be less than 3 V/m.			

INSTALLATION

Risk of incorrect installation.

Only a qualified technician may install the compressor and place it into operation for the first time. Their duty is to train operating personnel on the use and maintenance of the equipment. An entry is made in the equipment installation record to certify installation and operator training. (see the Annex chapter)

10. INSTALLATION CONDITIONS

- The compressor may only be installed and operating in dry, well-ventilated and clean environments under the conditions specified in the Technical Data chapter.

Risk of damage to the device.

The equipment may not be operated outdoors or in otherwise wet or damp environments.

Risk of explosion.

Do not use the equipment in the presence of explosive gases, dust or combustible liquids.

Burn or fire hazard! Caution! Hot surface!

Portions of the compressor, dryer may be hot and reach hazardous temperatures during compressor operation that may harm materials or operating staff.

- The compressor must be installed so that it is accessible at all times for operating and maintenance. Please ensure that the nameplate on the device is readily accessible.
- The compressor must stand on a flat, sufficiently stable base (be aware of the weight of the compressor, see the Technical Data chapter).

High temperature hazard.

The placement of air flow impediments upstream or downstream of the cooler is prohibited. The temperature of internal and external parts of the cooler may be hot and reach hazardous temperatures.

You may notice a “new product” odour when you first place the product into service (for a short period of time). This odour is temporary and does not impede the normal use of the product. Ensure the space is properly ventilated after installation.

11. COMPRESSOR ASSEMBLY

11.1. Handling and releasing the compressor

- Unpack the compressor from the packaging.
- For a compressor with a cabinet, remove the door held on the screws and for DK50 4VR/50/M compressors, remove the connecting strip (41) in the lower part of the cabinet. Remove the cabinet from the compressor.
- Detach the compressor from the pallet by removing the mounting bolts (Fig. 5).
- Position the compressor at the site of installation. (Fig. 6)


Use the handles on the compressor to move the compressor as needed. Do not use other parts of the compressor (the air pump, cooler, etc.) for grip


The number of persons required to handle the equipment must be matched to the weight of the device.

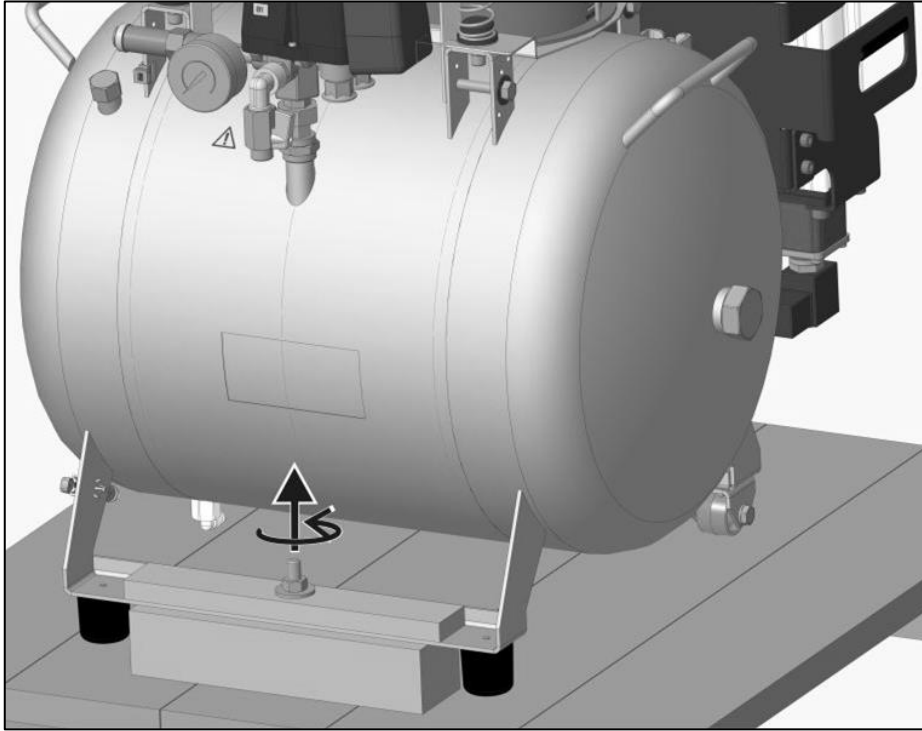Fig. 5: Removing the transport mounts




DK50 4VR/50


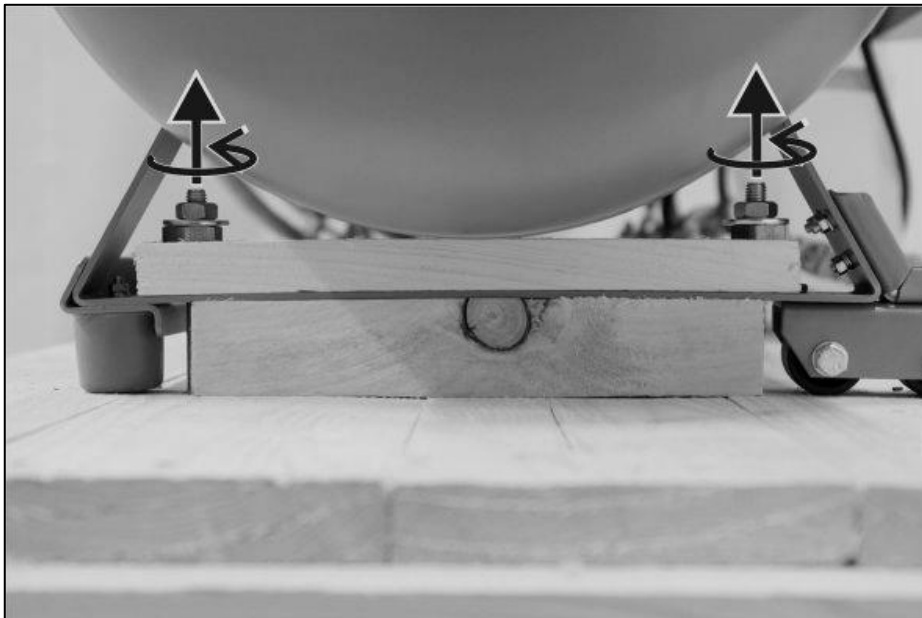
DK50 2x4VR/110

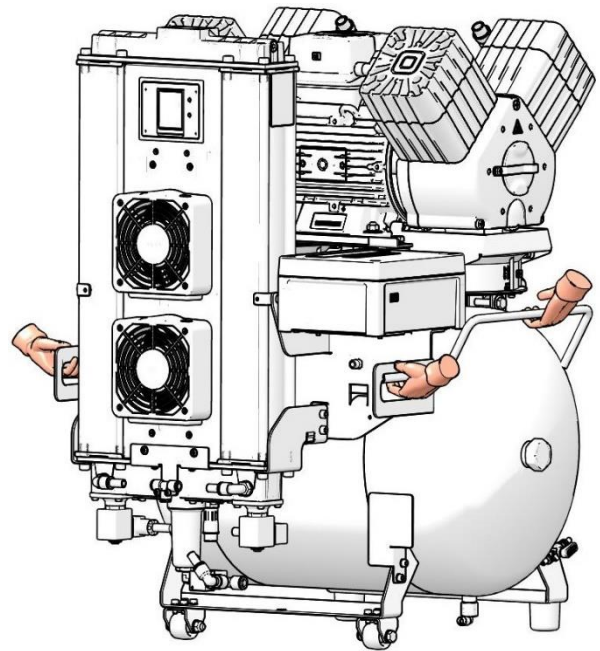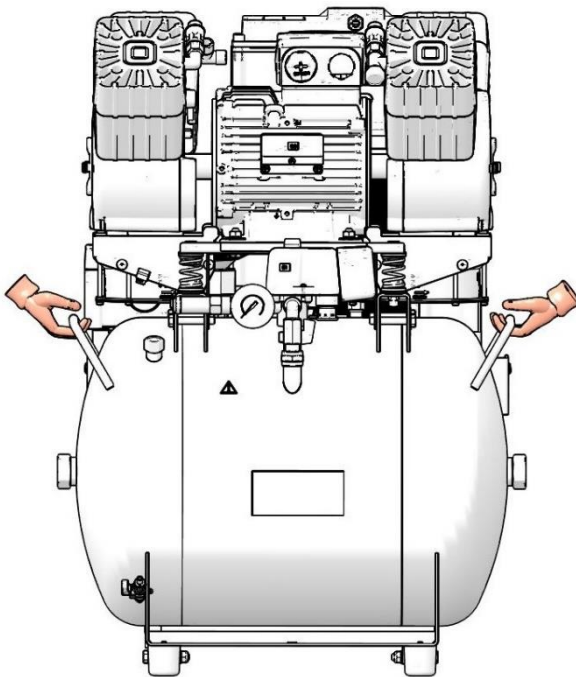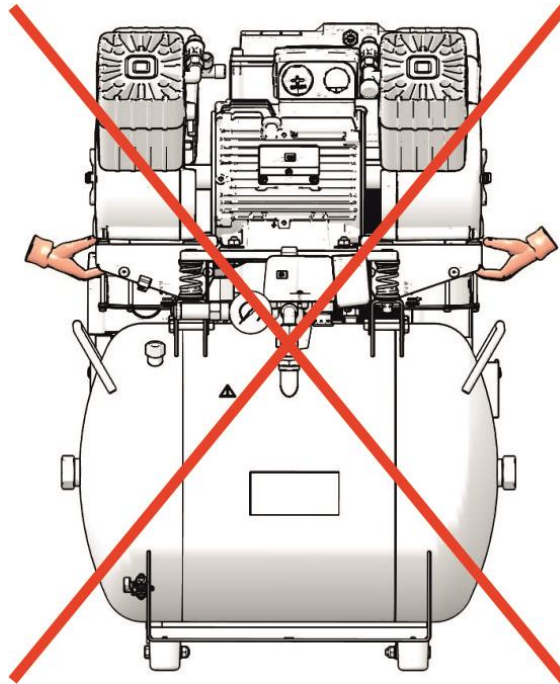Fig. 6: Handling the compressor

DK50 4VR/50

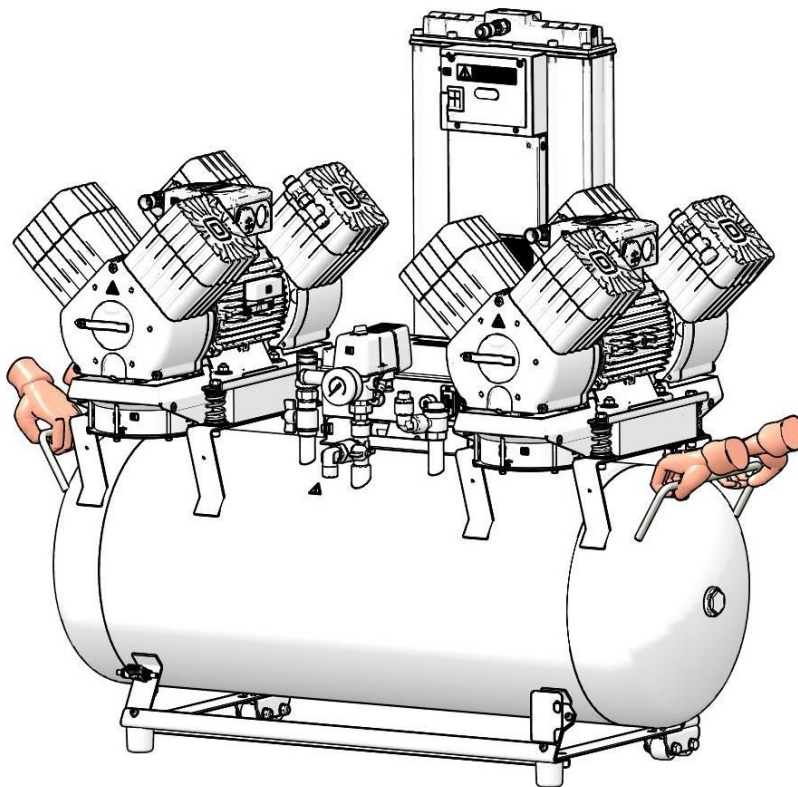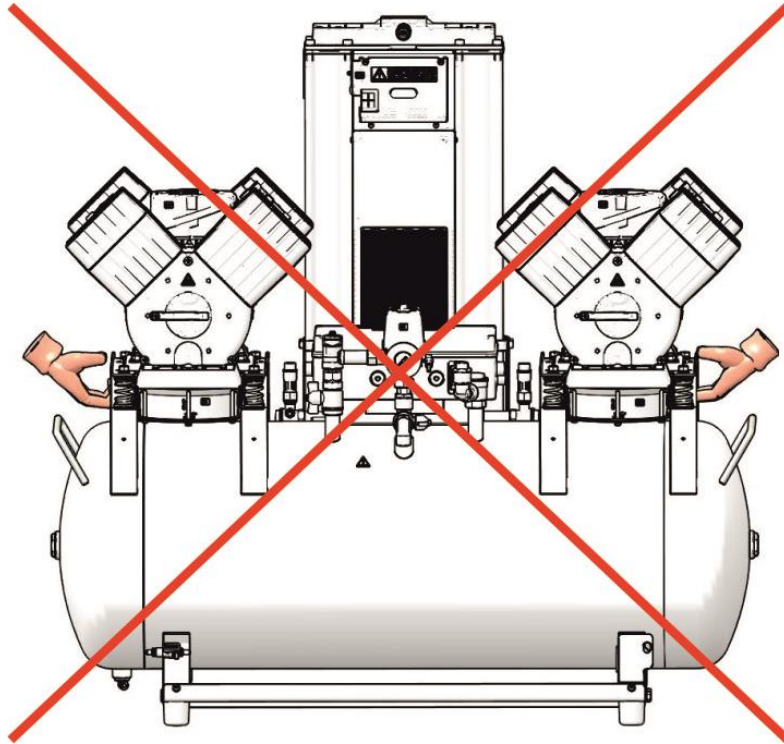DK50 2x4VR/110


- Remove the mounts used to secure the aggregate in transport: use a 10 mm wrench to remove the upper bolts, and then use a 13 mm wrench to remove the lower side bolts (Fig. 7).


Prior to installation, ensure that the compressor is free of all transport packaging and stabilizers to avoid any risk of damage to the product.

Remove all devices used to secure the air pumps once the compressor is installed and levelled at the site of final installation.

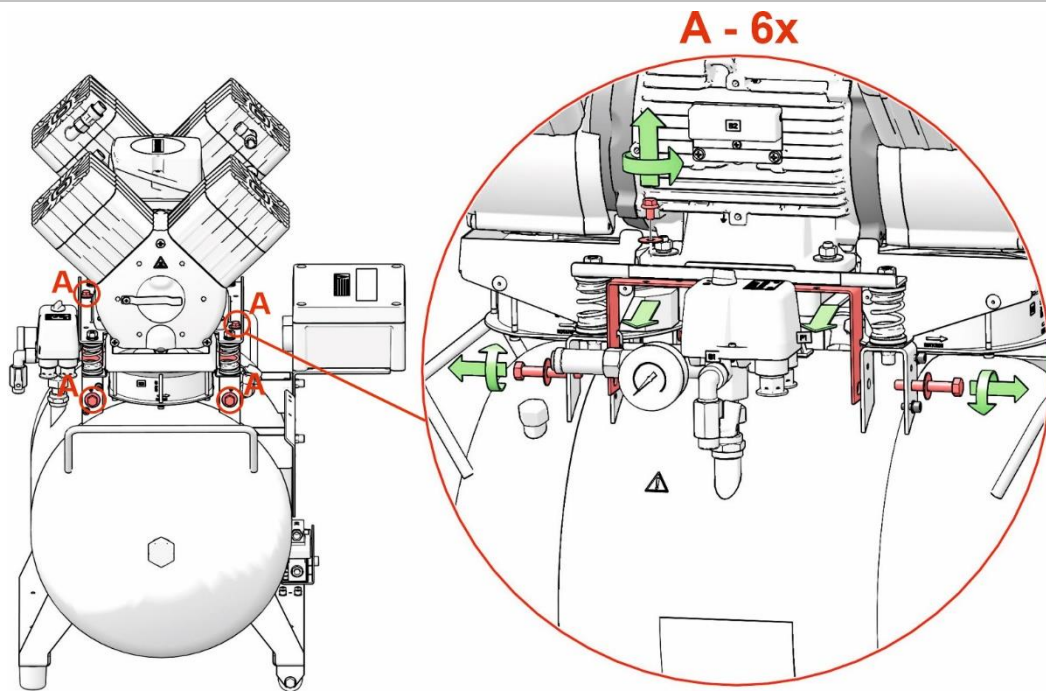Fig. 7: Releasing the air pumps

DK50 4VR/50

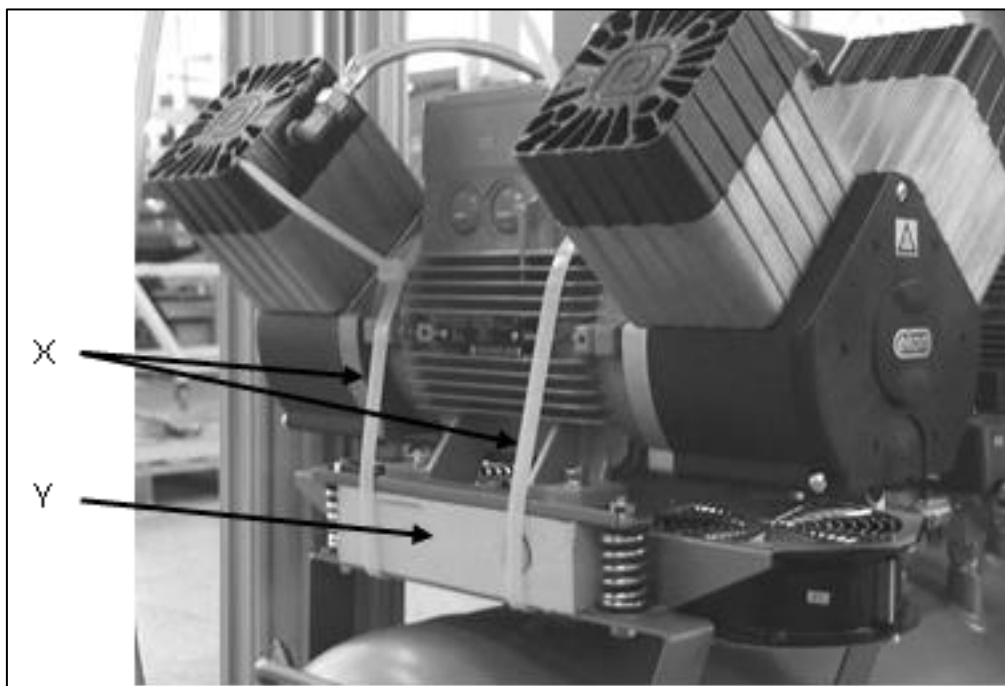DK50 2x4VR/110

12. PNEUMATIC CONNECTION

12.1. Connecting to the compressed air outlet

- Route the pressure hose from the compressed air outlet (1) to the connected equipment.
- A G 3/8" (DK50 4VR/50), G1/2" (DK50 2x4VR/110) connection is installed.
- Route the pressure hose through the opening in the rear wall of the cabinet for cabinet-mounted compressors.

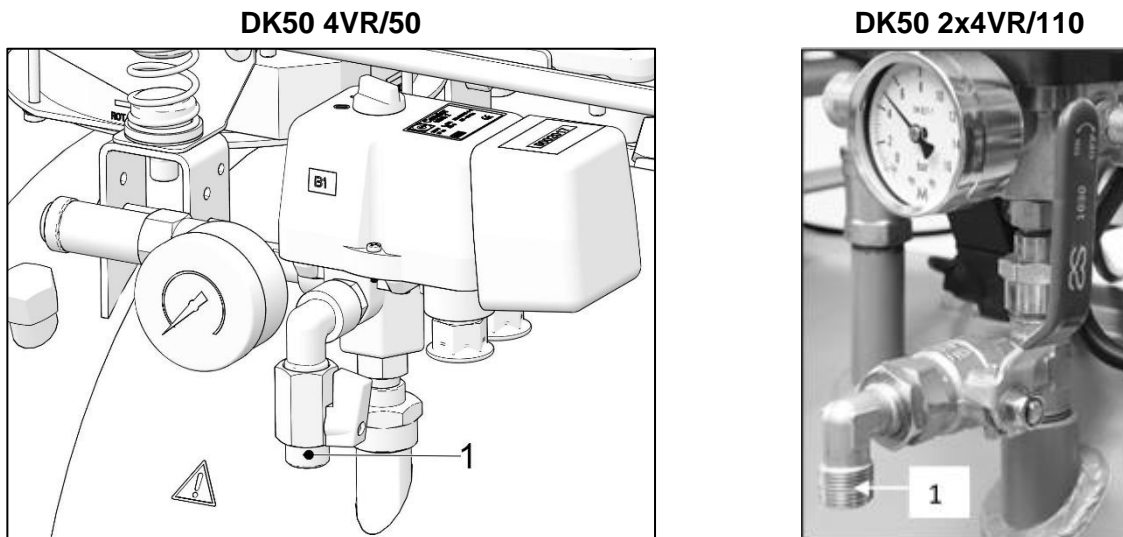
Fig. 8: Connecting to the compressed air outlet

12.2. Connecting the cabinet pressure gauge to the compressor

- Remove the plug (1) from the threads (2) on the pneumatic block of the compressor
- Connect the cabinet pressure gauge hose to the threaded fitting

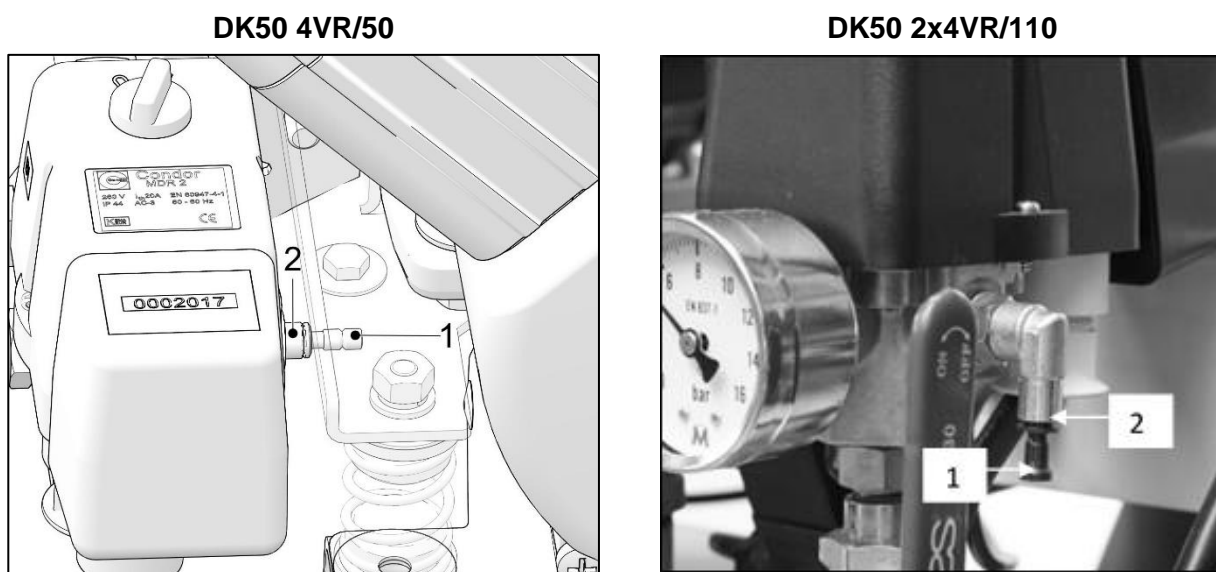
Fig. 9: Connecting the cabinet pressure gauge to the compressor

12.3. Condensate outlet from dryer


Injury hazard.


Do not connect the condensate drain directly to a waste drain! Passers-by may be injured!

- Connect a hose to the outlet (1) from the automatic condensate drain (2) to the provided collection vessel.
- For a compressor in a cabinet, route the hose through the rear opening in the cabinet. Mount the condensate collection tank to the side panel or rear panel of the cabinet.

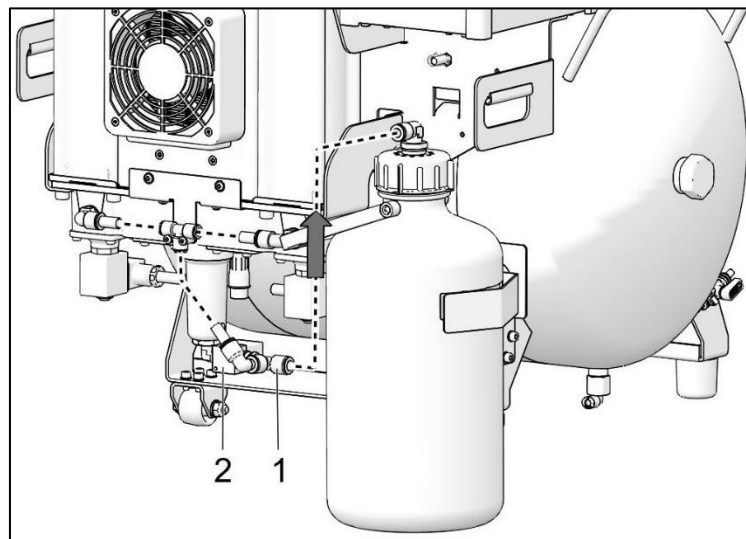Fig. 10: Condensate drain

DK50 4VR/50/M

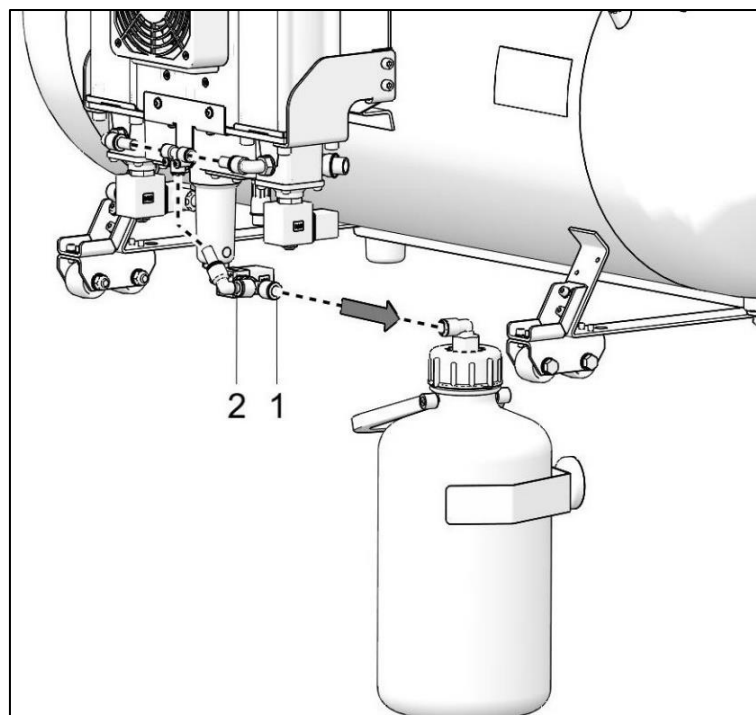DK50 2x4VR/110/M

Risk of damage to pneumatic components.
Air hoses must not be broken.

13. ELECTRICAL CONNECTION

- The product is delivered with a cord equipped with a plug with earthing pin.
- With cabinet-mounted compressors, route the power cord through the opening in the back wall of the cabinet.
- Keep the socket easily accessible to ensure that the device can be safely disconnected from the mains.
- Connection to the power distribution cabinet must be max. 16 A.

Risk of electric shock.

It is necessary to follow all local electro technical regulations. The mains voltage and frequency must comply with the data stated on the device label.

Risk of fire and electric shock.

Electrical cord must not be broken.

Risk of fire and electric shock.

Electrical cable must not be in contact with hot compressor components.

- Connect the pin for equipotential bonding 6Ø mm (1) to the electrical circuit using the defined method under valid electrical engineering regulations
- The socket for equipotential bonding (2) is an accessory and is not provided in the basic package

Fig. 11 :Equipotential bonding

- Mount the connector of the grounding conductor on the door and screw on the cabinet.

Power supply connection

Supply voltage	Electric wire type
230V/50 (60) Hz	3Gx1.5x4000
400V/50 (60) Hz	5Gx1.5x4000

13.1. Connecting a compressor not installed in a cabinet

- Insert the mains plug into a rated mains socket.
- The compressor is ready for operation.

13.2. Connecting a compressor installed in a cabinet

- Route the mains plug through the opening in the rear wall of the cabinet for cabinet-mounted compressors. (Fig. 12)
- Connect the cabinet electrically to the compressor by inserting the provided power cord with connector into an outlet. (Fig. 13)
- Disconnect the cabinet electrically from the compressor by pulling the connector from the outlet while the latch is released.
- Connect the display at the front door of the cabinet and then secure the cord to the display in the clamps (1). Secure the cord for DK50 4VR/50 S/M compressors in the clamp (1) in the cabinet door (Fig. 18). Secure the cord for DK50 2x4VR/110 S/M compressors in the clamps (1) in the cabinet enclosure (Fig. 22).

Fig. 12: Opening in the rear wall of the cabinet

Fig. 13: Connect the cabinet to the compressor

14. PLACEMENT OF THE COMPRESSOR IN THE CABINET

14.1. DK50 4VR/50 S/M (Fig. 4-A):

Disassembly cabinet door

- Remove the door held by 2 screws and disconnect the display connector and the earthing lead.

Check to ensure the display connector is removed prior to removing the front door as otherwise it could be damaged.

Compressor placement

- Remove the connecting strip (41) in front of the cabinet.
- Position the compressor within a distance of at least 500 mm from the cabinet to facilitate the movement of the hoses and the power cord within the cabinet (Fig. 14).

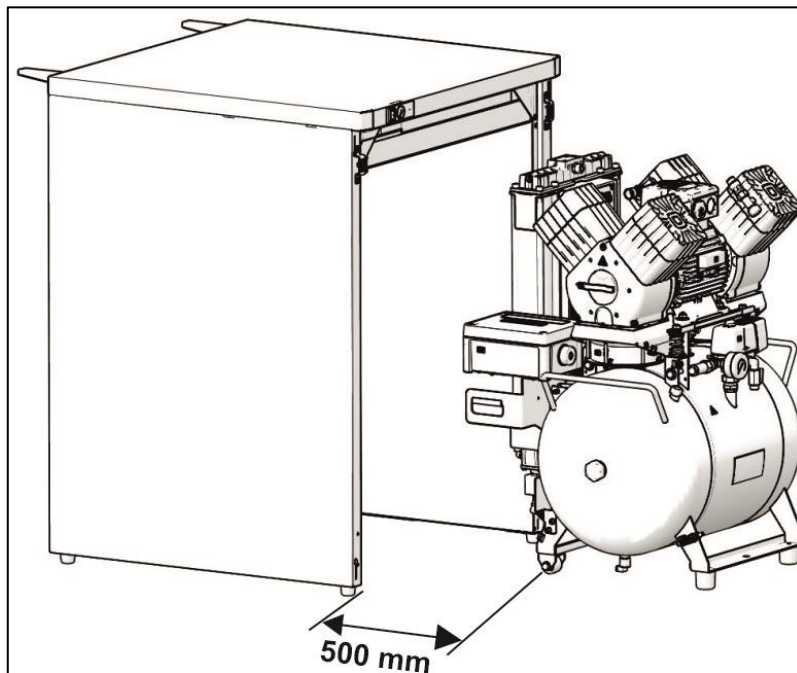
Fig. 14: Positioning of the compressor for connections

- Make the pneumatic connections per Chapter 12.1.
- Connect the condensate collection tank per Chapter 12.3.
- Route the pressure hose, the condensate drain hose, and the power cord through the opening in the rear wall of the cabinet.
- Position the compressor at such a distance from the cabinet so the WINSTA connector can make the electrical connection between the compressor and the cabinet (Fig. 13, Fig. 15).

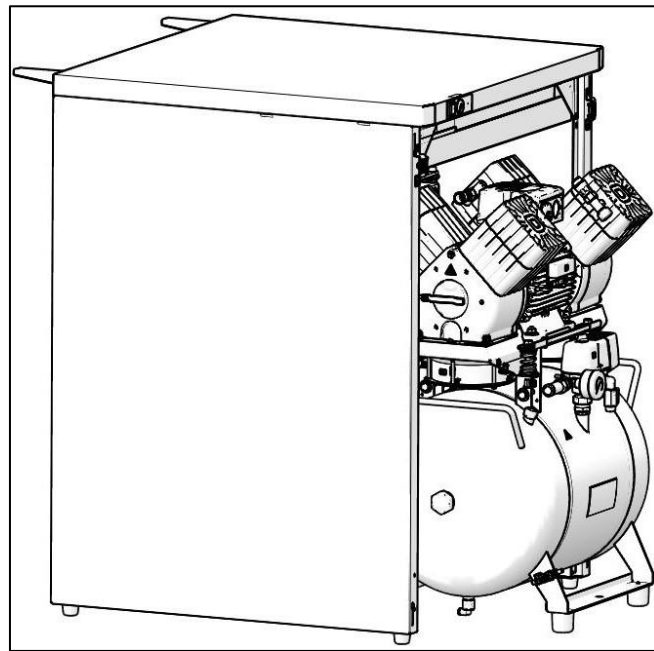
Fig. 15: Positioning the compressor for electrical connections

- Electrically connect the compressor per Chapter 13.
- Insert the compressor into the cabinet using the positioning truck (35), place the connecting strip in front of the cabinet and adjust the position of the compressor so that the two “V” notches on the connecting strip (41) align with the compressor legs (Fig. 4).
- Check to ensure the compressor is fully inserted into the cabinet using the control dimension (Fig. 16).

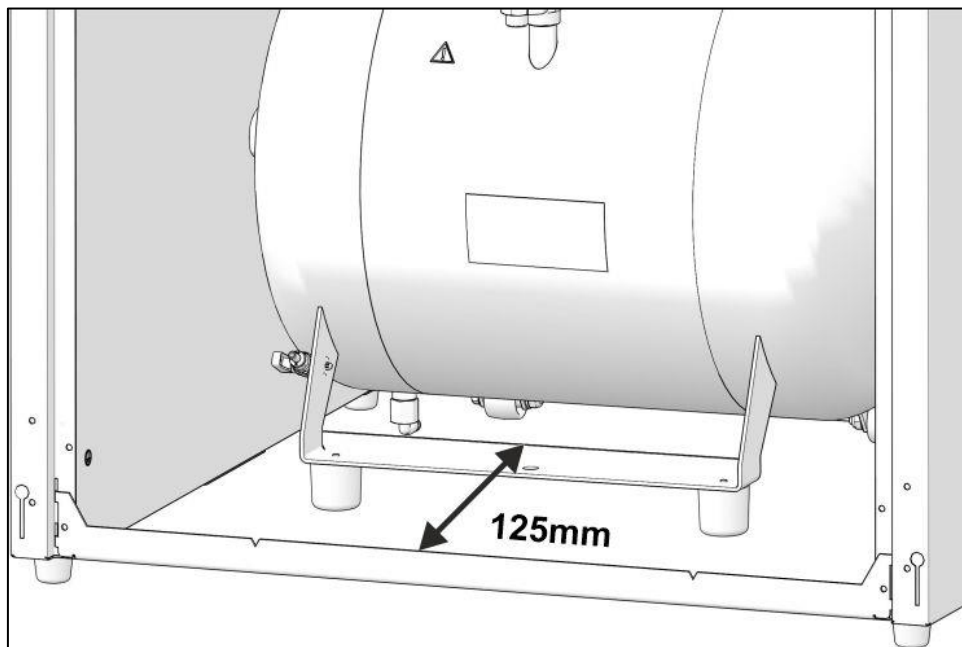
Fig. 16: Positioning of the DK50 4VR/50/M compressor in the cabinet

- Place the connecting strip (41) in its original position in the lower part of the cabinet.
- Connect the cabinet pressure gauge to the compressor per Chapter 12.2.

Do not push the compressor all the way to the back of the cabinet as it may permanently damage the dryer.

Cabinet door installation

- Attach the door to the cabinet, connect the display connector (Fig. 17) and secure the cord to the display in the clamp (1) in the front door (Fig. 18).
- Connect the earthing lead to the door and mount the door to the cabinet using 2 screws.

Fig. 17: Connecting the display connector

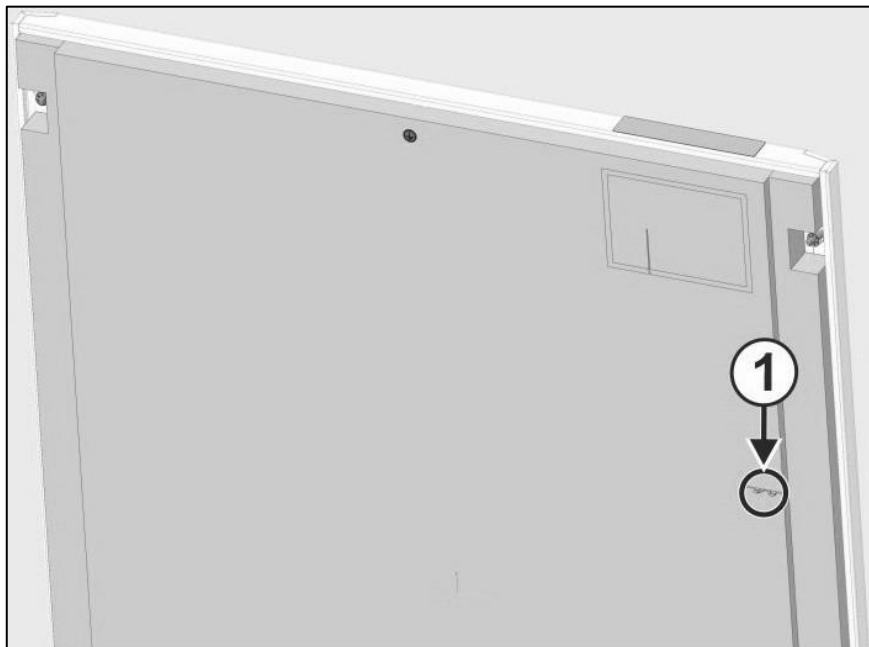
Fig. 18: DK50 4VR/50 S/M cabinet clamps

14.2. DK50 2x4VR/110 S/M (Fig. 4-B):

Opening the upper cover

- Lift the upper cabinet cover grabbing it by its handle (42) so that the gas springs keep the cover open.

**Be careful not to pinch your fingers when closing the cover on the cabinet.
Always lock with the quick releases after closing the cover on the cabinet!**

Removal of the front panel on the cabinet

- Remove the door held by 4 M5 screws and disconnect the display connector and the earthing lead.

Check to ensure the display connector is removed prior to removing the front door as otherwise it could be damaged.

Compressor placement

- Position the compressor within a distance of at least 500 mm from the cabinet to facilitate the movement of the hoses and the power cord within the cabinet (Fig. 19).

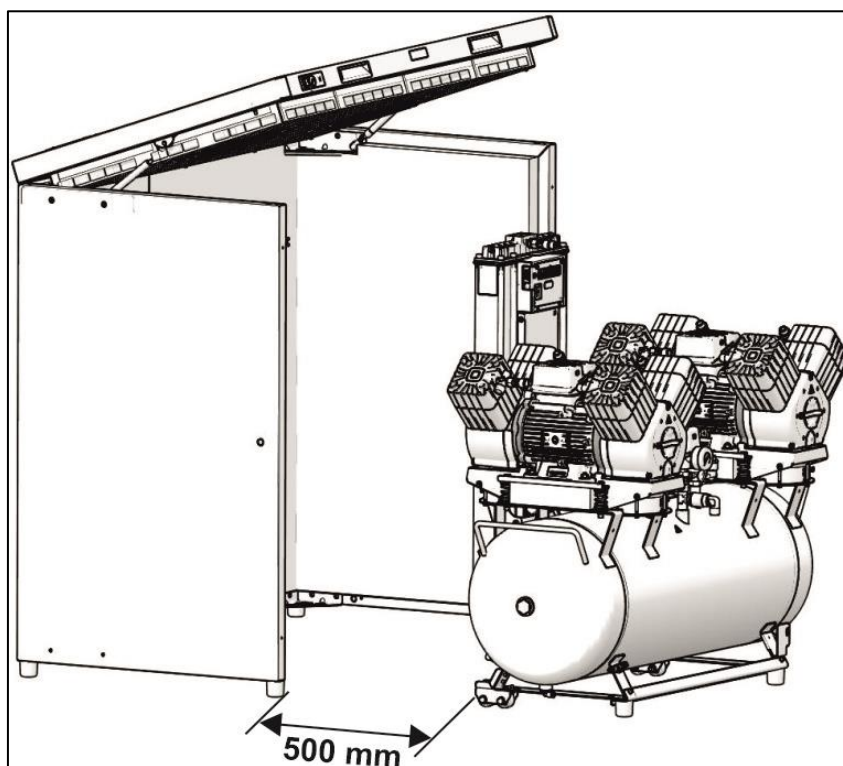
Fig. 19: Positioning of the compressor for connections

- Make the pneumatic connections per Chapter 12.1.
- Connect the condensate collection tank per Chapter 12.3.
- Route the pressure hose, the condensate drain hose, and the power cord through the opening in the rear wall of the cabinet.
- Position the compressor at such a distance from the cabinet so the WINSTA connector can make the electrical connection between the compressor and the cabinet (Fig. 13, Fig. 20).

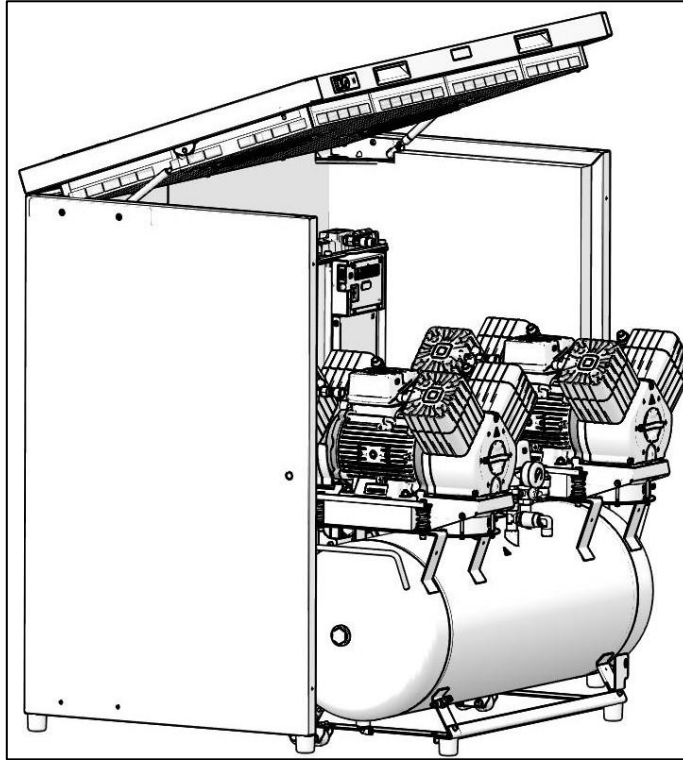
Fig. 20: Positioning the compressor for electrical connections

- Electrically connect the compressor per Chapter 13.
- Insert the compressor with dryer into the cabinet so that the dryer fan fits into the cooling tunnel opening in the cabinet. Check the positioning of the compressor in the cabinet against Fig. 21.

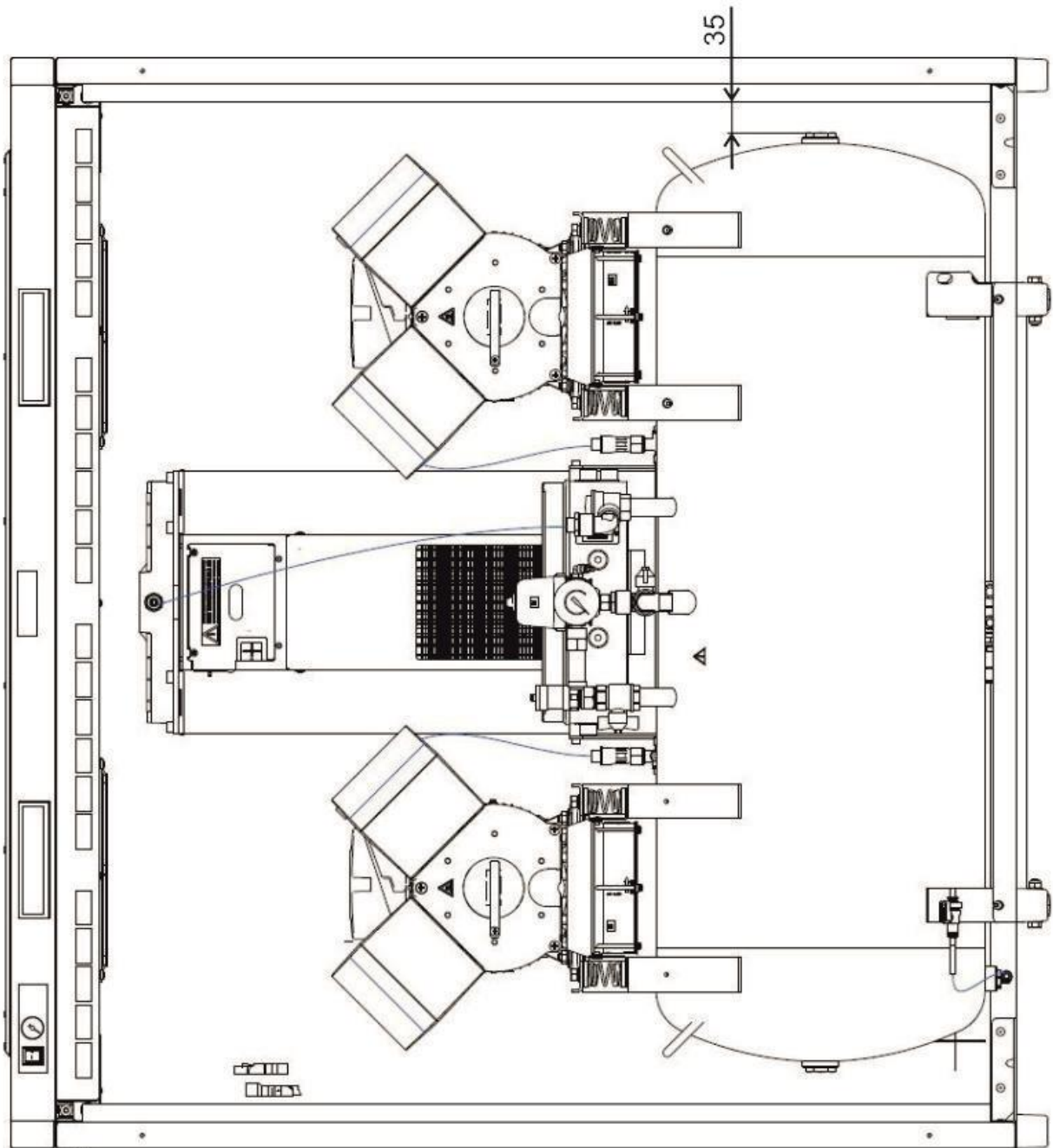
Fig. 21: Placement of the compressor in the cabinet

- Connect the cabinet pressure gauge to the compressor per Chapter 12.2.

Do not push the compressor all the way to the back of the cabinet as it may permanently damage the dryer.

Cabinet door installation

- Attach the door to the cabinet and connect the display of connector (Fig. 17) and secure the cord to the display in the clamp (1) in the front door (Fig. 22).

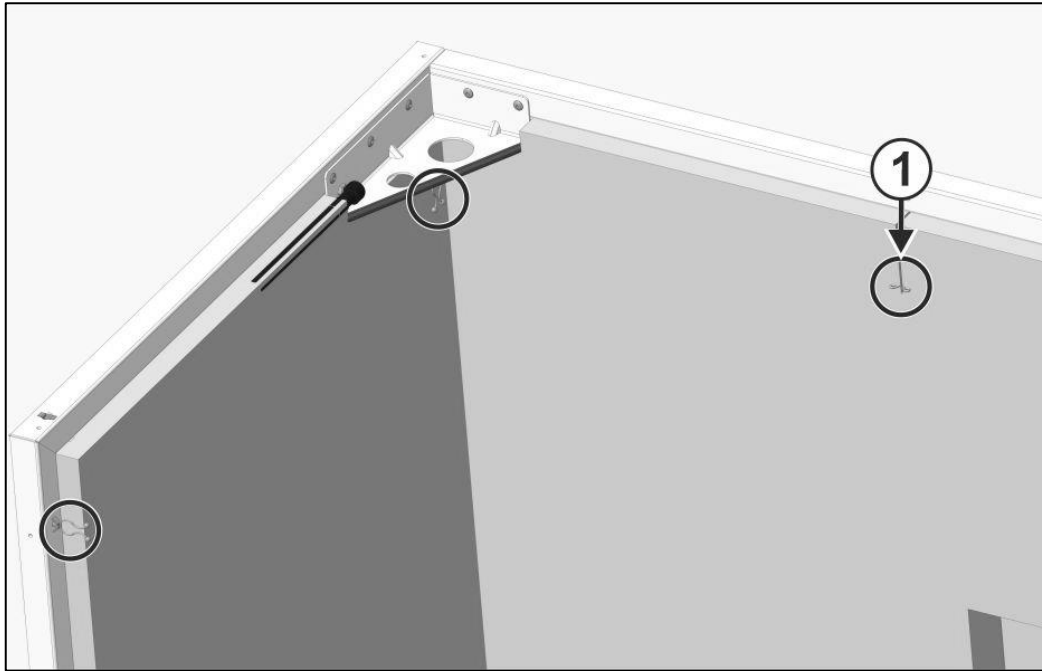
Fig. 22: DK50 2x4VR/110 S/M cabinet clamps

- Connect the earthing lead to the door. Install the door on the cabinet and mount using the 4 fasteners.

15. COMMISSIONING

- Make sure all transport stabilizers were removed.
- Check that all compressed air hose connections are correct. (see chap. 12)
- Check correct connection to the mains (see chap. 13).
- Check the circuit breaker position, it must be in the “I” position. If the main switch (2) is in the “0” position, turn it to the “I” position.(Fig. 23).
- For cabinet-mounted compressors, turn the main switch (4) on the front side of the cabinet into the “I” position; a green indicator indicates the equipment is in operation. (Fig. 23).
- Check connection of the cabinet connector to the compressor (Fig. 13).
- Check the connection of the display at the front door of the cabinet.
- Check connection of the cabinet manometer hose to the pneumatic block of the compressor (Fig. 9).
- Check to ensure the hose from the automatic condensate drain is connected to the collection vessel. (Fig. 10)
- Configuring the language on the dryer display (see chap. 20.2).
- Configuring the date and time on the dryer display (see chap. 20.3).
- Check the drying mode settings (see chap. 20.4).

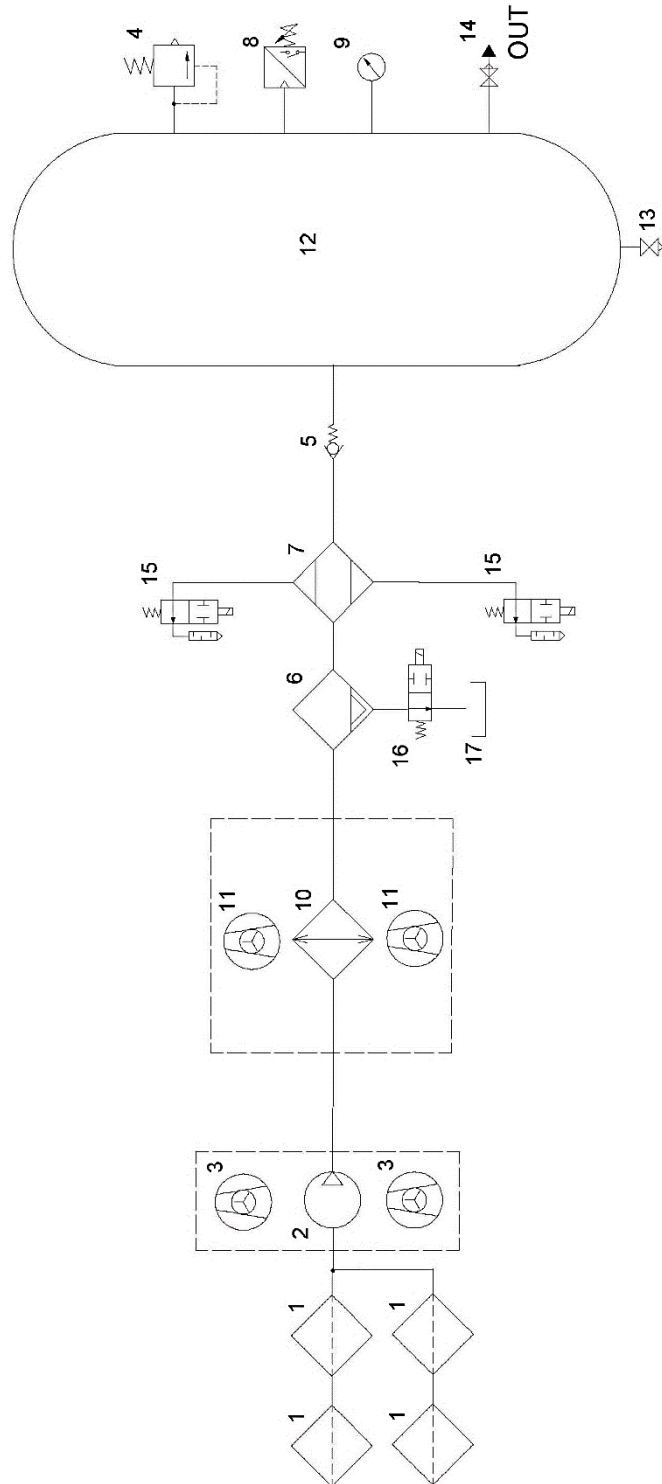
The compressor is not equipped with a backup power supply.


16. PNEUMATIC AND ELECTRICAL DIAGRAMS

16.1. Pneumatic diagram

DK50 4V/50/M

DK50 2x4VR/110/M

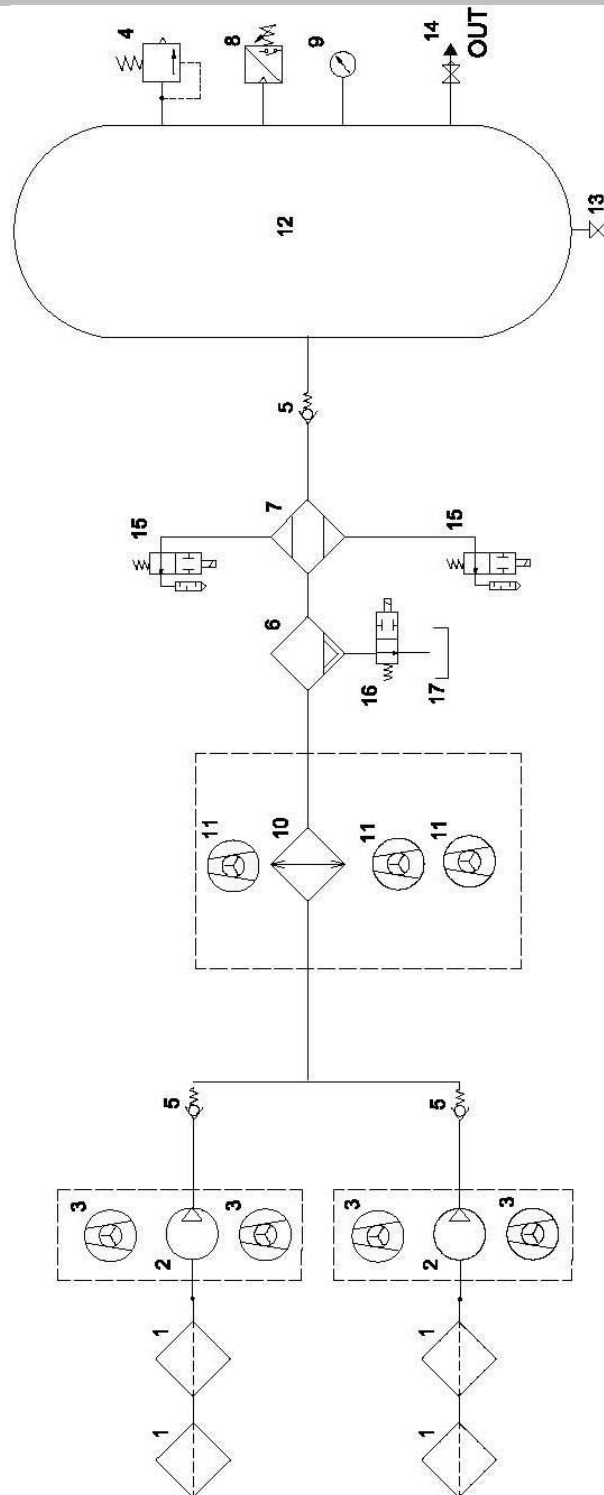Description to pneumatic diagram:

- | | |
|------------------------|--------------------------------------|
| 1 Inlet filter | 9 Pressure gauge |
| 2 Air pump | 10 Cooler |
| 3 Fan | 11 Cooler fan |
| 4 Safety valve | 12 Air tank |
| 5 Non-return valve | 13 Condensate drain valve |
| 6 Condensate separator | 14 Outlet valve |
| 7 Dryer | 15 Solenoid valve – regeneration |
| 8 Pressure switch | 16 Solenoid valve – condensate drain |
| | 17 Condensate collection vessel |

16.2. Electrical diagrams

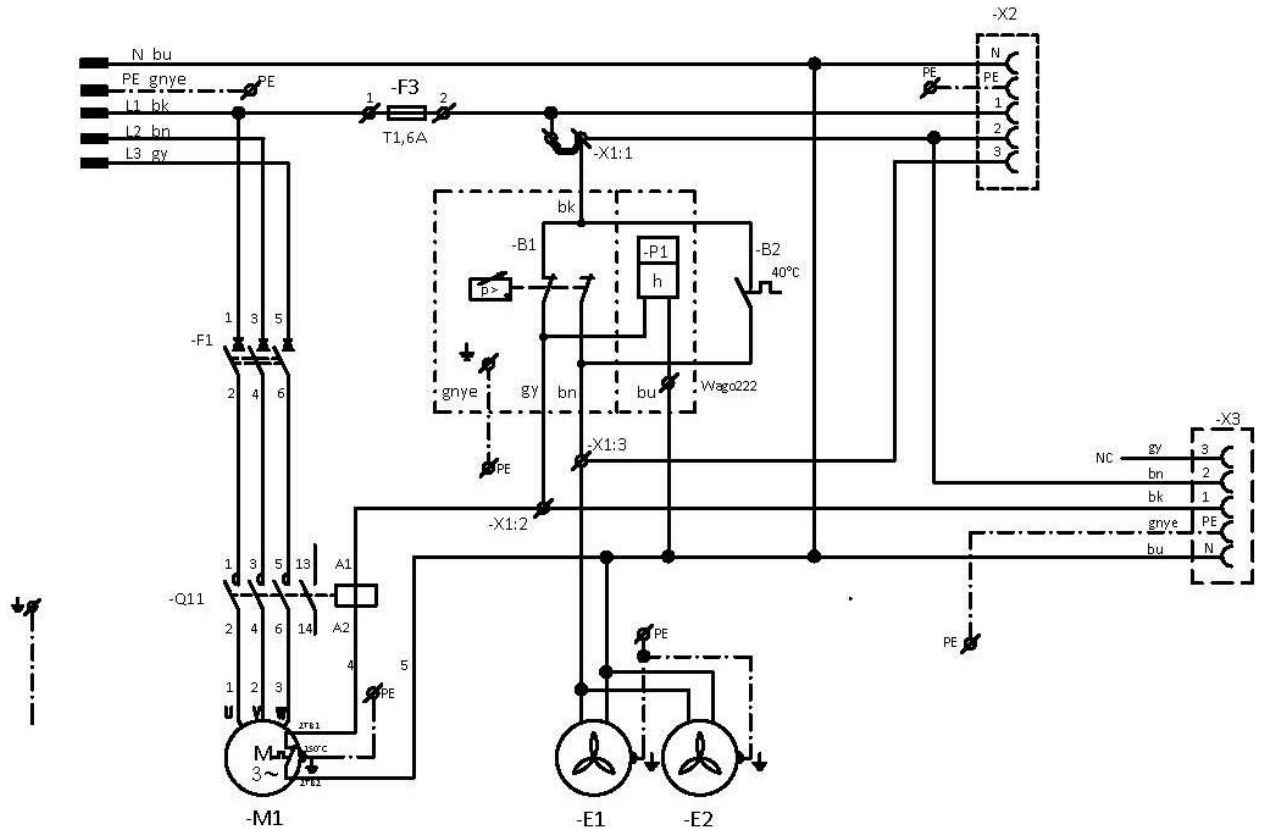
DK50 4VR/50/M

6 - 8 bar

3/N/PE~400V, 50Hz

ELEKTRIC MAIN TN-S [TN-C-S]

ELEKTRIC OBJECT OF 1st. CAT.


DK50 2x4VR/110/M

3/N/PE~400V, 50Hz

ELEKTRIC MAIN TN-S [TN-C-S]

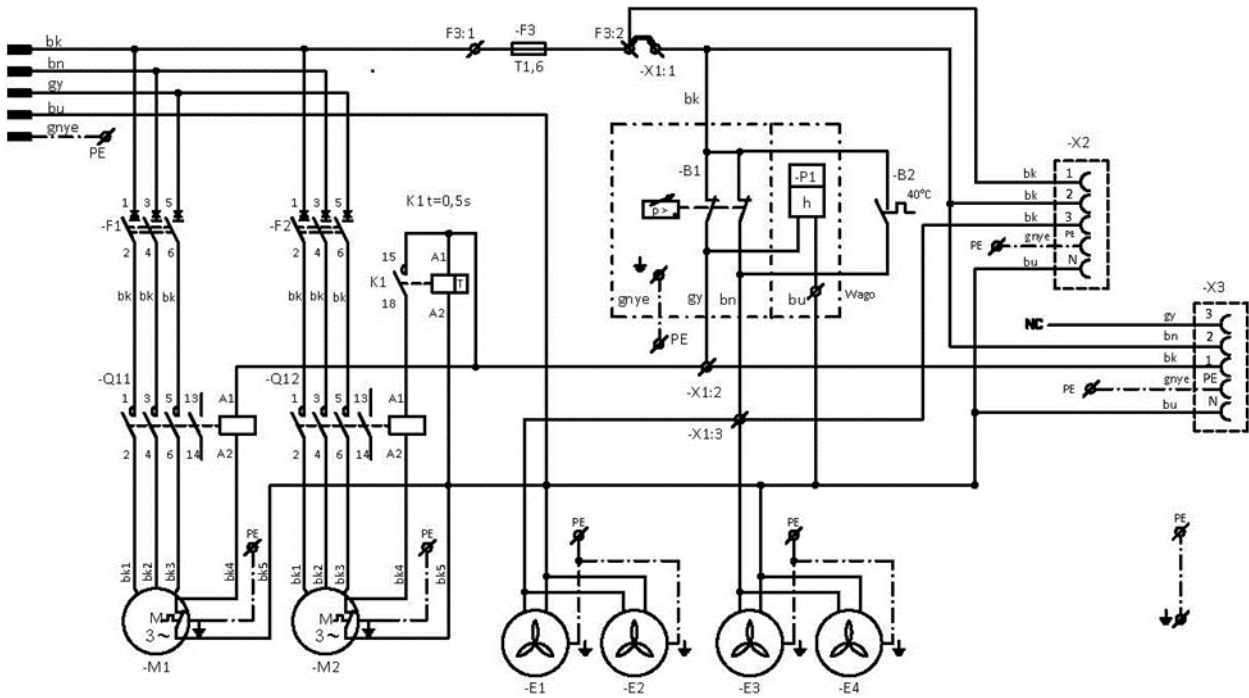
ELEKTRIC OBJECT OF 1st. CAT.

AD 280EK

1/N/PE ~ 230V, 50Hz, 230V, 60Hz

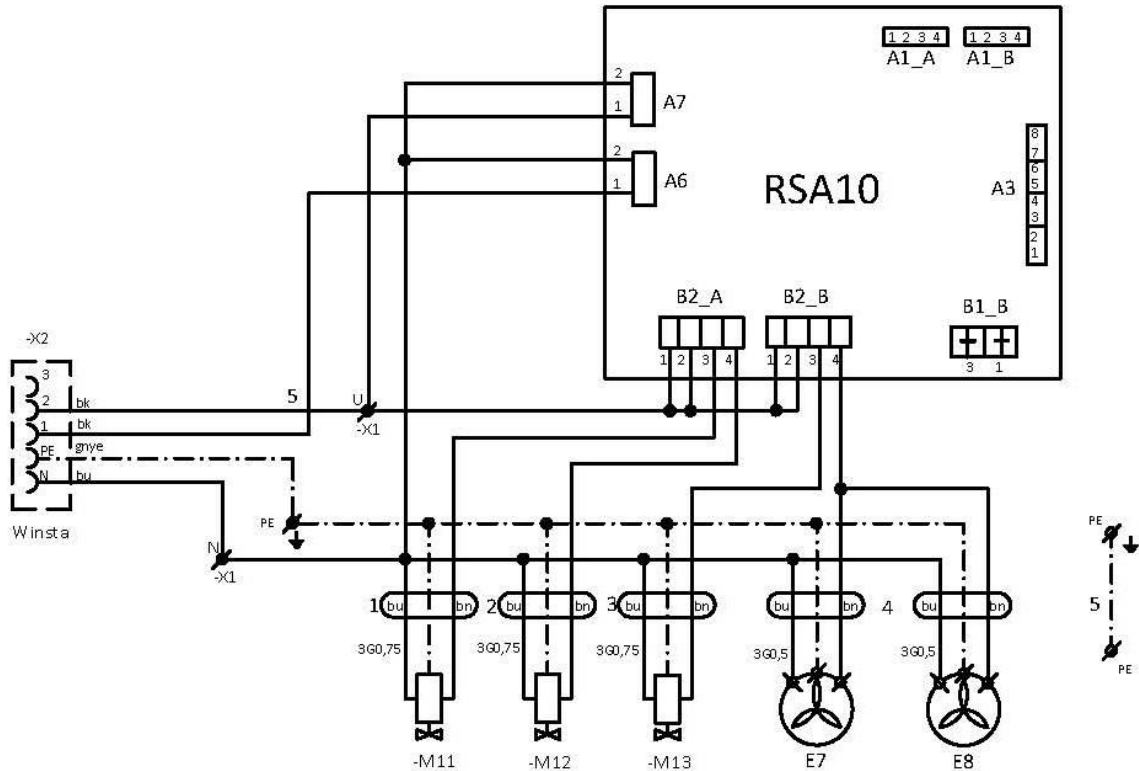ELEKTRIC OBJECT OF 1st. CAT


AD500

1/N/PE~230V, 50/60Hz

~120V, 60Hz

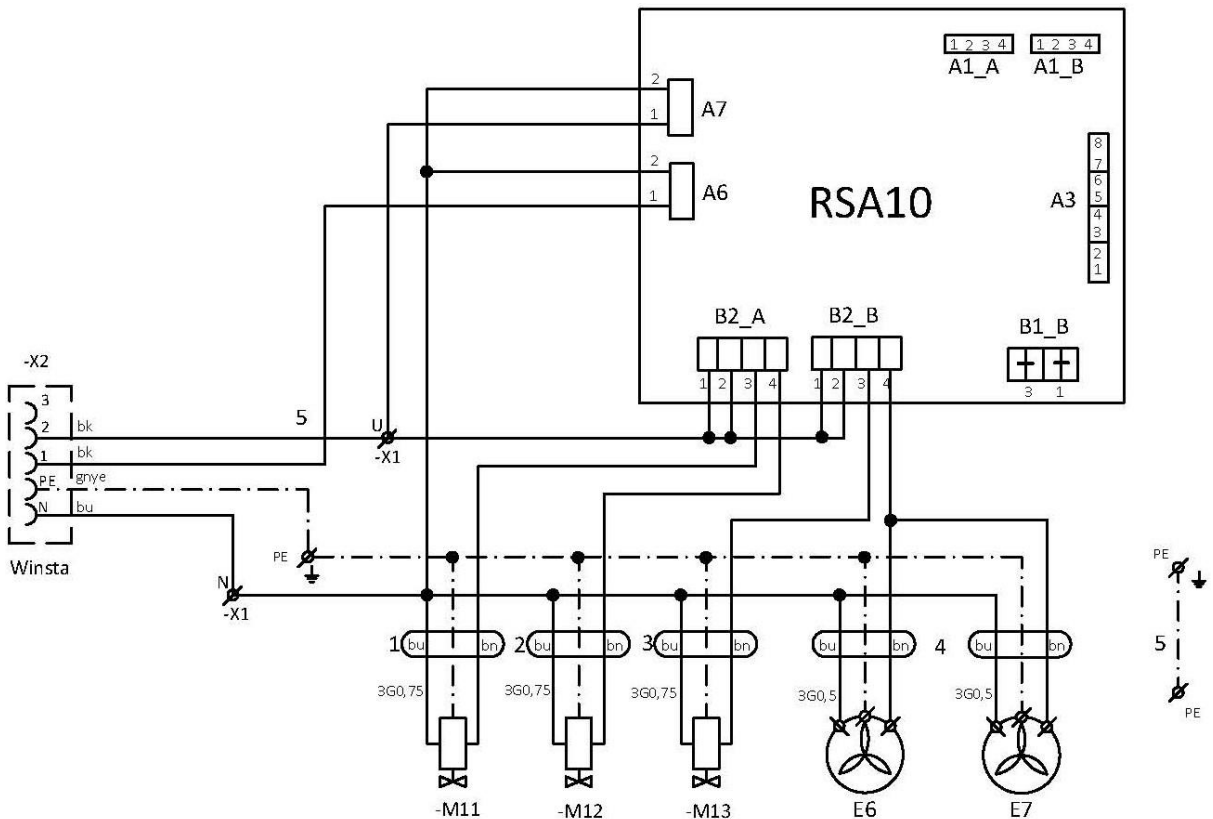
ELEKTRIC OBJECT OF 1st. CAT


Compressor cabinet

1/N/PE~230V, 50/60Hz

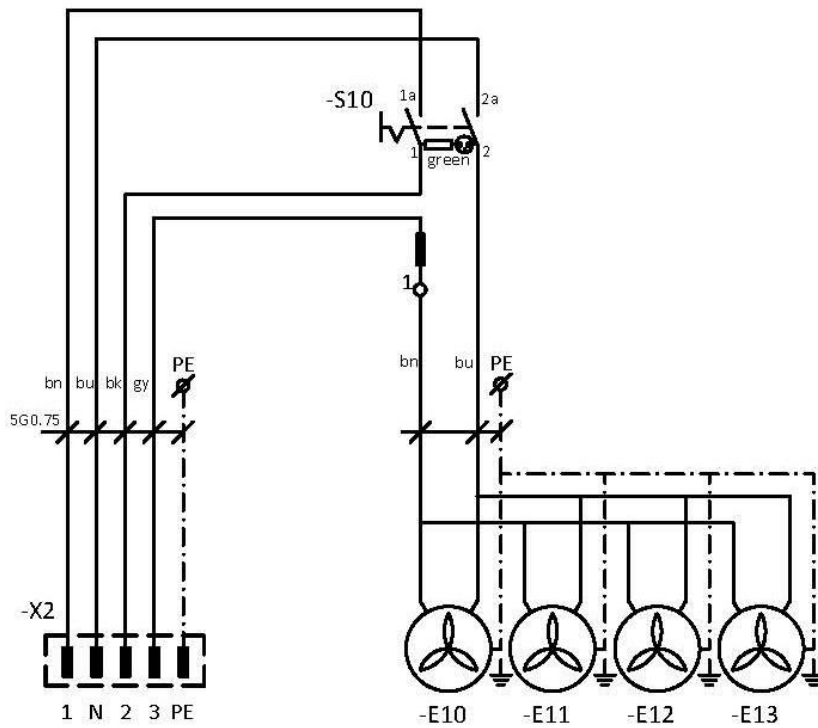ELEKTRIC OBJECT OF 1st. CAT.

DK50 4VR/50S/M

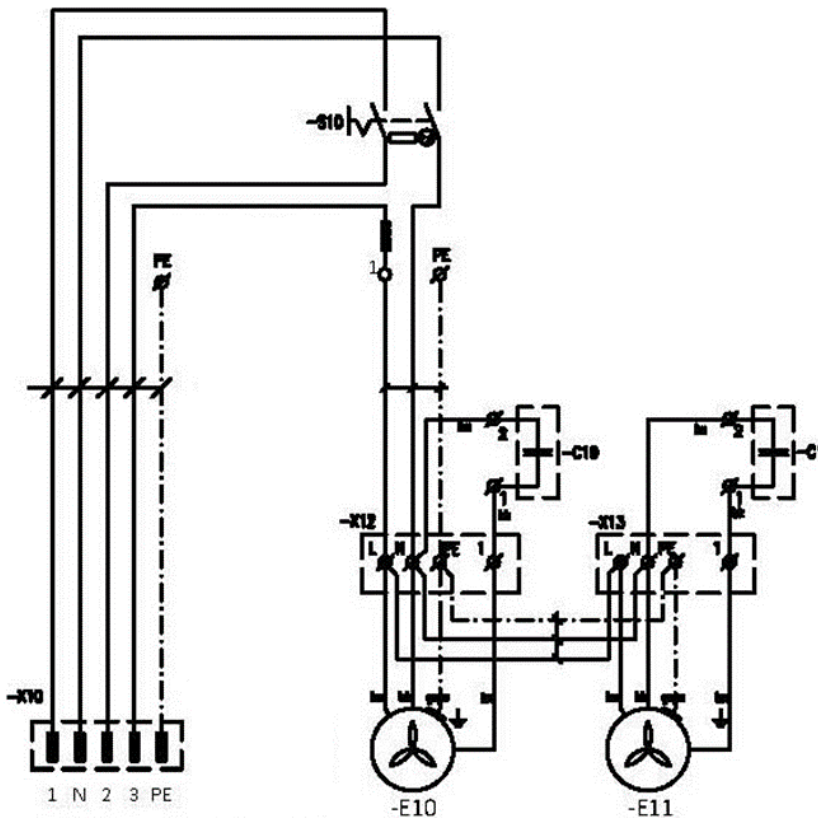DK50 2x4VR/110S/M

Description to electrical diagrams:

M1, M2	Compressor motor	X2	Socket
E1 – E4	Compressor fan	X3	Service indicator socket
F3	Fuse	S10	Switch
F1, F2	Breaker	X10,X11	Connector
B1	Pressure switch	E10, E11	Cabinet fan
B2	Temperature switch	E6-E8	Dryer fan
Q11,12	Contactora	M11, M13	Solenoid valve - regeneration
K1	Time relay	M12	Solenoid valve - venting
P1	Hour counter	RSA 10	Controller board
X1	Terminal box		

OPERATION

ONLY TRAINED PERSONNEL MAY OPERATE THE EQUIPMENT!

Risk of electric shock.

In case of emergency, disconnect the compressor from the mains (pull out the mains plug).

Burn or fire hazard.

Portions of the air pump and compressed air components between the air pump and dryer may be hot and reach hazardous temperatures during compressor operation that may harm materials or operating staff.

Warning – compressor is controlled automatically.

Automatic start-up. When pressure in the pressure tank drops to switch-on pressure, the compressor automatically switches on. The compressor automatically switches off, when pressure in the air tank reaches the switch-off pressure.

Potential for damage to pneumatic components.

The working pressure settings for the pressure switch set by the manufacturer cannot be changed. Compressor operation at a working pressure below the switching pressure indicates high air usage (see the Troubleshooting chapter).

Required drying performance can only be achieved when following the defined operating conditions.

Drying performance will decline and the achieved dew point will drop if the dryer is operated at any pressure below the minimum working pressure.

During prolonged operation of the compressor, the ambient temperature around the compressor may increase to over 40 °C. At this point the cooling fan automatically switches on. The fan switches off once the space is cooled to around 32 °C.

17. SWITCHING ON THE COMPRESSOR



Start the compressor (without a cabinet) at the pressure switch (1) by turning the switch (2) to position "I." This starts the compressor and fills the tank to the switching off pressure, which then shuts off the compressor.

Start the compressor (with a cabinet) at the switch (4) on the front side of the cabinet, the indicator will turn green. This starts the compressor and fills the tank to the switching off pressure, which then shuts off the compressor.

The compressor operates in automatic mode, switched on and off by the pressure switch, depending on compressed air consumption.

Check the values of switch-on and switch-off pressure using the pressure gauge (3). Tolerance of $\pm 10\%$ is acceptable. Pressure in the air tank must not exceed the allowed operating pressure.

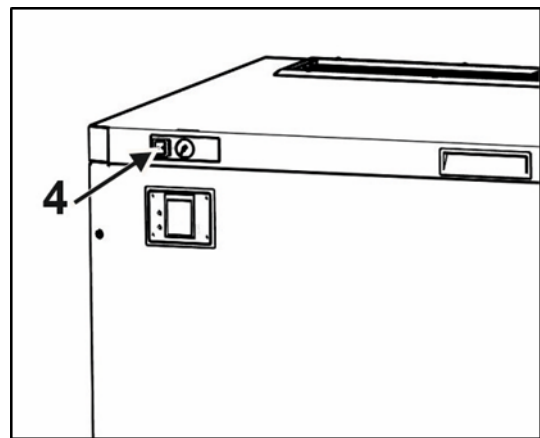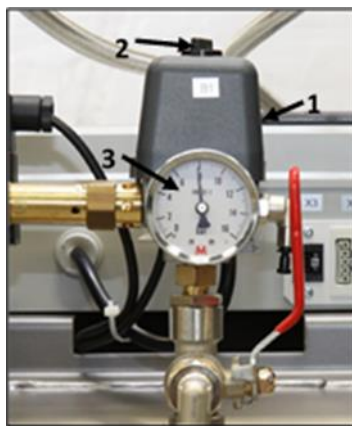Fig. 23: Switching the compressor

Risk of damage to pneumatic parts.

The pressure switch (1) was set up by the manufacturer and any other changes to its settings may be performed only by a qualified technician trained by the manufacturer.

18. SWITCHING OFF THE COMPRESSOR

- Switching off the compressor due to service or any other reason shall be performed using the pressure switch (1) by turning the switch (2) to position „0“ (Fig. 23) and pulling out the mains plug from the socket. This disconnects the compressor from the mains supply.
- Vent the pressure in the air tank to zero by opening the drain valve.

AD DRYER

19. PRINCIPLE OF OPERATION

The dryer is controlled by a signal from the compressor pressure switch.

When the compressor is running, compressed air enters the cooler where it is cooled, and a portion of the condensed moisture is separated in the integrated cyclical separator. Solenoid valve V3 is located in the bottom of the separator, and it drains off the condensate from the separator at regular intervals.

Air then passes to the chamber of logic valve OR1, which senses the pressure. If the pressure in the valve chamber does not reach $P1 = 3$ bar, regeneration solenoid valves V1 and V2 (normally closed type) remain closed and regeneration is not initiated. If the pressure in the logic valve chamber exceeds a minimum of 3 bar, the chambers switch based on the logic described in Subsection 19.1.

When the signal from the compressor pressure switch deactivates, regeneration valves V1 and V2 open for a short time to vent the compressor aggregate and to vent the dryer chambers.

Dryer venting and regeneration is completed via the hose shared with the condensate separator drain and for this reason the condensate outlet cannot be connected directly to a drain.

19.1. Dryer regeneration

Dryer regeneration is initiated when working pressure $P1 = 3$ bar. During regeneration, valves V1 and V2 switch cyclically, whereby there is a short delay (dT) to equalise the pressures in the chambers and during which both valves are closed before each switching cycle.

Working pressure is in chamber 1 and the chamber is in the drying phase, while there is atmospheric pressure in chamber 2 and this chamber is in the regeneration phase. The chamber is regenerated by air passing through the dryer nozzle.

19.2. Energy saving mode (only for models with a dew point sensor)

Dryers equipped with a dew point sensor have additional control over the purge air so as to maintain the defined dew point.

Once the required dew point is met, regeneration is stopped to optimise the quantity of purge air. When regeneration is stopped, both valves V1 and V2 are closed (and therefore chamber purging with dry air is stopped).

They continue to alternate following their standard interval to equalise the loading on the chambers.

If the dew point increases, Energy saving mode is deactivated and the dryer continues to operate with regeneration activated.

Duration of deactivated regeneration mode

Energy saving mode with deactivated regeneration is time limited. One cycle with the standard regeneration of both chambers is added after the end of a defined interval (4 chamber switches).

19.3. Alarm system

All device alarms are classified as low priority technical alarms pursuant to EN 60601-1-8. Every alarm is indicated by a corresponding visual signal (see Chapter 22) and the visual signal flashes. Call an authorised service provider to conduct repairs required in the event of a failure.

Alarm states indicate a potential failure of the device. Therefore, alarm signals may not be deactivated.

All alarm signals are intermittent – when the cause of the alarm no longer exists, the alarm signals are deactivated.

Operator's position. No specific operator's position is required. The operator may remain within range of a visual warning signal depending on actual operating conditions.

Low pressure. This signal is active when the dryer pressure sensor reads low pressure in the dryer under the defined limit of 3 bar. An air leak from the compressor/dryer, malfunction of the control solenoid valves on the dryer or a software error may activate this alarm during operation.

High pressure. This signal is active when the dryer pressure sensor reads an increase in pressure over the high limit. The air pump will then shut down and remain shut-down until the pressure decreases. If the aggregate does not shut down, excess pressure is vented by the safety valve on the compressor's air tank.

High dew point. This signal is active when the dew point sensor reads a high dew point value.

Expiration of service interval. This signal is activated until service is performed and the interval reset, or a new service interval is entered.

20. USER INTERFACE / SETTINGS

20.1. Meaning of symbols on the display

Symbol representing the user manual, or to read the user manual.

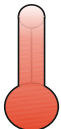
Symbol expressing the ambient temperature (from 25°C) related to the selected mode.

The program adjusts the chamber pre-filling in the given mode.

Symbol expressing the ambient temperature (from 25°C to 30°C) related to the selected mode.

The program adjusts the chamber pre-filling in the given mode.

Symbol expressing the ambient temperature (over 30°C) related to the selected mode.

The program adjusts the chamber pre-filling in the given mode.

Back to the main menu on the 4 main screens.

Go back

Symbol for the service interval

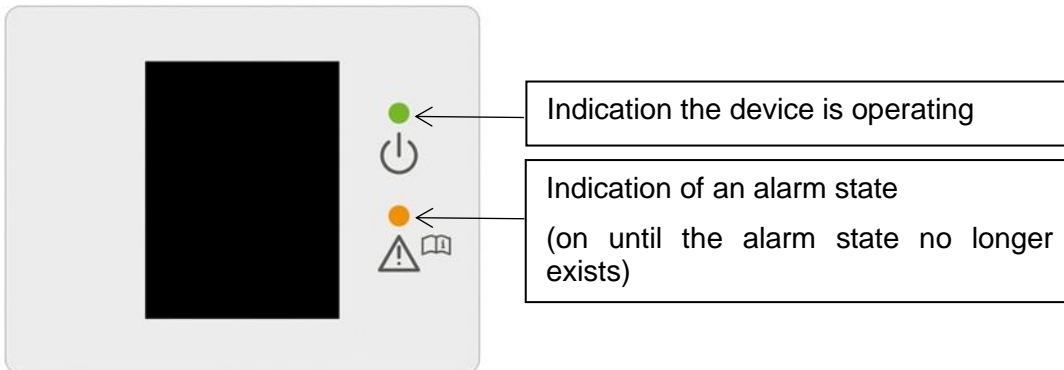
Symbol for when the dryer chamber is currently in regeneration

Symbol for when the dryer chamber is currently drying


Symbol for chamber pre-filling – equalisation of pressure in chambers.


20.2. Initial settings for the user interface

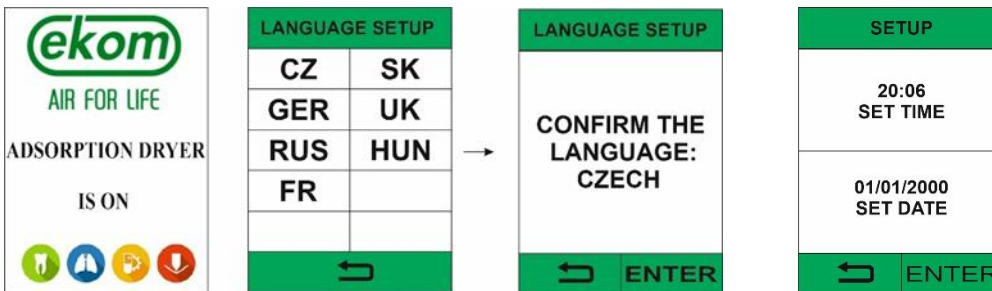The option to select language, time and date appears when the program is first launched. The service technician selects the given language, time and date during installation based on the geographic location of the dryer installation. The program saves and remembers this setting. The option does not appear when the program is restarted. Confirming this option will also start the service hours countdown.

20.3. Setting the date and time during commissioning

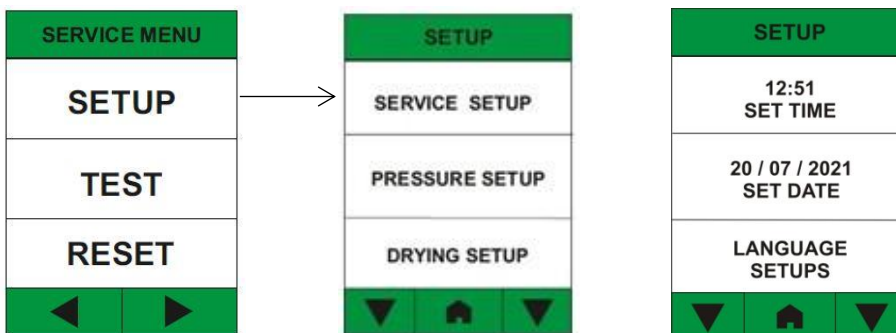
It is recommended to set the date and time during commissioning. These settings are changed in the service menu in the SETUP section. Access to the service menu is protected by a code (1992). In SETUP you have to tap down arrow to set date and time. In the same way, it is possible to reset the user interface language via LANGUAGE SETUP.

20.4. Setting the drying mode during commissioning

It is recommended to set the air-drying mode during commissioning. These settings are changed in

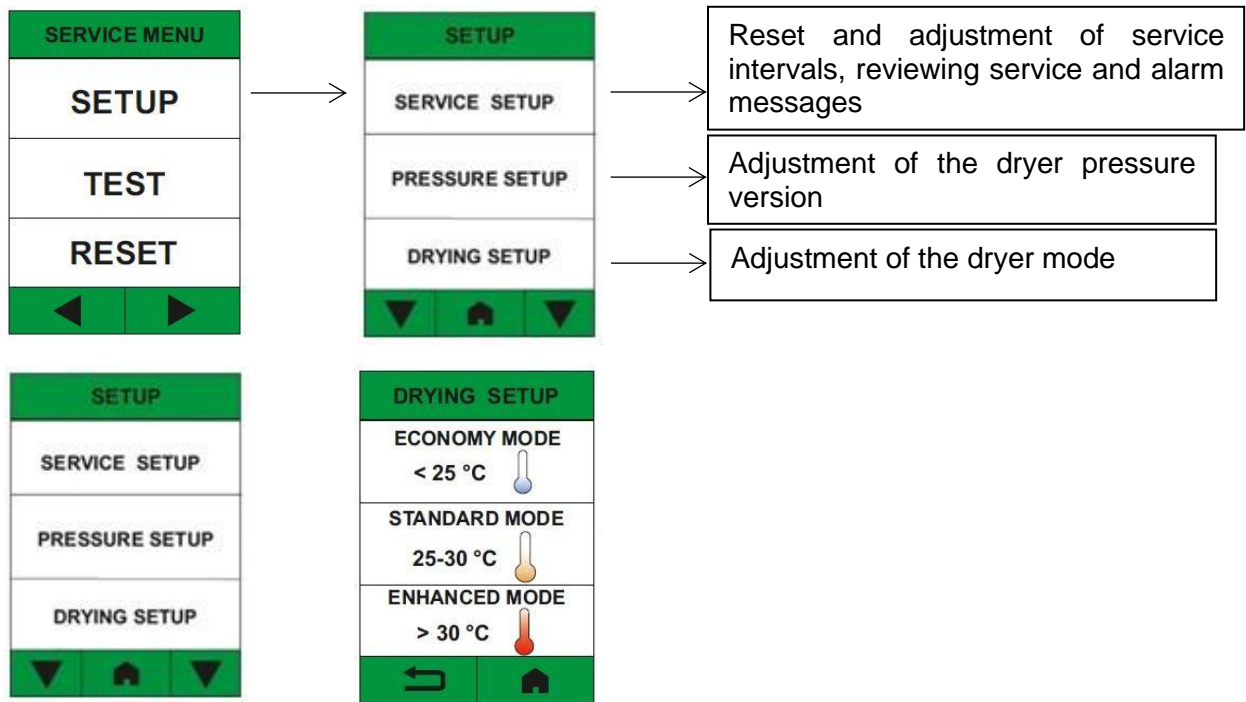the service menu in the SERVICE SETUP section. Access to the service menu is protected by a code.

The drying mode is set based on the expected pressure range in which the device will operate and the version of the compressor (with or without a cabinet).

- compressor without a cabinet $t_0 < 25\text{ °C}$ - set to ECONOMY mode
- compressor without a cabinet $t_0 = 25\text{ °C} - 30\text{ °C}$ - set to STANDARD mode
- compressor without a cabinet $t_0 > 30\text{ °C}$ - set to ENHANCED mode
- compressor in cabinet - set to ENHANCED mode

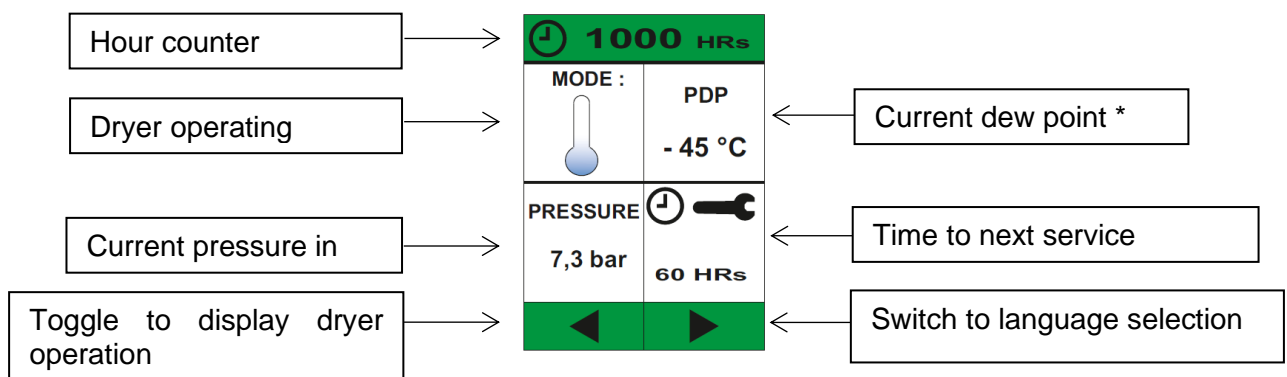
t_0 - ambient temperature

21. MAIN SCREENS

The home screen opens once the user interface language and drying mode are set. There are 4 main screens accessed from the home screen.

The arrows at the bottom of the screen are used to navigate between the individual screens.


21.1. Home screen

(*this information is only shown if the compressor is equipped with a dew point sensor)

- Basic information and measured parameters are shown on the dryer home screen. The dryer operating hours indicator is located at the top of the screen. The home screen has 4 main tiles that display the following information:
- Dryer operating mode (see chap. 20.1)
- Current pressure in the dryer (min. working pressure to activate dryer regeneration is 3 bar)
- Current pressure dew point value (only for compressors equipped with dew point sensors)
- Remaining time to the next service interval (see chap. 22.1, 22.2– service intervals).

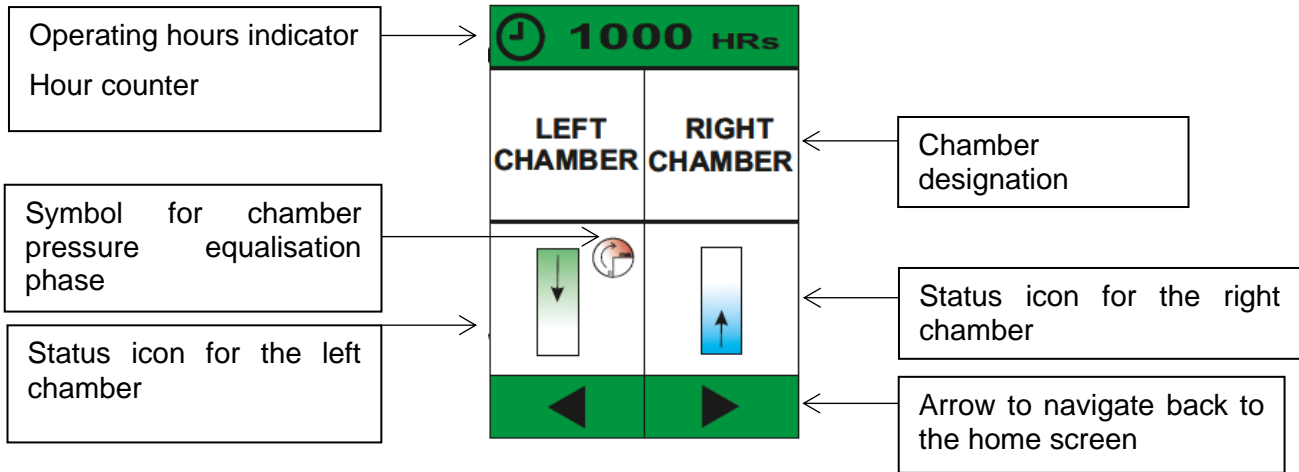21.2. Dryer operation screen

The dryer operating hours indicator is located at the top of the screen. The dryer operation screen is also divided into 4 tiles, which describe the dryer chamber working cycles using a graphical depiction of the operating states of each chamber.

Symbol for when the dryer chamber is currently in regeneration

Symbol for when the dryer chamber is currently drying


Symbol for chamber pre-filling – equalisation of pressure in chambers.

21.3. Language settings screen

LANGUAGE SETUP	
CZ	SK
GER	UK
RUS	HUN
FR	
◀ ▶	

The language settings screen is one of the main screens.

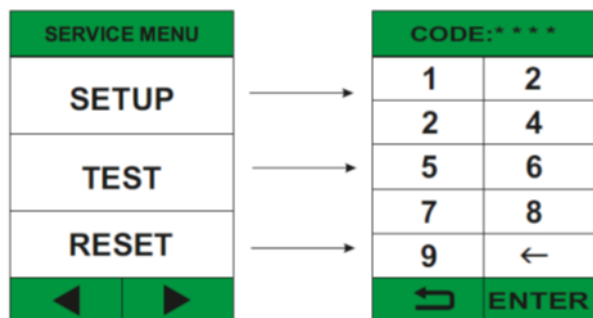21.4. Service menu screen


The service menu is intended exclusively for service personnel. Access to individual menu items is password (1992) protected.

22. INDICATION OF MAINTENANCE AND ALARM STATES

The adsorption dryer controller is capable of measuring selected parameters (pressure and pressure dew point in versions with a dew point sensor) and for assessing the basic dryer operating states.


Based on the measured parameters and service details, the controller displays information on faults using the following notifications and warnings.

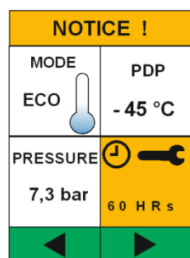22.1. Notification – Upcoming service interval

- 100 operating hours prior to the next service interval a **yellow notification appears (“Notice!”)**. The service tile also lights up orange.
- The remaining hours to the next service interval are also displayed.
- The warning is shown until service is completed and the service interval is reset.

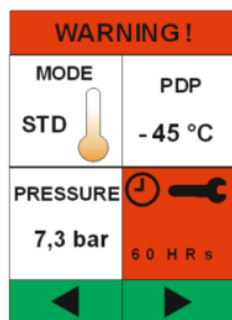
22.2. Warning – Service interval exceeded

- A red warning is shown (“Warning!”) once the service interval has lapsed. The service tile also lights up red.
- The warning is shown until service is completed and the service interval is reset.
- The hours to the next service interval are also displayed.
- A list of warnings is saved in the “Warning log”
- The dryer continues operating normally


22.3. Alarm conditions

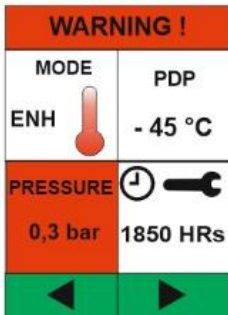Low / high pressure

- A red warning is shown (“Warning!”) if low pressure (<3 bar) or high pressure (> 8.5 bar, or > 10.5 bar depending on the pressure version) is detected. At the same time, the current pressure tile lights up red.
- The warning is displayed as long as the alarm condition endures.
- A list of warnings is saved in the “Warning log”
- The dryer is not operating normally, regeneration is not initiated, the compressor must be shut down

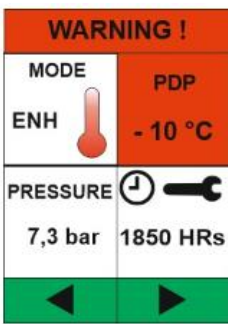High dew point


- This warning is only accessible on dryers with dew point sensors
- A red warning is shown (“Warning!”) if a high dew point is detected (> -20°C , or. > -40°C depending on the version of the dryer). At the same time, the current pressure dew point tile lights up red.
- The warning is displayed as long as the alarm condition endures.
- A list of warnings is saved in the “Warning log”
- The dryer does not operate properly or is used outside of recommended conditions for use, the compressor must be shut down.

After initial start-up, it may take up to 50 hours for the compressor to reach the required dew point. During this time, the high dew point warning is not displayed. If the dryer cannot meet the dew point set-point, please contact service personnel.

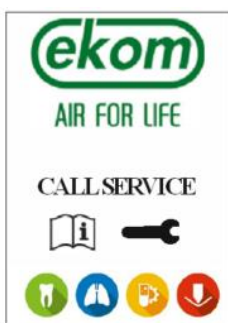22.4. „Call Service“ screen

- The “Call Service” screen alternates with the main screen if a notification or warning occurs.

PRODUCT MAINTENANCE**23. PRODUCT MAINTENANCE**

The operator shall ensure completion of repeated testing of the device at least once every 24 months (EN 62353) or at intervals defined by applicable national legal regulations. A record of these test results shall be completed (e.g. per EN 62353, Annex G) together with the measurement methods.

The equipment has been designed and manufactured to keep maintenance to a minimum. The following work must be performed to preserve the proper and reliable operation of the compressor.

Unauthorised interference hazard.

Repair work outside the framework of standard maintenance (see Chapter 23.1) may only be performed by a qualified technician (an organisation authorized by the manufacturer) or the manufacturer's customer service.

Standard maintenance work (see Chapter 23.1) may only be performed by the operator's trained personnel.

Only use manufacturer-approved replacement parts and accessories.

Danger of injury or equipment damage.

Prior to commencing compressor maintenance, it is necessary to:

- check if it is possible to disconnect the compressor from the appliance in order to avoid any risk of injury to the person using the appliance or other material damage;
- turn off the compressor;
- disconnect it from the mains (pulling the cord out of the mains socket);
- vent the compressed air from the air tank.

Venting compressed air poses an injury hazard.

Wear eye protection, i.e. goggles, when venting compressed air from the compressed air circuit (air tank).

Burn hazard.

When the compressor is running or shortly thereafter, certain portions of the air pump, the compressor's compressed air system, parts of the dryer and connecting hoses between the compressor and dryer may be hot - do not touch these components.

The removed grounding conductor during service must be connected back to the original position after completing the service.

- For any maintenance or repair work, the compressor may be removed from the cabinet by pulling it from the cabinet (on rolling casters as much as the cord between the compressor and the cabinet allows). (see chap. 23) The required maintenance or repair work may then be performed.
- If the cord between the compressor and the cabinet becomes disconnected (by pulling the mains plug, see chap. 13.2), the compressor shall not be functional. In order to bring the compressor into function again, please follow, see chap. 13.2 or 23.8 or 23.9.

23.1. Maintenance intervals

Time interval	Performed by operator				Performed by qualified technician												
	Set of replacement parts	Chap.	24 000 h	20 000 h	16 000 h	12 000 h	10 000 h	8 000 h	6 000 h	4 000 h	2 000 h	Once every 6 years	Once every 4 years	Once every 2 years	Once a year	Once a week	Once a day
Check of product operation	-	23.2															x
Drain condensate from air tank – at high humidity	-	23.5															x
Drain condensate from the air tank - at normal humidity	-	23.5														x	
Check of product function	-	9														x	
Check of pneumatic connections leakage and device inspection	-	23.3													x		
Inspection of electrical connections	-	23.4													x		
Check of cooler and fan	-	23.11													x		
Check of safety valve	-	23.6													x		
Replacement of pump inlet filter and pre-filter 4VR ^{a)}	025200139-000 025200150-000	23.7								x					x		

a) data is in hours, if not available, data is in years
b) check the operation of the relief valve on the dryer after every dryer malfunction
c) Whichever occurs first: 2 years/10000 hours.
d) Whichever occurs first: 6 years/24,000 hours.

Time interval	Performed by	qualified technician				
		Set of replacement parts	Chap.	24 000 h	20 000 h	16 000 h
Dew point sensor calibration	-	23.12				
Replacement of cassettes with adsorbent media AD280 dryer ^{c)}	603022493-000	See the service manual		x		
Replacement of cassettes with adsorbent media AD500 dryer ^{c)}	603022485-000			x		
Replacement of the dryer's logic valve ball	069000317-000				x	
Replacement of the dryer's solenoid valve ^{d)}	025300117-000					x
Conduct a "repeated test" per EN 62353	-	23				

a) data is in hours, if not available, data is in years
 b) check the operation of the relief valve on the dryer after every dryer malfunction
 c) Whichever occurs first: 2 years/10,000 hours.
 d) Whichever occurs first: 6 years/24,000 hours.

23.2. Check of product operation

- Check aggregate condition – the aggregates should be operating normally without excessive vibration or noise. Troubleshoot any problem or call in service personnel if trouble is detected.
- Visually inspect fan operation – the fans must be operating when the aggregates are running. Troubleshoot any problem or call in service personnel if trouble is detected.
- Check to ensure the power cord, the connecting compressed air hoses are undamaged. Replace damaged components or call in service personnel.
- Check the ambient temperature – the ambient temperature must be below the temperature limit (40°C). Cool the space if the temperature is high.
- Compressor with air dryer - open the plug on the condensate collecting vessel and drain the condensate.
- Check the service indicator on the display screen and schedule service as needed.
- Check the operating condition of the equipment (see Chapter 23.4).

23.3. Check the compressed air connections for leaks and inspect the equipment

Leak testing

- Check the compressor's compressed air lines for leaks during operation – pressure supplied by the compressor.
- Use a leak analyser or soapy water to check all joints and connections for leaks. Tighten or reseal the connection where leaks are found.

Inspecting the equipment

- Check the condition of the compressor aggregate for normal operation and noise levels.
- Fan operation check – the fans must be running during the defined compressor work cycles.
- Check function of the temperature switch (B2) – heat the temperature switch to temperature above 40°C (e.g. using a heat gun – Be careful, do not operate with high temperature on plastic components in the surrounding, it can cause deformation of plastic materials). Fan E1, E3 - the compressor must be under voltage.
- Check filters condition – filters must be undamaged and sufficiently clean.
- Check condition of the pump itself, check if there is no dirt inside the crankcase or clearance in the crankshaft.
- Replace any defective parts if there are any failures.
- Replace any defective parts as needed.
- Check the solenoid valves in the valve module – the valves should cyclically cycle between the chambers based on the description of the operation.
- Check the operating condition of the equipment (see Chapter 23.4).

23.4. Inspection of electrical connections

Risk of electric shock.

Inspect the product's electrical connections when the mains are disconnected.

- Check mechanical function of the main switch.
- Check if the power cable and conductors are not damaged.

- Visually check if cables are connected to the terminal box.
- Check all screw connections of the green-yellow PE grounding conductor.

Indication of an upcoming or expired service interval

Configuration of a new service interval

Press ESC + ► together and hold for 10 seconds to reset the service interval. Once complete, the display returns back to the home screen.

The SET UP function for a new service interval is only functional if the dryer indicates an upcoming service interval or that a service interval has expired. Once complete, the MAINT counter increases (number of completed service operations) by one. See the screen: Counters.

23.5. Condensate drain

A wet floor resulting from overflow from the vessel poses a slip hazard.

Regularly empty the condensate collection vessel.


Switch off the compressor every time before emptying the vessel!


Condensate from compressors with air dryers is automatically drained into a vessel to collect condensate.

- Monitor the level in the vessel using the markings (depending on the volume of the vessel), and empty at least once a day.

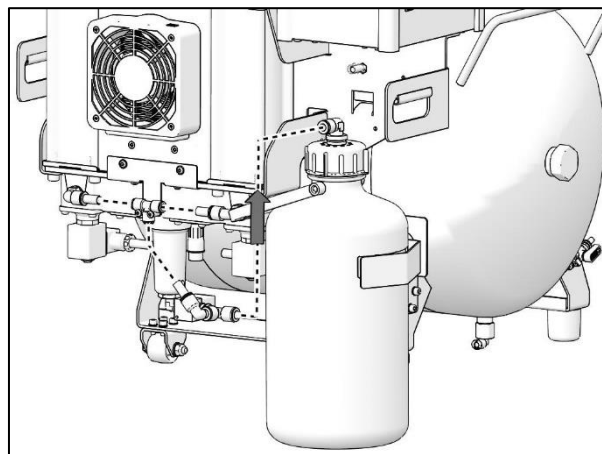Fig. 24: Check of the condensate collection vessel

DK50 4VR/50/M

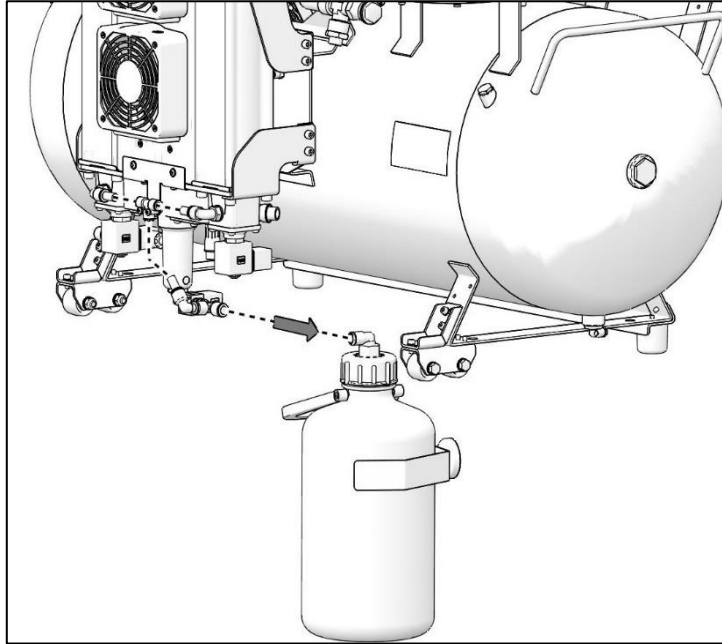DK50 2x4VR/110/M

Before the following checks it is required:

- For compressor variant with cabinet – unlock the door lock and open the cabinet door.

23.6. Check of safety valve

- Turn the screw (2) on the safety valve (1) several times to the left until the safety valve releases the air.
- Let the safety valve vent for a few seconds.
- Turn the screw (2) fully to the right, the valve must be closed now.

Fig. 25: Check of safety valve

Damage to the safety valve could cause pressure to rise to hazardous levels.

Never use the safety valve to release the air pressure in the air tank. This could damage the safety valve. The valve is set to the maximum permitted pressure by the manufacturer.

Never adjust a safety valve.

Venting compressed air poses an injury hazard.

Wear safety glasses when inspecting a safety valve.

23.7. Inlet filter replacement

The inlet filter (1) is located inside the compressor crankcase cover.

Inlet filter replacement:

- Pull out the rubber plug by hand (2).
- Remove the dirty intake filter (1).
- Insert a new filter and replace the rubber plug.

Pre-filter replacement:

- Pull out the pre-filter by hand (3).
- Replace with a new one and insert it back.

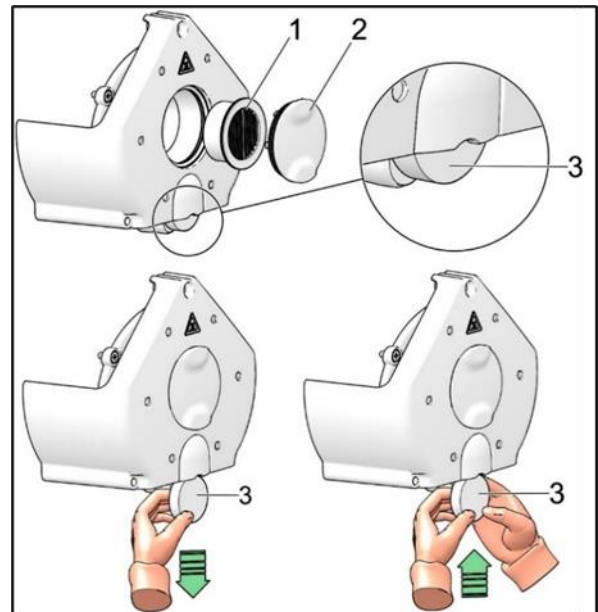
Fig. 26: Inlet filter replacement

23.8. Procedure for connecting a compressor disconnected from the cabinet

Prior to any maintenance or repair work, switch off the compressor and disconnect it from the mains (pull out the mains plug).

The compressor (without cabinet) requires that the jumper is always mounted in the terminal strip for correct operation (Fig. 27 pos. C,D, Fig. 28 pos. D). It then replaces the switch on the cabinet.

If there is no jumper in the terminal strip of the compressor (disconnected from the cabinet), the compressor will not work!



If the cord between the compressor and the cabinet becomes disconnected (by pulling the mains plug) and the compressor is removed from the cabinet, the compressor will not be functional. This is why it is necessary to first create a connection to the terminal strip with a jumper (replacing the circuit breaker function) (Fig. 27 pos. A,B, Fig. 28 pos. A).

PROCESS:

Jumper installation (A-B-C-D process)


- Disconnect the product from the mains by pulling the plug out of the socket.
- Remove the cover on the electrical panel (on the compressor).
- The jumper is not in the terminal strip – A.
- Lift up the fuse cover – B.
- Insert the jumper into the terminal strip – C.
- Close the fuse cover – D.
- Reinstall the cover on the electrical panel.
- Connect the device to the electrical mains.
- Activate the compressor by turning on the switch on the pressure switch.

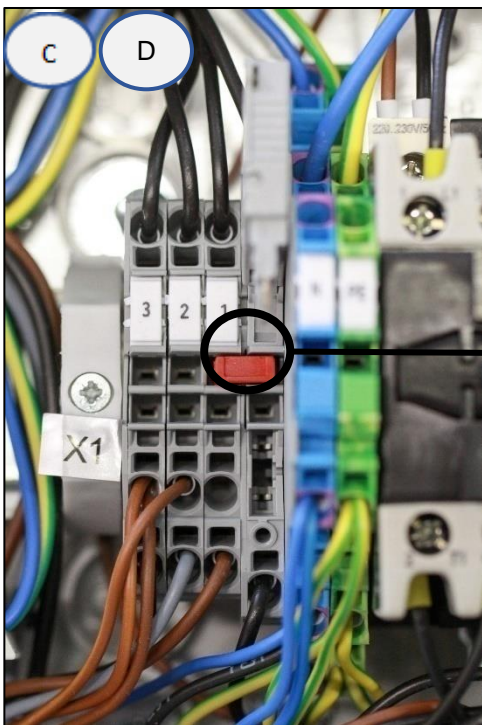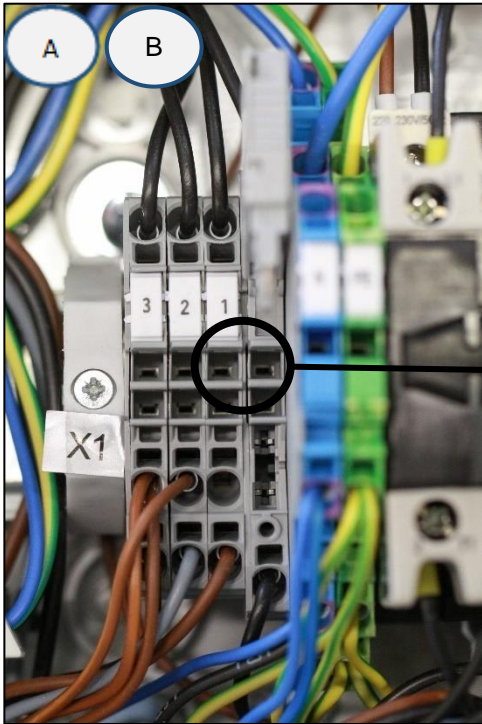Fig. 27: DK50 4VR/50/M

400V

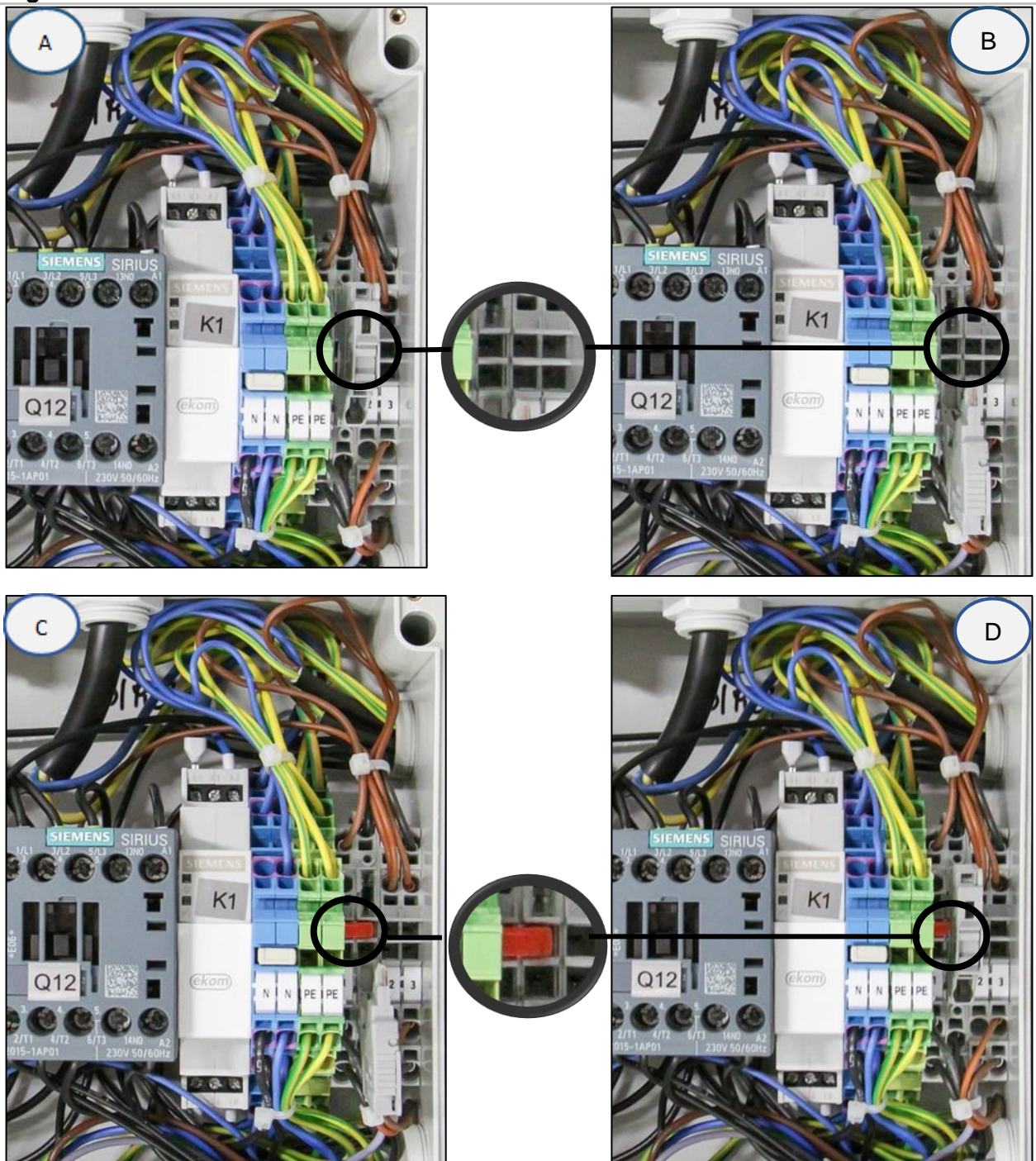Fig. 28: DK50 2x4VR/110/M

400 V

23.9. Procedure for connecting a compressor to a new cabinet

Prior to any maintenance or repair work, switch off the compressor and disconnect it from the mains (pull out the mains plug).

The compressor in a cabinet requires that the jumper is not mounted in the terminal strip for correct operation (Fig. 27 pos. A,B, Fig. 28 pos. A). The switch on the cabinet switches the entire product on and off, including the compressor.

If there is a jumper in the terminal strip of the compressor connected to a cabinet, the switch on the compressor cabinet will not work!

If we want to connect a new cabinet to a compressor that previously worked alone (a compressor without a cabinet), then it is necessary to cancel the connection in the compressor terminal strip by performing the procedure, see further. (Fig. 27, Fig. 28). (See also chap. 13.1 and 13.2).

PROCESS:

Jumper removal (D-C-B-A process)

- Disconnect the product from the mains by pulling the plug out of the socket.
- Remove the cover on the electrical panel (on the compressor).
- The jumper is in the terminal strip – D
- Lift up the fuse cover – C
- Remove the jumper from the terminal strip – B
- Close the fuse cover – A
- Reinstall the cover on the electrical panel.
- Insert the compressor into the box.
- Connect the cabinet to the compressor by inserting the power cord with connector into a power socket. (Fig. 13)
- Connect the connector to the display on the front door of the cabinet.
- Connect the device to the electrical mains.
- Activate the compressor by turning on the switch on the pressure switch and the switch (4) on the cabinet (Fig. 23).

Compressor with cabinet - when operating in the cabinet, there must not be an electrical connection. (Fig. 27 pos. A,B, Fig. 28 pos A).

Note: **In case of using a compressor without a cabinet or in case of maintenance, a jumper which is not installed into the terminal strip should be kept** in a designated place next to the electrical panel.

23.10. Cleaning and disinfection of the exterior surfaces of the product

Clean and disinfect the exterior surfaces with neutral cleaning products.

Use of aggressive detergents and disinfectants containing alcohol and chlorides can lead to surface damage and discolouration.

23.11. Inspecting the cooled and fan

The equipment, in particular the compressor fan, cooler fan, and the cooler, must be kept clean to ensure efficient drying. Remove dust from the surface of the cooling fins and fans by vacuuming or blowing down with compressed air.

23.12. Dew point sensor calibration

It is recommended to calibrate the dew point sensor every 2 years on compressors equipped with this sensor. Calibration is performed by the manufacturer (Vaisala) and must be ordered online.

Sensor calibration ordering process:

- Visit the website www.vaisala.com, and then navigate to the *Services* section and select *Calibration and repair services*.
- Then enter the type of sensor (sensor type: **DMT143**) and select a calibration type, then follow the provided instructions.
- After removing the dew point sensor, blank off the place where the sensor was installed using the provided plug, sealed with the provided adhesive (applies to DK50 4VR/50/M compressors).

24. LONG-TERM SHUTDOWN

If the compressor is not going to be used for long period, it is recommended to drain all condensate from the air tank and switch on the compressor for about 10 minutes, keeping the drain valve open (1). Thereafter switch off the compressor using the switch (2) on the pressure switch (1) (Fig. 23), close the drain valve and disconnect the device from the mains.

25. DISPOSAL OF DEVICE

- Disconnect the equipment from the mains.
- Release the air pressure in the pressure tank by opening the drain valve.
- Dispose of the equipment following all applicable regulations.
- Entrust a specialised company to sort and dispose of waste.
- Worn out components have no negative environmental impact.

TROUBLESHOOTING**Risk of electric shock.****Before interfering with the equipment, first disconnect it from the mains (remove the power socket).****Working with pressurised pneumatic components poses a risk of injury.****Before interfering with the equipment, vent the air tank and the compressed air system to zero pressure.****Troubleshooting may only be performed by a qualified service technician.****Damage to the safety valve could cause pressure to rise to hazardous levels.****Never adjust a safety valve.**

Malfunction	Possible cause	Solution
Compressor does not switch on	No voltage in the pressure switch	Check voltage in socket
		Check circuit breaker switch – switch to position switched-on „I“
		Loose conductor from terminal - repair
		Check electrical cord – replace defective cord
	Motor winding failure, damaged thermal protection	Replace motor or windings
	Capacitor failure	Replace capacitor
Compressor often switches on	Seized piston or other rotating part	Replace damaged components
	Pressure switch does not switch	Check function of pressure switch
	Air leakage in pneumatic system	Check pneumatic system – seal loose connections
	Non-return valve (SV) leakage	Clean non-return valve, replace seals, replace non-return valve
Low pressure in the air tank (compressor running constantly)	Large volume of condensed liquid in pressure tank	Drain condensed liquid
	Low compressor capacity	Check time of filling the air tank
	High air consumption of supplied equipment	Decrease air consumption Use compressor with higher capacity
	Leakage in pneumatic system	Check pneumatic system – seal loose connections
	Low capacity of the pump	Clean / replace the pump
Prolonged operation of the	Pump failure	Clean / replace the pump
	Dryer failure	Replace dryer
	Air leakage in pneumatic system	Check pneumatic system – seal loose connection

compressor	Worn piston ring	Replace worn piston ring
	Dirty inlet filter	Replace dirty filter with new filter
	Incorrect function of solenoid valve	Repair or replace fan or coil
Compressor is noisy (knocking, metal noises)	Damaged piston bearing, piston rods, motor bearing	Replace damaged bearing
	Loose (cracked) dampening element (spring)	Replace damaged spring
Dryer does not dry (condensed water in the air) *	Defective cooler fan	Replace fan Check power source
	Damaged dryer	Replace dryer
	Defective automatic condensate drain	Clean / replace
Degraded drying performance – high-pressure dew point (condensed water in the air)	Low operating pressure	Reduce the demand for air, check the output from the compressed air source, fix any leaks in the distribution system
	Regeneration solenoid valve not working	Check coil operation, replace if damaged Inspect the condition of the valve – clean the valve or replace if problems persist
	Air regeneration nozzle plugged	Clean or replace the nozzle (see product maintenance)
	Cooling fan not working	Check the power source to the fan Replace damaged fan
	Dirty cooler	Inspect the cooler and clean as necessary
	Silencer plugged at outlet from regeneration valve	Inspect the silencers. Clean or replace the silencer if flow resistance is too high or if heavily soiled.
Dryer emitting high levels of noise	Damaged fan	Replace damaged fan
	Damaged silencer	Replace the silencer
	Air leaking through relief valve at dryer inlet	Check the dryer connection to the mains and dryer connections, check the dryer operation, check the dryer's working pressure, and replace defective components.
Air leaking through relief valve at dryer inlet	High working pressure from air source	Check the pressure setting on the compressed air source
	Dryer inlet solenoid valve not working	Check the coil operation, replace if damaged Inspect the condition of the valve – clean the valve or replace if problems persist
	High pressure in equipment resulting from plugged filters	Check the internal filters and accessory filter assemblies. Clean or replace dirty filters.

)* Clean inside surfaces of the air tank thoroughly and remove all condensed liquid after dryer failure.

Once a dryer fault is cleared and after its reassembly, the dryer should quickly be regenerated, best when using continuous compressor operation at a pressure of around 1 bar for a period of at least

1 hour; then check the dryness of the compressed air.

Check the moisture content of the air exiting the air tank (see the Technical data chapter) to prevent damage to connected downstream equipment.

26. REPAIR SERVICE

Warranty and post-warranty repairs must be done by the manufacturer, its authorized representative, or service personnel approved by the supplier.

Attention.

The manufacturer reserves the right to make changes to the equipment without notice. Any changes made will not affect the functional properties of the equipment.

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**27. INSTALLATION RECORD**

1. Product: (model) DK50 4VR/50/M DK50 4VR/50S/M DK50 2x4VR/110/M DK50 2x4VRS/M		2. Serial number:	
3.1. User's name:			
3.2. Address of installation:			
4. Equipment connected to the compressor:			
5. Installation / Commissioning:		6. Contents of operator training:	
Product completeness check **	Y	Description of the product and functions**	Y
	N		N
Documentation completeness check **	Y	Product operation: turning on/off, controls, control procedures, data on the display panel, alarms, operation in alarm conditions**	Y
	N		N
Installation/connection to equipment **	Y	Product maintenance: maintenance intervals, maintenance procedure, service intervals, operating activities**	Y
	N		N
Functional test **	Y	Safety measures, warnings – their meaning and compliance **	Y
	N		N
Notes:			
7. Operator instructed on safety measures, operations and maintenance:			
Name:		Signature:	
Name:		Signature:	
Name:		Signature:	
8. Installation and instruction performed by: First name/Last name		Signature:	
Company:		Address:	
Phone:			
Email:		Date:	
9. Distributor:			
Company:		Address:	
Contact person:			
Phone:		Email:	

** mark with an "X" in points 5 and 6 (Y - yes /N - no). Enter any observations from points 5 and 6 into the "Notes" section

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