# DUO DUO 2V, DUO 2

- EN User manual
- DE Benutzerhandbuch
- FR Manuel de l'utilisateur
- RU Руководство пользователя
- PL Instrukcja obsługi
- SK Návod na použitie
- CS Návod k použití

CE 2460

COMPRESSOR  
KOMPRESSOR  
COMPRESSEUR  
КОМПРЕССОР  
SPRĘŻARKA  
KOMPRESOR

DUO  
DUO 2V, DUO 2

**EKOM spol. s r. o.**  
Priemyselná 5031/18  
SK-921 01 Piešťany  
Slovak Republic  
tel.: +421 33 7967255  
fax: +421 33 7967223

[www.ekom.sk](http://www.ekom.sk)  
email: [ekom@ekom.sk](mailto:ekom@ekom.sk)

DATE OF LAST REVISION  
DATUM DER LETZTEN ÜBERARBEITUNG  
DATE DE LA DERNIÈRE RÉVISION  
ДАТА ПОСЛЕДНЕЙ РЕДАКЦИИ  
DATA OSTATNIEJ AKTUALIZACJI  
DÁTUM POSLEDNEJ REVÍZIE  
DATUM POSLEDNÍ REVIZE

09/2024

NP-DUO-18\_09-2024-MD  
112000355-000

<b>EN</b>	<b><u>CONTENTS .....5</u></b>
<b>DE</b>	<b><u>INHALT.....46</u></b>
<b>FR</b>	<b><u>TABLES DES MATIÈRES .....88</u></b>
<b>RU</b>	<b><u>ОГЛАВЛЕНИЕ .....130</u></b>
<b>PL</b>	<b><u>SPIS TREŚCI.....174</u></b>
<b>SK</b>	<b><u>OBSAH.....216</u></b>
<b>CS</b>	<b><u>OBSAH.....258</u></b>

**CONTENTS**

<b>IMPORTANT INFORMATION .....</b>	<b>6</b>
1. CONFORMITY WITH THE REQUIREMENTS OF EUROPEAN UNION DIRECTIVES.....	6
2. INTENDED USE.....	6
3. CONTRAINDICATIONS AND SIDE-EFFECTS .....	6
4. WARNINGS AND SYMBOLS .....	6
5. WARNINGS .....	7
6. STORAGE AND TRANSPORT .....	9
7. TECHNICAL DATA .....	10
8. PRODUCT DESCRIPTION .....	16
9. PRODUCT FUNCTIONALITY .....	18
10. PNEUMATIC DIAGRAM.....	23
<b>INSTALLATION .....</b>	<b>24</b>
11. CONDITIONS FOR USE .....	24
12. PLACEMENT OF THE COMPRESSOR .....	25
13. PNEUMATIC CONNECTIONS .....	28
14. ELECTRICAL CONNECTIONS .....	31
15. CONNECTION DIAGRAM.....	33
<b>OPERATION .....</b>	<b>35</b>
16. COMMISSIONING .....	35
17. SWITCHING THE COMPRESSOR ON .....	36
18. COMPRESSOR SHUT-DOWN .....	36
<b>MAINTENANCE .....</b>	<b>37</b>
19. DEVICE MAINTENANCE .....	37
<b>TROUBLESHOOTING .....</b>	<b>44</b>
20. REPAIR SERVICE .....	45
21. STORAGE.....	45
22. DISPOSAL OF THE DEVICE .....	45

## **IMPORTANT INFORMATION**

### **1. CONFORMITY WITH THE REQUIREMENTS OF EUROPEAN UNION DIRECTIVES**

This product conforms to the requirements of the Medical Device Directive (93/42/EEC), and is safe for the intended use if all safety instructions are followed.

### **2. INTENDED USE**

The compressor and suction unit is used as a source of clean, oil-free compressed air to power and cool compressed air devices on dental sets, and as a source of vacuum for suction systems. The device can be used with all types of dental sets equipped with a suction block.

**Compressed air supplied by the compressor is unsuitable for use with artificial lung ventilation devices.**

Any other use of the product beyond its intended use is considered an incorrect use. The manufacturer is not liable for any damages or injuries resulting from the incorrect use.

### **3. CONTRAINDICATIONS AND SIDE-EFFECTS**

There are no contraindications or side-effects known.

### **4. WARNINGS AND SYMBOLS**

The following symbols are used in the user manual, device and its packaging to denote important details and information:

General warnings

Warning

Danger, electric shock hazard

Refer to instruction manual

CE-marking

Medical device

Serial number

Article number

Unique Device Identifier

Compressor is controlled automatically and may start without warning

**DUO, DUO 2V, DUO 2**

Caution! Hot surface

Ground connection

Terminal for ground connection

Handling mark on package – FRAGILE

Handling mark on package – THIS SIDE UP

Handling mark on package – KEEP DRY

Handling mark on package – TEMPERATURE LIMITATIONS

Handling mark on package – LIMITED STACKING

Mark on package – RECYCLABLE MATERIAL

Manufacturer

Danger of biological hazard

**5. WARNINGS**

The product is designed and manufactured to be safe for the user and the surrounding environment when used in the defined manner. Keep the following warnings in mind.

**5.1. General warnings**

**READ THE USER MANUAL CAREFULLY BEFORE USING THE DEVICE AND KEEP IT FOR FUTURE USE!**

- The user manual supports the correct installation, operation and maintenance of the product. Careful review of this manual will provide the information necessary for the correct operation of the product for its intended use.
- Original packaging should be retained for the return of the device. Only the original packaging ensures protection of the device during transport. If it is necessary to return the product during the warranty period, the manufacturer is not liable for damages caused by improper packaging.
- This warranty does not cover damages originating from the use of accessories other than those specified or recommended by the manufacturer.
- The manufacturer only guarantees the safety, reliability and function of the device if:
  - installation, new settings, changes, expansion, and repairs are performed by the manufacturer or an organisation authorised by the manufacturer.
  - the product is used pursuant to the user manual.

- The user manual corresponds to the configuration of the product and its compliance with the applicable safety and technical standards at the time of printing. The manufacturer reserves all rights for the protection of its configuration, methods and names.
- This user manual is the original instructions. Translation is performed in accordance with the best available knowledge.

### 5.2. General safety warnings

The manufacturer designed and manufactured the product to minimise all risks when used correctly for the intended use. The manufacturer considers it its obligation to lay down the following general safety precautions.

- Use and operation of the product must comply with all laws and local regulations valid in the place of use. The operator and user are responsible for following all the appropriate regulations in the interests of performing work safely.
- Only the use of original parts guarantees the safety of operating personnel and the flawless operation of the product itself. Only the accessories and parts mentioned in the technical documentation or expressly approved by the manufacturer should be used.
- The operator must ensure that the device is functioning correctly and safely before every use.
- The user must be familiar with the operation of the device.
- Do not use the product in environments with a risk of explosion.
- The user must inform the supplier immediately if any problem directly related to the operation of the device occurs.

### 5.3. Safety warnings on protection from electric current

- The device must only be connected to a properly installed, earthed socket.
- Before the product is plugged in, ensure that the mains voltage and frequency stated on the product are in compliance with the values of the mains.
- Prior to putting the device into operation check for any damage to the connected pneumatic lines and electrical wiring. Replace damaged pneumatic lines and electrical wirings immediately.
- Immediately disconnect the product from the mains (remove power cord from the socket) in hazardous situations or in the case of a technical malfunction.
- During all repairs and maintenance, ensure that:
  - the mains plug is removed from the power socket
  - pressure is vented from the air tank and pipes
- The product shall only be installed by a qualified technician.

## DUO, DUO 2V, DUO 2

### 6. STORAGE AND TRANSPORT

The compressor is shipped from the factory in a transport packaging. This protects the device from damage during transport.

**The original compressor packaging must be used for transport whenever possible. Transport the compressor in an upright position, always secured with transport fixation.**

Protect the compressor from moisture, dirt and extreme temperatures during transport and storage. Store the compressor in its original packaging in a warm, dry, and dust-free area. Do not store near any chemical substances.

Keep the packaging material, if possible. If not, please dispose of the packaging material in an environmentally-friendly way. Packaging cardboard can be recycled with old paper.

**The compressor may only be transported when all air pressure has been vented. Before moving or transporting the compressor, release all air pressure from the tank and hoses and drain condensate from the air tank.**

**Storing or shipping the equipment in any conditions other than those specified below is prohibited.**

#### Ambient conditions for storage and transport

Products may only be stored and transported in vehicles that are free of any traces of volatile chemicals under the following conditions:

<b>Temperature</b>	-25°C to +55°C, 24 h at up to +70°C
<b>Relative humidity</b>	max. 90% (non-condensing)

**7. TECHNICAL DATA**

Compressors are designed for dry and ventilated indoor environments with the following conditions:

<b>Temperature</b>	+5°C to +40°C
<b>Relative humidity</b>	max. 70%


		<b>DUO</b>	<b>DUO/M</b>
Compressor 6 – 8 bar		DK50 PLUS	DK50 PLUS/M
Suction unit		1	1
Nominal voltage / frequency (*)	V / Hz	230 / 50 230 / 60	230 / 50 230 / 60
Performance at 6 bar	Lit.min <sup>-1</sup>	70 80	60 70
Working pressure (**)	bar	6.0 – 8.0	6.0 – 8.0
Suction unit capacity at 5 kPa	Lit.min <sup>-1</sup>	800	800
Suction unit vacuum, max.	kPa	12	12
Max. current	A	7.0 8.0	7.2 8.1
Compressor motor rating	kW	0.55	0.55
Air tank volume	Lit.	25	25
Air quality – filtration	µm	-	0.1
Maximum operating pressure of safety valve	bar	11.0	11.0
Noise level at 5 bar	LpFA [dB]	≤47 ≤50	≤47 ≤50
Duty cycle		S1-100%	S1-100%
PDP drying performance at 7 bar		-	≤ +3°C
Time to fill air tank from 0 to 7 bar	s	149 126	198 159
Dimensions (net) w x d x h	mm	560x684x1251	560x684x1251
Net weight (***)	kg	114	119
Classification under EN 60601-1		Class I.	
Classification pursuant to MDD 93/42 EEC,2007/47 EC		IIa	

**Notes:**

- (\*) When ordering, state the version of the compressor
- (\*\*) For other range of pressure:consult with the supplier
- (\*\*\*) Information about the weight is for informative purposes only and applies to the product without any additional accessories.

# DUO, DUO 2V, DUO 2

## Dependence of compressor output on working pressure

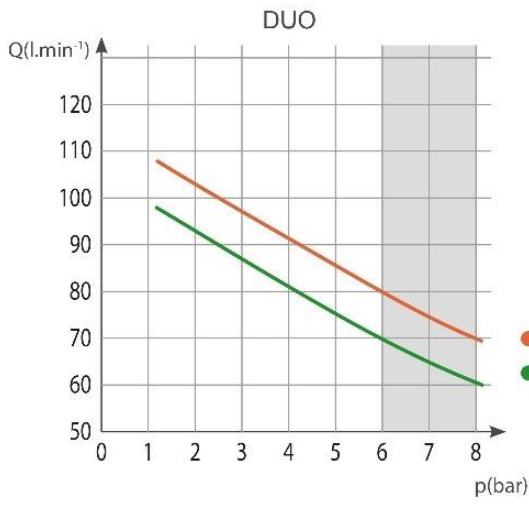		DUO 2V	DUO 2V/M	DUO 2	DUO 2/M
Compressor 6 – 8 bar		DK50 2V	DK50 2V/M	DK50 2V	DK50 2V/M
Suction unit		1	1	2	2
Nominal voltage / frequency (*)	V / Hz	230 / 50 230 / 60	230 / 50 230 / 60	230 / 50 230 / 60	230 / 50 230 / 60
Performance at 6 bar	Lit.min <sup>-1</sup>	135	104	135	104
Working pressure (**)	bar	6.0 – 8.0	6.0 – 8.0	6.0 – 8.0	6.0 – 8.0
Suction unit capacity at 5 kPa	Lit.min <sup>-1</sup>	800	800	2x800	2x800
Suction unit vacuum, max.	kPa	12	12	12	12
Max. current	A	11.8 12.5	12.0 12.7	14.8 15.4	15.0 15.6
Compressor motor rating	kW	1.2	1.2	1.2	1.2
Air tank volume	Lit.	25	25	25	25
Air quality – filtration	µm	-	0.1	-	0.1
Maximum operating pressure of safety valve	bar	11.0	11.0	11.0	11.0
Noise level at 5 bar	LpFA [dB]	≤ 51.5	≤ 55.0	≤ 51.5	≤ 55.0
Operating mode		S1-100%	S1-100%	S1-100%	S1-100%
PDP drying performance at 7 bar		-	≤ +3°C	-	≤ +3°C
Time to fill air tank from 0 to 7 bar	s	61	78	61	78
Dimensions (net) w x d x h	mm	560x684x1251	560x684x1251	560x684x1251	560x684x1251
Net weight (***)	kg	121	126	133	138
Classification under EN 60601-1		Class I.			
Classification pursuant to MDD 93/42 EEC,2007/47 EC		IIa			

**Notes:**

- (\*) When ordering, state the version of the compressor
- (\*\*) For other range of pressure:consult with the supplier
- (\*\*\*) Information about the weight is for informative purposes only and applies to the product without any additional accessories.

**Dependence of compressor output on working pressure**

## DUO, DUO 2V, DUO 2

### 7.1. Free air delivery (FAD) correction due to elevation

Capacity given in the form of FAD („Free Air Delivery“) applies to the following conditions:

Altitude	0 m.n.m.	Temperature	20°C
Atmospheric pressure	101325 Pa	Relative humidity	0%

To calculate FAD compressor capacity in dependence on altitude, it is necessary to apply correction factor according to the following table:

Altitude [m.n.m.]	0 -1500	1501 - 2500	2501 - 3500	3501 - 4500
FAD correction factor	1	0.80	0.71	0.60

### 7.2. Electromagnetic compatibility declaration

Medical equipment needs special precautions regarding electromagnetic compatibility (EMC) and needs to be installed and put into service according to the EMC information provided below.

Guidance and manufacturer's declaration - electromagnetic emissions		
Pursuant to IEC 60601-1-2:2014 - Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances		
The equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the equipment should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The equipment uses RF energy only for its internal functions. Therefore, the RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The equipment is suitable for use in all establishments, including domestic establishments and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	The equipment is not likely to cause any flicker, as the current flow is approx. constant after the start up.	

### Guidance and manufacturer's declaration - electromagnetic immunity

Pursuant to IEC 60601-1-2:2014 - Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances

The equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the equipment should assure that it is used in such an environment.

Immunity test	IEC 60601-1-2 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±15 kV air	±8 kV contact ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV 100 kHz repetition frequency Applied to mains connection	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV differential mode ±2 kV common mode	±1 kV L-N ±2 kV L-PE; N-PE Applied to mains connection	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruption, and voltage variations on power supply input lines IEC 60601-4-11	$U_T=0\%$ , 0,5 cycle (at 0, 45, 90, 135, 180, 225, 270 and 315°) $U_T=0\%$ , 1 cycle $U_T=70\%$ 25/30 cycles (at 0°) $U_T=0\%$ , 250/300 cycles	$U_T>95\%$ , 0,5 cycle (at 0, 45, 90, 135, 180, 225, 270 and 315°) $U_T>95\%$ , 1 cycle $U_T=70\%$ (30% dipp $U_T$ ), 25(50Hz)/30(60Hz) cycles (at 0°) $U_T>95\%$ , 250(50Hz)/300(60Hz) cycle	Mains power quality should be that of a typical commercial or hospital environment. The device stops and restarts automatically at each dip. In this case does not occur unacceptable pressure drop.
Power frequency (50/60 Hz) IEC 61000-4-8	N/A	N/A	The equipment does not contain magnetically sensitive components or circuits.

NOTE:  $U_T$  is the A.C. mains voltage prior to application of the test level.

## DUO, DUO 2V, DUO 2

### Guidance and manufacturer's declaration - electromagnetic immunity

Pursuant to IEC 60601-1-2:2014 - Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances

The equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the equipment should assure that it is used in such an environment.

Immunity test	IEC 60601-1-2 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the equipment, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.  <b>Recommended separation distance</b>  $d=1,2\sqrt{P}$  $d=1,2\sqrt{P}$ , 80 MHz to 800 MHz  $d=2,3\sqrt{P}$ , 800 MHz to 2,7 GHz
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2,7 GHz	3 V/m	where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).
Proximity fields from RF wireless communications equipment IEC 61000-4-3	9 to 28 V/m 15 specific frequencies (380 to 5800 MHz)	9 to 28 V/m 15 specific frequencies (380 to 5800 MHz)	Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey <sup>a</sup> , should be less than the compliance level in each frequency range <sup>b</sup> . Interference may occur in the vicinity of equipment marked with the following symbol:  

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

<sup>a</sup> Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the equipment is used exceeds the applicable RF compliance level above, the equipment should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the equipment.

<sup>b</sup> Over the frequency range 150 kHz to 80 kHz, field strengths should be less than 3 V/m.

**8. PRODUCT DESCRIPTION**

**8.1. Variants**

Compressor models are designed in the following variants:

<b>DUO</b>	This model is used for dental set models equipped with a suction block and designed for installation directly in the office
<b>DUO 2</b>	Models designed to operate with dental equipment sets equipped with a suction block. They are designed to supply two such dental equipment sets, i.e. two workstations
<b>DUO 2V</b>	Models designed to operate with dental equipment sets equipped with a suction block. They are designed to supply sets that use higher quantities of compressed air
<b>DUO/M</b>	Identical to the DUO model, but equipped with a membrane dryer
<b>DUO 2V/M</b>	Identical to the DUO 2V model, but equipped with a membrane dryer
<b>DUO 2/M</b>	Identical to the DUO 2 model, but equipped with a membrane dryer

**DUO**

**DUO 2V**

**DUO 2**

**8.2. Accessories**

Accessories not included in the standard order must be ordered separately!

**8.2.1. Automatic condensate drain**

The automatic condensate drain (AOK) automatically drains condensate from the compressor’s air tank based on a pre-set time interval. The condensate drain (AOK) is a suitable accessory for compressor models without dryers.

Type	Use	Article number
AOK 11	DK50 PLUS,DK50 2V	447000001-047

**8.2.1. Set of compressed air outlet filters**

The compressor may be equipped with a set of filters if specified. The filter set may be equipped with an air pressure regulator.

**Where a different level of air filtration is required, this requirement must be agreed upon with the supplier and specified in the order.**

## DUO, DUO 2V, DUO 2

Type	Use	Level of filtration ( $\mu\text{m}$ )	Article number
FS 30F	DK50 PLUS	1	604014119-014
FS 30M	Dk50 PLUS/M	1+0.1	604014119-015
FS 30S	DK50 2V	1+0.01	604014119-017
FS 30A	DK50 2V/M	1+0.01+A**	604014119-018

\*\* ) A – submicrofilter with activated carbon

### 8.2.2. Filter set regulator assembly

The compressor may be equipped with a pressure regulator of the compressed air outlet if specified. The regulator must be selected according to the application to the filter set, or separately. The regulator shall ensure constant pressure at the outlet.

Type	Use	Article number
Regulator complete (to the filter assembly)	DK50 PLUS, DK50 PLUS/M	604014130-000
REG11 (separate)	DK50 2V, DK50 2V/M	447000001-077

### 8.2.3. Filter set brackets

A suitable bracket must be ordered for every filter set.

Type	Use	Article number
Compressor-mounted bracket	DK50 PLUS, DK50 PLUS/M DK50 2V, DK50 2V/M	603014134-000

### 8.2.4. Condensation and filtration unit

The compressor may be additionally fitted with a condensation and filtration unit (KJF-1, KJF-2 or KJFR-1). The KJF-1, KJF-2 or KJFR-1 ensure that the compressed air from the air tank is cooled in the cooler and the condensate is captured in the filter and automatically separated from the pneumatic distribution system. The compressed air is filtered at the same time.

Type	Use	Level of filtration / $\mu\text{m}$ /	Pressure regulator	Article number
KJF-1	DK50 PLUS DK50 2V	5	no	450001011-001
KJFR-1			yes	450001011-002
KJF-2			no	450001021-001

### 8.2.5. Noise silencer

The noise silencer, with and without a filter, dampens the “whistling” noise caused by the suction pump impeller. Total noise reduction during suction unit operation is up to 4 dB.

- Noise silencer with a filter - a bacteriological filter is provided, making it suitable for installation directly in the office.
- Wall-mounted noise silencer - specifically designed for wall-mounted installation.
- Wall-mounted noise silencer with a filter - specifically designed for wall-mounted installation and equipped with a bacteriological filter.

Type	Variant	Use	Kit article no.
DS2	wall-mounted, <b>no</b> bacteriological filter	DUO, DUO 2V, DUO2	451000Z01-004
DS2	wall-mounted, <b>with</b> a bacteriological filter		451000Z01-005
DS4	<b>with</b> a bacteriological filter	DUO,DUO 2V	451000Z01-006
DS5	<b>with</b> a bacteriological filter	DUO2	451000Z01-007
DS6	<b>no</b> bacteriological filter	DUO,DUO 2V	451000Z01-008
DS7	<b>no</b> bacteriological filter	DUO2	451000Z01-009

### 8.2.6. Equipotential bonding socket

The socket allows an equipotential bonding. (Fig.10)

Type	Use	Name	Article no.
POAG-KBT6-EC	DUO, DUO 2V, DUO2	Zásuvka	033400075-000
FLEXI-S/POAG-HK6		Vodič (1 m)	034110083-000

## 9. PRODUCT FUNCTIONALITY

### Compressor with suction unit(Fig. 1)

The air pump (1) draws in atmospheric air through an inlet filter (8) and compresses it through a non-return valve (3) into an air tank (2) from which the device draws compressed air. If the pressure in the air tank drops to the switch-on pressure, the pressure switch (4) turns on the compressor and the compressor supplies compressed air into the air tank until it reaches the switch-off pressure, when the compressor switches off. The pressure hose is vented through the relief solenoid valve (13) once the air pump is shut off. The safety valve (5) prevents the pressure in the air tank from rising above the maximum allowed value. The drain valve (7) drains condensate from the air tank. Compressed, oil-free filtered air is stored in the air tank ready for use.

Condensate must be drained from the air tank at regular intervals (see Chapter 19.1).

The suction pump (36) (two aggregates on DUO2) draws in air, thereby creating vacuum in the suction lines that are connected to the device and is a source of vacuum for removing foreign substances from the doctor's work area. This air is then exhausted through an exhaust line routed outside the operator's work area.

### Compressor with suction unit and membrane dryer (Fig. 2)

The air pump (1) draws in air through an inlet filter (8) and compresses it, feeding it to the cooler (14) through the filter (15) and the check valve (3) into the dryer (9), and feeds the clean and dry air into the air tank (2). A part of the air is released from the dryer with the captured moisture, which is manifested as a light air stream along the body of the dryer (9). Condensate from the filter is automatically drained at regular intervals into the collecting bottle via the condensate drain solenoid valve (16). The dryer ensures continuous drying of the compressed air. The drain valve (7) drains condensate from the air tank when drying performance is checked. Compressed, oil-free filtered air is stored in the air tank ready for use.

For the membrane dryer to work as intended, the dryer is connected directly to the air tank without the use of a check valve. As the membrane dryer contains selective membrane fibres, there is a small amount of air leakage through them (about 0.5 bar/30 min).

As a result, there is a gradual, slow drop in pressure even when there is no demand for compressed air (from the air tank), however, this is not a malfunction.

The pressure vessel does not need to be drained.

The suction pump (36) (two aggregates on DUO2) draws in air, thereby creating vacuum in the suction lines that are connected to the device and is a source of vacuum for removing foreign

## DUO, DUO 2V, DUO 2

substances from the doctor's work area. This air is then exhausted through an exhaust line routed outside the operator's work area.

### Compressor cabinet(Fig.3)

The soundproof cabinet provides compact covering of the compressor, allowing sufficient exchange of cooling air. With its design it can be placed in a dentist's office as part of furniture. The fan under the air pump of a compressor provides cooling of compressor and it is in operation at the same time with an engine of the compressor. After prolonged use the temperature in the cabinet may rise above 40°C, causing the cooling fan blower to automatically turn on. After cooling the cabinet area down to 32°C the fan blower turns off automatically. Door of the casing with right opening may be changed to left opening (see Chapter 12.2).

**Make sure that nothing impedes the free flow of air under and around the compressor. Never cover the hot air outlet on the top back side of the cabinet.**


**If placing the compressor on a soft floor such as carpet, create space for ventilation between the base and floor or the cabinet and floor, e.g. underpin the footings with hard pads.**

### Description for Figures 1- 3

- |                                     |                                  |
|-------------------------------------|----------------------------------|
| 1. Air pump                         | 19. Bottle                       |
| 2. Air tank                         | 20. Magnetic holder              |
| 3. Non-return valve                 | 21. Fan                          |
| 4. Pressure switch                  | 22. Cabinet fan                  |
| 5. Safety valve                     | 23. Door pin                     |
| 6. Pressure gauge                   | 24. Rectifying screw             |
| 7. Drain valve                      | 25. Lock                         |
| 8. Inlet filter                     | 26. Supporting bracket           |
| 9. Dryer                            | 27. Wall stop                    |
| 10. -                               | 28. Switch                       |
| 11. -                               | 29. Pressure gauge               |
| 12. -                               | 30. Door hinge                   |
| 13. Solenoid valve                  | 31. Handle                       |
| 14. Dryer cooler                    | 32. Casters                      |
| 15. Filter                          | 33. Pressure gauge hose          |
| 16. Condensate drain solenoid valve | 34. Socket                       |
| 17. Compressed air outlet           | 35. Suction unit switching block |
| 18. Circuit breaker switch          | 36. Suction unit                 |

Fig.1 - DUO - Compressor with suction unit

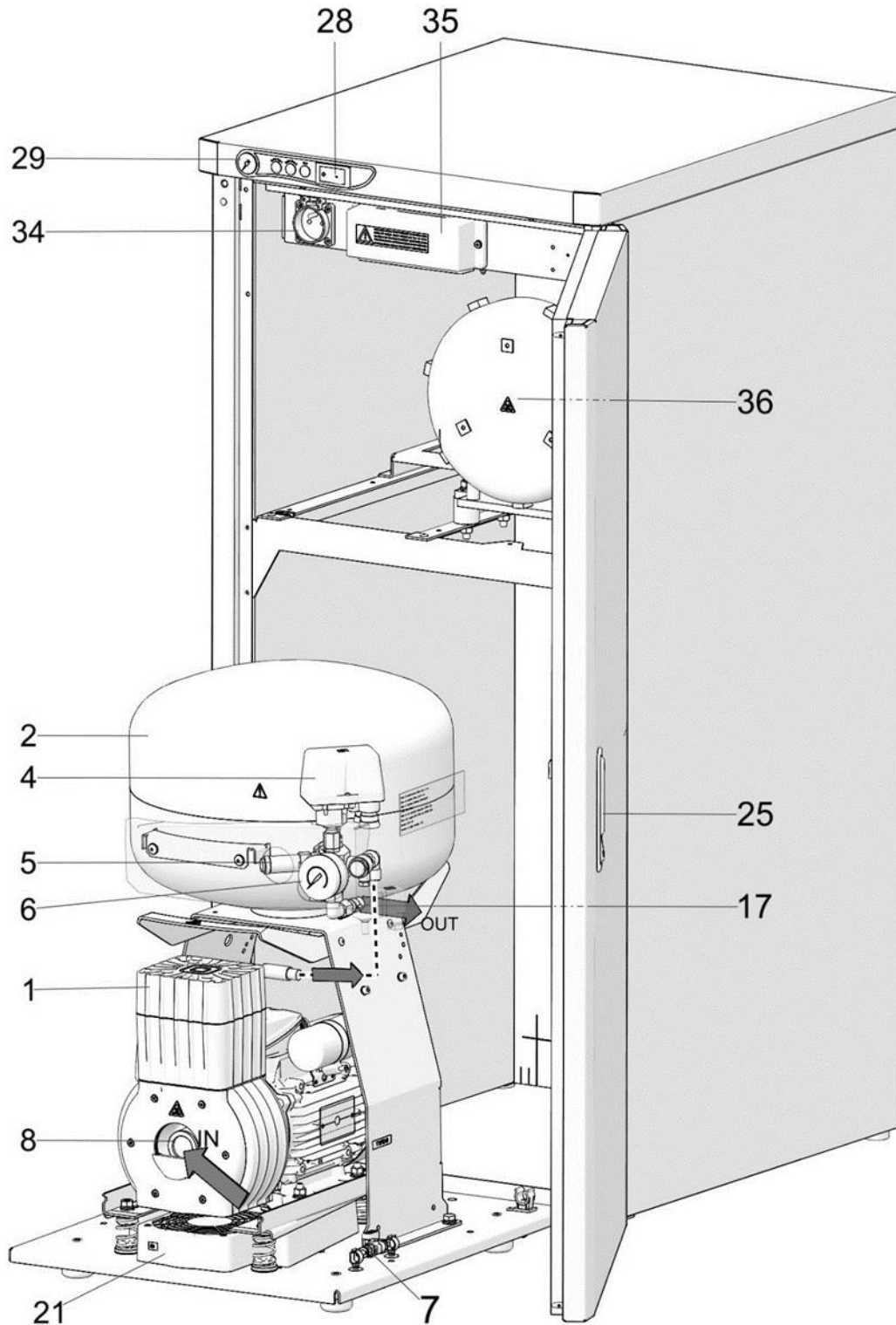**DUO, DUO 2V, DUO 2**

**Fig.2 - DK50 2V/M - Compressor with dryer**



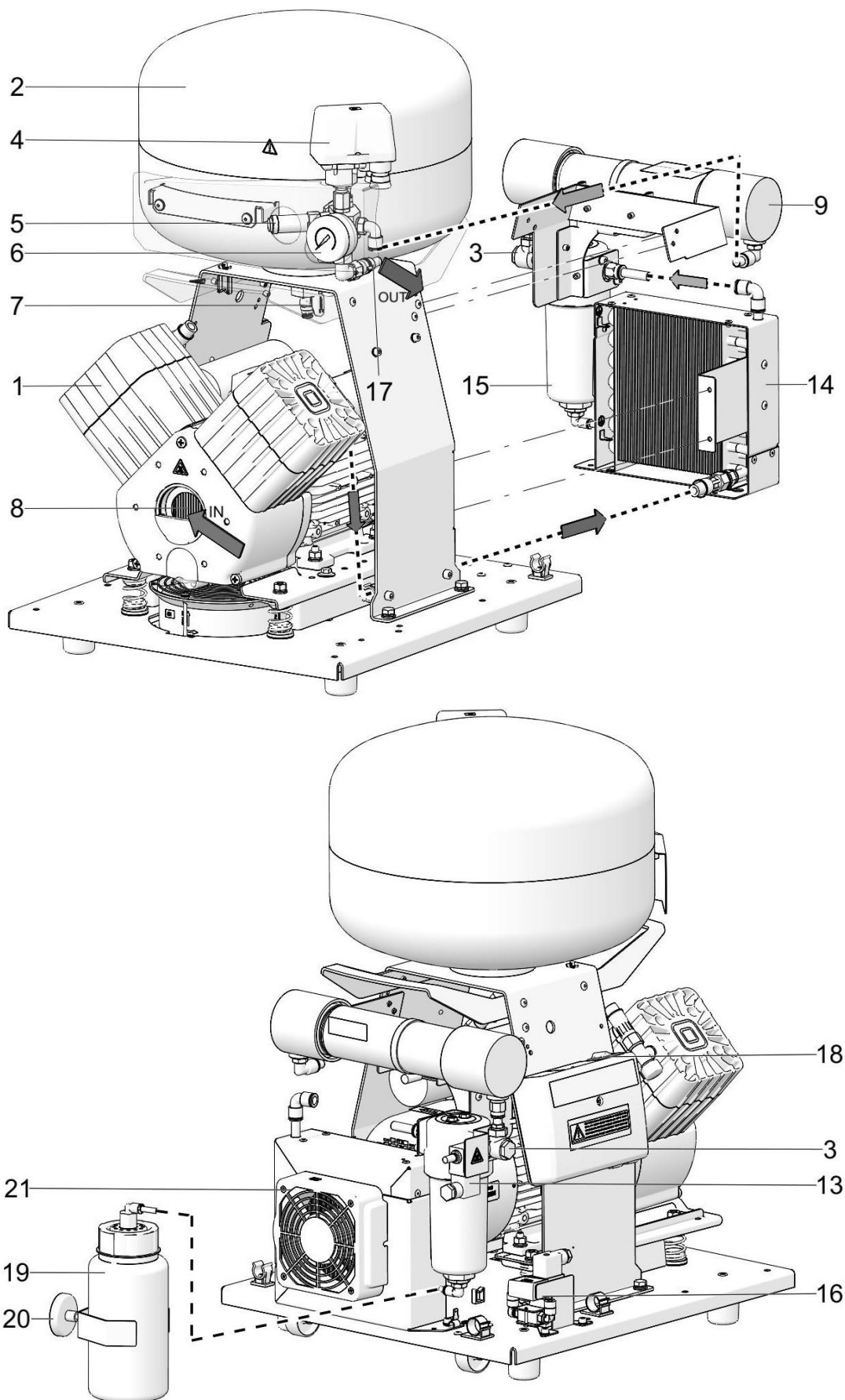
Fig.3 – Cabinet DUO

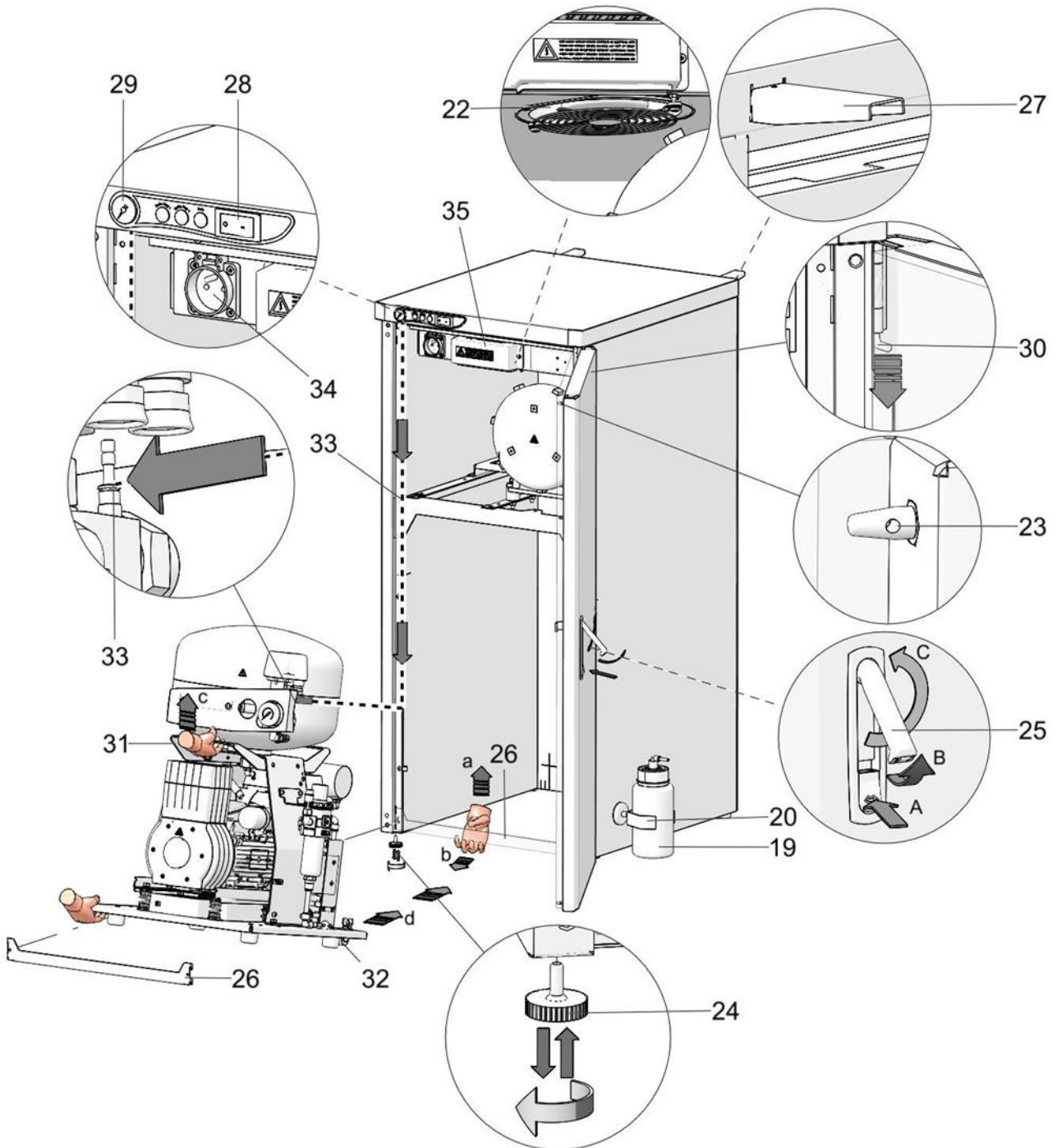
**DUO, DUO 2V, DUO 2**

**10. PNEUMATIC DIAGRAM**

**DK50 PLUS, DK50 2V**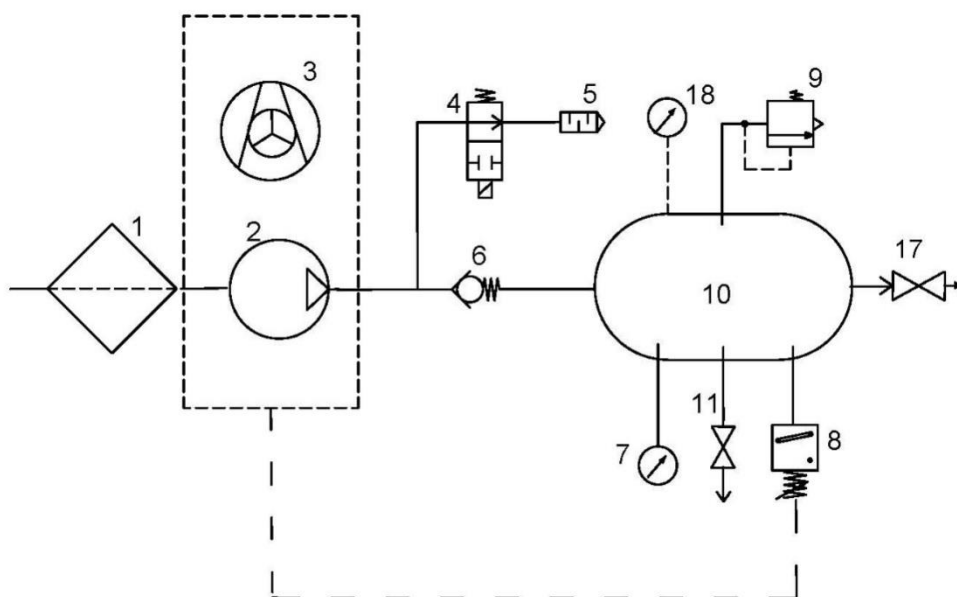

**DK50 PLUS/M, DK50 2V/M**

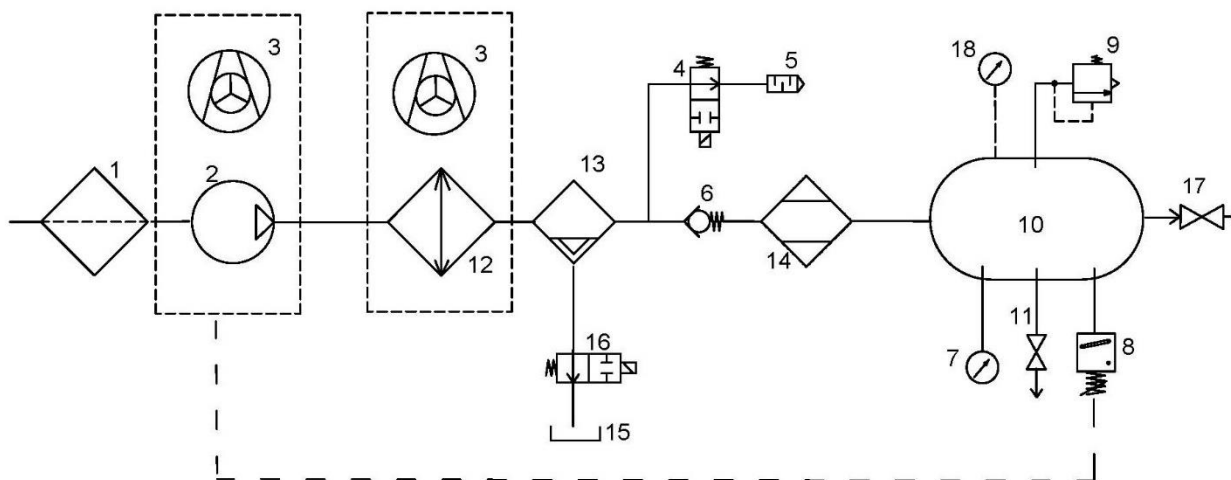**Legend for compressed air diagram**

- |                     |                                     |
|---------------------|-------------------------------------|
| 1. Inlet filter     | 10. Air tank                        |
| 2. Compressor       | 11. Drain valve                     |
| 3. Fan              | 12. Cooler                          |
| 4. Relief valve     | 13. Coalescence filter              |
| 5. Noise silencer   | 14. Membrane dryer                  |
| 6. Non-return valve | 15. Condensate collection bottle    |
| 7. Pressure gauge   | 16. Condensate drain solenoid valve |
| 8. Pressure switch  | 17. Outlet valve                    |
| 9. Safety valve     | 18. Cabinet pressure gauge          |

## INSTALLATION

### 11. CONDITIONS FOR USE

- The device may only be installed and operated in dry, well-ventilated and clean environments with environmental parameters that meet the requirements specified in Chapter 7, Technical data. The compressor must be installed so that it is accessible at all times for operating and maintenance. Please ensure that the label on the device is readily accessible.
- The device must stand on a flat, sufficiently stable base (be aware of the weight of the compressor, see Chapter 7, Technical data).
- The device may not be operated outdoors or in otherwise wet or damp environments. Do not use the compressor in the presence of explosive gases, dust, or flammable liquids.
- Before connecting the compressor to medical equipment, the supplier must confirm that it meets all requirements for its use. Refer to the technical data of the product for this purpose. When a unit is to be built-in, classification and evaluation of compatibility must be done by the manufacturer or supplier of the product to be used.
- Any other use or use outside this framework is not considered as intended use. The manufacturer is not responsible for any damages arising from such use. The operator/user assumes all risks.

**Only a qualified professional can install the compressor and place it into operation for the first time. This professional is obliged to train operating staff as to the use and maintenance of the device. Installation and training of all operators shall be confirmed by the installer's signature on the certificate of installation. (See warranty card)**

**Prior to installation, ensure that the compressor is free of all transport packaging and stabilizers to avoid any risk of damage to the product.**

**Parts of the air pump and suction unit may be hot and reach hazardous temperatures during compressor operation and may pose a contact risk for operators or materials. Burn or fire hazard! Caution! Hot surface!**

**You may notice a “new product” odour when you first place the product into service (for a short period of time). This odour is temporary and does not impede the normal use of the product. Ensure the space is properly ventilated after installation.**

### Ambient operating conditions


Temperature	+5°C to +40°C
Relative humidity	max. 70%
Absolute humidity	max. 15 g/m <sup>3</sup>


## 12.2. Changing door direction

- Disconnect the earthing conductor from the cabinet. (A)
- Remove the door, the rectifying screws on both sides of the cabinet (3) and the hinge (4) bracket (2). (B)
- Install the bracket for hinge (2) on the left side of the cabinet. (C)
- Rotate the door 180°. (B)
- Insert the spacer (5) between hinge (1) and the bottom of the door. (C)
- Install the door, the 2 rectification screws (3) and connect the earthing conductor. (C)
- Remove the door lock (6) and rotate 180°. (D)
- Remove the latch (7) and rotate 180°. (D)
- Install the lock. (D)



A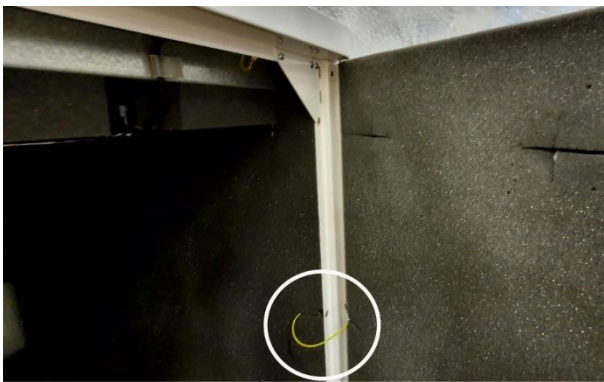

B

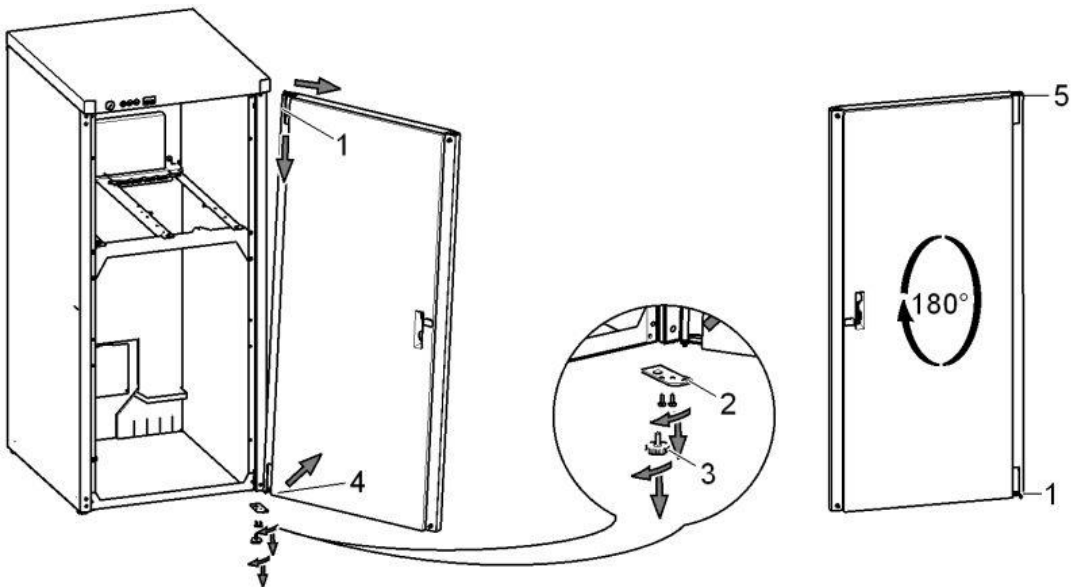**DUO, DUO 2V, DUO 2**

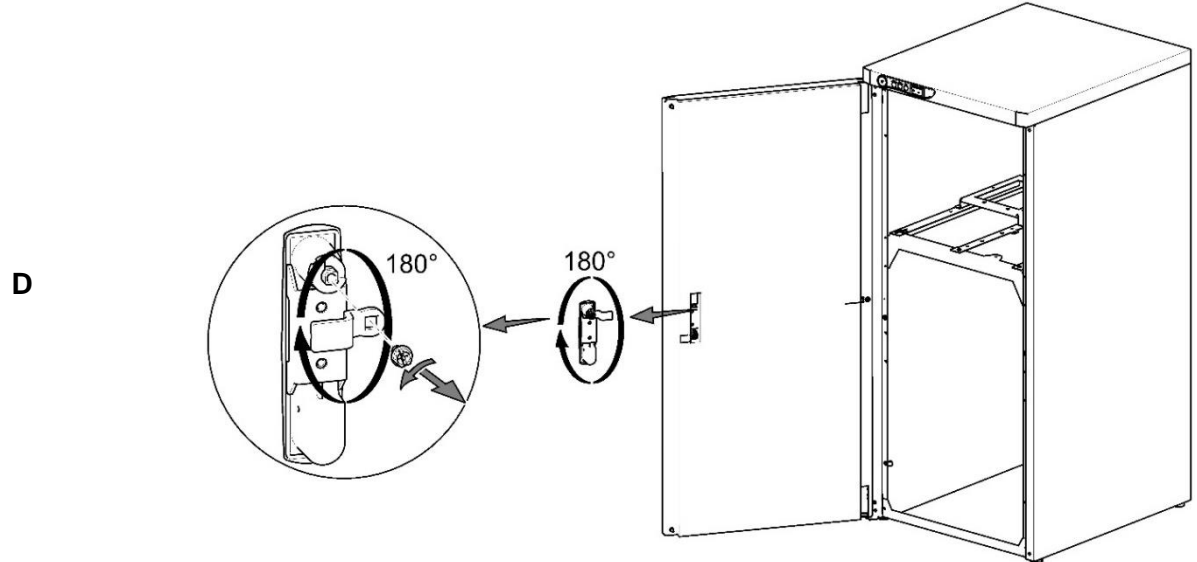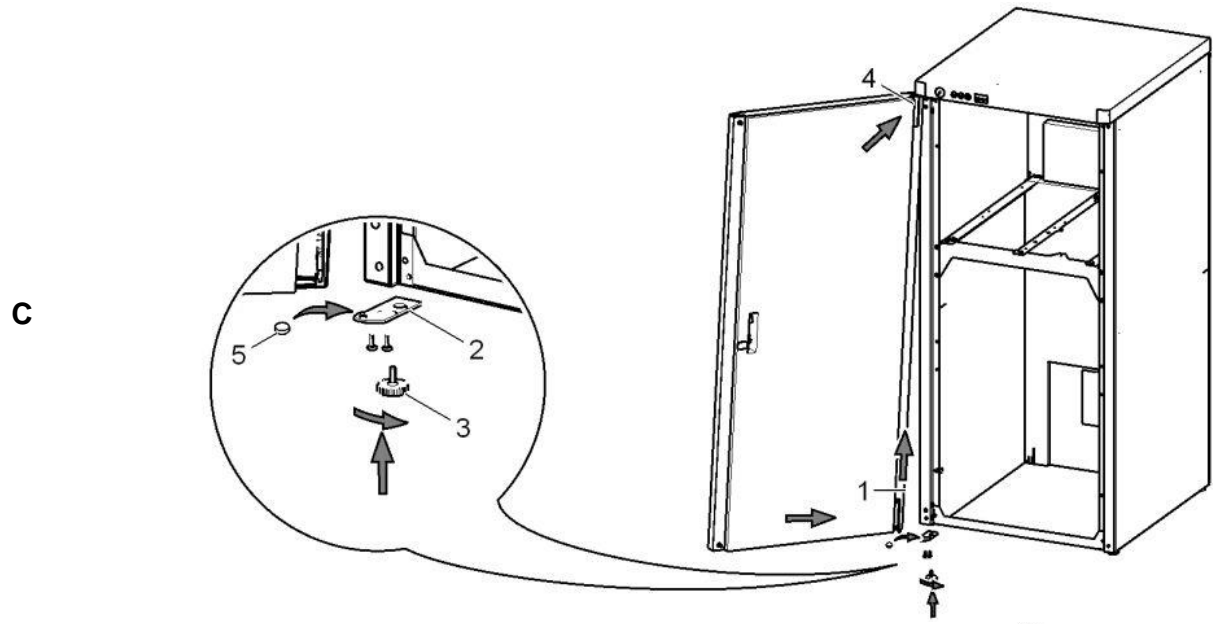
### 12.3. Adding sound-insulating material into the cabinet

A component from the sound-insulating material is to be installed in the opening of the cabinet of the compressor without a dryer.

This component is provided in the basic equipment delivered with the product.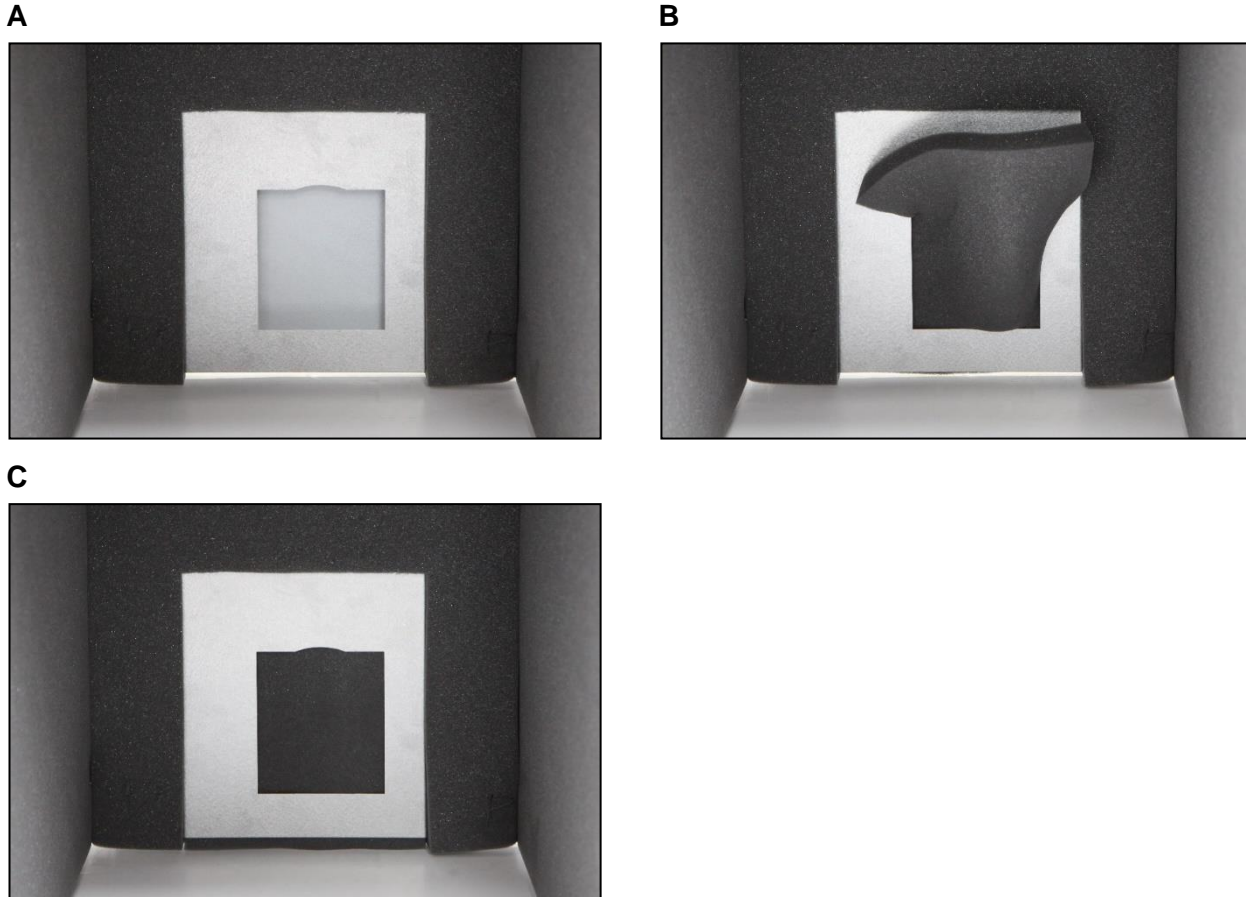

## 13. PNEUMATIC CONNECTIONS

### 13.1. Connecting to the compressed air outlet

- The pressure line (2) is attached to the compressed air outlet (1) on the compressor. Run the hose to the compressed air manifold or directly to the dental set..
- Route the pressure hose through the opening in the rear wall of the cabinet for cabinet-mounted compressors. (Fig. 8)

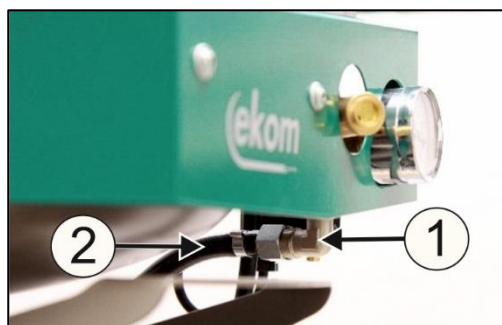
Fig. 6

## DUO, DUO 2V, DUO 2

### 13.2. Connecting the cabinet pressure gauge to the compressor

- Remove the plug (1) from the threads (2) on the pneumatic block of the compressor.
- Connect the cabinet pressure gauge hose to the threaded fitting.

Fig. 7

### 13.3. Connecting the condensate vessel

- Route the hose (1) through the opening in the rear wall of the cabinet for compressor models with a dryer and connect it to the vessel.

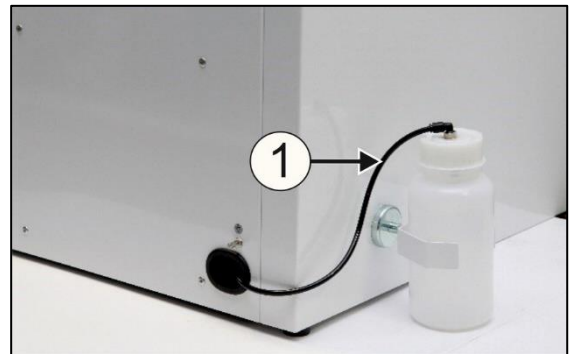
Fig. 8

### 13.4. Vacuum connector

(Fig. 9)


The suction pump is equipped with suction and discharge hoses. Hoses connected to the inlet/outlet to and from the suction aggregate are routed along the rear wall of the cabinet to its lower part. Suction pump hoses may be connected to the piping in the floor or routed through the rear opening in the cabinet (10). Connect the suction hose to the device and the discharge hose to the piping routed outside the operator's work area. A noise silencer can be added at the suction unit outlet to reduce the noise level caused by air passing through the hose (see Chapter 8.2. Accessories). A noise silencer with a bacteriological filter must be added if the air outlet from the suction unit is vented to an indoor area.

**Risk of biological contamination. The discharge hose from the suction pump needs to be routed outside the operator's working area otherwise a silencer with a bacteriological filter needs to be added.**


Fig. 9

**A: DUO,DUO2V installation (distribution systems in the floor)**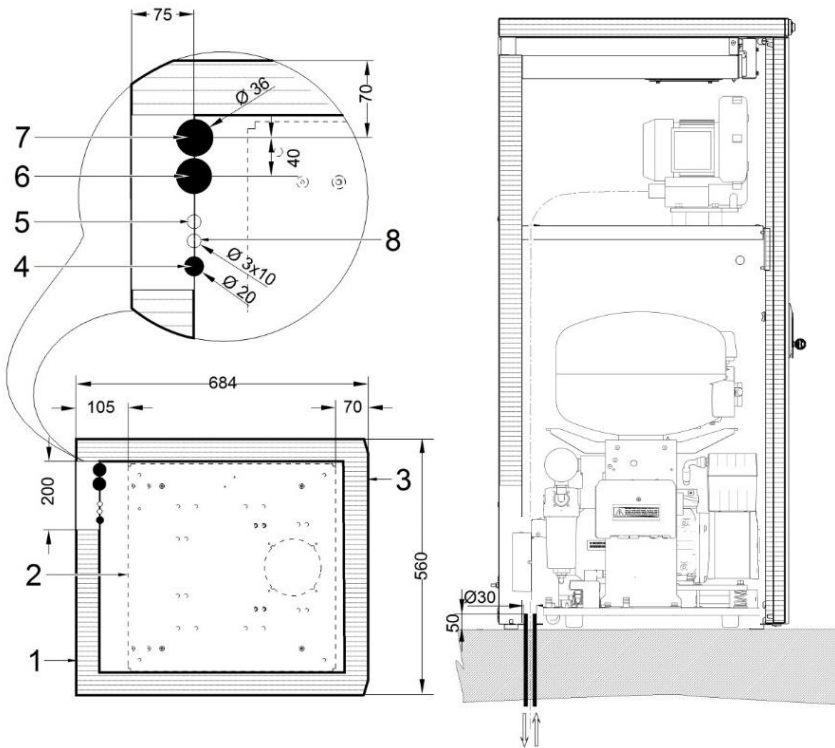


1. Outline of the cabinet
2. Outline of the base
3. Front section - door
4. Compressed air connection G3/8"
5. Suction unit control input 2A x 0.75
6. Suction unit intake
7. Suction unit discharge
8. Power supply input 230 V/50(60) Hz, 3G x 1.5
9. Condensate drain for version with dryer

(Connected via the opening in the rear of the cabinet)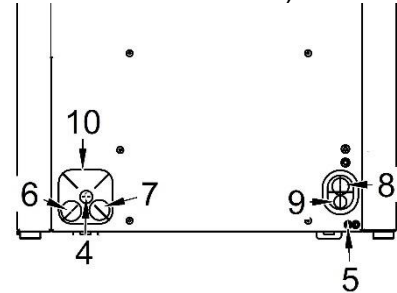

- All dimensions in millimetres
- Distance of the rear wall of the product from the obstacle - wall min. 100mm

**B: DUO2 installation (distribution systems in the floor)**

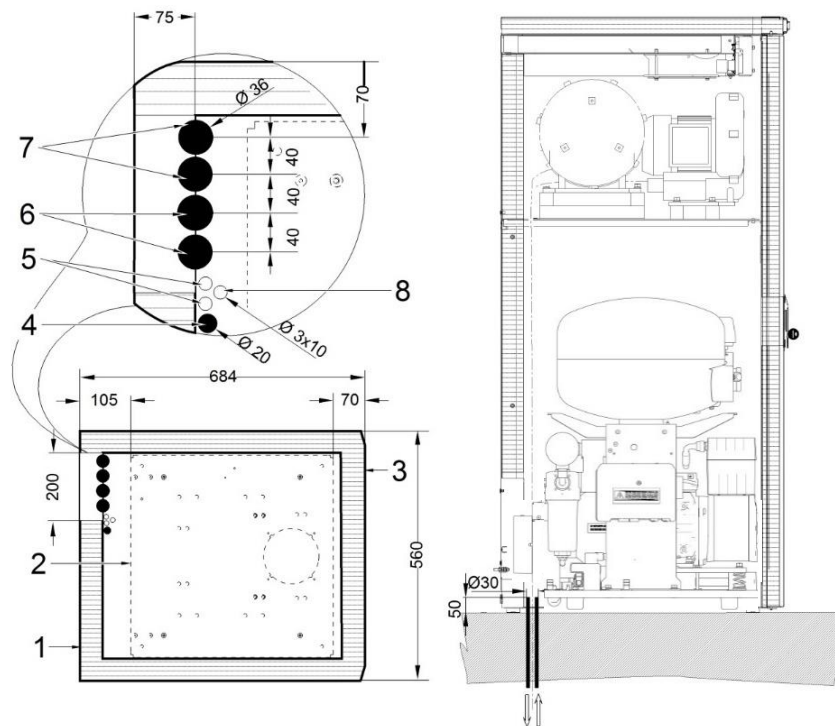1. Outline of the cabinet
2. Outline of the base
3. Front section - door
4. Compressed air connector G3/8"
5. Control voltage feed for suction unit 2A x 0.75
6. Intake + discharge from 1st suction unit
7. Intake + discharge from 2nd suction unit (DUO 2)
8. Power supply feed 230 V/50(60) Hz 3G x 1.5

- All dimensions in millimetres
- Spacing from wall min. 100 mm

## DUO, DUO 2V, DUO 2

### 14. ELECTRICAL CONNECTIONS

The product is equipped with a grounded plug. Insert the mains plug into a rated mains socket.

**Ensure full compliance with all local electrical codes. The mains voltage and frequency must comply with the data stated on the device's label.**

- Keep the socket easily accessible to ensure that in an emergency the device can be safely disconnected from the mains.
- Connection to the power distribution cabinet must be max.16 A.
- Connect the pin for equipotential bonding  $\varnothing$  6 mm (1) to the electrical circuit using the defined method under valid electrical engineering regulations. The socket for equipotential bonding (2) is an accessory and is not provided in the basic package.

Fig. 10

**Ensure the electrical cable does not touch hot compressor components. Risk of electric shock!**

**If any electrical cord or air hose is damaged it must be replaced immediately.**

- Route the power cord the rear opening in the cabinet or connect to the floor outlet (Fig. 8, 9).

#### 14.1. Suction unit control connector

(Fig. 11)

Route the suction unit control (24 V AC/DC) cord (12) through the opening in the rear of the cabinet (1) or the piping in the floor (11), and secure with the clips (2) located on the left panel of the cabinet at the bottom and through the cavity (3) in the front of the cabinet. Move the sound insulation (4) from the corners of the cabinet before inserting the cord into the clips. Remove the cover on the electrical circuit conduit (5) and the electrical panel cover (6). Connect the suction unit control cord to the printed circuit board as shown in the electrical diagram, insert into the cavity (3) in the front of the cabinet, behind the sound insulation on the side of the cabinet and into the electrical conduit (7).

**Ensure the electrical cable does not touch hot compressor components. It may damage the insulation!**

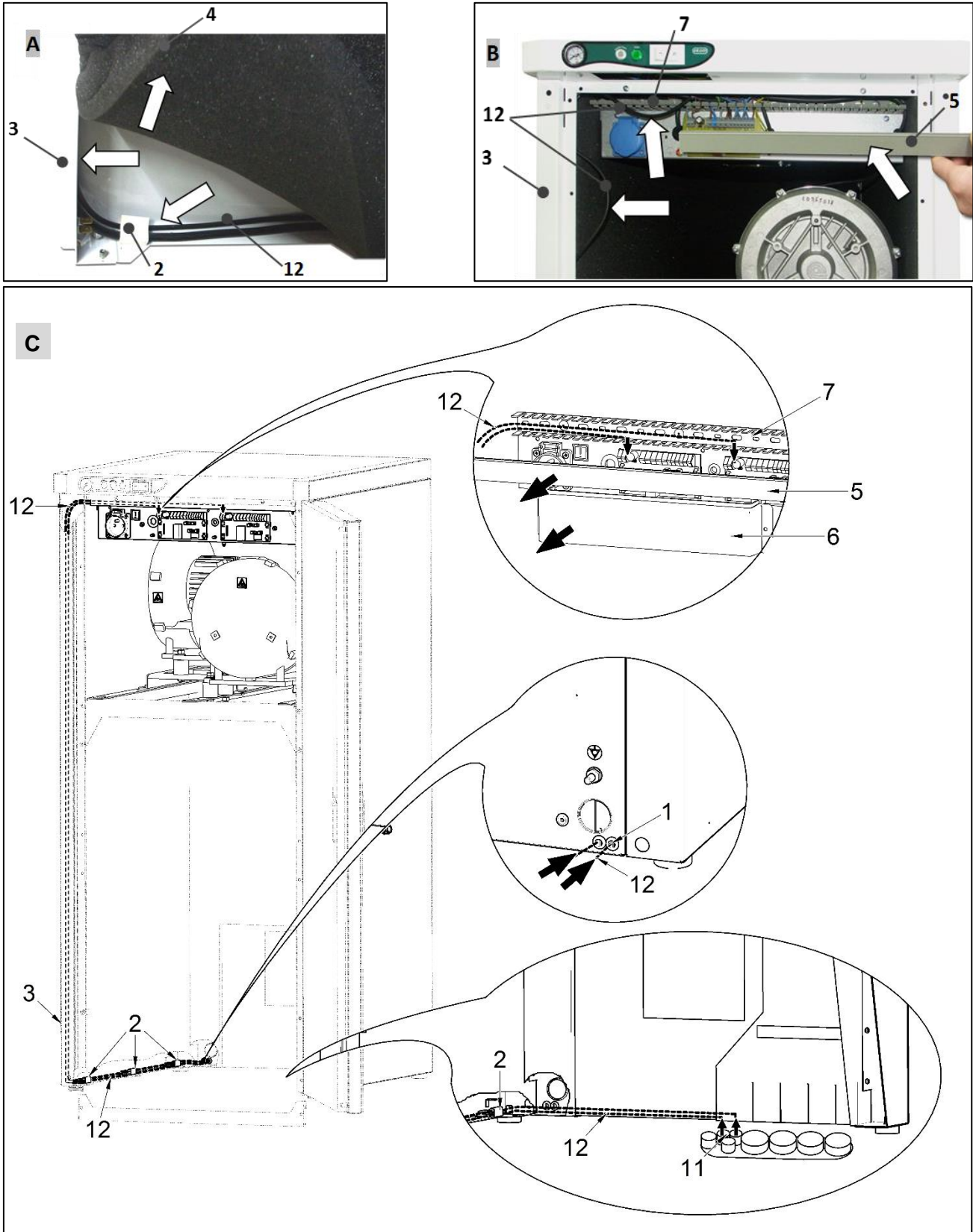
Fig. 11

**DUO, DUO 2V, DUO 2**


**15. CONNECTION DIAGRAM**

**DK50 PLUS, DK50 2V**

**6-8 bar**

1/N/PE ~ 230V/50Hz, 230V/60Hz

ELECTRIC OBJECT OF 1st. CAT.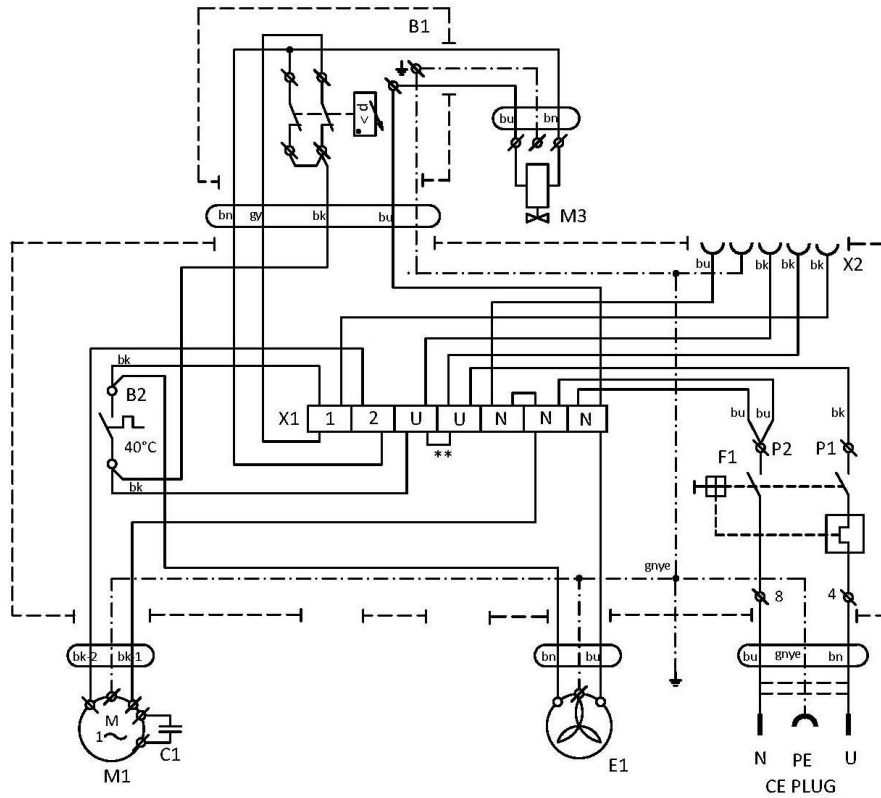


**DK50 PLUS/M, DK50 2V/M**

**6-8 bar**

1/N/PE ~ 230V/50Hz, 230V/60Hz

ELECTRIC OBJECT OF 1st. CAT.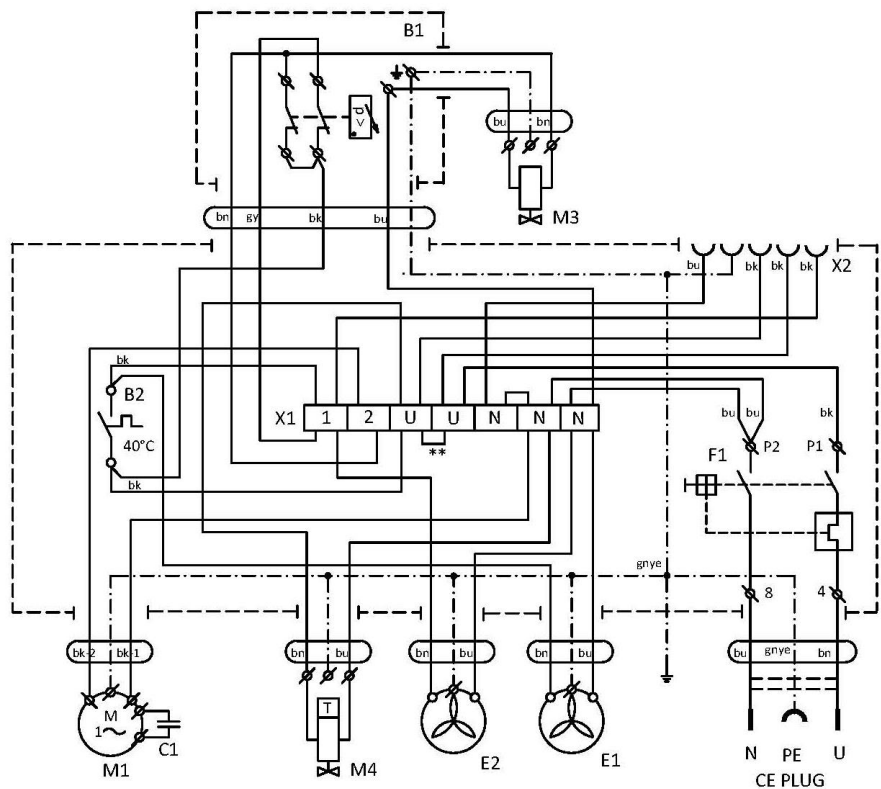


Cabinet Duo with suction unit

DUO

1/N/PE ~ 230V/50Hz, 230V/60Hz


ELECTRIC OBJECT OF 1st. CAT.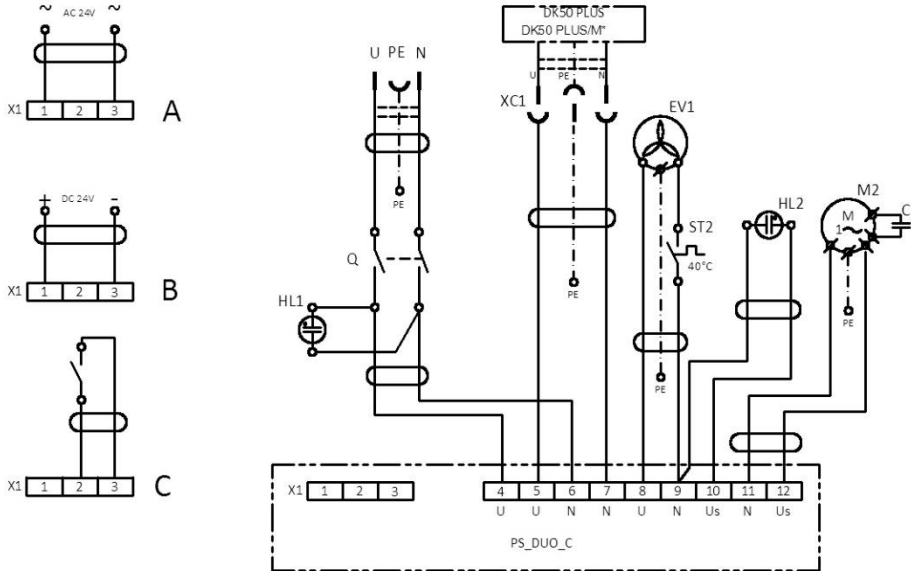

DUO2, DUO 2V

1/N/PE ~ 230V/50Hz, 230V/60Hz

ELECTRIC OBJECT OF 1st. CAT.

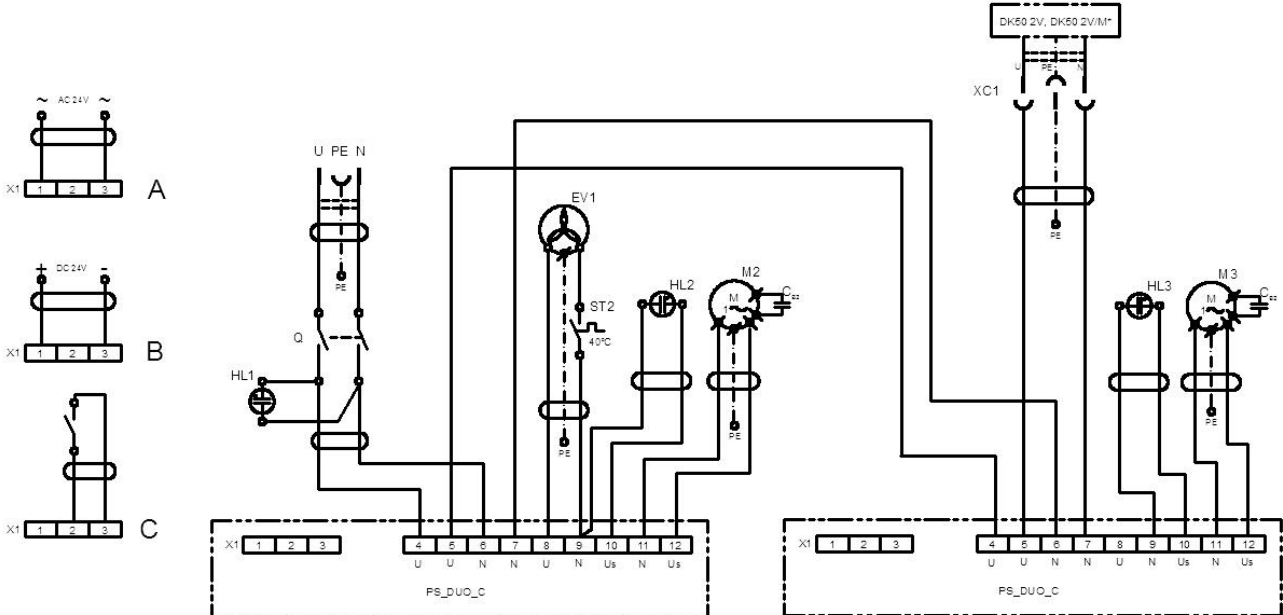Legend for electrical diagram

M1	Compressor motor	X1	Terminal box
E1	Compressor fan	F1	Breaker switch
E2	Dryer fan	M4	Condensate drain valve
M3	Reliefvalve	XC1	Socket
B2	Temperature switch	HL1,HL2	Indicators
EV1	Cabinet fan	HL3	Indicators
X2	Connector	M2, M3	Suction unit motor
C1, Cb	Capacitor	ST2	Temperature switch
B1	Pressure switch	Q	Switch

**OPERATION**

**THE EQUIPMENT MAY ONLY BE OPERATED BY THE TRAINED STAFF!**

**IN CASE OF EMERGENCY, DISCONNECT THE COMPRESSOR FROM THE MAINS (PULL OUT THE MAINS PLUG).**

**THE AIR PUMP HAS HOT SURFACES.  
BURNS OR FIRE MAY RESULT IF CONTACT IS MADE.**

**During prolonged operation of the compressor and suction unit, the temperature in the cabinet may increase to over 40°C. At this point the cooling fan automatically switches on. After cooling the space to under 32°C, the fan switches off.**

**Automatic start: when pressure in the pressure tank decreases below the switch-on pressure, the compressor automatically switches on. The compressor automatically switches off when pressure in the air tank reaches the switch-off pressure.**

**The working pressure settings for the pressure switch set by the manufacturer cannot be changed. Compressor operation at a working pressure below the switch-on pressure indicates high air consumption (see the chapter Troubleshooting -Malfunctions).**

**Required drying performance can only be achieved when following the defined operating conditions!**

**Drying performance will decline and the achieved dew point will drop if the dryer is operated at any pressure below the minimum working pressure!**

**WHEN THE DRYER IS OPERATED AT AMBIENT TEMPERATURE THAT IS HIGHER THAN MAXIMUM OPERATING TEMPERATURE, THE DRYER MAY BE DAMAGED!**

**16. COMMISSIONING**

- Make sure all transport stabilizers were removed.
- Check that all pressurized air line connections are secure.
- Check to ensure power is connected correctly to the compressor.
- Check the connection of the the cabinet pressure gauge hose to the compressor pneumatic block. (Fig.3 a 7)
- Check the compressor's electrical cord in the cabinet socket (34). (Fig. 3)
- Check the connection of the suction unit control cord (12). (Fig. 11)
- Check the connection of the intake and discharge hoses. (Fig.9)
- Check the position of the breaker switch to ensure it is in the "I" position. Turn the breaker switch (4) to the "I" position if it is found in the "0" position. (Fig. 13)
- Turn the switch (5) on the front side of the cabinet to the "I" position; a green indicator indicates the device is in operation. (Fig. 13)

**The compressor is not equipped with a backup power supply.**

## 17. SWITCHING THE COMPRESSOR ON

(Fig. 12): Start the compressor at the pressure switch (1) by turning the switch (2) to the “I” position. Check the position of the breaker switch to ensure it is in the “I” position. If not, turn the breaker switch (4) to the “I” position. Turn on the switch (5) on the front of the cabinet, the indicator will turn green. The compressor begins running, the air tank is filled, and the switching pressure and pressure switch turn the compressor off. The compressor operates in automatic mode, switched on and off by the pressure switch, depending on compressed air consumption. Check the values of the switch-on and switch-off pressure using the pressure gauge (3). A tolerance of  $\pm 10\%$  is acceptable. The pressure in the air tank must not exceed the permitted operating pressure.

Fig. 12

**Adjustments to the pressure range on the compressor’s pressure switch are prohibited. The pressure switch (1) was set up by the manufacturer and only a qualified technician trained by the manufacturer may make any changes to its settings.**

**Compressor with a suction unit** – when first started and placed into service, the compressor fills the air tank until the switching pressure and the pressure switch turn off the compressor. The compressor operates in automatic mode, switched on and off by the pressure switch, depending on compressed air usage.

The suction aggregate is controlled from the dental set. Suction unit operation is indicated by the white indicator on the front of the cabinet.

**Compressor with dryer** – the compressor operates in the same manner as above, only the compressed air passes through a cooler and a dryer that removes moisture from the compressed air.

## 18. COMPRESSOR SHUT-DOWN

(Fig.12)

Compressor shut-down for either service or any other reason is performed at the pressure switch (1) by turning the alternation switch (2) to the “0” position and **pulling the mains plug from the socket**. This disconnects the compressor from power supply. Vent the pressure in the air tank to zero by opening the drain valve. (Fig. 13)

**MAINTENANCE****19. DEVICE MAINTENANCE**

The operator shall ensure completion of repeated testing of the device at least once every 24 months (EN 62353) or at intervals defined by applicable national legal regulations. A record of these test results shall be completed (e.g. per EN 62353, Annex G) together with the measurement methods.

The device has been designed and manufactured to keep maintenance to a minimum. The following work must be performed to retain the proper and reliable operation of the compressor.

**Before starting compressor maintenance work, it is necessary to check if the compressor can be disconnected from the device to ensure that the person using the device is not at risk of health damage and there is no risk of any other material damages!**

**Air pump components (head, cylinder, pressure hose, etc.) are very hot during and shortly after compressor operation – do not touch these components!**

**Repair works beyond normal maintenance can be performed only by qualified personnel or the manufacturer's customer service. Use only spareparts and accessories approved by the manufacturer.**

**PROTECT EYESIGHT, WEAR GOGGLES, WHEN VENTING COMPRESSED AIR FROM THE COMPRESSED AIR CIRCUIT (AIR TANK).**

The work below may only be performed by trained personnel as follows:

**BEFORE STARTING ANY MAINTENANCE WORK, FIRST DISCONNECT THE COMPRESSOR FROM THE MAINS (PULL OUT THE MAINS PLUG) AND VENT THE COMPRESSED AIR FROM THE AIR TANK.**

**Allow the equipment to cool down before maintenance, service or connecting/disconnecting the compressed air supply!**

**Reconnect the grounding lead if removed during service work to its previous location once the work is complete.**

**The suction unit may be contaminated. In case of its repair it is necessary to send the suction unit to the specialized organisation for decontamination or to EKOM spol.s r.o.**

**19.1. Maintenance intervals**

Time interval <sup>c)</sup>	50 Hz		once a day		once a week		once a year		once every 2 years		2000 h		4000 h		6000 h		8000 h		10000 h		12000 h		Chapter	Set of spare parts	Performed by
	60 Hz																								
Product operational checks			x																				19.2	-	user
Drain the condensate from the air tank - At high RH ) <sup>b)</sup> - Pour the condensate out of the vessel <sup>d)</sup>			x																				19.5	-	user
Drain the condensate from the air tank - At normal RH ) <sup>b)</sup>					x																		19.5	-	user
Product function check					x																		9	-	user
Leak check connections and inspect device							x																19.3	-	qualified professional
Inspection of electrical connections							x																19.4	-	qualified professional
Cooler and fan check							x																19.9	-	qualified professional
Filter element replacement in the dryer filter <sup>d)</sup>								x															19.8	025200304-000	qualified professional
Safety valve check								x															19.6	-	qualified professional
Replacing the inlet filter on the DK50 PLUS air pump <sup>a)</sup>								x						x									19.7	025200126-000	qualified professional
Replacing the inlet filter and pre-filter on the DK50 2V air pump <sup>a)</sup>								x						x									19.7	025200139-000 025200150-000	qualified professional
Conduct a " repeated test" per EN 62353																							19	-	qualified professional

a) data is presented in hours; if not possible, then the data is considered in years

b) only for compressors without dryers

c) For 60 Hz compressor models, reduce the time interval in hours by 20%

(2000h./1600h., 4000h./3200h., 6000h./4800h., 8000h./6400h., 10000h./8000h., 12000h./9600h.)

d) only for compressors with dryers

## DUO, DUO 2V, DUO 2

---

### 19.2. Operational checks

- Check air pump condition – the air pumps should be operating normally without excessive vibration or noise. Troubleshoot any problem or call in service personnel if trouble is detected
- Visually inspect fan operation – the fans must be operating when the air pumps are running. Troubleshoot any problem or call in service personnel if trouble is detected
- Check to ensure the power cable and pneumatic hoses are undamaged. Replace damaged components or call in service personnel.
- Check the ambient temperature – the ambient temperature must be below the temperature limit (40 °C). Cool the space if the temperature is high.
- On the compressor with dryer – open the plug on the condensate collecting bottle to drain the condensate.

### 19.3. Pneumatic connection leakage check and device inspection

#### Leakage check:

- Perform the leakage check of pneumatic connections when the compressor is operating – pressurizing.
- Use a leakage analyser or soapy water to check all joints and connections for leaks. Tighten or reseal the connection where leaks are found.
- Device inspection:
  - Check the condition of the air pump for normal operation and noise levels.
  - Fan operation check - the fans must be running during the defined compressor work cycles
  - Check of temperature switch operation (B2) – heat the temperature switch to a temperature above 40°C (e.g. using a heat gun, being careful not to warp any plastic pieces nearby). Fan EV1 (and EV2 for a compressor with dryer) will start up once the temperature reaches 40°C so long as power is connected to the compressor.
  - Check filter condition – the filters must be undamaged and sufficiently clean.
  - Check the condition of the air pump itself - ensure there is no contamination inside the crankcase or play in the crankshaft
  - Replace any defective parts as needed.

### 19.4. Inspection of electrical connections

**Check all electrical connections on the device with the mains disconnected!**

#### Inspection

- Check the mechanical function of the main switch.
- Check to ensure the power cable and connected wires are undamaged.
- Visually inspect the connection of individual cables to the terminal strip.
- Inspect all screw terminals for the protective green and yellow PE grounding conductor.

### 19.5. Condensate drain

Risk of slipping on a wet floor in case of the collecting bottle overflow.

Switch off the compressor every time before emptying the vessel!

Carefully and slowly open the drain valve. Quickly opening the drain valve is accompanied by a loud sound and uncontrollable stream of accumulated condensate.

#### Compressors without air dryer

Draining the condensate from the pressure vessel is recommended during regular operation.

Switch off the compressor at the mains. Reduce air pressure in the device to max. 1 bar by releasing air via a connected device.

Release the drain valve from the clips on the compressor base. Place the open end of the hose into a suitable container and open the drain valve (1) to drain off the condensate from the container.

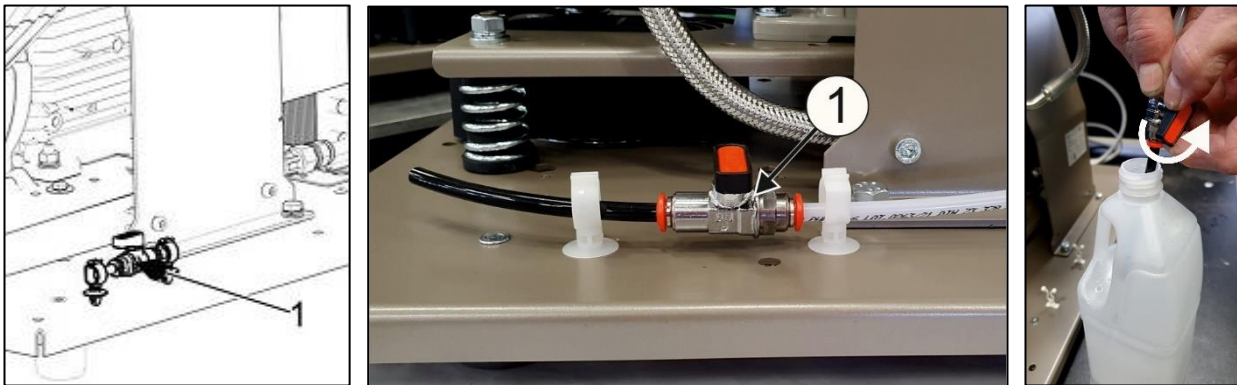
Fig.13

- Compressors without a dryer may be equipped with the Automatic Condensate Drain to allow the condensate to drain without operator intervention (see chap. 8.2.).

#### Compressors with air dryer

- Condensate from compressors with air dryers is automatically drained into a condensate collection vessel.
- The vessel must be drained regularly, see Chapter 19.1

Fig. 14

Monitor the level in the vessel using the 1 l or 2 l markings (depending on the volume of the vessel), and empty at least once a day. The vessel may overflow if the condensate is not drained out of the vessel at the defined interval.

## DUO, DUO 2V, DUO 2

If necessary, it is possible to drain the air tank using the drain valve (1) from the lower side of the air tank (Fig. 15).

Switch off the compressor at the mains and reduce the air pressure in the device to a maximum of 1 bar, e.g. by venting air through the connected equipment.

Use a hose to drain with the free end placed in a suitable container and then open the valve to drain condensate from the tank.

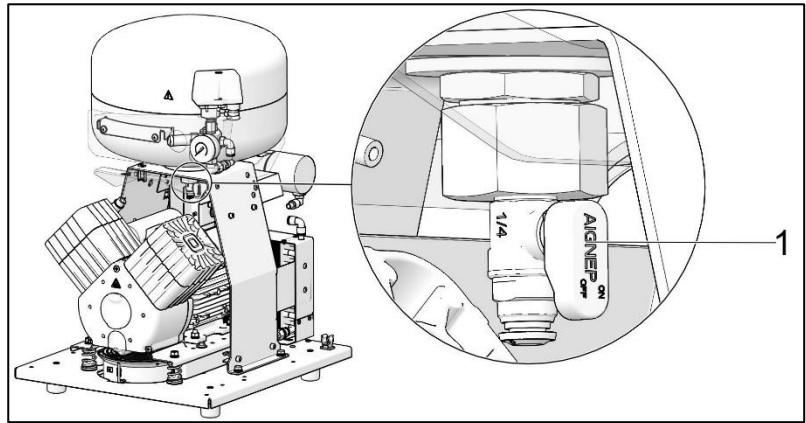
Fig. 15

**THE FOLLOWING IS REQUIRED BEFORE ANY FOLLOW-UP CHECKS:**

**Compressors with cabinet** - release the door lock and open the cabinet door.

### 19.6. Safety valve check

When the compressor is operated for the first time, make sure that the safety valve is working properly. Turn the screw (2) on the safety valve (1) several rotations to the left until the safety valve releases air. Let the safety valve vent for only a few seconds. Turn the screw (2) to the right until it seats, closing the valve.

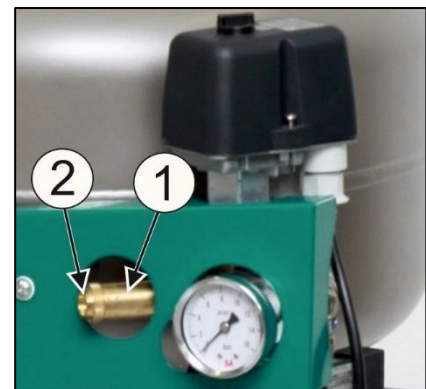
Fig. 16

**Never use the safety valve to vent air pressure in the air tank. It could damage the safety valve. The valve is set to the maximum permitted pressure by the manufacturer. Adjustments are not permitted!**

**WARNING! COMPRESSED AIR IS DANGEROUS BECAUSE OF THE RISK OF DAMAGE TO THE EYES! RISK OF DAMAGE TO THE EYES.**

### 19.7. Replacing the inlet filter and pre-filter

The inlet filter (1) is located inside the compressor crankcase cover.

Inlet filter replacement:

- Pull out the rubber plug by hand (2).
- Remove the used filter (1).
- Insert a new filter and replace the rubber plug.

Pre-filter replacement:

- Pull out the pre-filter by hand (3).
- Replace the element and re-insert.


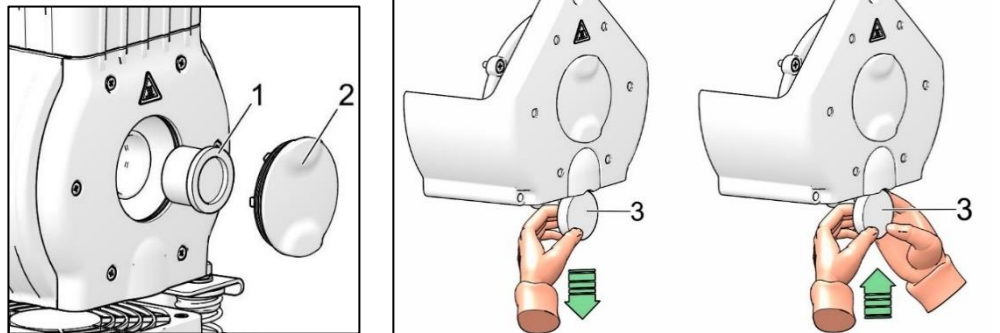
Fig. 17

### 19.8. Filter element replacement

- Remove the hose (1) from the quick connector.
- Use a wrench (2) to release the filter vessel (3) and remove.
- Pull down on the filter element (4) to remove.
- Insert a new filter element.
- Re-install the filter vessel.
- Gently tighten the filter vessel with the wrench.
- Re-install the hose on the quick connector.

Fig. 18

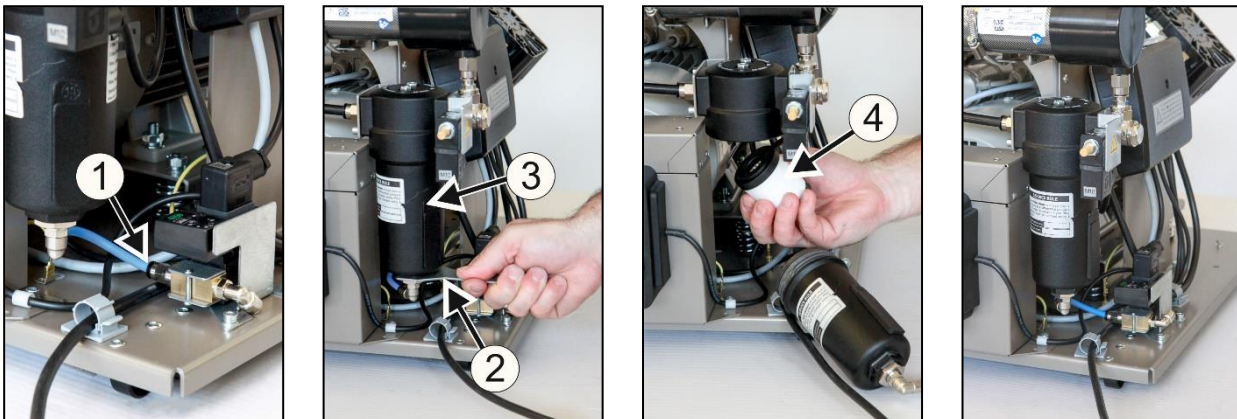## **DUO, DUO 2V, DUO 2**

---

### **19.9. Cooler and fan check (Fig. 2)**

The equipment, in particular the compressor fan, cooler fan (21), and the cooler (14), must be kept clean to ensure efficient drying. Remove dust from surface of the fan or cooling fins by vacuuming or blowing down with compressed air.

### **19.10. Cleaning and disinfection of the exterior surfaces of the product**

Use neutral agents for cleaning and disinfection of the external surfaces of the product.

**The use of aggressive cleaning agents and disinfectants comprising alcohol and chlorides may lead to the damage of the surface and the discoloration of the product.**

**TROUBLESHOOTING**

**Before proceeding, depressurize the air tank to zero and disconnect the device from the mains.**

**The equipment, in particular the cooling fan, must be kept clean to ensure the long-term efficiency of the cooler – the cooling fins must be cleaned occasionally to remove any dust.**

Troubleshooting can be performed only by qualified personnel.

**To follow the rules of personal hygiene for works with contaminated material. To separate, label, packing and providing for decontamination of contaminated parts by course of national regulations.**

MALFUNCTIONS	POSSIBLE CAUSE	REMEDY
Compressor does not start	No voltage at the pressure switch	Check voltage in socket Check the breaker switch - and switch to the "I" on position Loose conductor at terminal – tighten Check the electrical cord – replace defective cord
	Motor winding failure, damaged thermal overload protection Faulty capacitor Seized piston or other rotating part Pressure switch does not switch	Replace motor or windings  Replace capacitor Replace damaged parts Check the function of the pressure switch
Compressor often switches on	Air leak in compressed air distribution system Leaking non-return valve	Check compressed air distribution system – seal loose joints Clean non-return valve, replace seals, replace non-return valve
	Large volume of condensed liquid in pressure vessel Low compressor performance	Drain condensed liquid Check the time to fill the air tank
Low pressure in air tank (compressor running constantly)	High air usage by device, Leak from pneumatic distribution system, Low air pump output Air pump malfunction	Decrease air consumption Use compressor with higher capacity Check pneumatic system – seal loose connections Clean / replace the pump
	Dryer malfunction	Clean / replace the pump Replace dryer
Prolonged operation of the compressor	Leak from pneumatic distribution system Worn piston ring Inlet filter is plugged Defective solenoid valve	Check compressed air distribution system – seal loose joint Replace worn piston ring Replace old filter with a new filter Repair or change the valve or coil
Compressor is noisy (knocking, metal noises)	Damaged piston bearing, piston rod, motor bearing Loose or cracked spring	Replace damaged bearing  Replace damaged spring
The suction unit is not running, or is running irregularly	No voltage at the suction unit terminals	Check voltage in socket Check the fuses - replace faulty fuse Loose terminal - tighten Check the electrical cord - replace defective cord Check for control voltage
	Other mechanical defect in suction unit	Damaged bearing, damaged suction unit impeller - call a service centre

**DUO, DUO 2V, DUO 2**

	Suction unit overheating (thermal protection device tripped)	Check the cabinet fan for proper operation - replace if not working Inspect the suction and discharge lines for damage – (replace damaged hoses and remove foreign items)
Suction unit produces no or little suction, motor running	Leak in suction lines, foreign item in suction piping, plugged exhaust line	Check connections on the suction line, seal leaks, remove foreign items
Dryer doesn't dry (condensed water in the air) *	Cooling fan not working	Replace the fan Check the power source
	Damaged dryer	Replace the dryer
	Non-functioning automatic condensate drain	Clean / replace

)\* After a dryer malfunction, the interior surfaces of the air tank must be thoroughly cleaned and all condensed liquid removed.

**Check the dew point of the air leaving the air tank (see Chapter 7 - Technical data) to protect the device from damage!**

**20. REPAIR SERVICE**

Guaranteed and post-guarantee repairs must be done by the manufacturer, its authorized representative, or service personnel approved by the supplier.

**The manufacturer reserves the right to make changes to the device without notice. Any changes made will not affect the functional properties of the device.**

**21. STORAGE**

If the compressor will not be used for a prolonged time period, drain all condensate from the air tank. Then turn on the compressor for 10 minutes, keeping the drain valve open (1) (Fig. 13). Switch off the compressor using the switch (2) at the pressure switch (1) (Fig. 12), close the drain valve and disconnect the device from the mains.

**22. DISPOSAL OF THE DEVICE**

- Disconnect the device from the mains.
- Release air pressure in the pressure tank by opening the drain valve (1) (Fig. 13).
- Follow all personal hygiene rules when working with contaminated materials.
- Separate, label, pack, and secure the decontamination of contaminated parts as per national regulations.
- Dispose of the device following all applicable environmental regulations.
- Entrust a specialised company to sort and dispose of waste.
- Worn out components have no negative environmental impact.

**Internal parts of the suction unit may be contaminated with biological material as a result of inappropriate use. Hand over the device to a specialised organisation for decontamination before sorting and disposing of the unit as waste.**

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