DK50 2V/50 DK50 2x2V/110

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OBSAH 105

CONTENTS

GENERAL INFORMATION.....	6
1. CONFORMITY WITH THE REQUIREMENTS OF THE EUROPEAN UNION.....	6
2. SYMBOLS.....	6
3. DEVICE USE	7
4. GENERAL SAFETY INSTRUCTION	8
5. STORAGE AND TRANSPORT	9
PRODUCT DESCRIPTION	10
6. VARIANTS	10
7. ACCESORIES.....	11
8. PRODUCT FUNCTION	12
TECHNICAL DATA.....	18
INSTALLATION.....	24
9. INSTALLATION CONDITIONS	24
10. COMPRESSOR ASSEMBLY	24
11. PNEUMATIC CONNECTION	29
12. ELECTRICAL CONNECTION	30
13. COMMISSIONING	31
14. PNEUMATIC AND ELECTRICAL DIAGRAMS	32
OPERATION.....	41
15. SWITCHING THE COMPRESSOR	42
16. SWITCH OFF THE COMPRESSOR	42
PRODUCT MAINTENANCE	43
17. PRODUCT MAINTENANCE.....	43
18. LONG-TERM SHUTDOWN.....	51
19. DISPOSAL OF DEVICE	51
TROUBLESHOOTING.....	52
20. REPAIR SERVICE	53
ANNEX	154
21. INSTALLATION RECORD	154

GENERAL INFORMATION

Read the User manual carefully and keep it before use of the product. The User manual provides information on correct use – installation, operation and maintenance of the product.

The User manual corresponds with the design of the product and condition according to the applicable safety and technical standards at the time of printing. The manufacturer reserves all rights concerning the protection for the stated connections, procedures and names.

Slovak version is original of the User manual. The User manual has been translated in accordance with the best available knowledge. The Slovak version is to be used in the event of any uncertainties.

This user manual is the original instructions. Translation is performed in accordance with the best available knowledge.

1. CONFORMITY WITH THE REQUIREMENTS OF THE EUROPEAN UNION

This product conforms to the requirements of the European Union 2006/42/EC, 2014/29/EU, 2014/35/EU, 2014/30/EU, 2011/65/EU and is safe if used in compliance with the intended use and if all safety instructions are followed.

User manual is in compliance with requirements of Directive 2006/42/EC.

2. SYMBOLS

The following symbols and marks are used in the User manual, on the device and its packaging:

General warning

Warning – risk of electric shock

Warning - compressor is controlled automatically

Warning – hot surface

General caution

Refer to instruction manual

CE – marking

Serial number

Article number

Protecting earthing

Fuse

Package handling label – fragile

Package handling label – this side up

Package handling label – keep dry

Package handling label – temperature limits

Package handling label – limited stacking

Package label – recyclable material

Manufacturer

3. DEVICE USE

3.1. Intended use

The compressor is used as source of clean oil-free compressed air intended to be used in industry and laboratories, where parameters and properties of the compressed air are suitable.

The compressor is exclusively intended to compress air without content of explosive or chemically unstable substances.

The compressor is intended for operation in clean and dry rooms

3.2. Incorrect use

Contamination risk.

Air from the compressor is without additional treatment not suitable for breathing and direct contact with food

Explosion risk.

The product is not intended for operation in rooms with explosion risk.

The compressor must not be used to compress aggressive gases.

The compressor must not be operated in premises with occurrence of flammable vapors.

The compressor must not be operated in other conditions as mentioned in Technical data.

Any other use of the product beyond the intended use is considered as incorrect use. The manufacturer is not responsible for any damages or injuries as a result of incorrect use or disobedience to instructions stated in this User manual. All risks shall be solely borne by the user/operator.

4. GENERAL SAFETY INSTRUCTION

The product is designed and manufactured so that any risks connected with its use are minimized and the product is safe for the user and surrounding when used according to the intended use and the instructions stated below are followed.

4.1. Required qualification of the personnel

- Each user must be trained by the manufacturer or an organization authorized by the manufacturer or instructed on the device operation by other trained user.
- Installation, new settings, changes, extensions and repairs of the product may be performed by the manufacturer or an organization authorized by the manufacturer (hereinafter qualified technician).
- Otherwise the manufacturer is not responsible for safety, reliability and correct functioning of the product.

4.2. General instruction

- When operating the compressor, all acts and local regulations valid in the place of use must be observed. The operator and user are responsible for following the applicable regulations.
- Before every use, the user must check, if the device is functioning correctly and safely. Before building the compressor in other devices, the supplier must assess, if the supplied air and construction of the device comply with the requirements of the specified intended use. Taking this into account, follow the product technical data. Assessment of conformity shall be performed by the manufacturer – supplier of the final product.

4.3. Protection from dangerous voltage and pressure

- The device can be connected only to a properly mounted socket with protective earthing.
- Before connecting the product, check if mains voltage and mains frequency stated on the product are in compliance with the values of the mains.
- Before putting the product into operation, check eventual damages to connected pneumatic hoses and electrical cables. Replace damaged pneumatic hoses and electrical cables immediately.
- Immediately disconnect the product from the mains (pull out the mains plug) in hazardous situations or technical disorders.
- Safety valve settings must not be changed and used to depressurize the air tank.

4.4. Original spare parts and accessories

- Safety of operating staff and failure-free operation of the product are guaranteed only when original spare parts are used. Only accessories and spare parts stated in the technical documentation or expressly approved by the manufacturer may be used.
- The guarantee does not apply to damages arising from use of accessories and spare parts other than prescribed or recommended by the manufacturer and the manufacture is not responsible for them.

5. STORAGE AND TRANSPORT

The manufacturer ships the compressor in a transport packaging. This protects the device from damage during transport.

Risk of damage to pneumatic parts.

The compressor may be transported only depressurized. Vent air pressure from the pressure tank and pressure hoses and drain condensate from the air tank before transporting the compressor.

Original packaging must be kept for eventual return of the device. If possible, always use the original compressor packaging for optimal protection of the product. If it is necessary to return the product within the guarantee period, the manufacturer does not guarantee for damages caused by incorrect packaging of the product.

Transport the compressor in an upright position, always secured by transport fixing elements.

Protect the compressor from high humidity, dirt and extreme temperatures during transport and storage. Do not store together with volatile chemical substances.

If it is not possible to keep the original packaging, dispose of the packaging in an environmentally friendly way. Transport cardboard can be recycled with old paper.

It is not permitted to store and transport the device outside the defined conditions, see below.

5.1. Ambient conditions





Products can be stored in rooms and means of transport that are free from any traces of volatile chemical substances under the following climatic conditions:

Temperature	from -25°C to $+55^{\circ}\text{C}$, in 24h to $+70^{\circ}\text{C}$
Relative humidity	max. 90% (without condensation/ non-condensing)

PRODUCT DESCRIPTION**6. VARIANTS**

The compressor is manufactured according to its intended application in the following variants:

DK50 2V/50	Compressor for installation in areas where operations will not disturb the surroundings
DK50 2V/50/M	Compressor with a membrane air dryer
DK50 2V/50S	Compressor in a cabinet with efficient soundproofing
DK50 2V/50S/M	Compressor with membrane dryer in a cabinet
Cabinet S50	Used to reduce the level of noise generated by the compressor
DK50 2x2V/110	Compressor is suitable for installation in rooms, where its activity does not disturb surrounding.
DK50 2x2V/110/M	compressor with membrane dryer
DK50 2x2V/110 S	compressor in cabinet with effective noise dampening
DK50 2x2V/110 S/M	compressor in cabinet with membrane dryer
Cabinet S 110	It decreases noise level of the compressor.

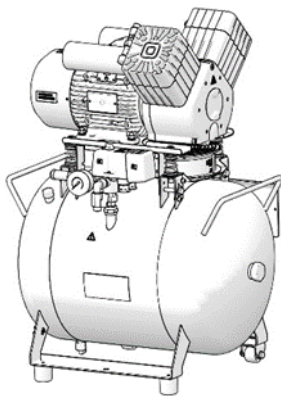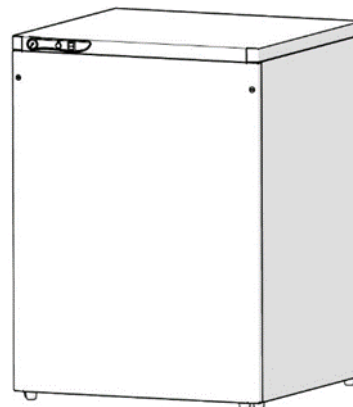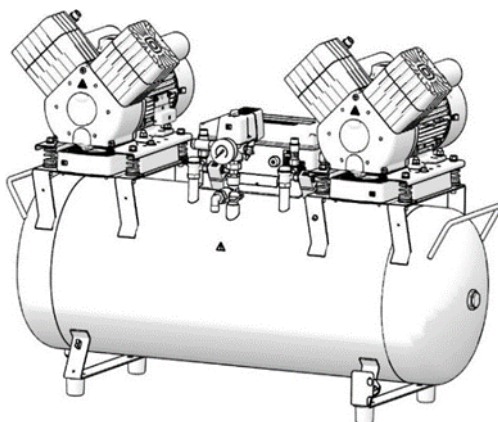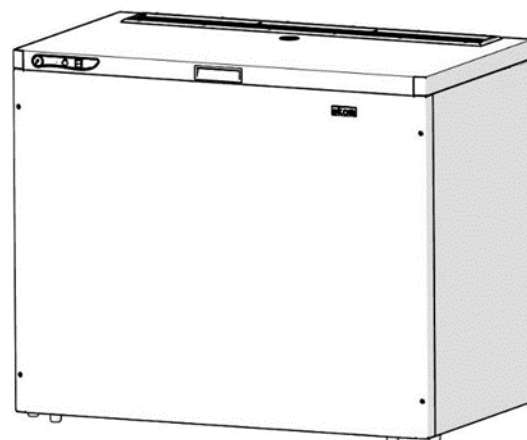**DK50 2V/50****S50
DK50 2V/50S****DK50 2x2V/110****S 110
DK50 2x2V/110 S**

7. ACCESORIES

Accessories are not included in a standard order, they must be ordered separately.

Automatic condensate drain

Automatic condensate drain (ACD) automatically drains condensate from the compressor air tank in pre-set time interval. ACD is suitable accessory for compressor without dryer.

Type	Use	Article number
AOK 13	DK50 2V/50	447000001-154
AOK 12	DK50 2x2V/110	447000001-123

Set of compressed air outlet filters

The compressor may be equipped with a set of filters if specified. The filter set may be equipped with an air pressure regulator.

Where a different level of air filtration is required, this requirement must be agreed upon with the supplier and specified in the order.

Type	Use	Level of filtration (µm)	Article number
FS 30F		1	604014119-014
FS 30M	DK50 2V/50, DK50 2V/50/M	1+0.1	604014119-015
FS 30S	DK50 2x2V/110, DK50 2x2V/110/M	1+0.01	604014119-017
FS 30A		1+0.01+A**	604014119-018

**) A – submicrofilter with activated carbon

*) - set **FS 30A**, in the standard configuration, this cannot be combined with a pressure regulator on products DK50 2V/50, DK50 2V/50/M; please contact your supplier if you are interested in such set (with a regulator)

Filters set regulator assembly

The compressor may be equipped with a pressure regulator of the compressed air outlet if specified. The regulator must be selected according to the application to the filter set, or separately. The regulator shall ensure constant pressure at the outlet.

Type	Use	Article number
Regulator complete (to the filter assembly)	DK50 2V/50, DK50 2V/50/M DK50 2x2V/110, DK50 2x2V/110/M	604014130-X00
REG12 (separate)	DK50 2V/50, DK50 2V/50/M	447000001-088
REG13 (separate)	DK50 2x2V/110, DK50 2x2V/110/M	447000001-096

Filter set brackets

A suitable bracket must be ordered for every filter set.

Type	Use	Article number
Compressor-mounted bracket	DK50 2V/50, DK50 2V/50/M	603014177-000
Compressor-mounted bracket	DK50 2x2V/110, DK50 2x2V/110/M	604014169-000

Condensation and filtration unit

The compressor may be additionally fitted with a condensation and filtration unit (KJF).

The KJF ensure that the compressed air from the air tank is cooled in the cooler and the condensate is captured in the filter and automatically separated from the pneumatic distribution system. The compressed air is filtered at the same time.

Type	Use	Level of filtration / μm /	Pressure regulator	Article number
KJF-2	DK50 2x2V/110	5	no	450001021-002

3KOM modules for controlling multiple compressors in Master-Slave functionality

A 3KOM control module may be ordered to manage the operation of 2 or 3 identical compressors with the same pressure range in Master-Slave functionality. In use, the controller in the module takes over the compressor switching function. The control module may be ordered in the configurations shown in the following table.

Compressor model	Control schematic	Article number	System air outlet
DK50 2V/50/ (M)	1+1	603002152-000	G 3/4"
DK50 2x2V/110/ (M)	1+1+1		

Equipotential bonding socket

The socket provides protective bonding. (Fig. 11)

Type	Use	Name	Article number
POAG-KBT6-EC	DK50 2V/50S, DK50 2V/50S/M	Mains plug connector	033400075-000
FLEXI-S/POAG-HK6	DK50 2x2V/110S, DK50 2x2V/110S/M	Conductor (1 m)	034110083-000

8. PRODUCT FUNCTION

8.1. Compressor

Fig. 1: Air pump (1) sucks the atmospheric air in through the inlet filter (8) and compresses it through the non-return valve (3) into the air tank (2) from which the device draws in the compressed air. If the pressure in the air tank drops to the switch-on pressure, the pressure switch (4) switches the compressor on and the compressor compresses the air into the air tank until it reaches the switch-off pressure, when the compressor switches off. The pressure hose is vented through the solenoid relief valve (13) once the compressor pump is switched off. The safety valve (5) prevents the pressure in the air tank from rising above the maximum allowed value. The drain valve (7) drains the condensate from the air tank. Compressed, oil-free filtered air is stored in the air tank ready for use.

Condensate must be drained from the air tank at regular intervals (see chapter 17.1).

8.2. Compressor with membrane dryer

Fig. 2: The air pump (1) draws in air through an inlet filter (8) and compresses it, feeding it to the cooler (14) through the filter (15) and the check valve (3) into the dryer (9), and feeds the clean and dry air into the air tank (2). A part of the air is released from the dryer with the captured moisture, which is manifested as a light air stream along the body of the dryer (9). Condensate from the filter is automatically drained at regular intervals into the collecting bottle via the condensate drain solenoid valve (16). The dryer ensures continuous drying of the compressed air. The drain valve (7) drains condensate from the air tank when drying performance is checked. Compressed, oil-free filtered air is stored in the air tank ready for use.

For the membrane dryer to work as intended, the dryer is connected directly to the air tank without the use of a check valve. As the membrane dryer contains selective membrane fibres, there is a

small amount of air leakage through them (about 0.5 bar/30 min)

This is why there is a gradual, slow drop in pressure even when there is no demand for compressed air (from the air tank), however, this is not a malfunction.

The pressure vessel does not need to be drained.

8.3. Compressor cabinet

Fig. 3: The soundproof cabinet provides compact covering of the compressor, dampens noise efficiently, providing sufficient exchange of cooling air. The fan under the compressor pump provides cooling of the compressor. It is in operation at the same time with the compressor motor or after switching on the temperature switch at temperature above 40°C. After cooling the cabinet area approximately under 32°C, the fan turns off automatically.

Risk of compressor overheating.

Make sure that there are no obstacles at the cooling air inlet into the cabinet (around the bottom part of the cabinet) and at the hot air outlet on the top back side of the cabinet.


If the compressor is placed on a soft floor, e.g. carpet, create space between the base and the floor or the cabinet and the floor, e.g. underlay the footings with hard pads to ensure sufficient cooling of the compressor.

The 8 - 10 bar compressor model is equipped with hours counter (Fig. 3).


Description for Figures 1-3:

- | | |
|-------------------------------------|----------------------------------|
| 1. Air pump | 18. Socket |
| 2. Air tank | 19. Condensate collection vessel |
| 3. Non-return valve | 20. Magnetic holder |
| 4. Pressure switch | 21. Fan |
| 5. Safety valve | 22. Cabinet fan |
| 6. Pressure gauge | 23. M5 screw |
| 7. Drain valve | 24. Cabinet gas springs |
| 8. Inlet filter | 25. Cabinet enclosure |
| 9. Dryer | 26. Wall stop |
| 10. - | 27. Switch |
| 11. - | 28. Pressure gauge |
| 12. - | 29. Power cord |
| 13. Solenoid valve | 30. Pressure gauge hose |
| 14. Dryer cooler | 31. Handle S110 |
| 15. Filter | 32. Wheels |
| 16. Condensate drain solenoid valve | 33. Connecting reinforcement |
| 17. Ball valve | 34. Connector of cabinet |
| | 35. Pulling system |

Fig. 1: Compressor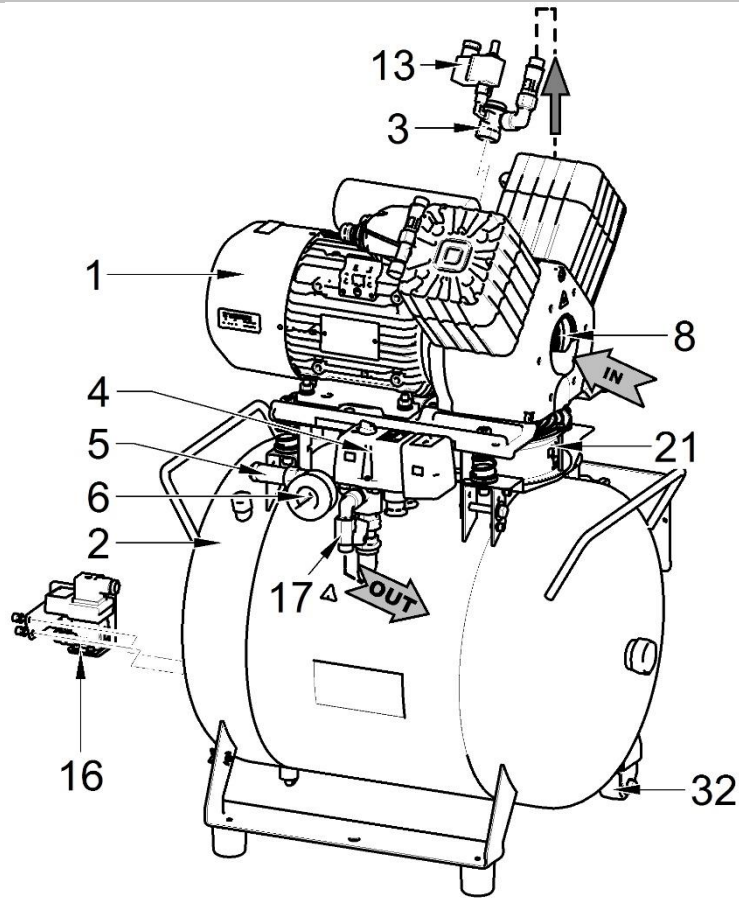


DK50 2V/50

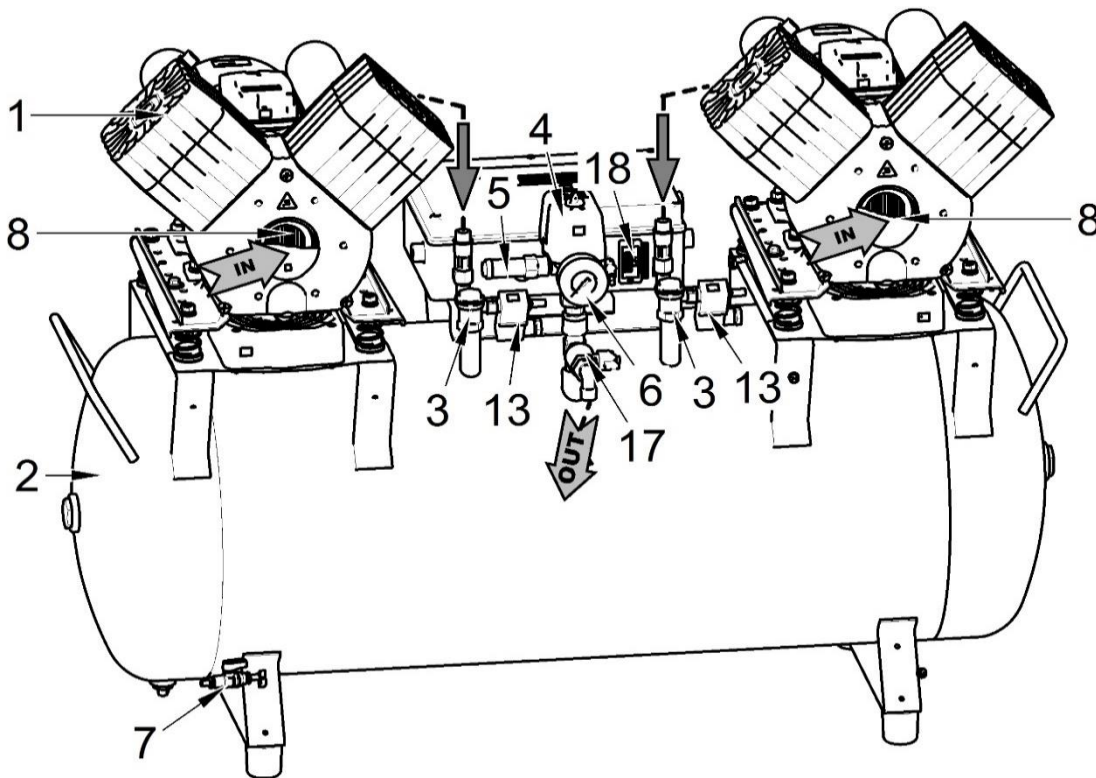
DK50 2x2V/110

Fig. 2: Compressor with MD dryer
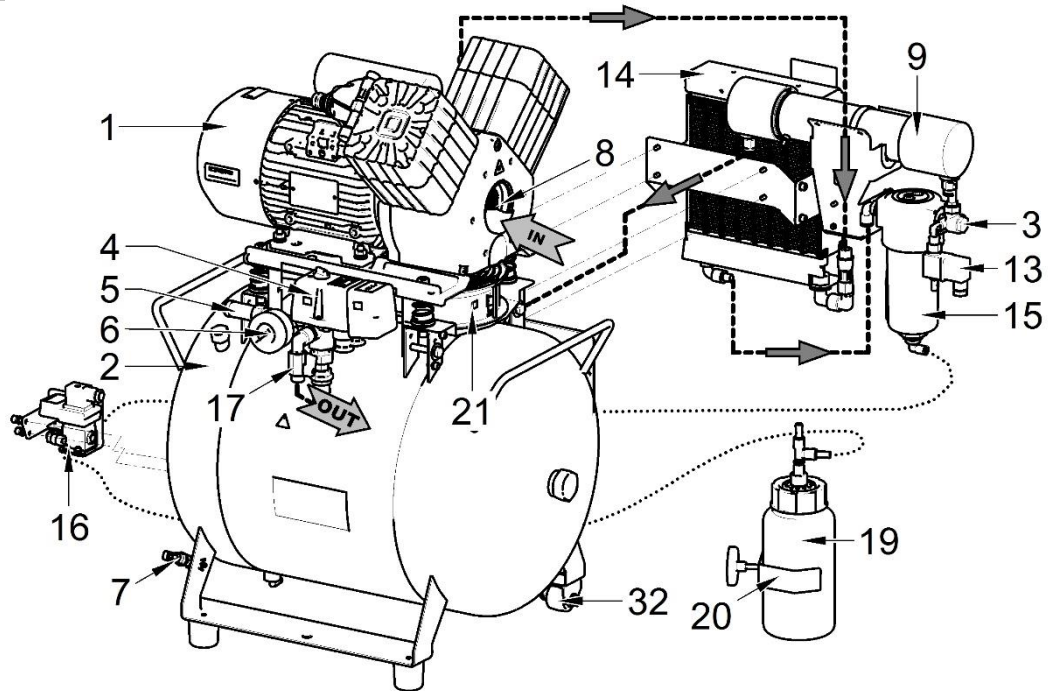


DK50 2V/50/M

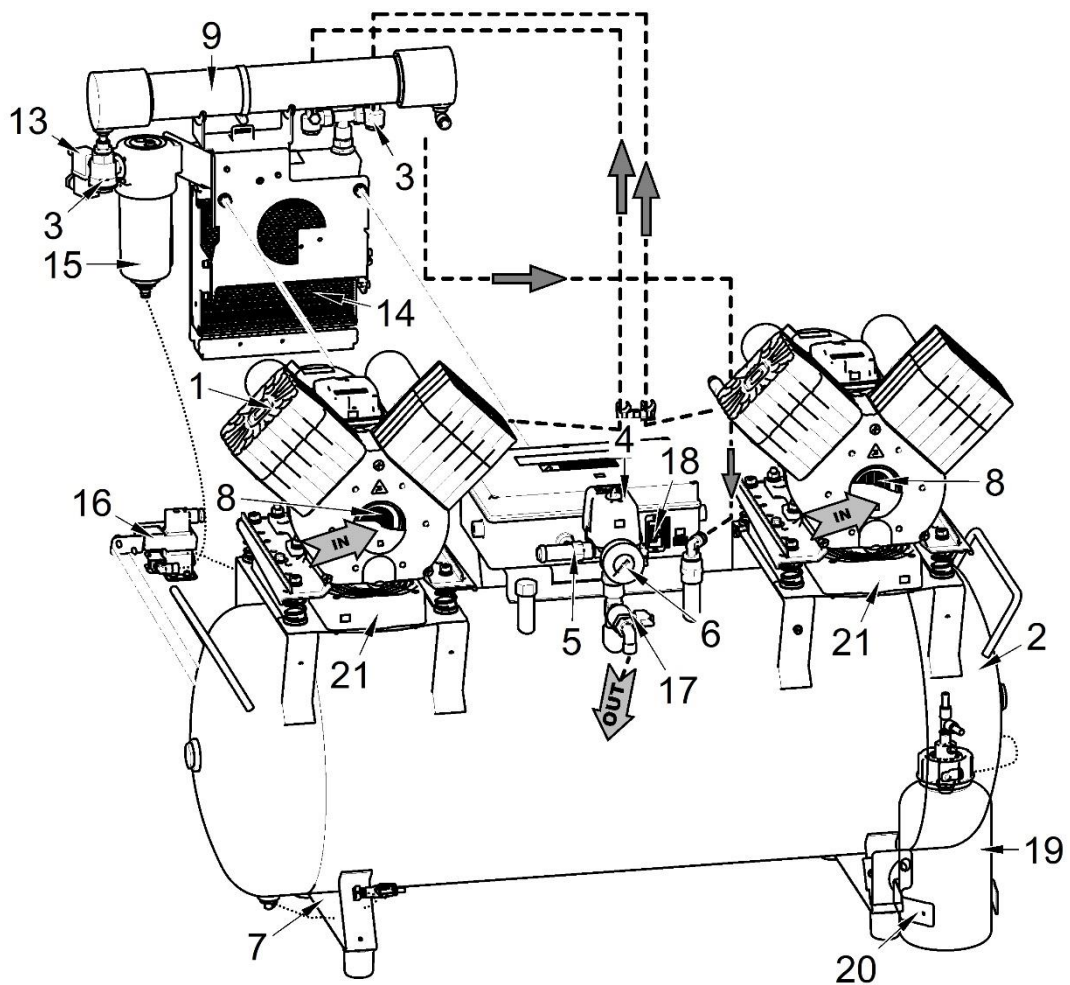
DK50 2x2V/110/M

Fig. 3: Cabinet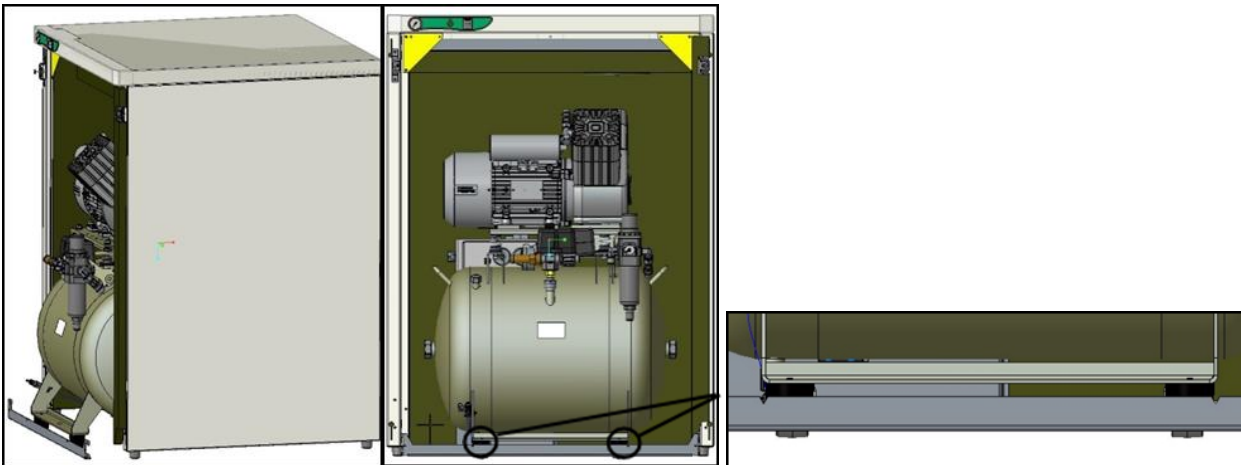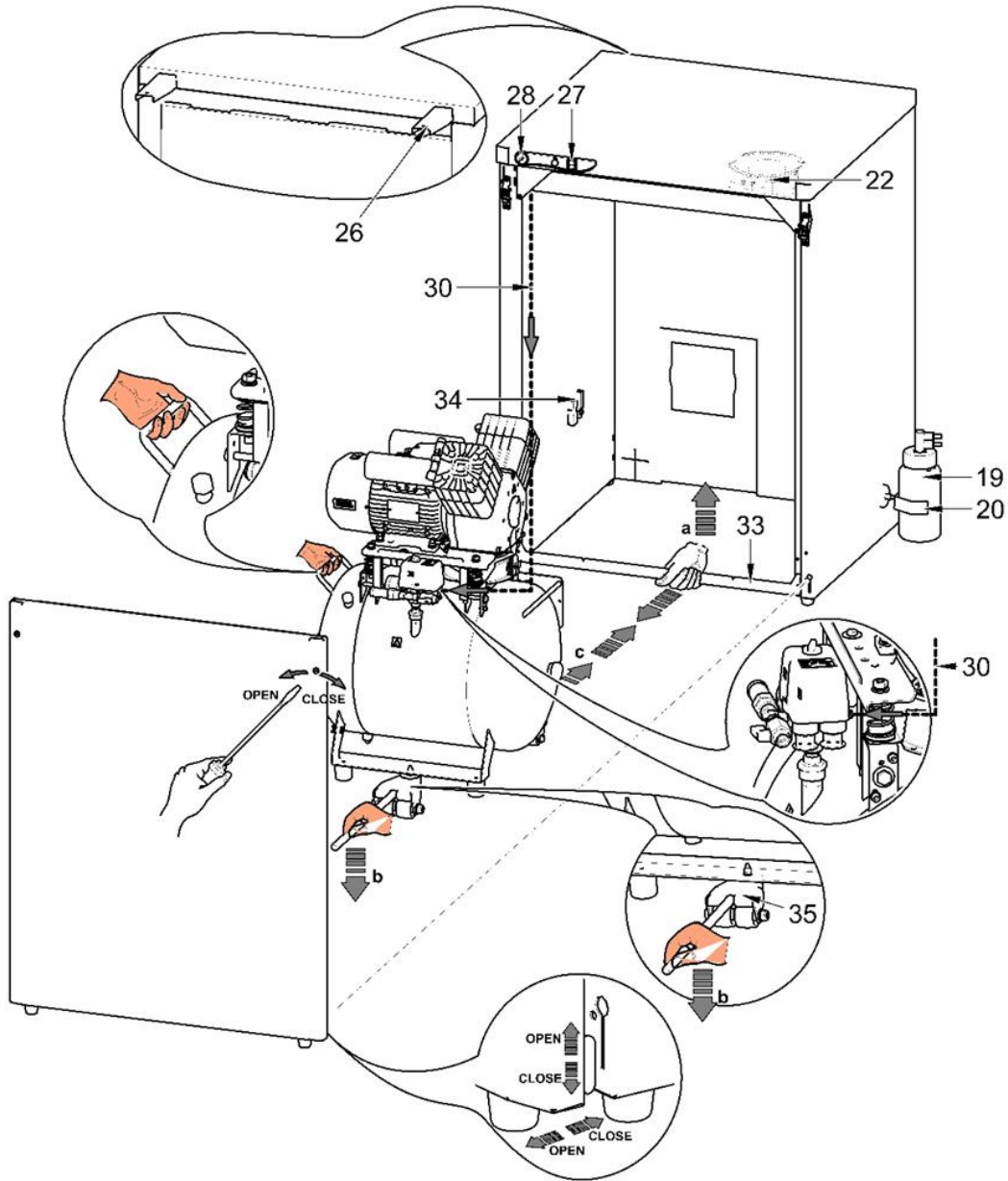

3A – DK50 2V/50S

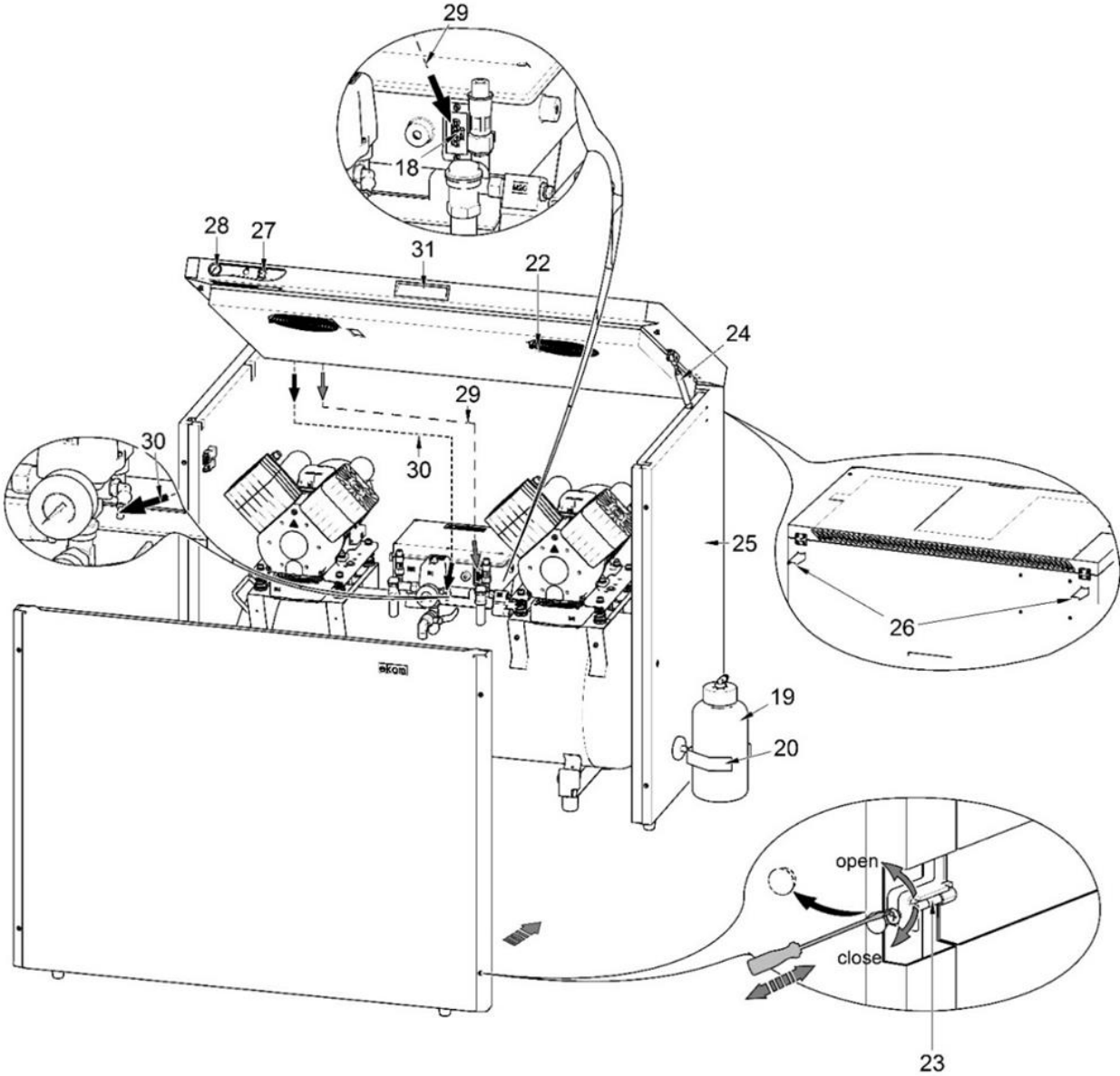3B – DK50 2x2V/110S

TECHNICAL DATA

Compressors are designed to operate in dry, ventilated and indoor dust-free rooms with the following climatic conditions:

Temperature from +5°C to +40°C

Relative humidity max. 70%



Working pressure 6 – 8 bar		DK50 2V/50		DK50 2V/50S		DK50 2V/50/M		DK50 2V/50S/M	
Nominal voltage Frequency ^{a)}	V, Hz	230,50 3x400, 50	230,60	230,50 3x400, 50	230,60	230,50 3x400, 50	230,60	230,50	230,60
Capacity at 6 bar (FAD)	l/min	135		135		104		104	
Working pressure ^{b)}	bar	6.0 – 8.0		6.0 – 8.0		6.0 – 8.0		6.0 – 8.0	
Rated current	A	8.6 3.8	9.2	8.8 4.0	9.4	8.7 3.8	9.3	8.9	9.7
Motor power	kW	1.2		1.2		1.2		1.2	
Air tank volume	l	50		50		50		50	
Air quality - filtration	µm	-		-		0.1		0.1	
Maximum operating pressure of safety valve	bar	9.0		9.0		9.0		9.0	
Noise level at 5 bar (L _{pA})	dB	≤67.0		≤51.5		≤69.0		≤55.0	
Operating mode	%	S1-100		S1-100		S1-100		S1-100	
PDP drying at 7 bar	°C	-		-		≤ +3		≤ +3	
Time to fill air tank from 0 to 7 bar	s	149		149		218		218	
Dimensions (net) w x d x h	mm	577x587x804		750x715x1015		577x628x804		750x715x1015	
Net weight	kg	58 54		108 105		61		114	

a) State the compressor variant in the order

b) For other range of pressure consult with the supplier



c) Weight value is only informative data and applies only to a product without any accessories

Dependence of compressor capacity on working pressure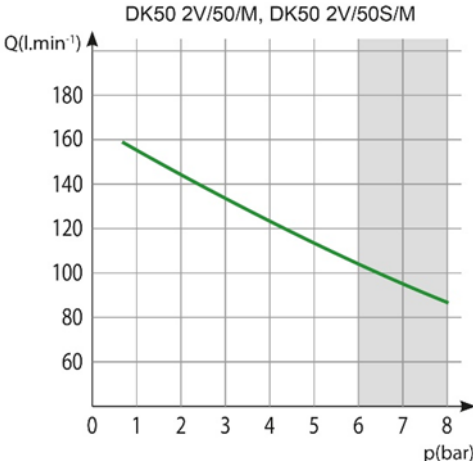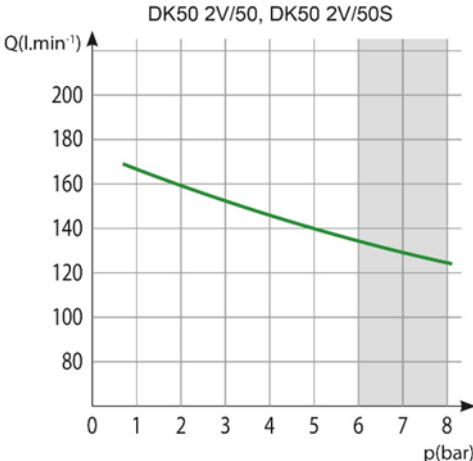

Working pressure 8 – 10 bar		DK50 2V/50	DK50 2V/50S	DK50 2V/50/M	DK50 2V/50S/M
Nominal voltage Frequency ^{a)}	V, Hz	230, 50	230, 50	230, 50 3x400, 50	230, 50
Capacity at 8 bar (FAD)	l/min	110	110	90	90
Working pressure ^{b)}	bar	8.0 – 10.0	8.0 – 10.0	8.0 – 10.0	8.0 – 10.0
Rated current	A	8.4	8.6	8.5 4.6	8.7
Motor power	kW	1.2	1.2	1.2	1.2
Air tank volume	l	50	50	50	50
Air quality - filtration	µm	-	-	0.1	0.1
Maximum operating pressure of safety valve	bar	12.0	12.0	12.0	12.0
Noise level at 5 bar (L _{pA})	dB	≤67.0	≤51.5	≤69.0	≤57.0
Operating mode	%	S1-100	S1-100	S1-100	S1-100
PDP drying at 7 bar	°C	-	-	≤ +3	≤ +3
Time to fill air tank from 0 to 7 bar	s	226	226	250	250
Dimensions (net) w x d x h	mm	577x587x804	750x715x1015	577x628x804	750x715x1015
Net weight	kg	58	108	61	114

- a) State the compressor variant in the order
- b) For other range of pressure consult with the supplier
- c) Weight value is only informative data and applies only to a product without any accessories

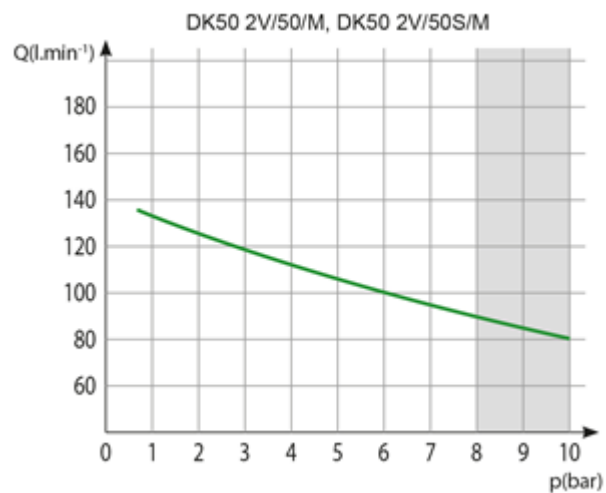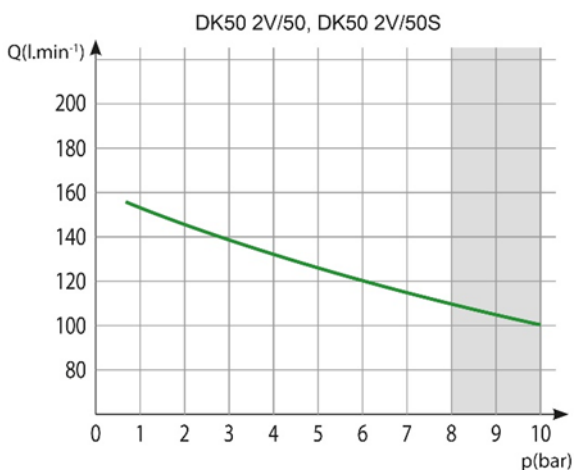Dependence of compressor capacity on working pressure

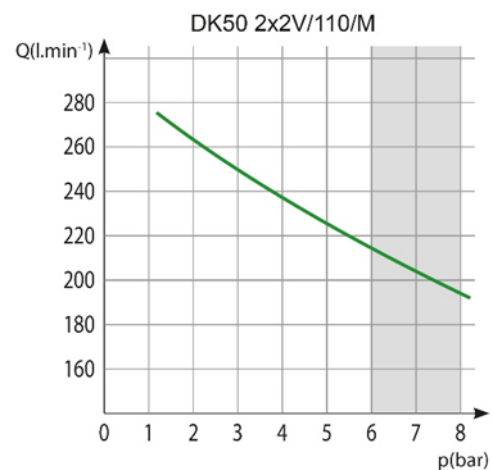
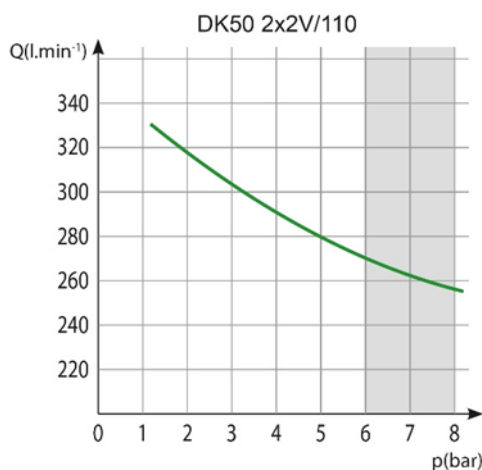
Working pressure 6 – 8 bar		DK50 2x2V/110		DK50 2x2V/110S		DK50 2x2V/110/M		DK50 2x2 V/110 S/M	
Nominal voltage Frequency ^{a)}	V, Hz	230,50 3x400, 230,60 50		230,50 3x400, 230,60 50		230,50 3x400, 230,60 50		230,50 3x400, 230,60 50	
Capacity at 6 bar (FAD)	l/min	270		270		215		215	
Working pressure ^{b)}	bar	6.0 – 8.0		6.0 – 8.0		6.0 – 8.0		6.0 – 8.0	
Rated current	A	15.2 8.0	18.5	15.5 8.4	18.8	15.3 8.0	18.7	15.6 8.4	19.0
Motor power	kW	2x1.2		2x1.2		2x1.2		2x1.2	
Air tank volume	l	110		110		110		110	
Air quality – filtration	µm	-		-		0.1		0.1	
Maximum operating pressure of safety valve	bar	9.0		9.0		9.0		9.0	
Noise level at 5 bar (L _{pA})	dB	≤73		≤55		≤73		≤56	
Operating mode	%	S1-100		S1-100		S1-100		S1-100	
PDP drying at 7 bar	°C	-		-		≤ +3		≤ +3	
Tie to fill air tank from 0 to 6 bar	s	159		159		198		198	
Dimensions (net) w x d x h	mm	1085x490x815		1200x730x990		1085x625x815		1200x730x990	
Net weight	kg	106 104		177 175		113 111		184 182	

a) State the compressor variant in the order

b) For other range of pressure consult with the supplier

c) Weight value is only informative data and applies only to a product without any accessories

Dependence of compressor capacity on working pressure



Working pressure 8 – 10 bar		DK50 2x2V/110		DK50 2x2V/110S		DK50 2x2V/110/M		DK50 2x2 V/110 S/M	
Nominal voltage Frequency ^{a)}	V, Hz	230, 50 3x400, 50	230,60	230,50 3x400, 50	230,60	230,50 3x400, 50	230,60	230,50 3x400, 50	230,60
Capacity at 8 bar (FAD)	l/min	210		210		165		165	
Working pressure ^{b)}	bar	8.0 – 10.0		8.0 – 10.0		8.0 – 10.0		8.0 – 10.0	
Rated current	A	14.7 8.5	17.9	14.9 8.9	18.2	14.8 8.5	18.0	15.0 8.9	18.3
Motor power	kW	2x1.2		2x1.2		2x1.2		2x1.2	
Air tank volume	l	110		110		110		110	
Air quality – filtration	µm	-		-		0.1		0.1	
Maximum operating pressure of safety valve	bar	12.0		12.0		12.0		12.0	
Noise level at 5 bar (L _{pA})	dB	≤73		≤55		≤73		≤56	
Operating mode	%	S1-100		S1-100		S1-100		S1-100	
PDP drying at 7 bar	°C	-		-		≤ +3		≤ +3	
Time to fill air tank from 0 to 6 bar	s	256		256		363		363	
Dimensions (net) w x d x h	mm	1085x490x815		1200x730x990		1085x625x815		1200x730x990	
Net weight	kg	110 108		181 179		117 115		188 186	

a) State the compressor variant in the order

b) For other range of pressure consult with the supplier

c) Weight value is only informative data and applies only to a product without any accessories

Dependence of compressor capacity on working pressure

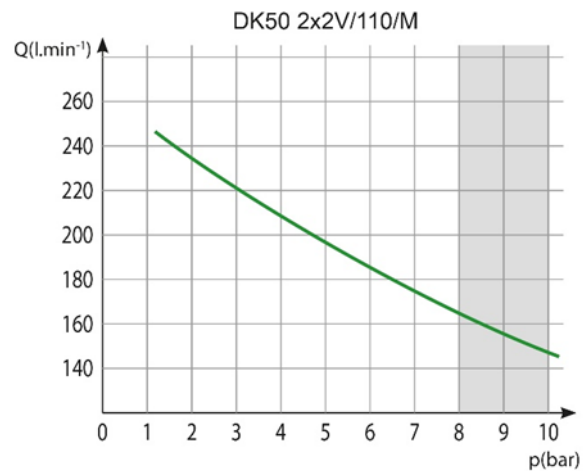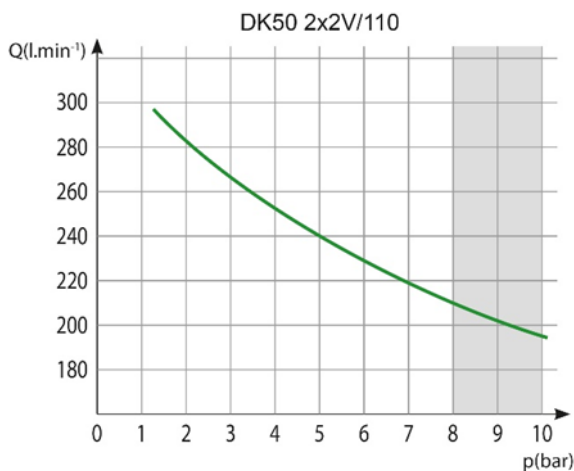FAD correction of capacity for altitude

Capacity given in the form of FAD („Free Air Delivery“) applies to the following conditions:

Altitude	0 m.n.m.	Temperature	20°C
Atmospheric pressure	101325 Pa	Relative humidity	0%

To calculate FAD compressor capacity in dependence on altitude, it is necessary to apply correction factor according to the following table:

Altitude [m.n.m.]	0 -1500	1501 - 2500	2501 - 3500	3501 - 4500
FAD correction factor	1	0,8	0,71	0,60

INSTALLATION

Risk of incorrect installation.

Only a qualified professional can install the compressor and place it into operation for the first time. His obligation is to train the operating personnel on the use and maintenance of the device. He shall confirm installation and training of operators by an entry into the installation record.

9. INSTALLATION CONDITIONS

- The compressor may only be installed and operated in dry, well-ventilated and dust-free rooms with conditions stated in technical data

Risk of damage to the device.

Do not operate the device in outdoor environment or in humid or wet rooms.

Explosion risk.

Do not install the device in any rooms, in which explosive gases, dusts or flammable liquids may be present.

- The compressor must be installed in such a way, that it is accessible at all times for operation and maintenance and label on the device is accessible.
- The compressor must stand on a flat, sufficiently stable base (be aware of the compressor weight, see technical data).

You may notice a “new product” odour when you first place the product into service (for a short period of time). This odour is temporary and does not impede the normal use of the product. Ensure the space is properly ventilated after installation.

10. COMPRESSOR ASSEMBLY

10.1. Manipulation and fixation removal

- Unpack the compressor from the packaging.
- Put the compressor at the place of operation (Fig. 4).


Use the compressor base for gripping, when manipulating with the compressor. Do not use any other compressor parts (pump, cooler, etc.) for gripping the compressor.


The number of persons required to handle the equipment must be matched to the weight of the device.

Fig. 4: Manipulation with the compressor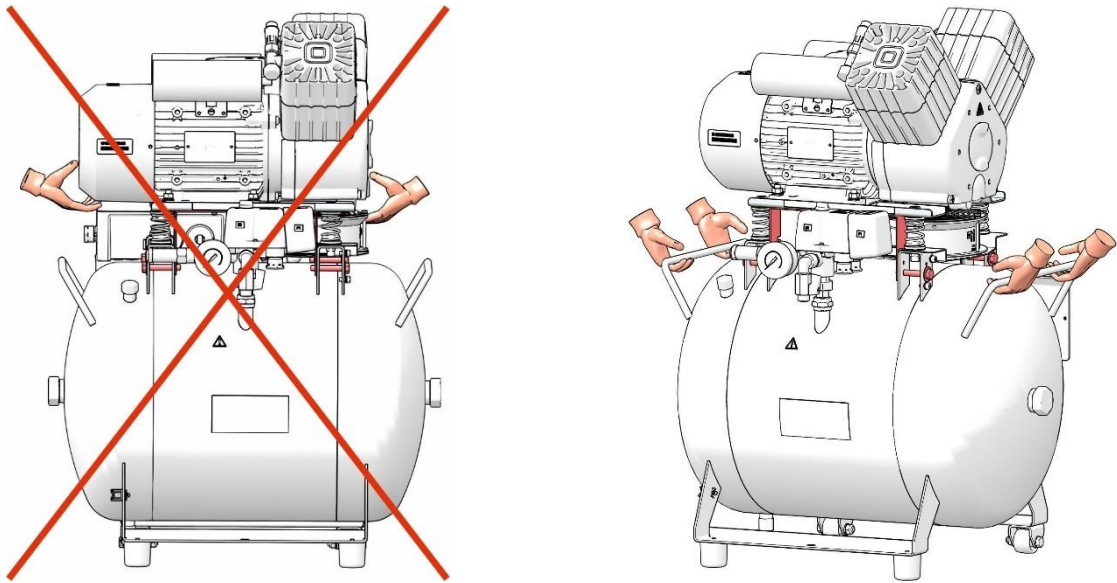

DK50 2V/50

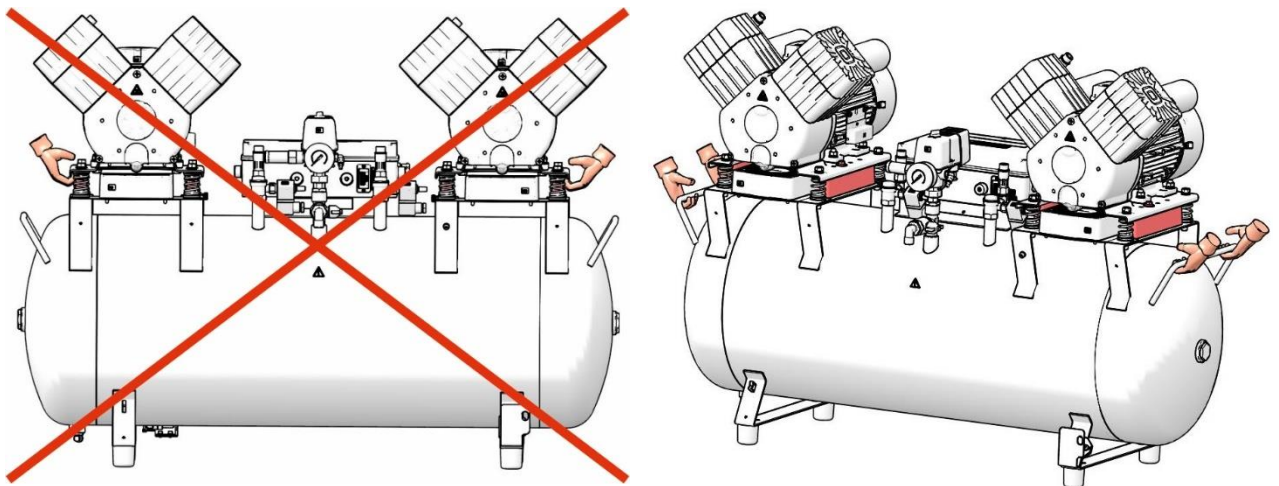DK50 2x2V/110


- Remove transport fixing elements from the pumps (Fig. 5).


Prior to installation, ensure that the compressor is free of all transport packaging and stabilizers to avoid any risk of damage to the product.

Remove fixing elements of the pumps once the compressor has been assembled and balanced at the final installation place.

Fig. 5: Releasing the pump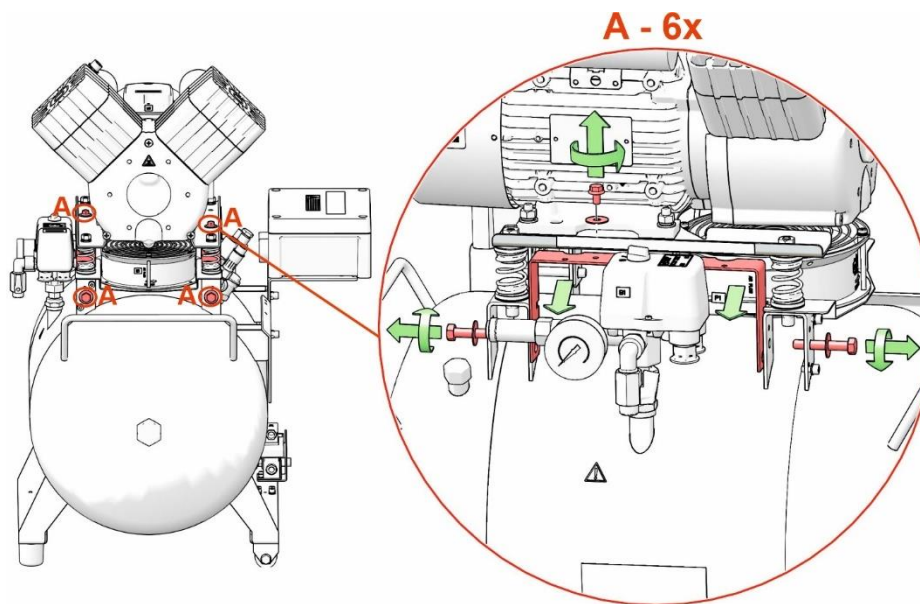

DK50 2V/50

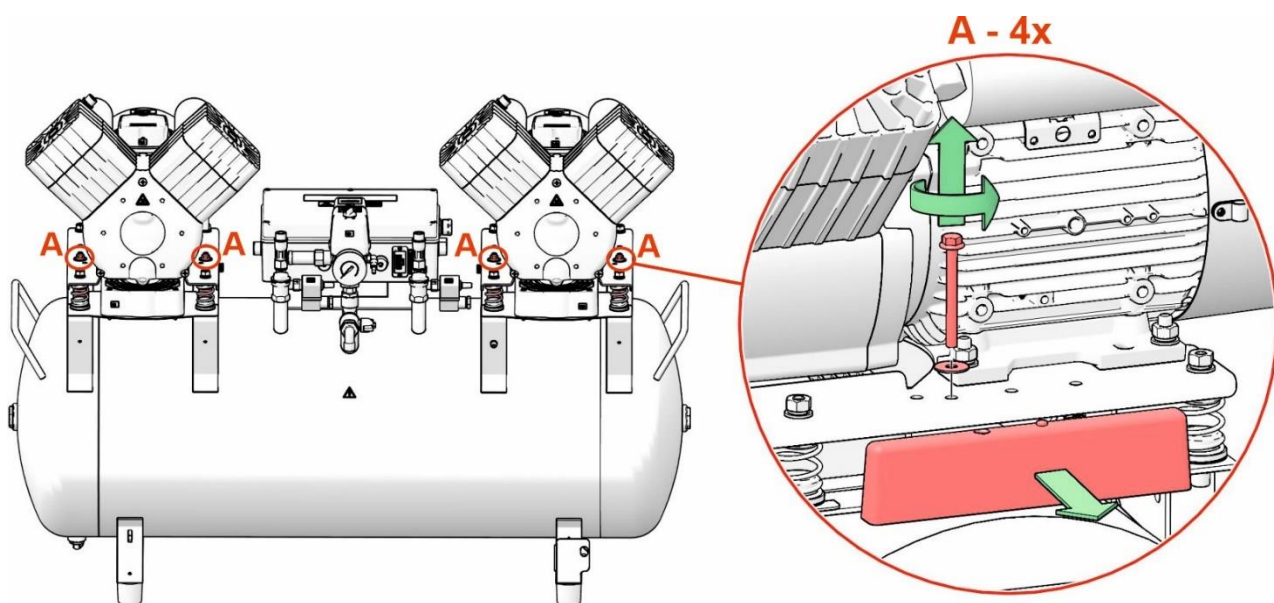DK50 2x2V/110

10.2. Placement of the compressor in the cabinet

Disassembly cabinet door (DK50 2V/50 - Fig. 3)

- Remove the door held by 2 screws and disconnect the grounding lead.

Compressor placement

- Remove the connecting strip (33) in front of the cabinet.
- Before placing the compressor without the dryer (DK50 2V/50) into the cabinet, please follow Chapter 12.3 to add sound insulation foam into the cabinet.
- Insert the compressor into the cabinet using the positioning truck (35), place the connecting strip in front of the cabinet and adjust the position of the compressor so that the two “V” notches on the connecting strip align with the compressor legs.

- This puts the compressor in the right position for the cabinet. Insert the compressor into the cabinet only so far as to be able to install the connecting strip and so the compressor doesn't obstruct its installation.
- Insert the compressor with a dryer into the cabinet so that the dryer fan fits into the cooling tunnel opening in the cabinet. **WARNING**, do not insert the compressor down to the stop. For dryer versions of the compressor, the flow of cooling air into the cooler will be restricted and there is a risk of permanent damage to the dryer.
- Place the connecting strip (33) in its original position in the lower part of the cabinet.

Cabinet door installation

- Align the door with the cabinet, connect the grounding lead to the door and install the door to the cabinet using 2 screws.

A compressor with air dryer must not be inserted to the cabinet to the stop as there is a risk of permanent damage to the dryer.

Opening the upper cover (DK50 2x2V/110 - Fig. 3)

- Lift the upper cabinet cover grabbing it by its handle (31) so that the gas springs keep the cover open.

Be careful not to pinch your fingers when closing the cover on the cabinet.

Always lock with the quick release after closing the cover on the cabinet!

Removal of the front panel on the cabinet

- Remove the door held on by the 4 (four) M5 screws and disconnect the earthing conductor

Compressor placement

- Insert the compressor into the cabinet so the compressed air outlet is pointing towards the operator
- The dryer fan must be inserted into the ventilation tunnel in the cabinet on compressor models with a dryer

Cabinet door installation

- Align the door with the cabinet, connect the grounding lead to the door and install the door to the cabinet using 4 screws.

10.3. Adding sound-insulating material into the cabinet (DK50 2V/50)

A component from the sound-insulating material is to be installed in the opening of the cabinet of the compressor without a dryer.


This component is provided in the basic equipment delivered with the product.

Fig. 6: DK50 2V/50S

A

B

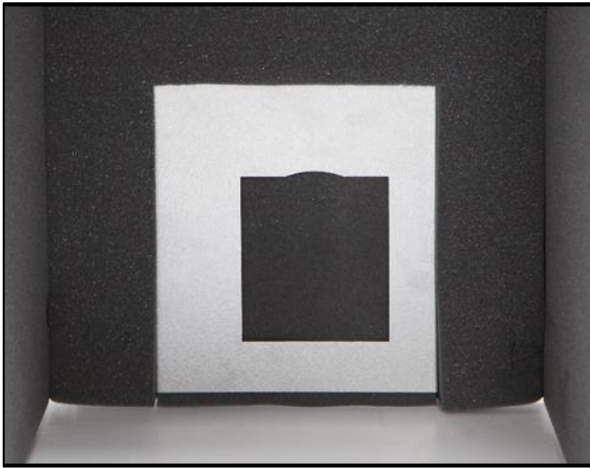C

10.4. Installation of condensate drain valve on cabinet (DK50 2x2V/110)

For compressor variant with cabinet install the drain cock (1) into a hole on the side of the cabinet and install a tube PA Ø8 / Ø6 (2). Insert the second end of the tube into the fitting (4) under the air tank; the valve (3) with the tube shall be dismantled from this fitting.

Blind the hole with a plug Ø16.5 at the opposite side.

The drain cock is part of the cabinet basic equipment. The condensate drain valve can be mounted on any side of the compressor cabinet.

Fig. 7: Installation of condensate drain valve on cabinet

11. PNEUMATIC CONNECTION

11.1. Connecting to the compressed air outlet

- Route the pressure hose from the compressed air outlet (1) to the connected equipment.
- When the compressor is in the cabinet, route out the power cord through the opening in the rear wall of the cabinet.

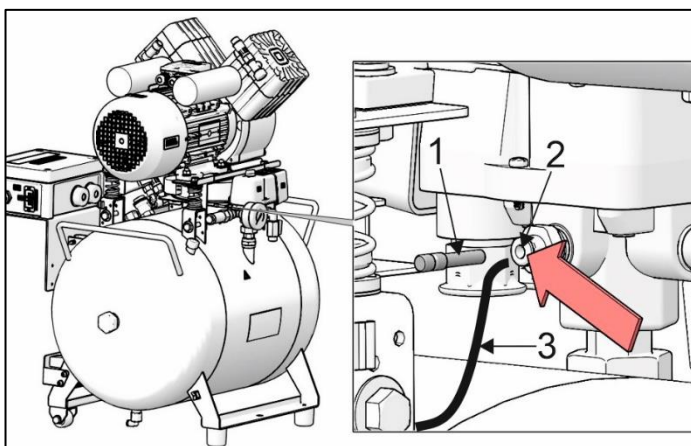
Fig. 8: Connection of pressure hose


11.2. Connecting the cabinet pressure gauge to the compressor

- Remove the plug (1) from the threads (2) on the pneumatic block of the compressor.
- Connect the cabinet pressure gauge hose (3) to the threaded fitting.

Fig. 9: Connecting the cabinet pressure gauge to the compressor


DK50 2V/50S
DK50 2V/50S/M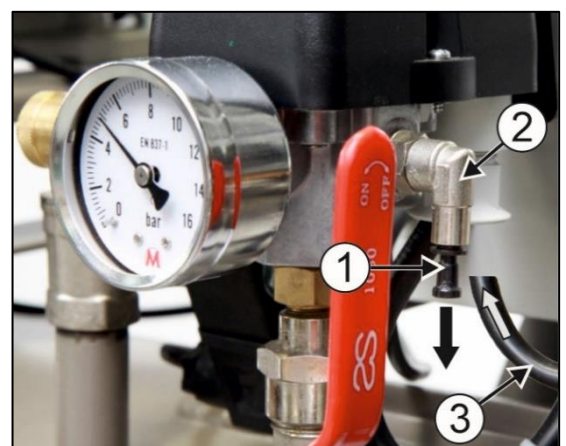

DK50 2x2V/110S
DK50 2x2V/110S/M

11.3. Connecting the condensate vessel

- Connect the drain hose (1) to the condensate vessel on compressors with dryers (Fig. 10)
- Route the hose through the opening in the rear wall of the cabinet for compressor models with a dryer and connect it to the vessel.

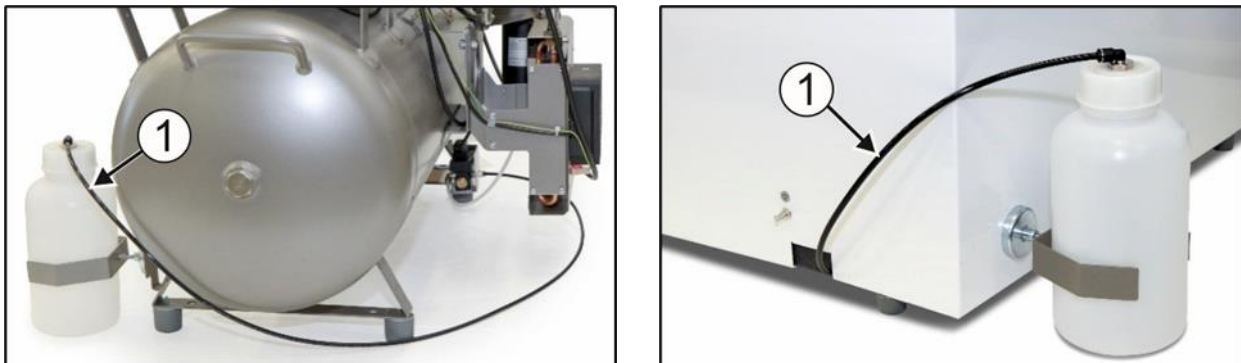Fig. 10: Connecting the condensate vessel

Risk of damage to pneumatic components.

Air hoses must not be broken.

12. ELECTRICAL CONNECTION

- The product is delivered with a cord equipped with a plug with earthing pin.
- Keep the socket easily accessible to ensure that the device can be safely disconnected from the mains.
- Connection to the power distribution cabinet must be max.16 A

Risk of electric shock.

it is necessary to follow all local electrotechnical regulations. The mains voltage and frequency must comply with the data stated on the device label.

Risk of fire and electric shock.

Electrical cable must not be in contact with hot compressor components.

Risk of fire and electric shock.

Electrical cord must not be broken.

- Connect the pin for equipotential bonding 6Ø mm (1) to the electrical circuit using the defined method under valid electrical engineering regulations
- The socket for equipotential bonding (2) is an accessory and is not provided in the basic package.

**Fig. 11: Equipotential bonding**

- Mount the connector of the grounding conductor on the door and screw on the cabinet.

Power supply connection – DK50 2x2V/110

Supply mains	Type of electrical conductor
230V/50 (60) Hz	3Gx1.5x4000
400V/50 (60) Hz	5Gx1.5x4000


12.1. Connecting a compressor not installed in a cabinet

- Insert the mains plug into a rated mains socket.
- The compressor is ready for operation.

12.2. Connecting a compressor installed in a cabinet

- Route the mains plug through the opening in the rear wall of the cabinet for cabinet-mounted compressors.
- Connect the cabinet electrically to the compressor by inserting the provided power cord with connector (1) into an outlet (2). (Fig. 8)
- Disconnect the cabinet electrically from the compressor by pulling the connector (1) from the outlet while the latch is released.

Fig. 12: Connect the cabinet to the compressor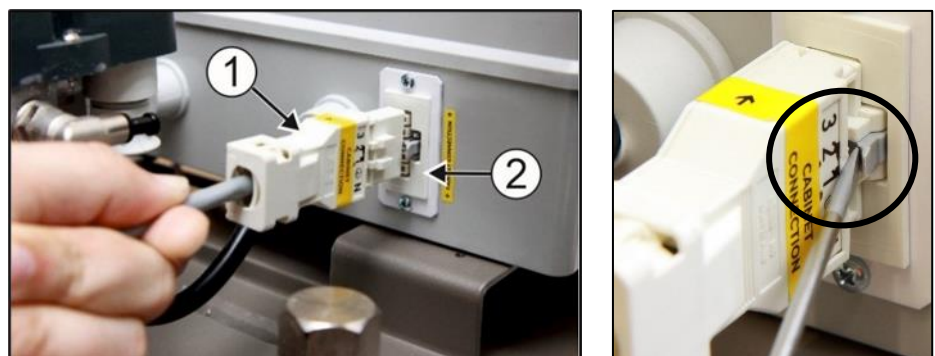

13. COMMISSIONING

- Check if all fixing elements used during transport have been removed.
- Check correct connection of compressed air (see chapter 11).
- Check correct connection to the mains (see chapter 12).
- Check position of the circuit breaker switch, it must be in position „I“. If it is in position „0“, turn the switch (4) to position „I“. (Fig. 13).
- On DK50-10S/M (8 - 10bar) compressor, switch the switch (4) on the front side of the cabinet to position „I“, the green light indicates the device status in operation. (Fig. 13)
- Check connection of the cabinet connector to the compressor. (Fig. 12)
- Check connection of the cabinet manometer hose to the pneumatic block of the compressor.. (Fig. 9)


The compressor is not equipped with a backup power supply.


14. PNEUMATIC AND ELECTRICAL DIAGRAMS

14.1. Pneumatic diagrams

DK50 2V/50/M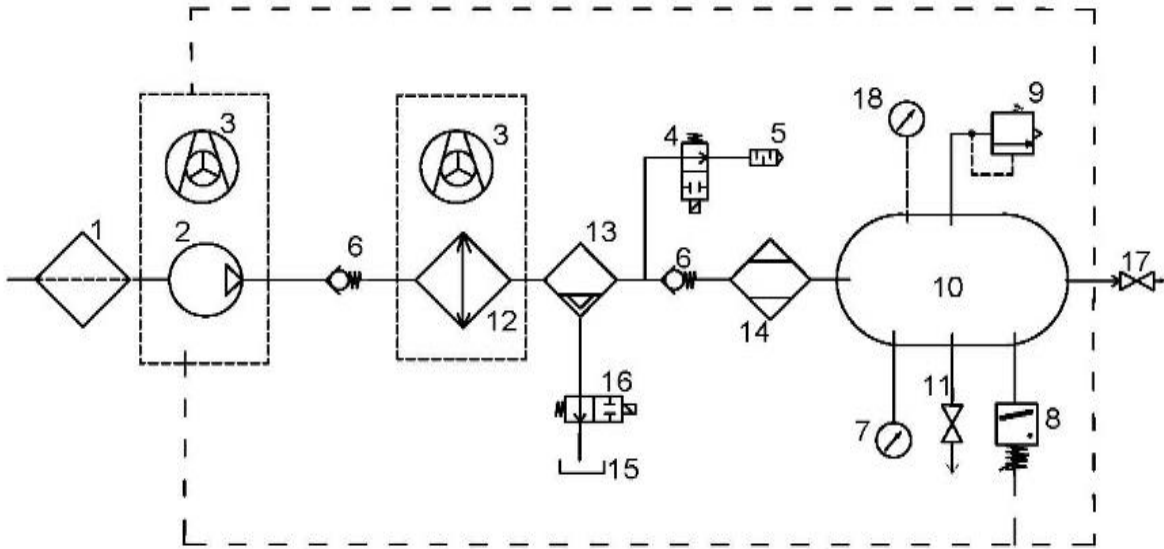


DK50 2V/50

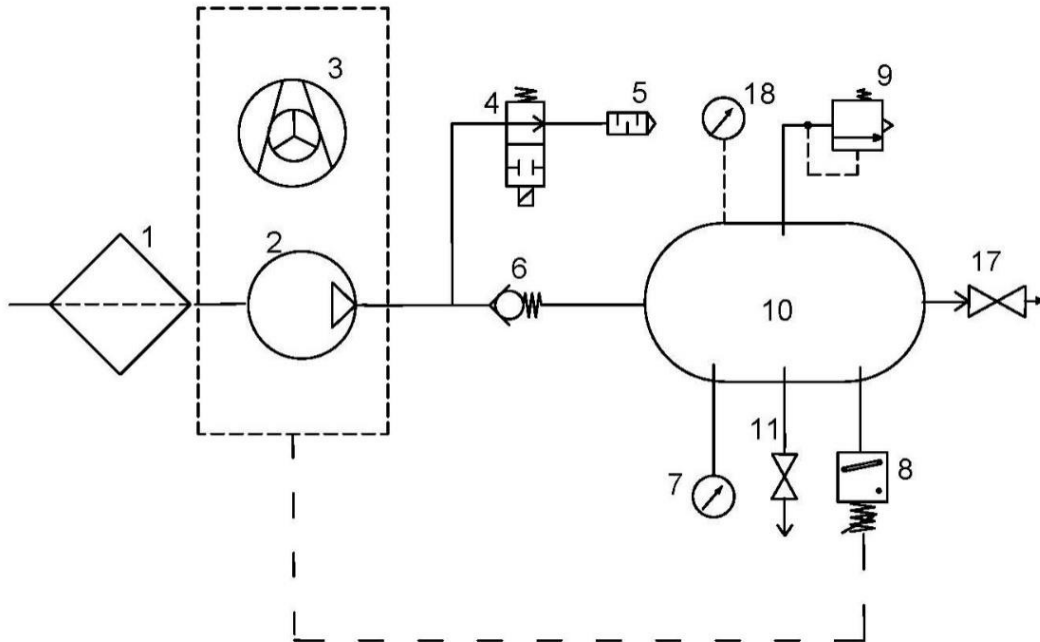
DK50 2x2V/110/M

230V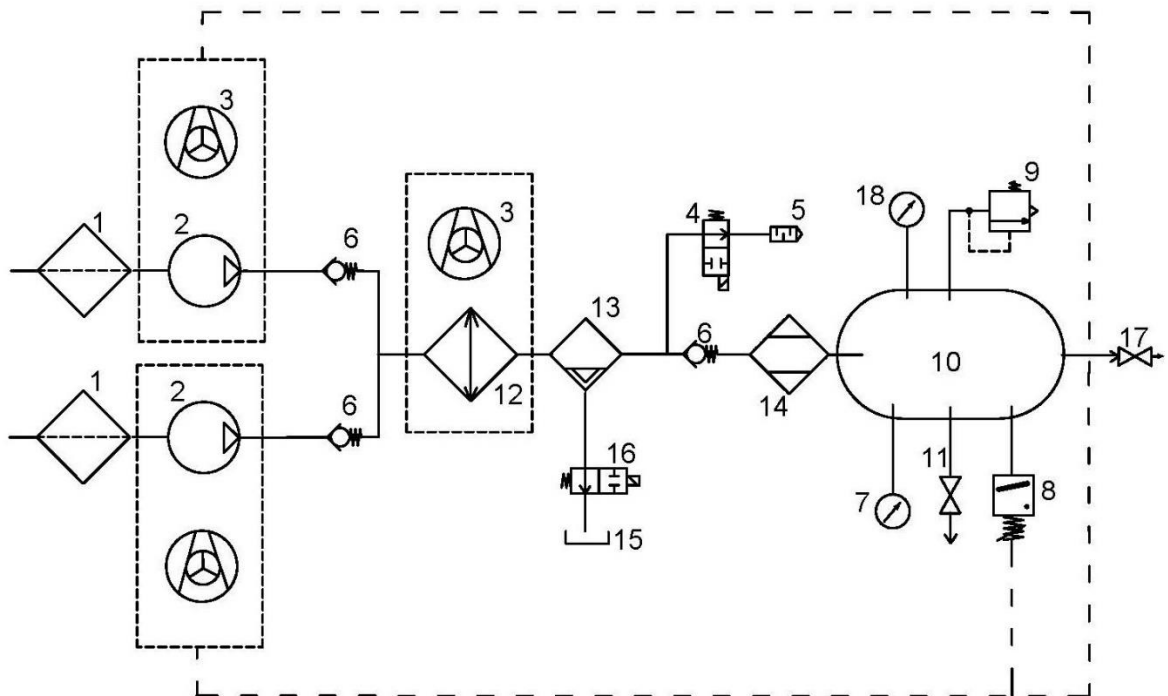


DK50 2x2V/110/M

400V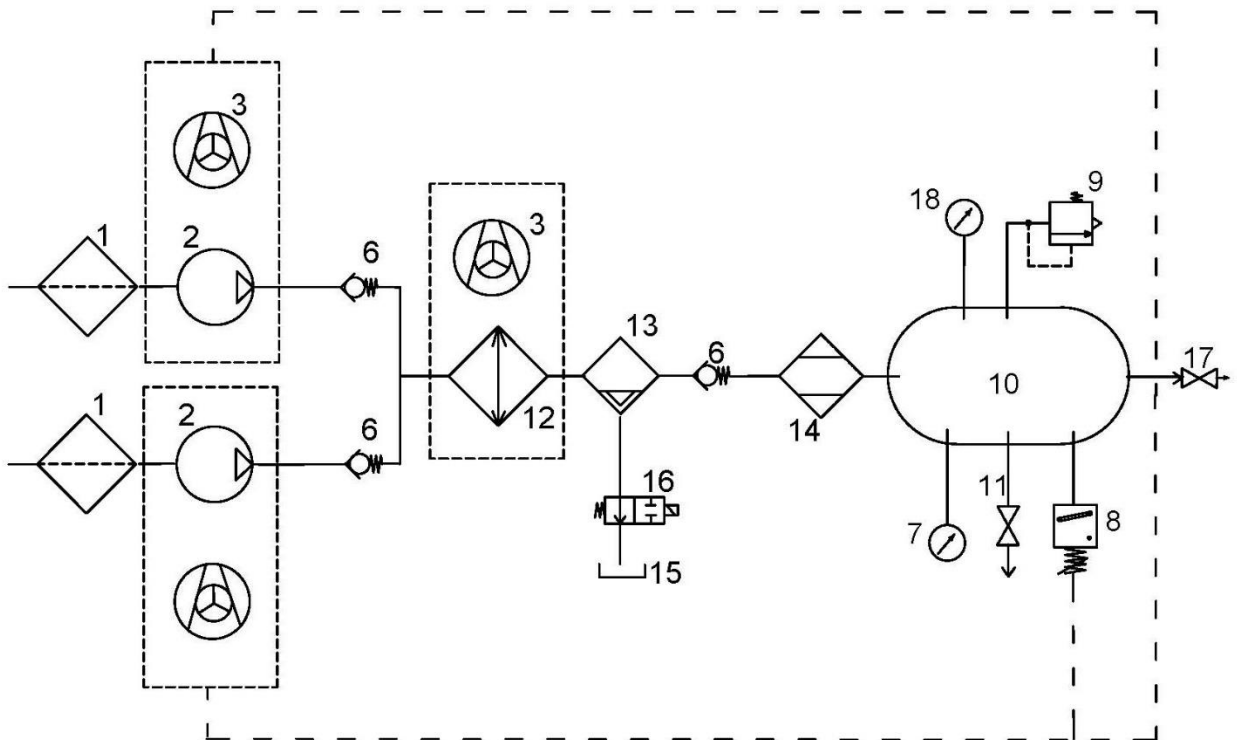


DK50 2x2V/110

230V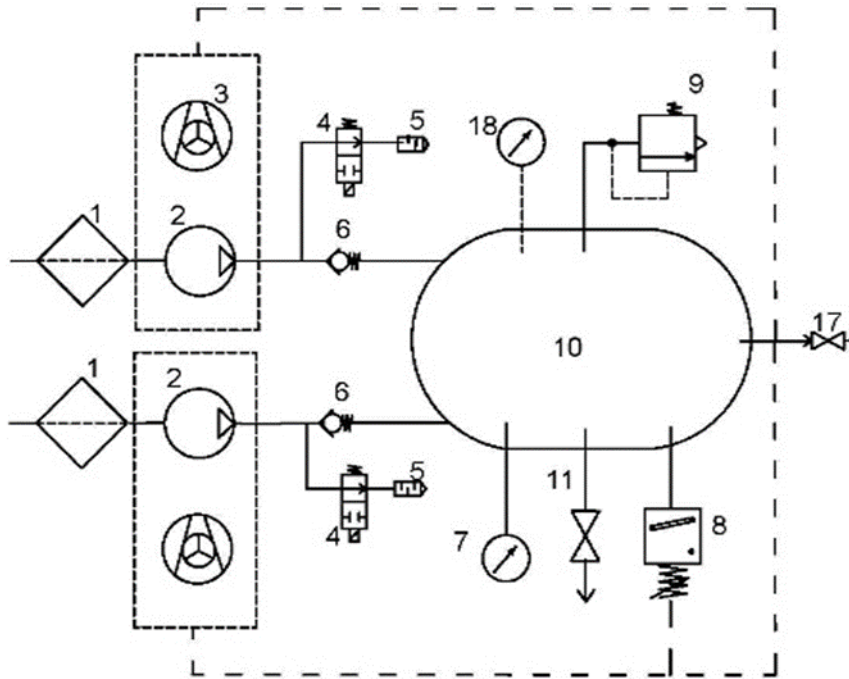

DK50 2x2V/110

400V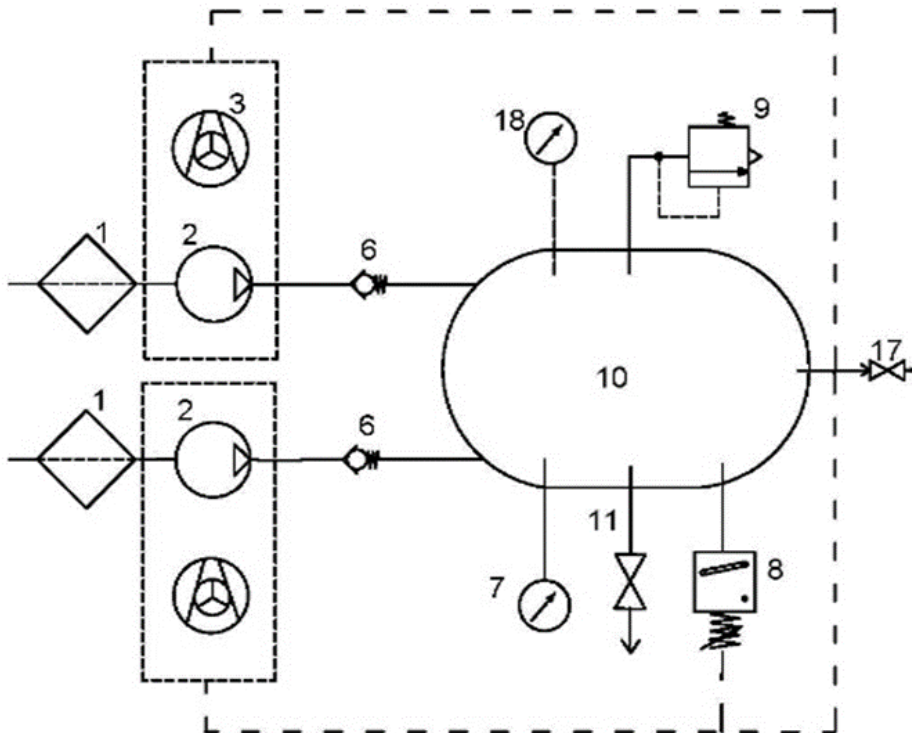

Description to pneumatic diagrams:

- | | | | |
|---|------------------|----|---------------------------------|
| 1 | Inlet filter | 10 | Air tank |
| 2 | Compressor | 11 | Drain valve |
| 3 | Fan | 12 | Cooler |
| 4 | Relief valve | 13 | Coalescence filter |
| 5 | Noise silencer | 14 | Membrane filter |
| 6 | Non-return valve | 15 | Condensate collection vessel |
| 7 | Pressure gauge | 16 | Condensate drain solenoid valve |
| 8 | Pressure switch | 17 | Outlet valve |
| 9 | Safety valve | 18 | Cabinet pressure gauge |


14.2. Electrical diagrams

DK50 2V/50

6 - 8 bar, 8 - 10 bar

1/N/PE ~ 230V/50Hz, 230V/60Hz

ELEKTRICAL OBJECT OF 1st. CAT.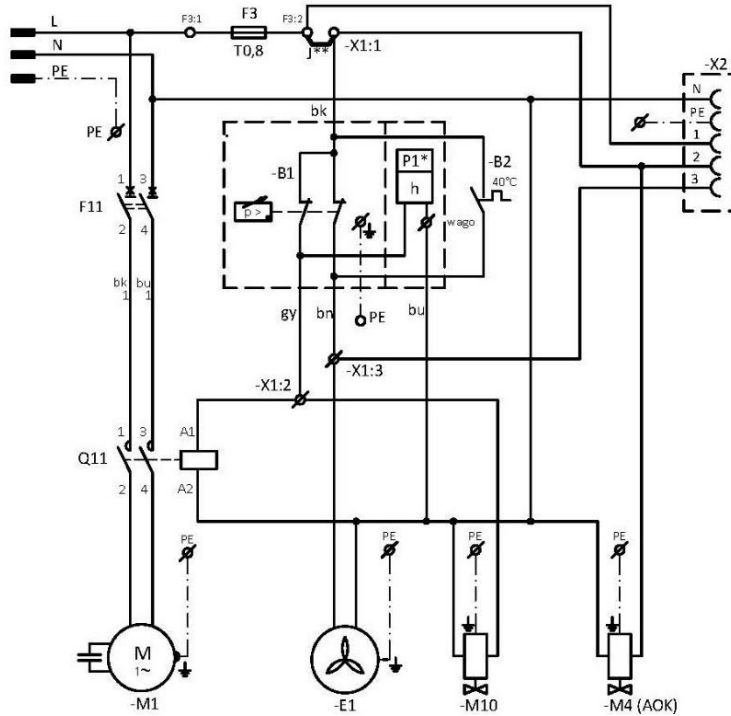


DK50 2V/50/M

6 - 8 bar, 8 - 10 bar

1/N/PE ~ 230V/50Hz, 230V/60Hz

ELEKTRICAL OBJECT OF 1st. CAT.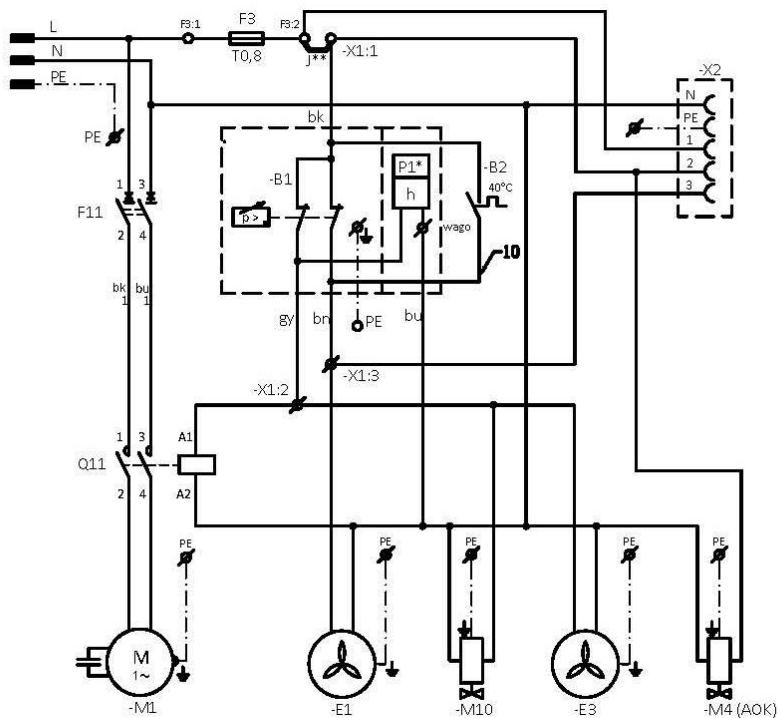


DK50 2V/50

6 - 8 bar

1/N/PE ~ 400V 50 Hz

ELECTRICAL MAINS TN-S [TN-C-S]

ELEKTRICAL OBJECT OF 1st. CAT.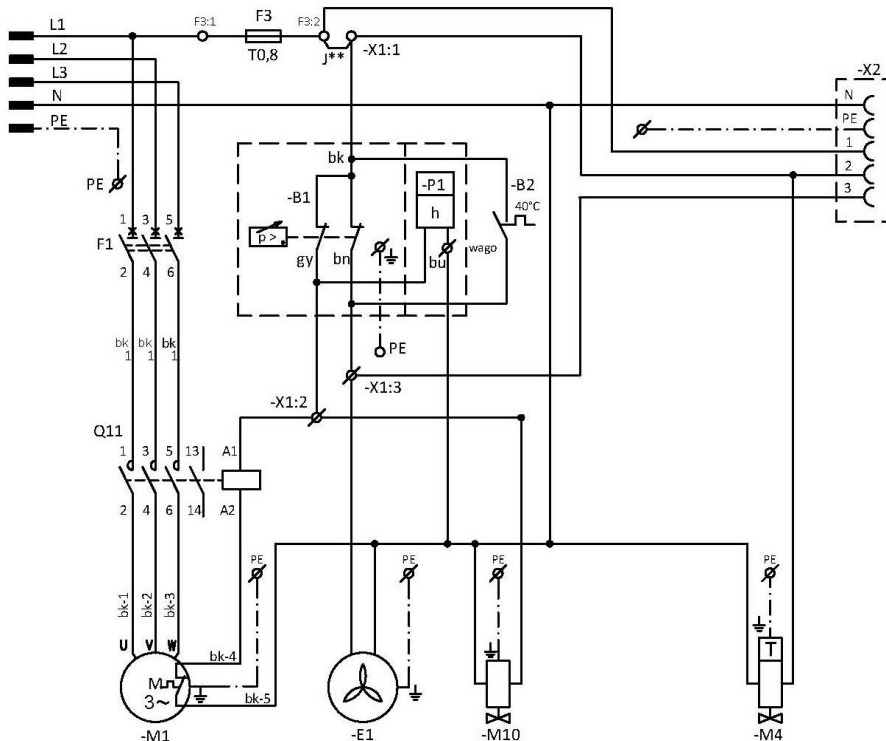


DK50 2V/50/M

6 - 8 bar

1/N/PE ~ 400V 50 Hz

ELECTRICAL MAINS TN-S [TN-C-S]

ELEKTRICAL OBJECT OF 1st. CAT.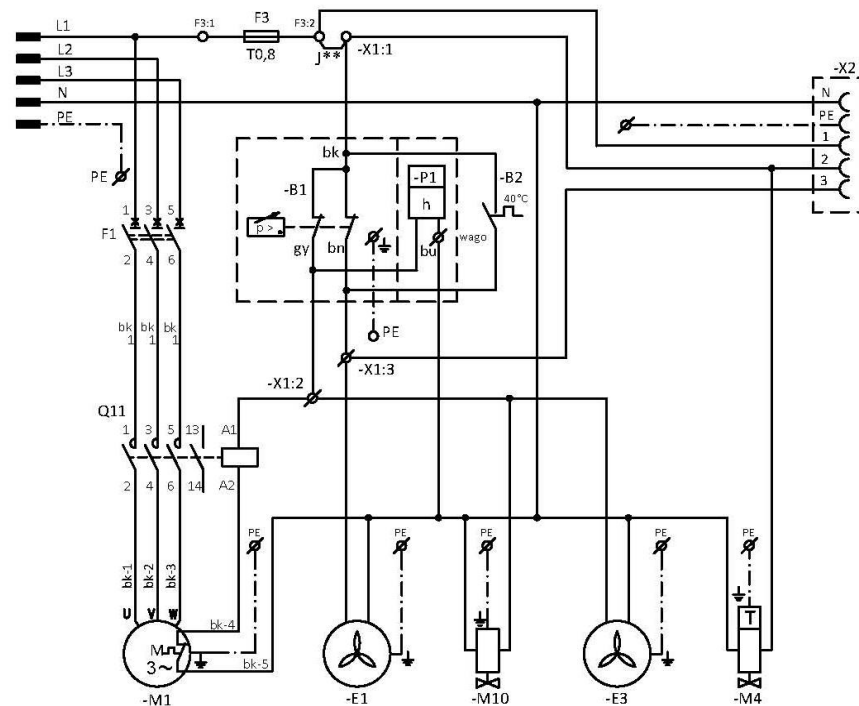


DK50 2x2V/110

5 - 7 bar, 6 - 8 bar, 8 - 10 bar

1/N/PE 230 V, 50/60 Hz

ELEKTRICAL OBJECT OF 1st. CAT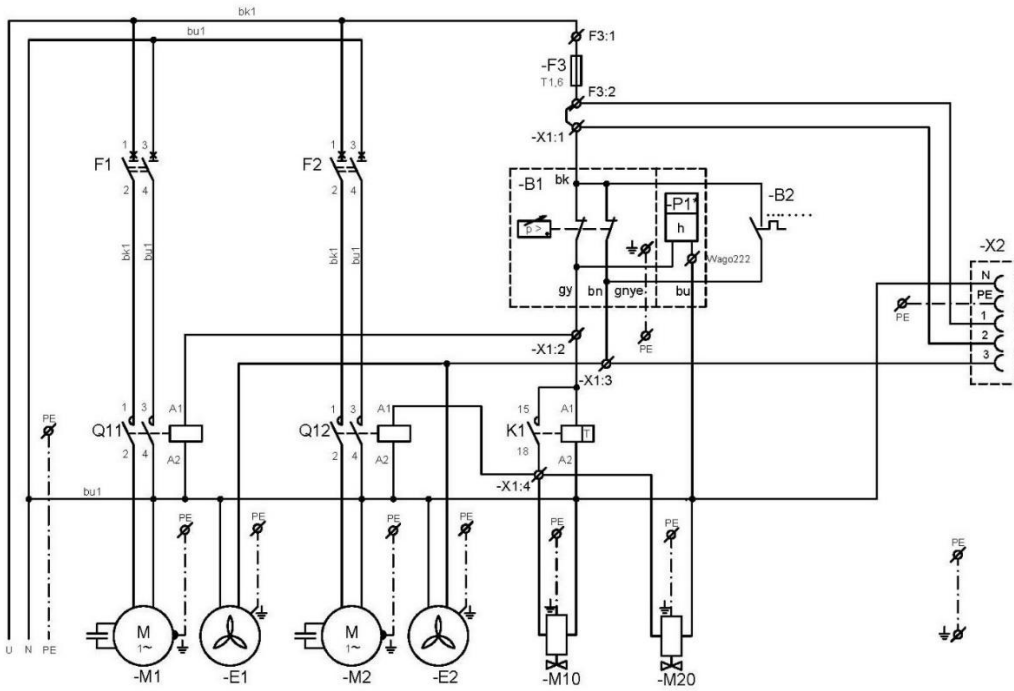


DK50 2x2V/110/M

5 - 7 bar, 6 - 8 bar

1/N/PE 230 V, 50/60 Hz

ELEKTRICAL OBJECT OF 1st. CAT.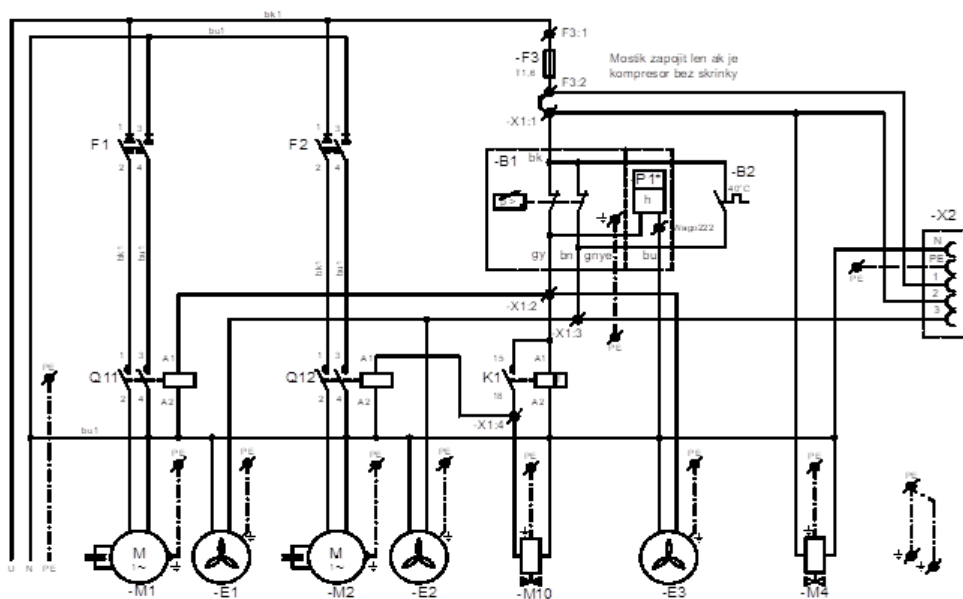


DK50 2x2V/110

5 - 7 bar, 6 - 8 bar, 8 - 10 bar

1/N/PE 230 V, 50/60 Hz

ELEKTRICKAL OBJECT OF 1st. CAT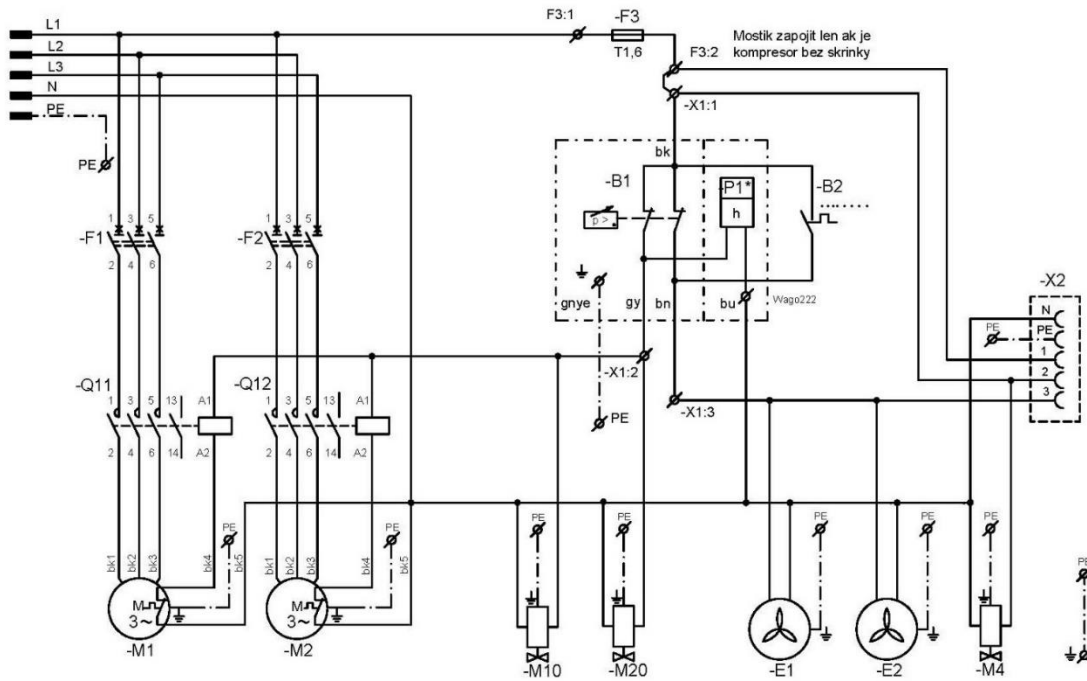


DK50 2x2V/110/M

5 - 7 bar, 6 - 8 bar, 8 - 10 bar

1/N/PE 230 V, 50/60 Hz


ELECTRICAL MAINS TN-S [TN-C-S]

ELECTRICAL OBJECT OF 1st. CAT.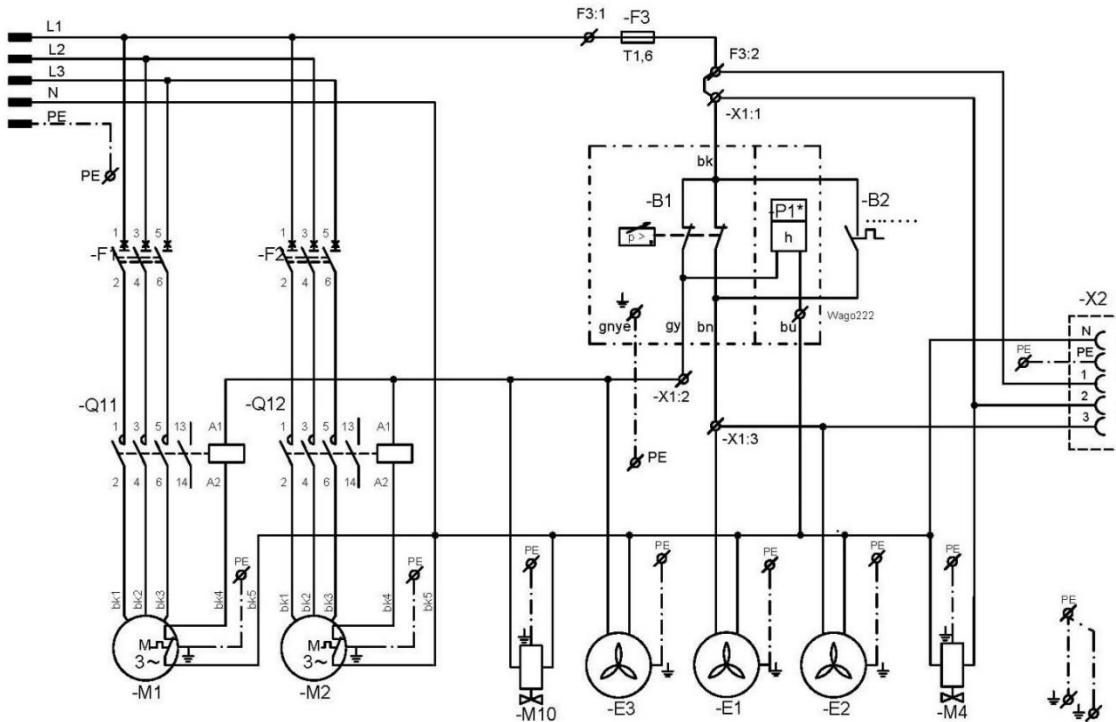


Compressor cabinet

ELEKTRICTRICAL OBJECT OF 1st. CAT.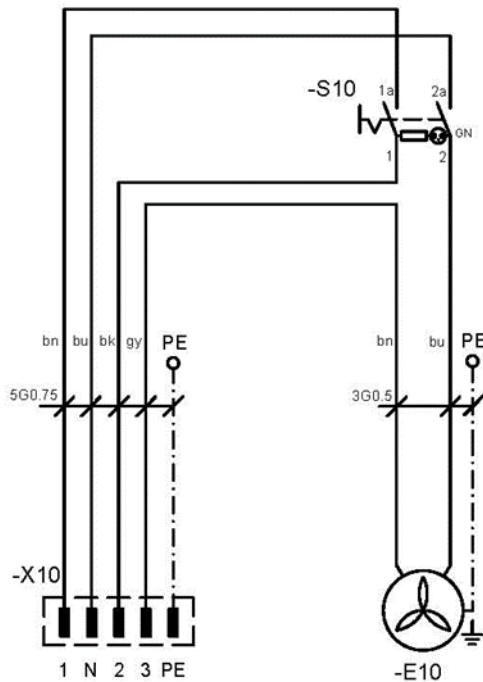

DK50 2V/50S

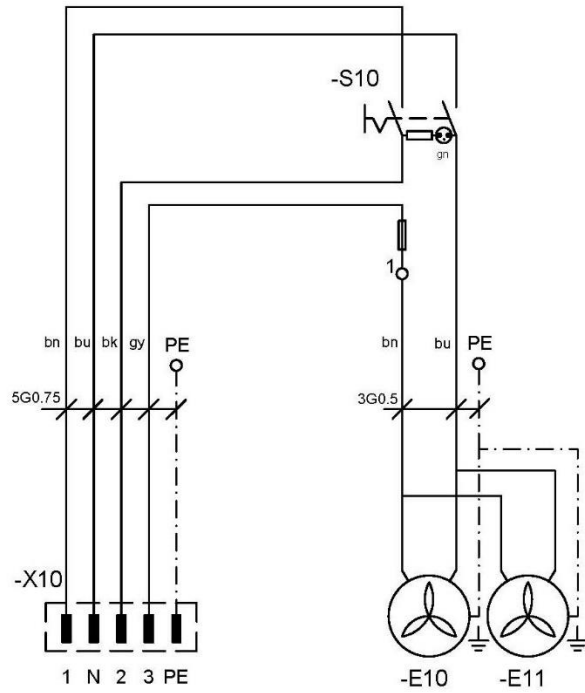DK50 2x2V/110S

Legend for electrical diagram

M1,M2	Compressor motor	B1	Pressure switch
E1,E2	Compressor fan	X1	Terminal strip
E3	Dryer fan	M4	Condensate drain valve
M10,M20	Relief valve	P1*	Hour meter
B2	Temperature switch	S10	Switch
E10,E11	Cabinet fan	K1	Timer relay
X10	Connector	F1,2	Breaker
X2	Socket	F3	Fuse
		Q11,12	Contactors

Note:

- J** - Only connect the jumper for compressors without cabinet (see chapter 17.10)
- P1* - Hour meter installed only for the 8-10bar compressor model

OPERATION

THE EQUIPMENT MAY ONLY BE OPERATED BY THE TRAINED STAFF!

Risk of electric shock

In case of any danger, disconnect the compressor from the mains (pull out the mains plug).

Risk of burn or fire.

During compressor operation, the pump parts may heat to temperatures dangerous for contact with persons or materials.

Warning – compressor is controlled automatically.

Automatic start-up. When pressure in the pressure tank drops to switch-on pressure, the compressor automatically switches on. The compressor automatically switches off, when pressure in the air tank reaches the switch-off pressure.

Hazard of damage to pneumatic components.

Working pressures of the pressure switch set by the manufacturer must not be changed. If the compressor operates at higher working pressure than the switch-on pressure, it means that the air consumption is too high (see Chapter Malfunctions)

Hazard of damage to dryer.

When the dryer is operated at ambient temperature that is higher than maximum operating temperature stated in Technical data, the dryer can be damaged.

Required drying performance can only be achieved when the defined operating conditions are followed.

Drying performance decreases and the achieved dew point drops if the dryer is operated at any pressure below the minimum working pressure.

During longer operation of the compressor, the temperature in the surrounding of the compressor increases above 40 °C and the cooling fan switches on automatically. After cooling the space below 32 °C, the fan switches off.

15. SWITCHING THE COMPRESSOR


Start the compressor (without a cabinet) at the pressure switch (1) by turning the switch (2) to position "I." This starts the compressor and fills the tank to the switching off pressure, which then shuts off the compressor.

Start the compressor (with a cabinet) at the switch (4) on the front side of the cabinet, the indicator will turn green. This starts the compressor and fills the tank to the switching off pressure, which then shuts off the compressor.

The compressor operates in automatic mode, switched on and off by the pressure switch, depending on compressed air consumption.

Check the values of switch-on and switch-off pressure using the pressure gauge (3). Tolerance of $\pm 10\%$ is acceptable. Pressure in the air tank must not exceed the allowed operating pressure.

Fig. 13: Switching the compressor

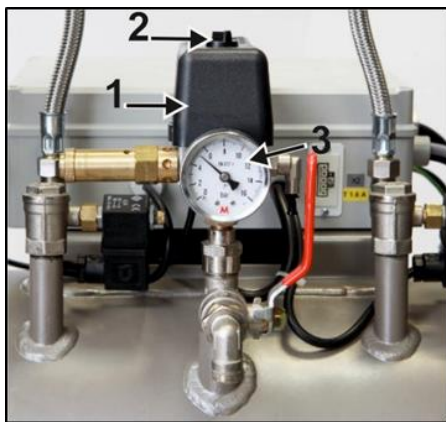The compressor starts to operate according to the description of operation as follows:

- **Compressor (without dryer)** – when first started and put into operation, the compressor fills the air tank to the switch-off pressure and the pressure switch turns off the compressor. Then the compressor operates in automatic mode, switches on and off depending on compressed air consumption.
- **Compressor with dryer** – the compressor operates in the same manner as above, only the compressed air passes through the cooler that removes moisture from the compressed air.

Risk of damage to pneumatic parts.

The pressure switch (1) was set up by the manufacturer and any other changes to its settings may be performed only by a qualified technician trained by the manufacturer.

16. SWITCH OFF THE COMPRESSOR

- Switching off the compressor due to service or any other reason shall be performed using the pressure switch (1) by turning the switch (2) to position „0“ (Fig. 13) and **pulling out the mains plug from the socket**. This disconnects the compressor from the mains supply.
- Vent the pressure in the air tank to zero by opening the drain valve. (Fig. 7).

PRODUCT MAINTENANCE**17. PRODUCT MAINTENANCE**

The operator should carry out device checks regularly in the intervals defined by applicable regulations. Test results must be recorded.

The device has been designed and manufactured to keep its maintenance to a minimum. For correct and reliable operation of the compressor perform the following operations:

Risk of servicing by persons without required qualification.

Repair works beyond standard maintenance (see chapter.17.1) may only be performed by a qualified technician (organization authorized by the manufacturer) or manufacturer's customer service.

Standard maintenance works (see chapter 17.1) may only be carried out by a trained operator.

Only use spare parts and accessories approved by the manufacturer.

Risk of injury and damage to the device.

Before any maintenance work, it is necessary to:

- **check, if it is possible to disconnect compressor from the supplied equipment, so that there is no risk of damage to a person using the given appliance or any other material damages**
- **switch off the compressor**
- **disconnect it from the mains (pull out the mains plug)**
- **vent the compressed air from the air tank**

Risk of injury when venting the compressed air.

Protect eyesight – wear goggles when venting the compressed air from the compressor pneumatic system (air tank).

Risk of burn.

Pump components (head, cylinder, pressure hose) have high temperature during and shortly after compressor operation – do not touch these components!

Let the device cool before any product maintenance, service or connection/disconnection of pressurized air!

The removed grounding conductor during service must be connected back to the original position after completing the service.

- For any maintenance or repair work, the compressor may be removed from the cabinet by pulling it from the cabinet (on rolling casters as much as the cord between the compressor and the cabinet allows). See also chap. 17). The required maintenance or repair work may then be performed.
- If the cord between the compressor and the cabinet becomes disconnected (by pulling the mains plug, chap. 12.2), the compressor shall not be functional. In order to bring the compressor into function again, please follow chap. 12.2 or 17.10 or 0.

17.1. Maintenance intervals

Time interval ^{e)}	50 Hz		once a day		once a week		once a year		once every 2 years		2000 hours		4000 hours		6000 hours		8000 hours		10000 hours		12000 hours		Chap.	Spare parts set	Performed by
	60 Hz	once a day	once a week	once a year	once every 2 years	1600 hours	3200 hours	4800 hours	6400 hours	8000 hours	9600 hours														
Check of product operation		x																				17.2	-	operator	
Drain condensate from air tank ^{b)} - at high humidity		x																				17.5	-	operator	
Drain condensate from the air tank ^{b)} - at normal humidity			x																			17.5	-	operator	
Check of product function			x																			8	-	operator	
Check of pneumatic connections leakage and device inspection				x																		17.3	-	qualified technician	
Check of electrical connection				x																		17.4	-	qualified technician	
Check of fan and cooler				x																		17.9	-	qualified technician	
Replacement of filter element in dryer filter DK50 2x2V/110				x																		17.8	025200305-000	qualified technician	
Replacement of filter element in dryer filter DK50 2V/50				x																		17.8	025200304-000	qualified technician	
Check of safety valve				x																		17.6	-	qualified technician	
Replacement of pump inlet filter and pre-filter 2V ^{a)}				x																		17.7	025200126-000 025200150-000	qualified technician	

^{a)} data is in hours, if not available, data is in years

^{b)} applies to compressor without dryer

^{c)} time interval reduces by 20 % for compressor variants 60 Hz (2000h./1600h., 4000h./3200h., 6000h./4800h., 8000h./6400h., 10000h./8000h., 12000h./9600h.)

17.2. Check of product operation

- Check pump condition – the pumps must operate evenly without excessive vibrations or noise. If there is any negative result, find the cause of the given condition or call service engineers.
- Check fans operation (visually) – the fans must be operating when the fans are running. If there is a negative result, look for the cause of the given condition or call service engineers.
- Check if the power cord and pneumatic hoses are not damaged. Replace the damaged parts or call service engineers.
- Check ambient temperature – ambient temperature must be below temperature limit (40°C). Improve cooling in the room if the temperature is higher.
- Compressor with air dryer - open the plug on the condensate collecting bottle and drain the condensate.

17.3. Check of pneumatic connections leakage and device inspection

Leakage check

- Check pneumatic connections of the compressor when the compressor is operating – pressurizing.
- Use a leakage analyser or soapy water to check all connections for leaks. Tighten or release the connection where leaks are found.

Device inspection

- Check condition of the compressor pump – even operation, appropriate noise level.
- Check fans operation – fans must be running in the defined cycles of the compressor operation.
- Check function of the temperature switch (B2) – heat the temperature switch to temperature above 40°C (e.g. using a heat gun – Be careful, do not operate with high temperature on plastic components in the surrounding, it can cause deformation of plastic materials). Fan EV1, EV2 (and EV3 on compressor with dryer) starts up once the temperature reaches 40°C – the compressor must be under voltage.
- Check filters condition – filters must be undamaged and sufficiently clean.
- Check condition of the pump itself, check if there is no dirt inside the crankcase or clearance in the crankshaft.
- Replace any defective parts if there are any failures.

17.4. Check of electrical connections

Risk of electric shock.

Check electrical connections on the device disconnected from the mains.

- Check mechanical function of the main switch.
- Check if the power cable and conductors are not damaged.
- Visually check if cables are connected to the terminal box.
- Check all screw connections of the green-yellow PE grounding conductor.

17.5. Condensate drain

Risk of slipping on a wet floor in case of the collecting vessel overflow.

Switch off the compressor every time before emptying the vessel!

Open the drain valve carefully and slowly. If the drain valve is opened too much, it is accompanied by high noise level and uncontrolled jet of the accumulated condensate.

Compressors

During regular operation, it is recommended to drain condensate from the pressure tank.

- Switch off the compressor from the mains and decrease air pressure in the device to max. 1 bar, e.g. by releasing air through the connected device
- Install the hose with condensate drain valve into the prepared bottle and drain condensate from the tank by opening the drain valve (1)

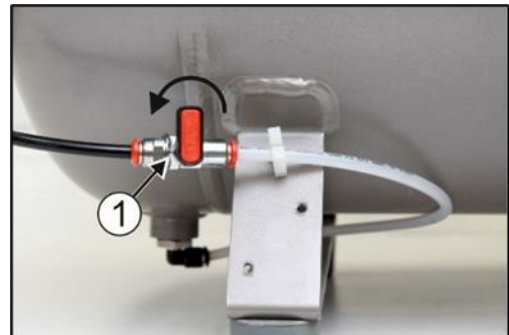
Fig. 14: Condensate drain from the pressure tank

Compressors can be equipped with Automatic condensate drain (AOK), which provides draining of condensate from the air tank without the need of operator action (see chapter accessories).

Compressors with air dryer

Condensate from compressors with air dryers is automatically drained into the condensate collection vessel.

- Monitor the level in the vessel using the 1 l or 2 l markings (depending on the volume of the vessel), and empty at least once a day.

Fig. 15: Check of condensate collection vessel

Before the following checks it is required:

- For compressor variant with cabinet – unlock the door lock and open the cabinet door.

17.6. Check of safety valve

- Turn the screw (2) on the safety valve (1) several times to the left until the safety valve releases the air.
- Let the safety valve vent for a few seconds.
- Turn the screw (2) fully to the right, the valve must be closed now.

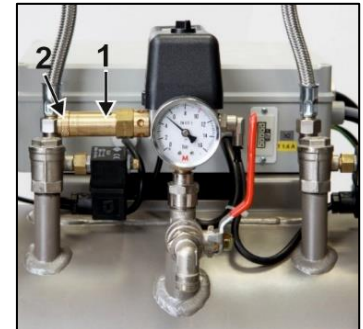
Fig. 16: Check of safety valve

Risk of pressure increase when safety valve is damaged.

Safety valve must not be used to depressurize the air tank. It could damage the safety valve function. The valve is set to the maximum allowed pressure, it is tested and marked.

Safety valve setting must not be adjusted.

Risk of injury when compressed air is vented.

Protect your eyesight when checking the safety valve – wear protective goggles.

17.7. Inlet filter and pre-filter replacement

The inlet filter (1) is located inside the compressor crankcase cover.

Inlet filter replacement:

- Pull out the rubber plug by hand (2).
- Remove the used filter (1).
- Insert a new filter and replace the rubber plug.

Pre-filter replacement:

- Pull out the pre-filter by hand (3).
- Replace the element and re-insert.

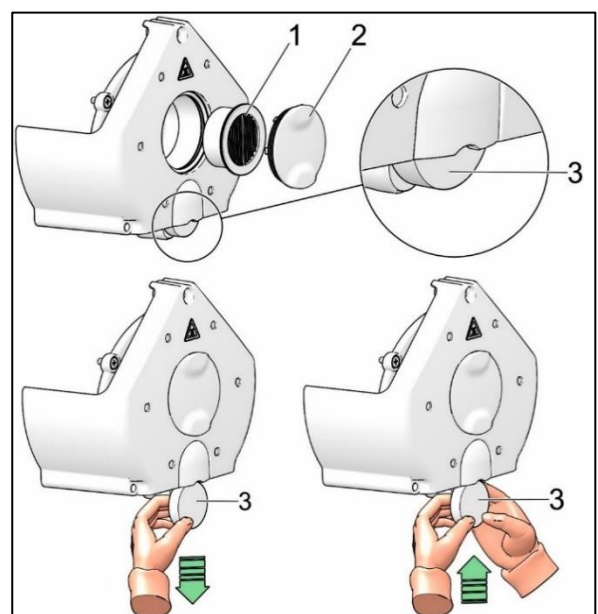
Fig. 17: Inlet filter and pre-filter

17.8. Replacement of filter element

Risk of injury during work with pneumatic components under pressure.

Before any of the following operations on the device, disconnect the device from the mains and decrease pressure in the air tank and pneumatic system to zero.

- Remove the tube (1) from the quick coupling.
- Use a wrench (2) to release the filter vessel (3) and remove
- Dismantle the filter element (4) by pulling it downwards.
- Insert a new filter element.
- Mount the filter bowl.
- Slightly tighten the filter bowl using a wrench.
- Mount the tube back on the quick coupling.

Fig. 18: Replacement of filter element

17.9. Check of fan and cooler

The entire device and especially the compressor fan, cooler fan (21) and cooler (14) must be kept clean (Fig. 2) – vacuum or blown down the cooling fins and fans with compressed air to remove any dust from the surface.

17.10. Procedure for connecting a compressor disconnected from the cabinet

Prior to any maintenance or repair work, switch off the compressor and disconnect it from the mains (pull out the mains plug).

The compressor (without cabinet) requires that the jumper is always mounted in the terminal strip for correct operation (Fig. 19, Fig. 20 pos. D). It then replaces the switch on the cabinet.

If there is no jumper in the terminal strip of the compressor (disconnected from the cabinet), the compressor will not work!

If the cord between the compressor and the cabinet becomes disconnected (by pulling the mains plug) and the compressor is removed from the cabinet, the compressor will not be functional. This is why it is necessary to first create a connection to the terminal strip with a jumper (replacing the circuit breaker function) (Fig. 19, Fig. 20 pos. A).


PROCESS:

Jumper installation (A-B-C-D process)

- Disconnect the product from the mains by pulling the plug out of the socket.
- Remove the cover on the electrical panel (on the compressor).
- The jumper is not in the terminal strip – A.
- Lift up the fuse cover – B.
- Insert the jumper into the terminal strip – C.
- Close the fuse cover – D.
- Reinstall the cover on the electrical panel.
- Connect the device to the electrical mains.
- Activate the compressor by turning on the switch on the pressure switch.


Compressor - when operating outside the cabinet, the electrical connection with an installed jumper **MUST** be created (Fig. 19, Fig. 20 Pos. D).

Fig. 19: DK50 2V/50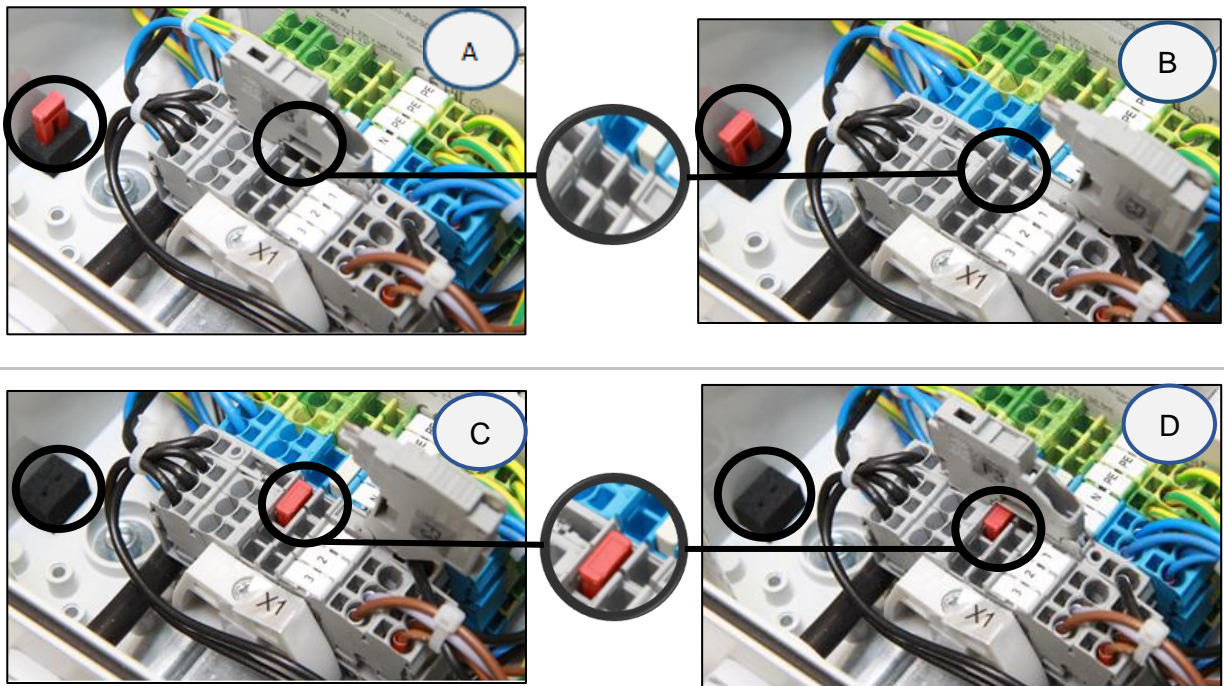

230 V

Fig. 20:DK50 2x2V/110

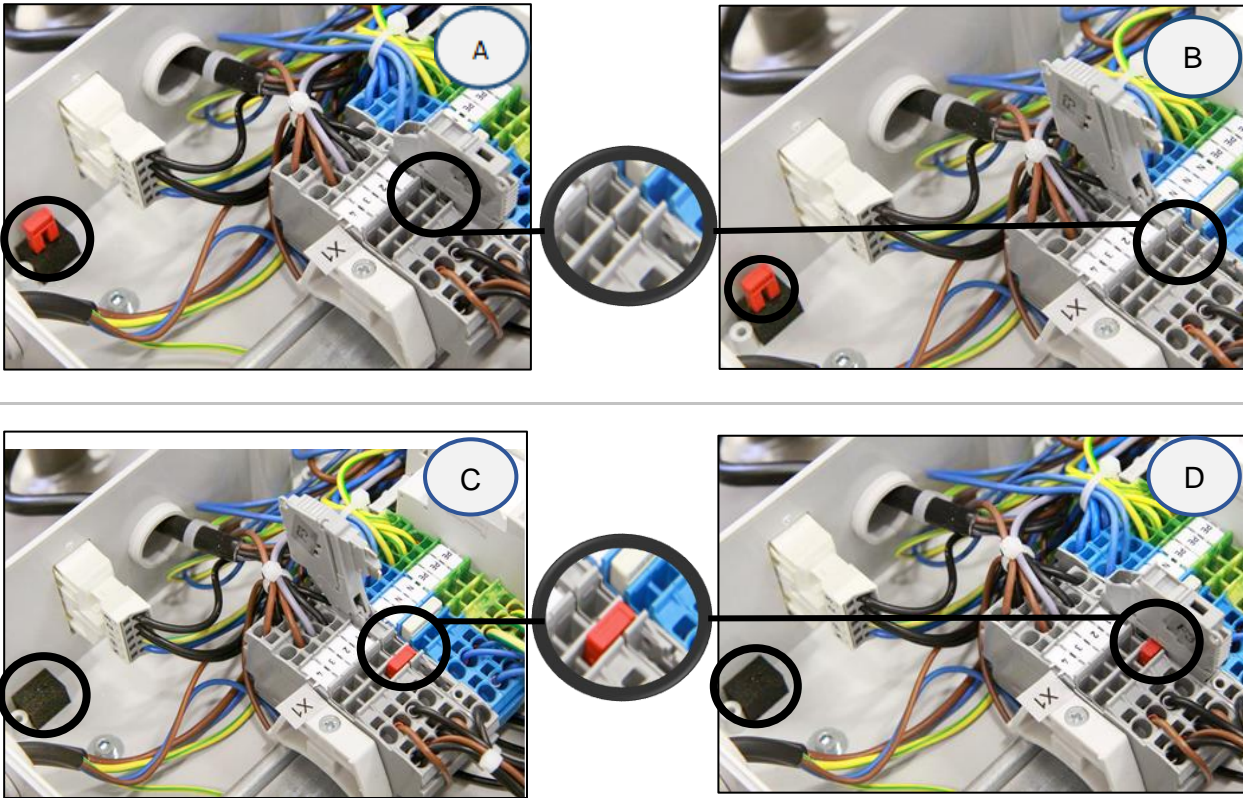230 V

17.11. Procedure for connecting a compressor to a new cabinet

Prior to any maintenance or repair work, switch off the compressor and disconnect it from the mains (pull out the mains plug).

The compressor in a cabinet requires that the jumper is not mounted in the terminal strip for correct operation (Fig. 19, Fig. 20 pos. A). The switch on the cabinet switches the entire product on and off, including the compressor.

If there is a jumper in the terminal strip of the compressor connected to a cabinet, the switch on the compressor cabinet will not work!

If we want to connect a new cabinet to a compressor that previously worked alone (a compressor without a cabinet), then it is necessary to cancel the connection in the compressor terminal strip by performing the procedure, see further. (Fig. 19, Fig. 20). (See also chap.12.1. and 12.2).

PROCESS:

Jumper removal (D-C-B-A process)

- Disconnect the product from the mains by pulling the plug out of the socket.
- Remove the cover on the electrical panel (on the compressor).
- The jumper is in the terminal strip – D

- Lift up the fuse cover – C
- Remove the jumper from the terminal strip – B
- Close the fuse cover – A
- Reinstall the cover on the electrical panel.
- Insert the compressor into the box.
- Connect the cabinet to the compressor by inserting the power cord with connector (1) into a power socket (2). (Fig. 12)
- Connect the device to the electrical mains
- Activate the compressor by turning on the switch on the pressure switch and the switch (4) on the cabinet (Fig. 13).

Compressor with cabinet - when operating in the cabinet, there **must not** be an electrical connection. (Fig. 19, Fig. 20 pos. A).

Note : **In case of using a compressor without a cabinet or in case of maintenance, a jumper which is not installed into the terminal strip should be kept** in a designated place next to the electrical panel.

17.12. Cleaning of the exterior surfaces of the product

Use neutral agents for cleaning of the external surfaces of the product.

The use of aggressive cleaning agents and comprising alcohol and chlorides may lead to the damage of the surface and the discoloration of the product.

18. LONG-TERM SHUTDOWN

If the compressor is not going to be used for long period, it is recommended to drain all condensate from the air tank and switch on the compressor for about 10 minutes, keeping the drain valve open (1) (Fig. 14). Thereafter switch off the compressor using the switch (2) on the pressure switch (1) (Fig. 13), close the drain valve and disconnect the device from the mains.

19. DISPOSAL OF DEVICE

- Disconnect the device from the mains.
- Release air pressure in the pressure tank by opening the drain valve (1) (Fig. 14).
- Dispose of the device according to the applicable local rules.
- Order a specialized organization to sort and dispose of waste.
- Product components after its operational lifetime have no negative environmental effect.

TROUBLESHOOTING**Risk of electric shock.****Before any of the following operations on the device, disconnect the device from the mains (pull out the mains plug).****Risk of injury during work with pneumatic components under pressure.****Before any of the following operations on the device, it is necessary to decrease pressure in the air tank and in the pneumatic system to zero.****Any operations concerning the troubleshooting may be performed only by a qualified technician.**

Malfunction	Possible cause	Solution
Compressor does not switch on	No voltage in the pressure switch	Check voltage in socket
		Check circuit breaker switch – switch to position switched-on „I“
		Loose conductor from terminal - repair
		Check electrical cord – replace defective cord
	Motor winding failure, damaged thermal protection	Replace motor or windings
	Capacitor failure	Replace capacitor
Compressor often switches on	Seized piston or other rotating part	Replace damaged components
	Pressure switch does not switch	Check function of pressure switch
Cabinet switch does not switch off the compressor	There is a compressor in the cabinet that previously worked separately (there is a jumper in the compressor terminal strip)	Remove the jumper from the terminal strip, see chap. 0
Compressor often switches on	Air leakage in pneumatic system	Check pneumatic system – seal loose connections
	Non-return valve (SV) leakage	Clean non-return valve, replace seals, replace non-return valve
	Large volume of condensed liquid in pressure tank	Drain condensed liquid
	Low compressor capacity	Check time of filling the air tank
Low pressure in the air tank (compressor running constantly)	High air consumption of supplied equipment	Decrease air consumption Use compressor with higher capacity
	Leakage in pneumatic system	Check pneumatic system – seal loose connections
	Low capacity of the pump	Clean / replace the pump
	Pump failure	Clean / replace the pump
	Dryer failure	Replace dryer

Prolonged operation of the compressor	Air leakage in pneumatic system	Check pneumatic system – seal loose connection
	Worn piston ring	Replace worn piston ring
	Dirty inlet filter	Replace dirty filter with new filter
	Incorrect function of solenoid valve	Repair or replace fan or coil
Compressor is noisy (knocking, metal noises)	Damaged piston bearing, piston rods, motor bearing	Replace damaged bearing
	Loose (cracked) dampening element (spring)	Replace damaged spring
Dryer does not dry (condensed water in the air) *	Defective cooler fan	Replace fan Check power source
	Damaged dryer	Replace dryer
	Defective automatic condensate drain	Clean / replace

)* Clean inside surfaces of the air tank thoroughly and remove all condensed liquid after dryer failure.

To protect the connected equipment from any damage, it is necessary to check humidity of the released air from the air tank (see chapter Technical data)

20. REPAIR SERVICE

Guarantee and post-guarantee repairs are provided by the manufacturer or organizations and technicians approved by the manufacturer.

Warning

The manufacturer reserves the right to make changes on the device, which will not significantly affect properties of the device.

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ANNEX / ANHANG / PRÍLOHA**21. INSTALLATION RECORD**

1. Product: (model) DK50 2V/50 DK50 2V/50S DK50 2V/50/M DK50 2V/50S/M DK50 2x2V/110 DK50 2x2V/110S DK50 2x2V/110/M DK50 2x2V/110S/M		2. Serial number:	
3.1. User's name:			
3.2. Address of installation:			
4. Equipment connected to the compressor:			
5. Installation / Commissioning:		6. Contents of operator training:	
Product completeness check **	Y	Description of the product and functions**	Y
	N		N
Documentation completeness check **	Y	Product operation: turning on/off, controls, control procedures, data on the display panel, warnings, operation in warning conditions**	Y
	N		N
Installation/connection to equipment **	Y	Product maintenance: maintenance intervals, maintenance procedure, service intervals, operating activities**	Y
	N		N
Functional test **	Y	Safety measures, warnings – their meaning and compliance **	Y
	N		N
Notes:			
7. Operator instructed on safety measures, operations and maintenance:			
Name:		Signature:	
Name:		Signature:	
Name:		Signature:	
8. Installation and instruction performed by: First name/Last name		Signature:	
Company:		Address:	
Phone:			
Email:		Date:	
9. Distributor:			
Company:		Address:	
Contact person:			
Phone:		Email:	

** mark with an "X" in points 5 and 6 (Y - yes /N - no). Enter any observations from points 5 and 6 into the "Notes" section

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