DK50 B

- EN User manual
- RU Руководство пользователя
- PL Instrukcja obsługi
- SK Návod na použitie

COMPRESSOR
КОМПРЕССОР
SPREŽARKA
KOMPRESOR

DK50 B

EKOM spol. s r. o.
Priemyselná 5031/18
SK-921 01 Piešťany
Slovak Republic
tel.: +421 33 7967255
fax: +421 33 7967223

www.ekom.sk
email: ekom@ekom.sk

DATE OF LAST REVISION
ДАТА ПОСЛЕДНЕЙ РЕДАКЦИИ
DATA OSTATNIEJ AKTUALIZACJI
DÁTUM POSLEDNEJ REVÍZIE

09/2024

NP-DK50 B-A-3_09-2024
112000504-000

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CONTENTS

GENERAL INFORMATION.....	6
1. CONFORMITY WITH THE REQUIREMENTS OF THE EUROPEAN UNION.....	6
2. SYMBOLS.....	6
3. DEVICE USE	7
4. GENERAL SAFETY INSTRUCTIONS.....	7
5. STORAGE AND TRANSPORT CINDITIONS.....	9
PRODUCT DESCRIPTION	10
6. VARIANTS.....	10
7. PRODUCT FUNCTION	10
TECHNICAL DATA.....	12
INSTALLATION.....	13
8. INSTALLATION CONDITIONS	13
9. INSTALATION CONDITIONS	14
10. PNEUMATIC CONNECTION	15
11. ELECTRICAL CONNECTION	15
12. COMMISSIONING	16
13. ELECTRICAL DIAGRAMS	16
OPERATION.....	17
14. SWITCHING ON THE COMPRESSOR.....	18
15. SWITCHING OFF THE COMPRESSOR.....	18
PRODUCT MAINTENANCE	19
16. PRODUCT MAINTENANCE.....	19
17. LONG-TERM SHUTDOWN.....	23
18. DISPOSAL OF DEVICE	23
TROUBLESHOOTING.....	24
19. REPAIR SERVICE	25
ANNEX	89
20. INSTALLATION RECORD	89

GENERAL INFORMATION

Carefully read this user manual before using the product and carefully store it for future reference. The user manual aids in the proper use, including installation, operation and maintenance, of the product.

The user manual corresponds to the configuration of the product and its compliance with applicable safety and technical standards at the time of its printing. The manufacturer reserves all rights for the protection of its configuration, processes and names.

The Slovak version represents the original version of the user manual. The translation of the user manual is performed in accordance with the best available knowledge. The Slovak version is to be used in the event of any uncertainties.

The user manual is original and the translation is performed with the best available knowledge.

1. CONFORMITY WITH THE REQUIREMENTS OF THE EUROPEAN UNION

This product conforms to the requirements of the European Union 2006/42/EC, 2014/29/EU, 2014/35/EU, 2014/30/EU, 2011/65/EU and is safe if used in compliance with the intended use and if all safety instructions are followed.

User manual is in compliance with requirements of Directive 2006/42/EC.

2. SYMBOLS

The following symbols and marks are used in the User manual, on the device and its packaging:

General warning

Warning - risk of electric shock

Warning - compressor is controlled automatically

Warning - hot surface

General caution

Refer to instruction manual

CE – marking

Serial number

Article number

Protecting earthing

Terminal for ground connection

Package handling label – fragile

Package handling label – this side up

Package handling label – keep dry

Package handling label – temperature limits

Package handling label – limited stacking

Package label – recyclable material

Manufacturer

3. DEVICE USE

3.1. Intended use

The compressor is used as source of clean oil-free compressed air intended to be used in industry and laboratories, where parameters and properties of the compressed air are suitable.

The compressor is exclusively intended to compress air without content of explosive or chemically unstable substances.

The compressor is intended for operation in clean and dry rooms.

3.2. Incorrect use

Contamination risk.

Air from the compressor is without additional treatment not suitable for breathing and direct contact with food

Explosion risk.

The product is not intended for operation in rooms with explosion risk.

The compressor must not be used to compress aggressive gases.

The compressor must not be operated in premises with occurrence of flammable vapors.

The compressor must not be operated in other conditions as mentioned in Technical data.

Any other use of the product beyond the intended use is considered as incorrect use. The manufacturer is not responsible for any damages or injuries as a result of incorrect use or disobedience to instructions stated in this User manual. All risks shall be solely borne by the user/operator.

4. GENERAL SAFETY INSTRUCTIONS

The product is designed and manufactured so that any risks connected with its use are minimized and the product is safe for the user and surrounding when used according to the intended use and the instructions stated below are followed.

4.1. Required qualification of the personnel

- Each user must be trained by the manufacturer or an organization authorized by the manufacturer or instructed on the device operation by other trained user.
- Installation, new settings, changes, extensions and repairs of the product may be performed by the manufacturer or an organization authorized by the manufacturer (hereinafter qualified technician).
 - Otherwise the manufacturer is not responsible for safety, reliability and correct functioning of the product.

4.2. General instructions

- When operating the compressor, all acts and local regulations valid in the place of use must be observed. The operator and user are responsible for following the applicable regulations.
- Before every use, the user must check, if the device is functioning correctly and safely. Before building the compressor in other devices, the supplier must assess, if the supplied air and construction of the device comply with the requirements of the specified intended use. Taking this into account, follow the product technical data. Assessment of conformity shall be performed by the manufacturer – supplier of the final product.

4.3. Protection from dangerous voltage and pressure

- The equipment may only be connected to a properly installed socket connected to earth (grounded).
- Before the product is plugged in, make sure that the mains voltage and frequency stated on the product are the same as the power mains.
- Check for any damage to the connected compressed air system and electrical circuits before use. Replace damaged pneumatic and electrical conductors immediately.
- Immediately disconnect the product from the mains (remove the power cord from the socket) in hazardous situations or when a technical malfunction occurs.
- Never adjust or use the safety valve to release the air pressure in the air tank.
- Never adjust or use pressure relief valves to release air pressure from the device.

4.4. Original spare parts and accessories

- Only the use of original parts guarantees the safety of operating personnel and flawless operation of the product itself. Only accessories and replacement parts specified in the technical documentation or expressly approved by the manufacturer may be used.
- The warranty does not cover damage resulting from the use of other accessories and replacement parts as specified or recommended by the manufacturer and the manufacturer has no related liability.

5. STORAGE AND TRANSPORT CINDITIONS

The compressor is shipped from the manufacturer in transport packaging. This protects the product from damage during transport.

Potential for damage to pneumatic components.

The compressor must be transported only when all air has been vented. Before moving or transporting the compressor, release all the air pressure from the tank and pressure hoses, from dryer chambers and drain condensate from the tank and from the condensate separator on the dryer.

Keep the original factory packaging in case the device needs to be returned Use the original factory packaging during transport as it provides optimum protection for the product. If it is necessary to return the product during the warranty period, the manufacturer is not liable for damages caused by improper packaging.

The compressor is shipped in a vertical position and must be secured using transport straps.

Protect the compressor from humid and dirty environments and extreme temperatures during transport and storage. Do not store near any volatile chemical substances.

If not, please dispose of the original packaging material in an environmentally-friendly way. The packaging cardboard can be recycled with old paper.

Storing or shipping the equipment in any conditions other than those specified below is prohibited.

5.1. Ambient conditions

Products may only be stored and transported in vehicles that are free of any traces of volatile chemicals under the following climactic conditions:

Temperature

–25°C to +55°C, 24 h at up to +70°C

Relative humidity

max. 90% (non-condensing)

PRODUCT DESCRIPTION

6. VARIANTS

The compressor is manufactured according to its intended application in the following variants:

- DK50 B** compressor on base for stand-alone room installations
- DK50 BS** compressor in cabinet with effective noise dampening for room installations

DK50 B

DK50 BS

7. PRODUCT FUNCTION

7.1. Compressor

The compressor (1) draws in air through a filter (8) and compresses it through a check valve (3) into an air tank (2). The connected apparatus draws the compressed air from the air tank until the pressure drops to a default preset level on the air-pressure switch (4) switching the compressor on. The compressor again compresses air into the nozzle until the maximum pressure is reached and the compressor switches off. After compressor aggregate is switched off, pressure hose shall be pressure-release solenoid valve (13). Safety valve (5) prevents the pressure in air chamber from rising above the maximal allowed value. The time of work of a compressor is recorded on an operation clock. The drain valve (7) releases the condensate from the air nozzle. Compressed, clean air free from oil traces is stored in the air tank ready for use.

7.2. Compressor cabinet

The soundproof box is compact yet allows sufficient exchange of cooling air. It can be placed in a dentist's office. The ventilator under the aggregate of a compressor provides cooling of compressor and it is in operation at the same time with an engine of the compressor. After prolonged use the temperature in the case may rise above 40°C, causing the cooling fan blower to automatically turn on. After cooling the case area to 32°C the fan blower turns off automatically.


Risk of compressor overheating.

Make sure that there are no obstacles at the cooling air inlet into the cabinet (around the bottom part of the cabinet) and at the hot air outlet on the top back side of the cabinet.

If the compressor is placed on a soft floor, e.g. carpet, create space between the base and the floor or the cabinet and the floor, e.g. underlay the footings with hard pads to ensure sufficient cooling of the compressor.

Fig. 1: DK50 B - Compressor

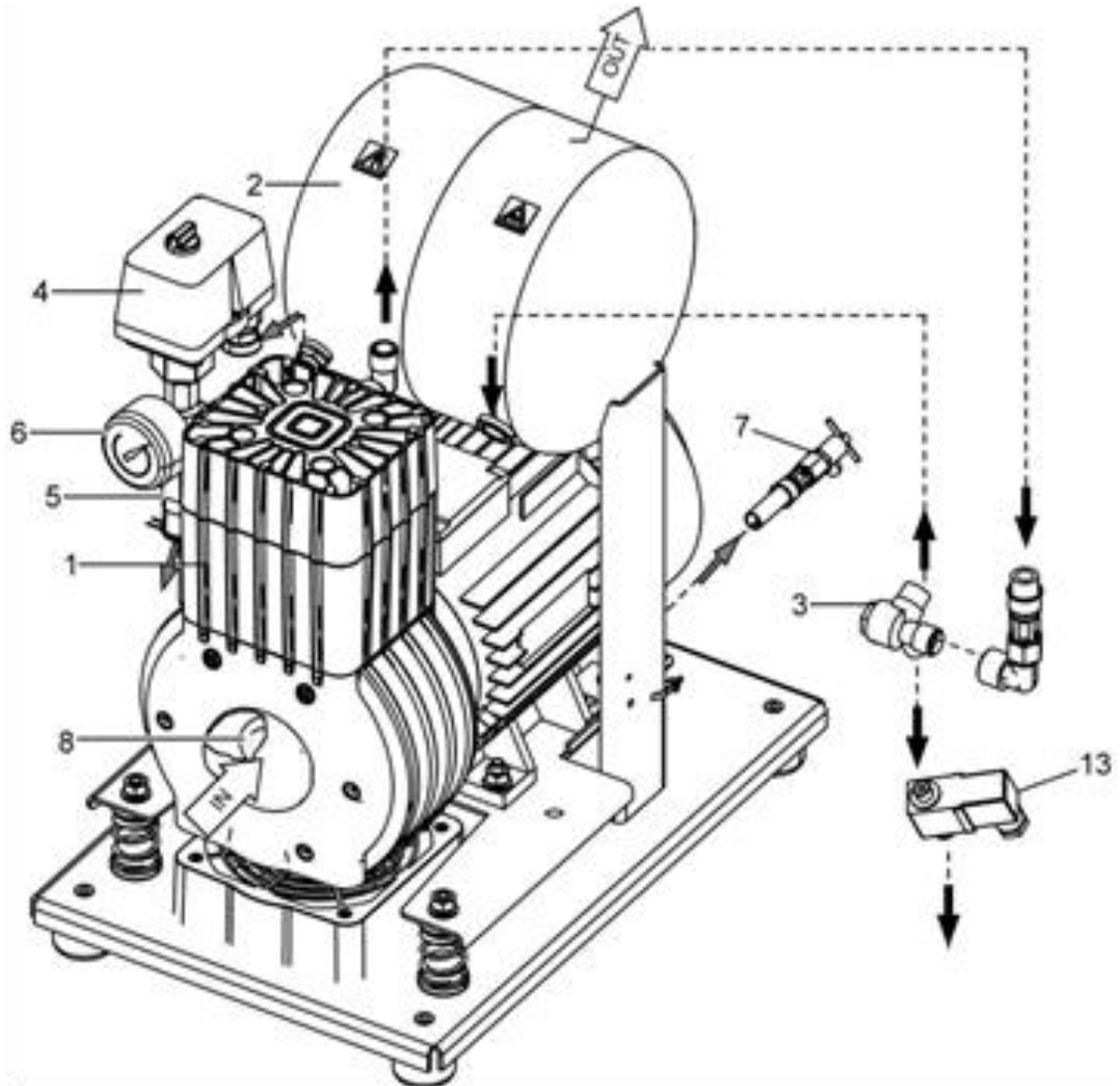Description to figure:

- 1 Air pump
- 2 Air tank
- 3 Non-return valve
- 4 Pressure switch
- 5 Safety valve
- 6 Pressure gauge
- 7 Outlet valve
- 8 Inlet filter
- 9 Solenoid valve

TECHNICAL DATA

Compressors are designed for operation in dry, ventilated and dust-free indoor rooms under the following climactic conditions:

Temperature	+5°C to +40°C
Relative humidity	max. 70%

Working pressure 9.5 – 11.5 bar		DK50 B	DK50 BS
Rated voltage, Frequency	V, Hz	230, 50/60 ^{a)}	230, 50/60 ^{a)}
Capacity at 10 bar	l/min	50/60	50/60
Working pressure	bar	9.5 – 11.5	9.5 – 11.5
Rated current	A	4.4/4.6	4.4/4.6
Motor power	kW	0.55	0.55
Air tank volume	l	4	4
Maximum operating pressure of safety valve	bar	12	12
Noise level at 5 bar (L _{pA})	dB	≤65	≤45
Operating mode	%	Intermittent S 3-50	Intermittent S 3-50
Dimensions (net) w x d x h	mm	290x430x490	380x525x575
Net weight	kg	35	47
Net weight in packaging	kg	37	49

^{a)} Specify the compressor version when ordering

FAD correction of capacity for altitude

Capacity given in the form of FAD („Free Air Delivery“) applies to the following conditions:

Altitude	0 m.n.m.	Temperature	20°C
Atmospheric pressure	101325 Pa	Relative humidity	0%

To calculate FAD compressor capacity in dependence on altitude, it is necessary to apply correction factor according to the following table:

Altitude [m.n.m.]	0 - 1500	1501 - 2500	2501 - 3500	3501 - 4500
FAD correction factor	1	0.80	0.71	0.60

INSTALLATION**Risk of incorrect installation.**

Only a qualified technician may install the compressor and place it into operation for the first time. Their duty is to train operating personnel on the use and maintenance of the equipment. An entry is made in the equipment installation record to certify installation and operator training.

8. INSTALLATION CONDITIONS

- The compressor may only be installed and operating in dry, well-ventilated and clean environments under the conditions specified in the Technical Data chapter.

**Risk of damage to the device.**

The equipment may not be operated outdoors or in otherwise wet or damp environments.

**Risk of explosion.**

Do not use the equipment in the presence of explosive gases, dust or combustible liquids.

**Burn or fire hazard! Caution! Hot surface!**

Portions of the compressor may be hot and reach hazardous temperatures during compressor operation that may harm materials or operating staff.


You may notice a “new product” odour when you first place the product into service (for a short period of time). This odour is temporary and does not impede the normal use of the product. Ensure the space is properly ventilated after installation.

- The compressor must be installed so that it is accessible at all times for operating and maintenance. Please ensure that the nameplate on the device is readily accessible.
- The compressor must stand on a flat, sufficiently stable base (be aware of the weight of the compressor, see the Technical Data chapter).
- Before connecting the compressor to equipment, the supplier must confirm that it meets all requirements for its use. Refer to the technical data of the product for this purpose. When a unit is to be built-in, classification and evaluation of compatibility must be done by the manufacturer or supplier of the product to be used.
- Any use other than that described in this manual is not covered by the guarantee, and the manufacturer is not liable for any damages that may result. The operator/user assumes all risk.

9. INSTALATION CONDITIONS


9.1. Handling and releasing the compressor

Fig. 2: Handling the compressor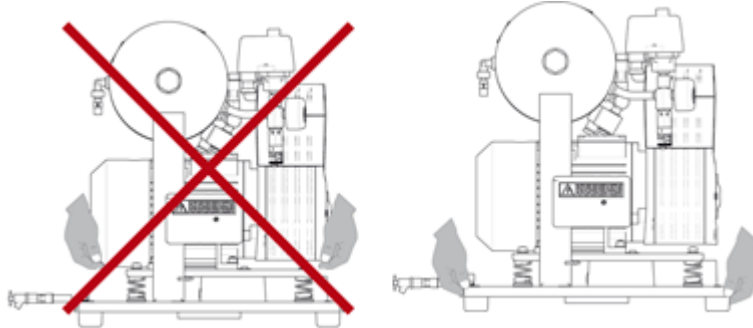

Prior to installation, ensure that the compressor is free of all transport packaging and stabilizers to avoid any risk of damage to the product

Fig. 3: Releasing the air pumps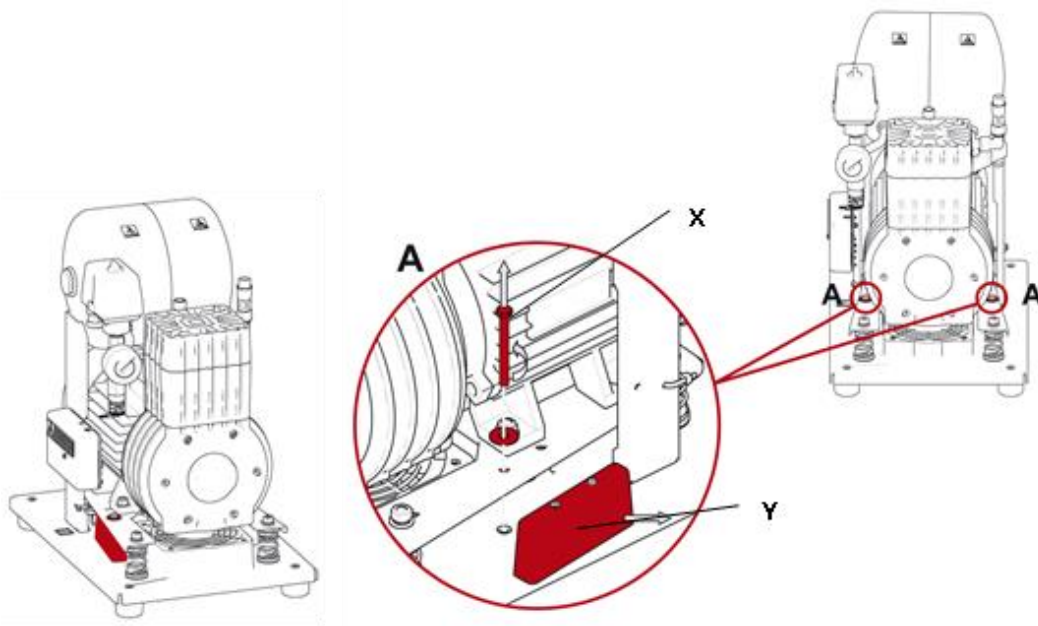

Compressor with base DK50 B (Fig. 3)

After removing all packaging material, place the product on the floor and remove stabilization parts X and Y (Detail A). Direct the output pressure hose, drain hose and power cord out the back of the compressor.

Compressor in box DK50 BS (Fig. 3)

After removing all packaging material, place the product on the floor and remove stabilization parts X and Y (Detail A). Direct the output pressure hose, drain hose and power cord out the back of the compressor. Slide the box over the compressor so that the front face of the box matches the front part of the compressor and the box is fully seated. Make sure that the pressure hose, drain hose and electric cord come out via the opening at the back of the box. Position the drain hose with its valve in the holder at the rear of the box.

10. PNEUMATIC CONNECTION

10.1. Connecting to the compressed air outlet

- The pressure hose (1) is attached to the compressed air outlet (2) on the compressor and connect the hose to the pneumatic hose or directly to the supplied equipment.

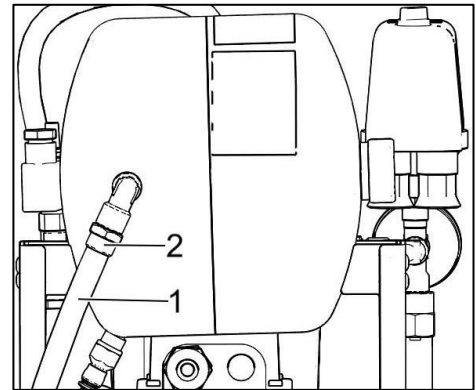
Fig. 4: Connecting to the compressed air outlet

11. ELECTRICAL CONNECTION

- The product is delivered with a cord equipped with a plug with earthing pin.
- Keep the socket easily accessible to ensure that the device can be safely disconnected from the mains.
- Connection to the power distribution cabinet must be max.16 A

Risk of electric shock.

It is necessary to follow all local electro technical regulations. The mains voltage and frequency must comply with the data stated on the device label.

Risk of fire and electric shock.

Electrical cord must not be broken.

Risk of fire and electric shock.

Electrical cable must not be in contact with hot compressor components.

11.1. Connecting a compressor not installed in a cabinet

- Insert the mains plug into a rated mains socket.
- The compressor is ready for operation.

11.2. Connecting a compressor installed in a cabinet

- Route the mains plug through the opening in the rear wall of the cabinet for cabinet-mounted compressors.

12. COMMISSIONING

- Make sure all transport stabilizers were removed. (Fig. 3)
- Check that all compressed air hose connections are correct. (Fig. 4)
- Check correct connection to the mains (see chap. 11).
- Start compressor at pressure switch (2) by turning switch (3) to position "I."


The compressor is not equipped with a backup power supply.

13. ELECTRICAL DIAGRAMS

DK50 B

1/N/PE~230 V, 50/60 Hz

ELEKTRIC OBJECT OF 1st. CAT.

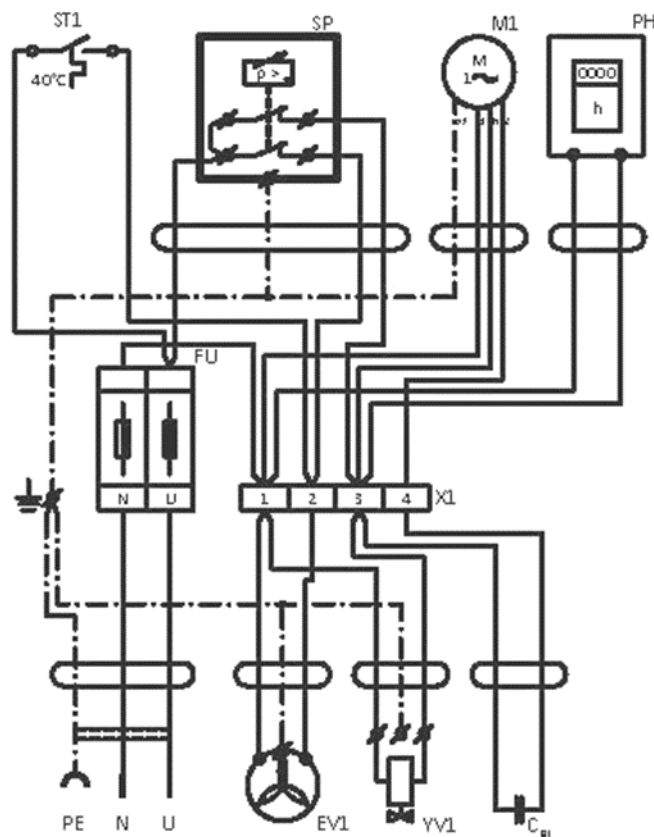Description to electrical diagrams:

M1 Motor of compressor
 EV1 Fan of compressor
 YV1 Solenoid valve
 FU Fuses

ST Temperature switch
 C1 Capacitor
 SP Pressure switch
 X1 Terminal box
 PH Hour counter

OPERATION

ONLY TRAINED PERSONNEL MAY OPERATE THE EQUIPMENT!

Risk of electric shock.

In case of emergency, disconnect the compressor from the mains (pull out the mains plug).

Burn or fire hazard.

Portions of the air pump may be hot and reach hazardous temperatures during compressor operation that may harm materials or operating staff.

Warning – compressor is controlled automatically.

Automatic start-up. When pressure in the pressure tank drops to switch-on pressure, the compressor automatically switches on. The compressor automatically switches off, when pressure in the air tank reaches the switch-off pressure.

Potential for damage to pneumatic components.

The working pressure settings for the pressure switch set by the manufacturer cannot be changed. Compressor operation at a working pressure below the switching pressure indicates high air usage (see the Troubleshooting chapter).

During prolonged operation of the compressor, the ambient temperature around the compressor may increase to over 40 °C. At this point the cooling fan automatically switches on. The fan switches off once the space is cooled to around 32 °C.

14. SWITCHING ON THE COMPRESSOR

Fig. 5: Switching the compressor

Switch on the compressor at the pressure switch (1) by turning the knob (2) to position "I." The compressor sends pressurized air to the air tank. As the compressed air is used, the pressure in the air nozzle drops to a preset level, the compressor switches on and the air nozzle fills with compressed air. After reaching the cut-off pressure the compressor turns off automatically and the cycle is repeated. Check the value of switching-on and switching-off pressure on pressure gauge (3). The values may be within a tolerance of $\pm 10\%$. Air pressure in air chamber must not exceed maximal permitted operation pressure.

Risk of damage to pneumatic parts.

The pressure switch (1) was set up by the manufacturer and any other changes to its settings may be performed only by a qualified technician trained by the manufacturer.

15. SWITCHING OFF THE COMPRESSOR

- Switching off the compressor due to service or any other reason shall be performed using the pressure switch (1) by turning the switch (2) to position „0“ and **pulling out the mains plug from the socket**. This disconnects the compressor from the mains supply.(Fig. 5)
- Vent the pressure in the air tank to zero by opening the drain valve.

PRODUCT MAINTENANCE**16. PRODUCT MAINTENANCE**

The operator should carry out device checks regularly in the intervals defined by applicable regulations. Test results must be recorded.

The equipment has been designed and manufactured to keep maintenance to a minimum. The following work must be performed to preserve the proper and reliable operation of the compressor.

Unauthorised interference hazard.

Repair work outside the framework of standard maintenance (see Chapter 16.1) may only be performed by a qualified technician (an organisation authorized by the manufacturer) or the manufacturer's customer service.

Standard maintenance work (see Chapter 16.1) may only be performed by the operator's trained personnel.

Only use manufacturer-approved replacement parts and accessories.

Danger of injury or equipment damage.

Prior to commencing compressor maintenance, it is necessary to:

- **check if it is possible to disconnect the compressor from the appliance in order to avoid any risk of injury to the person using the appliance or other material damage;**
- **turn off the compressor;**
- **disconnect it from the mains (pulling the cord out of the mains socket);**
- **vent the compressed air from the air tank.**

Venting compressed air poses an injury hazard.

Wear eye protection, i.e. goggles, when venting compressed air from the compressed air circuit (air tank) and from the dryer chamber.

Burn hazard.

Pump components (head, cylinder, pressure hose) have high temperature during and shortly after compressor operation – do not touch these components!

Let the device cool before any product maintenance, service or connection/disconnection of pressurized air!

16.1. Maintenance intervals

Time interval ^{c)}	50 Hz		60 Hz		Once a day	Once a week	Once a year	Once every 2 years	2000 hours	4000 hours	6000 hours	8000 hours	10000 hours	12000 hours	Chap.	Set of replacement parts			Performed by
	50 Hz	60 Hz	1600 hours	3200 hours	4800 hours	6400 hours	8000 hours	10000 hours	12000 hours	operator	qualified technician								
Check of product operation	x														16.2	-			operator
Drain condensate from air tank ^{b)} – at high humidity	x														16.5	-			operator
Check of product function			x												7	-			qualified technician
Check of pneumatic connections leakage and device inspection							x								16.3	-			qualified technician
Inspection of electrical connections							x								16.4	-			qualified technician
Check of cooler and fan							x								16.8	-			qualified technician
Check of safety valve							x								16.6	-			qualified technician
Replacement of pump inlet filter ^{a)}								x				x		x	16.7	025200126-000			qualified technician

^{a)} data is in hours, if not available, data is in years

^{b)} applies to compressor without dryer

^{c)} time interval reduces by 20% for compressor variants 60 Hz (2000h./1600h., 4000h./3200h., 6000h./4800h., 8000h./6400h., 10000h./8000h., 12000h./9600h.)

To ensure that the compressor works correctly, perform the following maintenance tasks at regular intervals.

16.2. Check of product operation

- Check pump condition – the pumps must operate evenly without excessive vibrations or noise. If there is any negative result, find the cause of the given condition or call service engineers.
- Check fans operation (visually) – the fans must be operating when the fans are running. If there is a negative result, look for the cause of the given condition or call service engineers.
- Check if the power cord and pneumatic hoses are not damaged. Replace the damaged parts or call service engineers.
- Check ambient temperature – ambient temperature must be below temperature limit (40°C). Improve cooling in the room if the temperature is higher.
- Compressor with air dryer - open the plug on the condensate collecting bottle and drain the condensate

16.3. Check the compressed air connections for leaks and inspect the equipment

Leak testing

- Check the compressor's compressed air lines for leaks during operation – pressure supplied by the compressor.
- Use a leak analyser or soapy water to check all joints and connections for leaks. Tighten or reseal the connection where leaks are found.

Inspecting the equipment

- Check condition of the compressor pump – even operation, appropriate noise level.
- Check fans operation – fans must be running in the defined cycles of the compressor operation.
- Check function of the temperature switch (B2) – heat the temperature switch to temperature above 40°C (e.g. using a heat gun – Be careful, do not operate with high temperature on plastic components in the surrounding, it can cause deformation of plastic materials). Fan EV1 (and EV2 on compressor with dryer) starts up once the temperature reaches 40°C – the compressor must be under voltage.
- Check filters condition – filters must be undamaged and sufficiently clean.
- Check condition of the pump itself, check if there is no dirt inside the crankcase or clearance in the crankshaft.
- Replace any defective parts if there are any failures.

16.4. Inspection of electrical connections

Risk of electric shock.

Inspect the product's electrical connections when the mains are disconnected.

- Check mechanical function of the main switch.
- Check if the power cable and conductors are not damaged.
- Visually check if cables are connected to the terminal box.
- Check all screw connections of the green-yellow PE grounding conductor.

16.5. Condensate drain

Open the drain valve carefully and slowly. If the drain valve is opened too much, it is accompanied by high noise level and uncontrolled jet of the accumulated condensate.

A wet floor resulting from overflow from the vessel poses a slip hazard.

During regular operation, it is recommended to drain condensate from the pressure tank.

- Switch off the compressor from the mains and decrease air pressure in the device to max. 1 bar, e.g. by releasing air through the connected device.
- Install the hose with condensate drain valve into the prepared bottle and drain condensate from the tank by opening the drain valve (1).

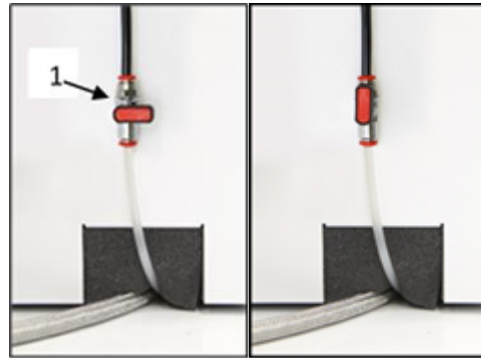
Fig. 6: Check of the condensate collection vessel

For compressor DK50 BS the case must be removed before beginning the following procedures.

16.6. Check of safety valve

- Turn the screw (2) on the safety valve (1) several times to the left until the safety valve releases the air.
- Let the safety valve vent for a few seconds.
- Turn the screw (2) fully to the right, the valve must be closed now.

Fig. 7: Check of safety valve

Damage to the safety valve could cause pressure to rise to hazardous levels.

Never use the safety valve to release the air pressure in the air tank. This could damage the safety valve. The valve is set to the maximum permitted pressure by the manufacturer.

Never adjust a safety valve.

Venting compressed air poses an injury hazard.

Wear safety glasses when inspecting a safety valve.

16.7. Inlet filter replacement

The inlet filter (1) is located inside the compressor crankcase cover.

Inlet filter replacement:

- Pull out the rubber plug (2) by hand.
- Remove the dirty inlet filter (1).
- Insert a new filter and mount the rubber plug.

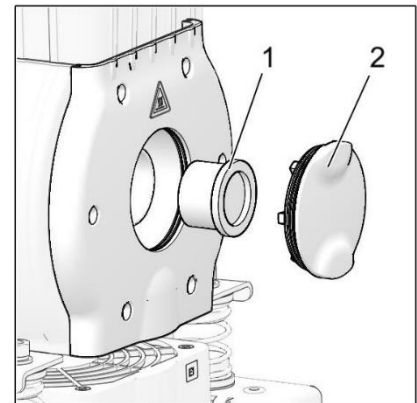
Fig. 8: Inlet filter replacement

16.8. Inspecting the cooled and fan

The equipment, in particular the compressor fan, cooler fan, and the cooler, must be kept clean to ensure efficient drying. Remove dust from the surface of the cooling fins and fans by vacuuming or blowing down with compressed air.

16.9. Cleaning and disinfection of the exterior surfaces of the product

Clean and disinfect the exterior surfaces with neutral cleaning products.

Use of aggressive detergents and disinfectants containing alcohol and chlorides can lead to surface damage and discolouration.

17. LONG-TERM SHUTDOWN

If the compressor will not be used for a prolonged time period, drain any condensate from the air tank. Then turn on the compressor for 10 minutes, keeping the drain valve open (1) (Fig. 6). Switch off the compressor by switch (2) at pressure switch (1) (Fig. 5), close the drain valve and disconnect the appliance from the mains.

18. DISPOSAL OF DEVICE

- Disconnect the equipment from the mains.
- Release the air pressure in the pressure tank by opening the drain valve (1) (Fig. 6).
- The components of the product are non-toxic.
- Dispose of the appliance following all environmental regulations.

TROUBLESHOOTING**Risk of electric shock.****Before interfering with the equipment, first disconnect it from the mains (remove the power socket).****Working with pressurised pneumatic components poses a risk of injury.****Before interfering with the equipment, vent the air tank and the compressed air system to zero pressure.****Troubleshooting may only be performed by a qualified service technician.**

Malfunction	Possible cause	Solution
Compressor does not switch on	No voltage in the pressure switch	Check voltage in socket
		Check circuit breaker switch – switch to position switched-on „I“
		Loose conductor from terminal - repair
		Check electrical cord – replace defective cord
	Motor winding failure, damaged thermal protection	Replace motor or windings
	Capacitor failure	Replace capacitor
Compressor often switches on	Seized piston or other rotating part	Replace damaged components
	Pressure switch does not switch	Check function of pressure switch
	Air leakage in pneumatic system	Check pneumatic system – seal loose connections
	Non-return valve (SV) leakage	Clean non-return valve, replace seals, replace non-return valve
Low pressure in the air tank (compressor running constantly)	Large volume of condensed liquid in pressure tank	Drain condensed liquid
	Low compressor capacity	Check time of filling the air tank
	High air consumption of supplied equipment	Decrease air consumption Use compressor with higher capacity
	Leakage in pneumatic system	Check pneumatic system – seal loose connections
	Low capacity of the pump	Clean / replace the pump
Prolonged operation of the compressor	Pump failure	Clean / replace the pump
	Dryer failure	Replace dryer
	Air leakage in pneumatic system	Check pneumatic system – seal loose connection
	Worn piston ring	Replace worn piston ring
	Dirty inlet filter	Replace dirty filter with new filter
	Incorrect function of solenoid valve	Repair or replace fan or coil

Compressor is noisy (knocking, metal noises)	Damaged piston bearing, piston rods, motor bearing	Replace damaged bearing
	Loose (cracked) dampening element (spring)	Replace damaged spring

19. REPAIR SERVICE

Warranty and post-warranty repairs must be done by the manufacturer, its authorized representative, or service personnel approved by the supplier.

Attention.

The manufacturer reserves the right to make changes to the equipment without notice. Any changes made will not affect the functional properties of the equipment.

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20. INSTALLATION RECORD

1. Product: (model) DK50 B DK50 BS		2. Serial number:	
3.1. User's name:			
3.2. Address of installation:			
4. Equipment connected to the compressor:			
5. Installation / Commissioning:		6. Contents of operator training:	
Product completeness check **	Y	Description of the product and functions**	Y
	N		N
Documentation completeness check **	Y	Product operation: turning on/off, controls, control procedures, data on the display panel, alarms, operation in alarm conditions**	Y
	N		N
Installation/connection to equipment **	Y	Product maintenance: maintenance intervals, maintenance procedure, service intervals, operating activities**	Y
	N		N
Functional test **	Y	Safety measures, warnings – their meaning and compliance **	Y
	N		N
Notes:			
7. Operator instructed on safety measures, operations and maintenance:			
Name:		Signature:	
Name:		Signature:	
Name:		Signature:	
8. Installation and instruction performed by: First name/Last name		Signature:	
Company:		Address:	
Phone:			
Email:		Date:	
9. Distributor:			
Company:		Address:	
Contact person:			
Phone:		Email:	

** mark with an "X" in points 5 and 6 (Y - yes /N - no). Enter any observations from points 5 and 6 into the "Notes" section

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