DK50 PLUS/M DK50 2V/M

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OBSAH 91

CONTENTS

GENERAL INFORMATION.....	6
1. CONFORMITY WITH THE REQUIREMENTS OF THE EUROPEAN UNION.....	6
2. SYMBOLS.....	6
3. DEVICE USE	7
4. GENERAL SAFETY INSTRUCTION	8
5. STORAGE AND TRANSPORT	9
PRODUCT DESCRIPTION	10
6. VARIANTS	10
7. ACCESORIES.....	10
8. PRODUCT FUNCTION	12
TECHNICAL DATA.....	16
INSTALLATION.....	21
9. INSTALLATION CONDITIONS	21
10. COMPRESSOR ASSEMBLY	21
11. PNEUMATIC CONNECTION	25
12. ELECTRICAL CONNECTION	27
13. COMMISSIONING	28
14. PNEUMATIC AND ELECTRICAL DIAGRAMS	29
OPERATION.....	33
15. SWITCHING ON THE COMPRESSOR.....	34
16. SWITCHING OFF THE COMPRESSOR.....	34
PRODUCT MAINTENANCE	35
17. PRODUCT MAINTENANCE.....	35
18. LONG-TERM SHUTDOWN.....	44
19. DISPOSAL OF DEVICE	44
TROUBLESHOOTING.....	45
20. REPAIR SERVICE	46
ANNEX	134
21. INSTALLATION RECORD	134

GENERAL INFORMATION

Read the User manual carefully and keep it before use of the product. The User manual provides information on correct use – installation, operation and maintenance of the product.

The User manual corresponds with the design of the product and condition according to the applicable safety and technical standards at the time of printing. The manufacturer reserves all rights concerning the protection for the stated connections, procedures and names.

Slovak version is original of the User manual. The User manual has been translated in accordance with the best available knowledge. The Slovak version is to be used in the event of any uncertainties.

This user manual is the original instructions. Translation is performed in accordance with the best available knowledge.

1. CONFORMITY WITH THE REQUIREMENTS OF THE EUROPEAN UNION

This product conforms to the requirements of the European Union 2006/42/EC, 2014/29/EU, 2014/35/EU, 2014/30/EU, 2011/65/EU and is safe if used in compliance with the intended use and if all safety instructions are followed.

User manual is in compliance with requirements of Directive 2006/42/EC.

2. SYMBOLS

The following symbols and marks are used in the User manual, on the device and its packaging:

General warning

Warning – risk of electric shock

Warning - compressor is controlled automatically

Warning – hot surface

General caution

Refer to instruction manual

CE – marking

Serial number

Article number

Protecting earthing

Terminal for ground connection

Package handling label – fragile

Package handling label – this side up

Package handling label – keep dry

Package handling label – temperature limits

Package handling label – limited stacking

Package label – recyclable material

Manufacturer

3. DEVICE USE

3.1. Intended use

The compressor is used as source of clean oil-free compressed air intended to be used in industry and laboratories, where parameters and properties of the compressed air are suitable.

The compressor is exclusively intended to compress air without content of explosive or chemically unstable substances.

The compressor is intended for operation in clean and dry rooms

3.2. Incorrect use

Contamination risk.

Air from the compressor is without additional treatment not suitable for breathing and direct contact with food

Explosion risk.

The product is not intended for operation in rooms with explosion risk.

The compressor must not be used to compress aggressive gases.

The compressor must not be operated in premises with occurrence of flammable vapors.

The compressor must not be operated in other conditions as mentioned in Technical data.

Any other use of the product beyond the intended use is considered as incorrect use. The manufacturer is not responsible for any damages or injuries as a result of incorrect use or disobedience to instructions stated in this User manual. All risks shall be solely borne by the user/operator.

4. GENERAL SAFETY INSTRUCTION

The product is designed and manufactured so that any risks connected with its use are minimized and the product is safe for the user and surrounding when used according to the intended use and the instructions stated below are followed.

4.1. Required qualification of the personnel

- Each user must be trained by the manufacturer or an organization authorized by the manufacturer or instructed on the device operation by other trained user.
- Installation, new settings, changes, extensions and repairs of the product may be performed by the manufacturer or an organization authorized by the manufacturer (hereinafter qualified technician).
- Otherwise the manufacturer is not responsible for safety, reliability and correct functioning of the product.

4.2. General instruction

- When operating the compressor, all acts and local regulations valid in the place of use must be observed. The operator and user are responsible for following the applicable regulations.
- Before every use, the user must check, if the device is functioning correctly and safely. Before building the compressor in other devices, the supplier must assess, if the supplied air and construction of the device comply with the requirements of the specified intended use. Taking this into account, follow the product technical data. Assessment of conformity shall be performed by the manufacturer – supplier of the final product.

4.3. Protection from dangerous voltage and pressure

- The device can be connected only to a properly mounted socket with protective earthing.
- Before connecting the product, check if mains voltage and mains frequency stated on the product are in compliance with the values of the mains.
- Before putting the product into operation, check eventual damages to connected pneumatic hoses and electrical cables. Replace damaged pneumatic hoses and electrical cables immediately.
- Immediately disconnect the product from the mains (pull out the mains plug) in hazardous situations or technical disorders.
- Safety valve settings must not be changed and used to depressurize the air tank.

4.4. Original spare parts and accessories

- Safety of operating staff and failure-free operation of the product are guaranteed only when original spare parts are used. Only accessories and spare parts stated in the technical documentation or expressly approved by the manufacturer may be used.
- The guarantee does not apply to damages arising from use of accessories and spare parts other than prescribed or recommended by the manufacturer and the manufacture is not responsible for them.

5. STORAGE AND TRANSPORT

The manufacturer ships the compressor in a transport packaging. This protects the device from damage during transport.

Risk of damage to pneumatic parts.

The compressor may be transported only depressurized. Vent air pressure from the pressure tank and pressure hoses and drain condensate from the air tank before transporting the compressor.

Original packaging must be kept for eventual return of the device. If possible, always use the original compressor packaging for optimal protection of the product. If it is necessary to return the product within the guarantee period, the manufacturer does not guarantee for damages caused by incorrect packaging of the product.

Transport the compressor in an upright position, always secured by transport fixing elements.

Protect the compressor from high humidity, dirt and extreme temperatures during transport and storage. Do not store together with volatile chemical substances.

If it is not possible to keep the original packaging, dispose of the packaging in an environmentally friendly way. Transport cardboard can be recycled with old paper.

It is not permitted to store and transport the device outside the defined conditions, see below.

5.1. Ambient conditions

Products can be stored in rooms and means of transport that are free from any traces of volatile chemical substances under the following climatic conditions:

Temperature	from -25°C o $+55^{\circ}\text{C}$, in 24h to $+70^{\circ}\text{C}$
Relative humidity	max. 90% (without condensation/ non-condensing)

PRODUCT DESCRIPTION

6. VARIANTS

The compressor is manufactured according to its intended application in the following variants:

DK50 PLUS	compressor on base for stand-alone room installations
DK50 PLUS/M	compressor on base with air dryer
DK50 PLUS S	compressor in cabinet with effective noise dampening for room installations
DK50 PLUS S/M	compressor in cabinet with air dryer
DK50 2V	compressor on base for stand-alone room installations
DK50 2V/M	compressor on base with air dryer
DK50 2V S	compressor in cabinet with effective noise dampening for room installations
DK50 2V S/M	compressor in cabinet with air dryer

DK50 PLUS

DK50 2V

DK50 PLUS S
DK50 2V S

7. ACCESORIES

Accessories are not included in a standard order, they must be ordered separately.

Automatic condensate drain

The automatic condensate drain (AOK) automatically drains condensate from the compressor's air tank based on a pre-set time interval. The condensate drain (AOK) is a suitable accessory for compressor models without dryers.

Type	Use	Article number
AOK 11	DK50 PLUS, DK50 2V	447000001-047

Set of compressed air outlet filters

The compressor may be equipped with a set of filters if specified. The filter set may be equipped with an air pressure regulator.

Where a different level of air filtration is required, this requirement must be agreed upon with the supplier and specified in the order.

PRODUCT DESCRIPTION

Type	Use	Level of filtration (µm)	Article number
FS 30F	DK50 PLUS	1	604014119-014
FS 30M	Dk50 PLUS/M	1+0.1	604014119-015
FS 30S	DK50 2V	1+0.01	604014119-017
FS 30A	DK50 2V/M	1+0.01+A**	604014119-018

**) A – submicrofilter with activated carbon

Filters set regulator assembly

The compressor may be equipped with a pressure regulator of the compressed air outlet if specified. The regulator must be selected according to the application to the filter set, or separately. The regulator shall ensure constant pressure at the outlet.

Type	Use	Article number
Regulator complete (to the filter assembly)	DK50 PLUS, DK50 PLUS/M	604014130-000
REG11 (separate)	DK50 2V, DK50 2V/M	447000001-077

Filter set brackets

A suitable bracket must be ordered for every filter set.

Type	Use	Article number
Compressor-mounted bracket	DK50 PLUS, DK50 PLUS/M DK50 2V, DK50 2V/M	603014134-000

Condensation and filtration unit

The compressor can be equipped with a condensation and filtration unit (KJF-1 or KJFR-1). The unit is used to cool the compressed air from the air tank in the cooler, to capture condensate in the filter, to drain condensate automatically from the pneumatic system and to filter the compressed air.

Type	Use	Level of filtration /µm/	Pressure regulator	Article number
KJF-1	DK50 PLUS	5	no	450001011-001
KJFR-1			yes	450001011-002
KJF-2	DK50 2V		no	450001021-001

Equipotential bonding socket

The socket provides protective bonding.

Type	Use	Name	Article number
POAG-KBT6-EC	DK50 PLUS S	Mains plug connector	033400075-000
	DK50 PLUS S/M		
FLEXI-S/POAG-HK6	DK50 2VS, DK50 2VS/M	Conductor (1 m)	034110083-000

8. PRODUCT FUNCTION

8.1. Compressor

Fig. 1: Compressor air pump (1) sucks the atmospheric air in through the inlet filter (8) and compresses it through the non-return valve (3) into the air tank (2) from which the device draws in the compressed air. If the pressure in the air tank drops to the switch-on pressure, the pressure switch (4) switches the compressor on and the compressor compresses the air into the air tank until it reaches the switch-off pressure, when the compressor switches off. The pressure hose is vented through the solenoid relief valve (13) once the compressor pump is switched off. The safety valve (5) prevents the pressure in the air tank from rising above the maximum allowed value. The drain valve (7) drains the condensate from the air tank. Compressed, oil-free filtered air is stored in the air tank ready for use.

Condensate must be drained from the air tank at regular intervals (see chapter 17.1).

8.2. Compressor with membrane dryer

Fig. 2: Compressor air pump (1) draws in air through an inlet filter (8) and compresses it, feeding it to the cooler (14) through the filter (15) and the check valve (3) into the dryer (9), and feeds the clean and dry air into the air tank (2). A part of the air is released from the dryer with the captured moisture, which is manifested as a light air stream along the body of the dryer (9). Condensate from the filter is automatically drained at regular intervals into the collecting bottle via the condensate drain solenoid valve (16). The dryer ensures continuous drying of the compressed air. The drain valve (7) drains condensate from the air tank when drying performance is checked. Compressed, oil-free filtered air is stored in the air tank ready for use.

For the membrane dryer to work as intended, the dryer is connected directly to the air tank without the use of a check valve. As the membrane dryer contains selective membrane fibres, there is a small amount of air leakage through them (about 0.5 bar/30 min).

As a result, there is a gradual, slow drop in pressure even when there is no demand for compressed air (from the air tank), however, this is not a malfunction.

The pressure tank is dry and does not need to be drained.

8.3. Compressor cabinet

Fig. 3: The soundproof cabinet provides compact covering of the compressor, dampens noise efficiently, providing sufficient exchange of cooling air. The fan under the compressor pump provides cooling of the compressor. It is in operation at the same time with the compressor motor or after switching on the temperature switch at temperature above 40°C. After cooling the cabinet area approximately under 32°C, the fan turns off automatically.

Door of the casing with right opening may be changed to left opening (see Chapter 10.2).

Risk of compressor overheating.


Make sure that there are no obstacles at the cooling air inlet into the cabinet (around the bottom part of the cabinet) and at the hot air outlet on the top back side of the cabinet.

If the compressor is placed on a soft floor, e.g. carpet, create space between the base and the floor or the cabinet and the floor, e.g. underlay the footings with hard pads to ensure sufficient cooling of the compressor.

The 8-10 bar compressor model is equipped with an hour counter.

Fig. 1: DK50 PLUS – Compressor


Description to Figures 1 - 3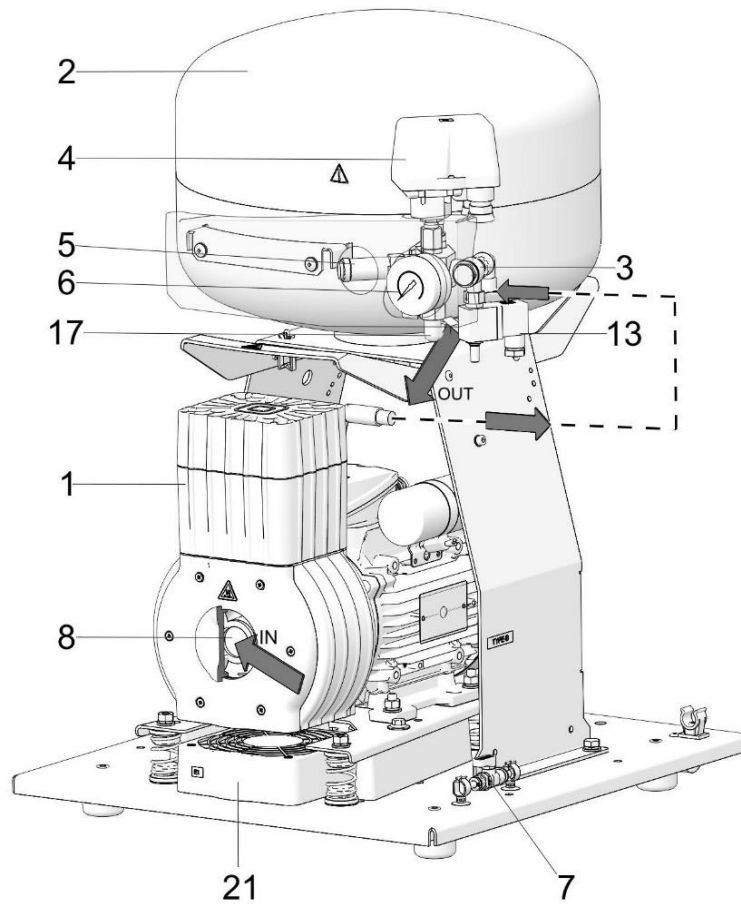

- 1 Air pump
- 2 Air tank
- 3 Non-return valve
- 4 Pressure switch
- 5 Safety valve
- 6 Pressure gauge
- 7 Drain valve
- 8 Inlet filter
- 9 Dryer
- 10 –
- 11 –
- 12 –
- 13 Solenoid valve
- 14 Dryer cooler
- 15 Filter
- 16 Solenoid valve of condensate drain
- 17 Compressed air outlet
- 18 Circuit breaker switch
- 19 Condensate collection vessel
- 20 Magnetic holder
- 21 Fan
- 22 Cabinet fan
- 23 Door pin
- 24 Rectifying screw
- 25 Lock
- 26 Supporting bracket
- 27 Wall stop
- 28 Switch
- 29 Pressure gauge
- 30 Door hinge
- 31 Handle
- 32 Casters
- 33 Pressure gauge hose
- 34 Socket
- 35 Cabinet connector with cord
- 36 *Hour counter


* Hour counter installed only for the 8-10bar compressor model.

Fig. 2: DK50 2V/M – Compressor with dryer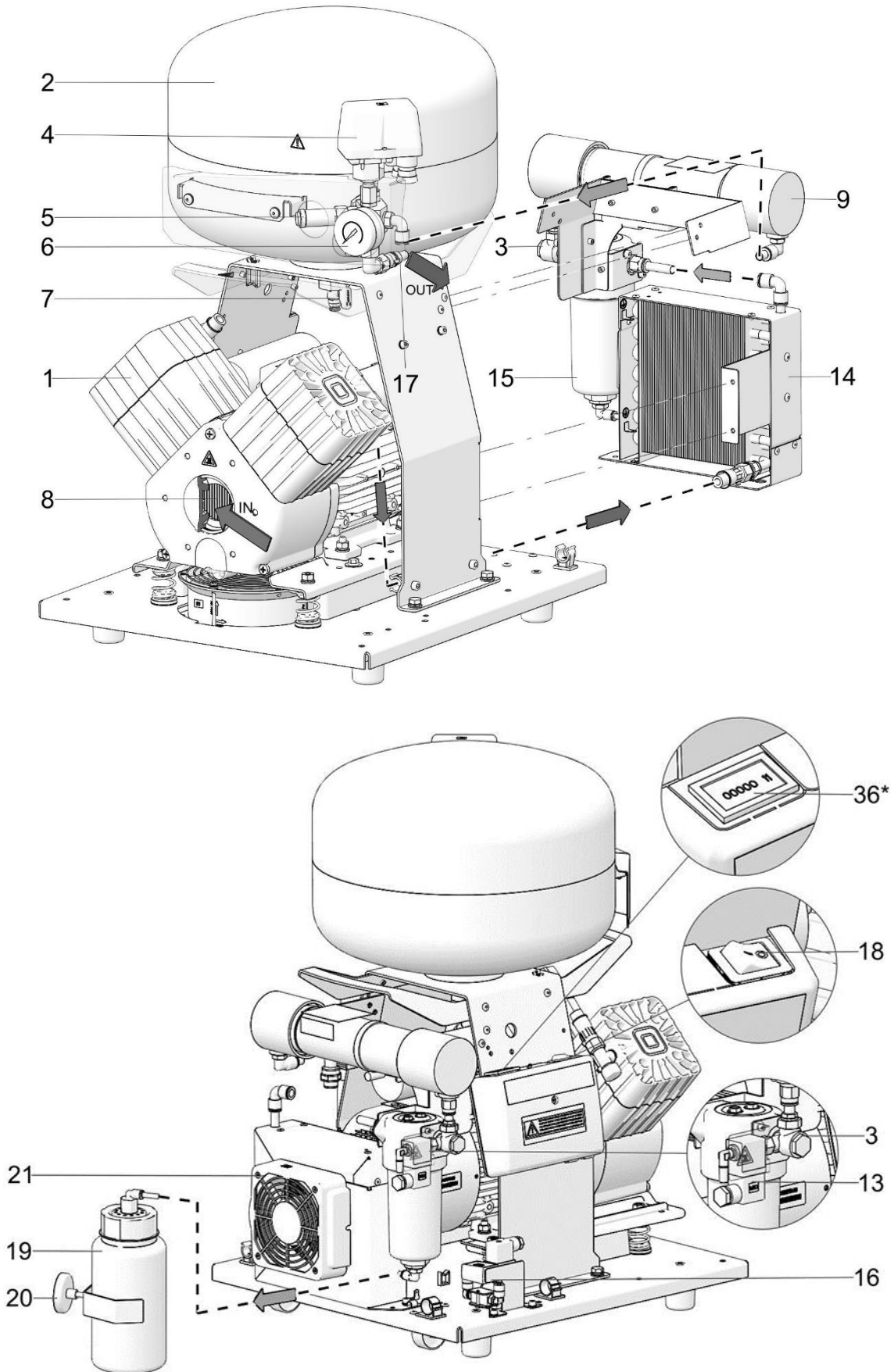

* Hour counter installed only for the 8-10bar compressor model.

Fig. 3: – Cabinet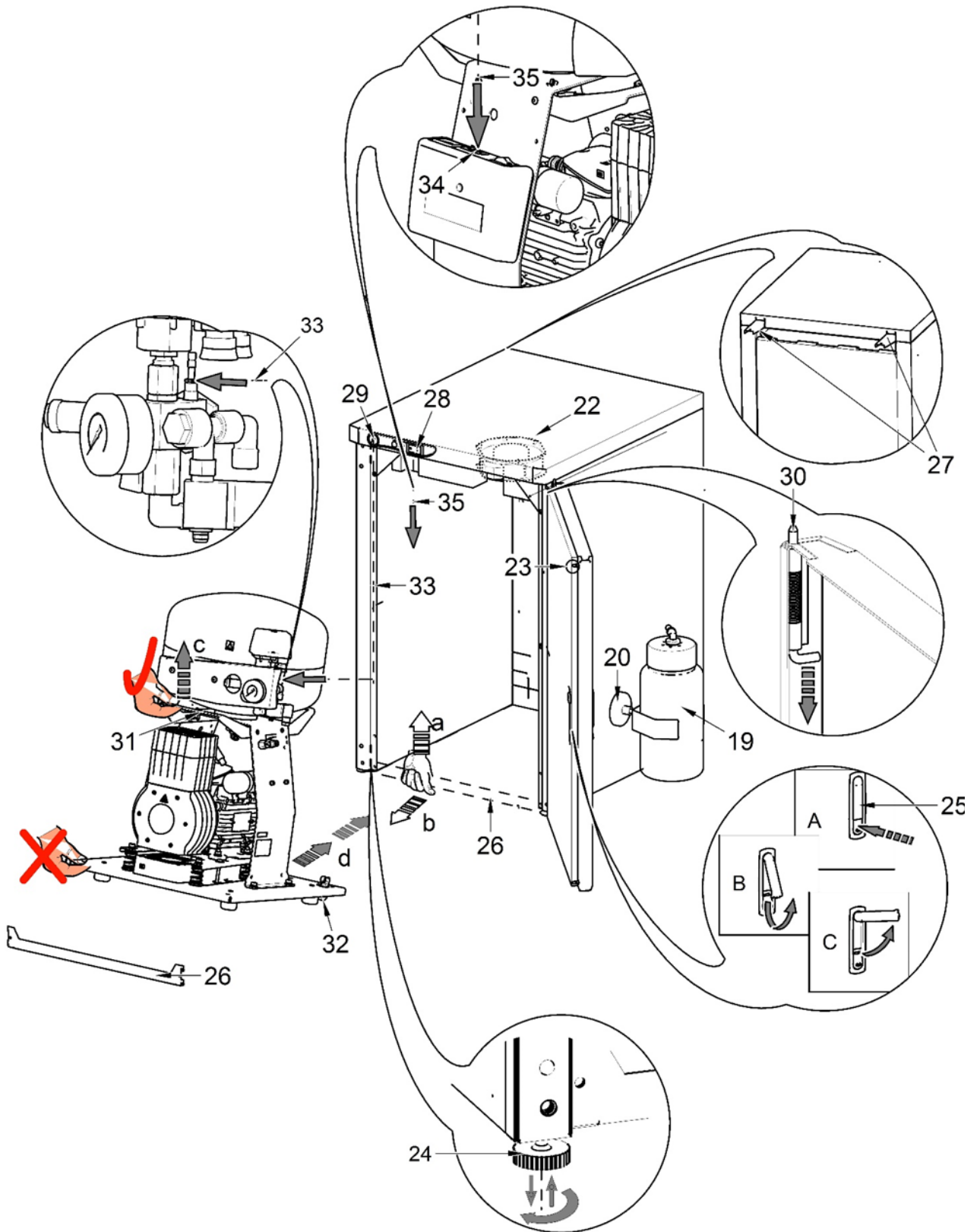

TECHNICAL DATA

Compressors are designed to operate in dry, ventilated and indoor dust-free rooms with the following climatic conditions:

Temperature from +5°C to +40°C



Relative humidity max. 70%

Working pressure 6 – 8 bar		DK50 PLUS		DK50PLUS S		DK50 PLUS/M		DK50 PLUS S/M	
Nominal voltage Frequency ^{a)}	V, Hz	230, 50/60	115,6 0	230, 50/60	115, 60	230, 50/60	115,6 0	230, 50/60	115, 60
Capacity at 6 bar (FAD)	l/min	70/80	80	70/80	80	60/70	70	60/70	70
Working pressure ^{b)}	bar	6.0 – 8.0		6.0 – 8.0		6.0 – 8.0		6.0 – 8.0	
Max. current	A	4.8/5.2	9.8	4.9/5.3	10.0	4.8/5.2	10.0	4.9/5.3	10.2
Motor power	kW	0.55		0.55		0.55		0.55	
Air tank volume	l	25		25		25		25	
Air quality - filtration	µm	-		-		0.1		0.1	
Maximum operating pressure of safety	bar	11.0		11.0		11.0		11.0	
Noise level at 5 bar (L _{pA})	dB	≤66/68	≤68	≤47/50	≤50	≤66/68	≤68	≤47/ 50	≤50
Operating mode	%	S1-100		S1-100		S1-100		S1-100	
PDP drying at 7 bar	°C	-		-		≤+3		≤+3	
Time to fill air tank from 0 to 7 bar	s	149/126	126	149/126	126	198/15 9	159	198/ 159	159
Dimensions (net) w x d x h	mm	460x500x708		560x675x877		460x565x708		560x675x877	
Net weight	kg	48 ^{c)}		87 ^{c)}		53		92	

^{a)} State the compressor variant in the order

^{b)} For other range of pressure consult with the supplier

^{c)} Weight value is only informative data and applies only to a product without any accessories

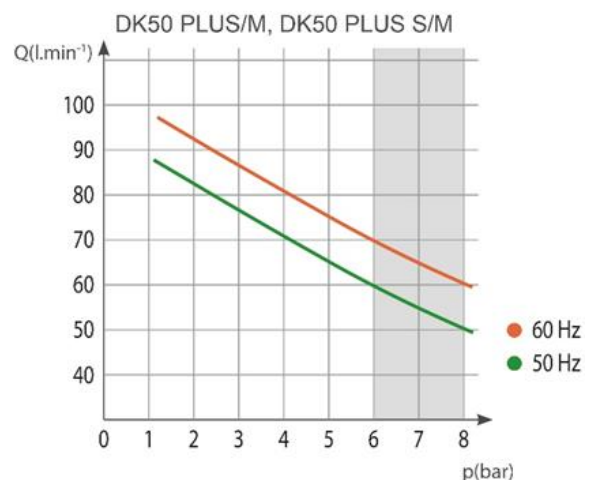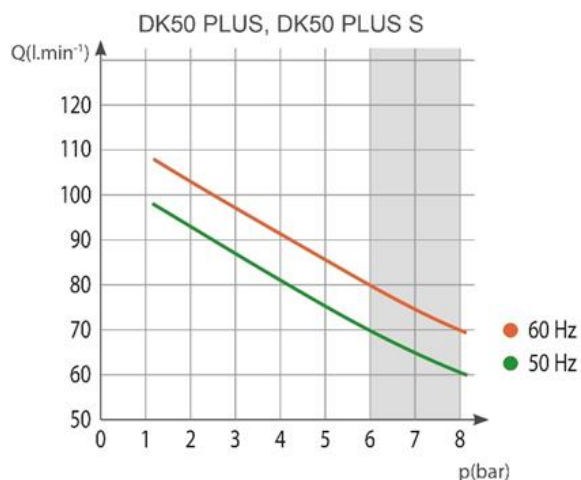Dependence of compressor capacity on working pressure

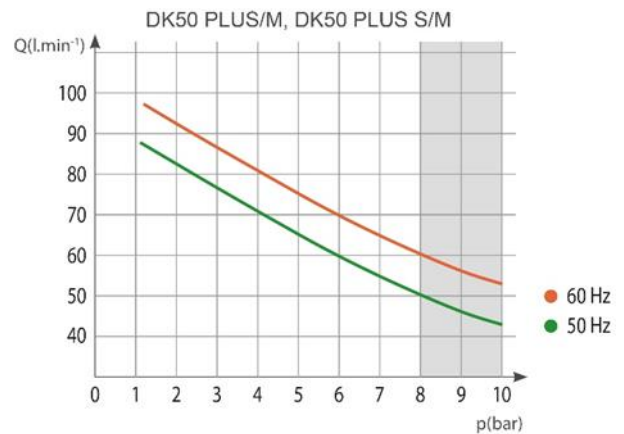
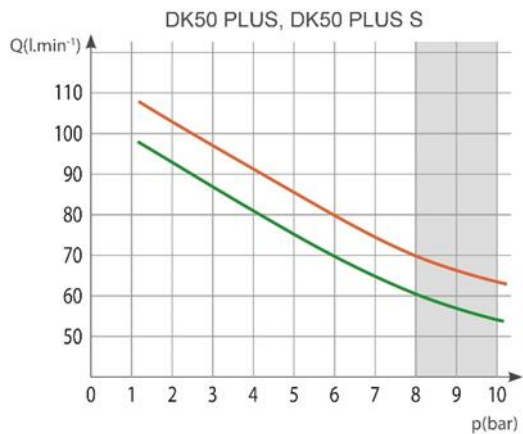
Working pressure 8 – 10 bar		DK50 PLUS		DK50PLUS S		DK50 PLUS/M		DK50 PLUS S/M	
Nominal voltage Frequency ^{a)}	V, Hz	230, 50/60	115, 60	230, 50/60	115, 60	230, 50/60	115, 60	230, 50/60	115, 60
Capacity at 8 bar (FAD)	l/min	60/70	70	60/70	70	50/60	60	50/60	60
Working pressure ^{b)}	bar	8.0 – 10.0		8.0 – 10.0		8.0 – 10.0		8.0 – 10.0	
Max. current	A	4.1/5.1	10.2	4.2/5.2	10.5	4.2/5.2	10.5	4.3/ 5.3	10.7
Motor power	kW	0.55		0.55		0.55		0.55	
Air tank volume	l	25		25		25		25	
Air quality – filtration	µm	-		-		0.1		0.1	
Maximum operating pressure of safety valve	bar	11.0		11.0		11.0		11.0	
Noise level at 5 bar (L _{pA})	dB	≤66/68	≤68	≤47/50	≤50	≤66/68	≤68	≤47/ 50	≤50
Operating mode	%	S1-100		S1-100		S1-100		S1-100	
PDP drying at 7 bar	°C	-		-		≤+3		≤+3	
Time to fill air tank from 0 to 7 bar	s	208/176	176	208/176	176	304/239	239	304/ 239	239
Dimensions (net) w x d x h	mm	460x500x708		560x675x877		460x565x708		560x675x877	
Net weight	kg	48 ^{c)}		87 ^{c)}		53		92	

^{a)} State the compressor variant in the order

^{b)} For other range of pressure consult with the supplier

^{c)} Weight value is only informative data and applies only to a product without any accessories

Dependence of compressor capacity on working pressure



Working pressure 6 – 8 bar		DK50 2V		DK50 2V S		DK50 2V/M		DK50 2V S/M	
Nominal voltage Frequency ^{a)}	V, Hz	230, 50 3x400, 50	230,60 115,60	230,50 3x400, 50	230,60 115,60	230,50 3x400, 50	230,60 115,60	230,50 3x400, 50	230, 60 115, 60
Capacity at 6 bar (FAD)	l/min	135		135		104		104	
Working pressure ^{b)}	bar	6.0 – 8.0		6.0 – 8.0		6.0 – 8.0		6.0 – 8.0	
Max. current	A	8.6 3.8	9.2 19.9	8.8 4.0	9.4 20.1	8.7 3.9	9.3 20.0	8.9 4.1	9.5 20.2
Motor power	kW	1.2 1.2	1.2 1.1	1.2 1.2	1.2 1.1	1.2 1.2	1.2 1.1	1.2 1.2	1.2 1.1
Air tank volume	l	25		25		25		25	
Air quality – filtration	µm	-		-		0.1		0.1	
Maximum operating pressure of safety valve	bar	11.0		11.0		11.0		11.0	
Noise level at 5 bar (L _{pA})	dB	≤67.0		≤51.5		≤69.0		≤55.0	
Operating mode	%	S1-100		S1-100		S1-100		S1-100	
PDP drying at 7 bar	°C	-		-		≤+3		≤+3	
Time to fill air tank from 0 to 7 bar	s	61		61		78		78	
Dimensions (net) w x d x h	mm	460x500x720		560x675x877		460x565x720		560x675x877	
Net weight	kg	55 53	55	93 91	93	60 58	60	98 96	98

a) State the compressor variant in the order

b) For other range of pressure consult with the supplier

c) Weight value is only informative data and applies only to a product without any accessories


Dependence of compressor capacity on working pressure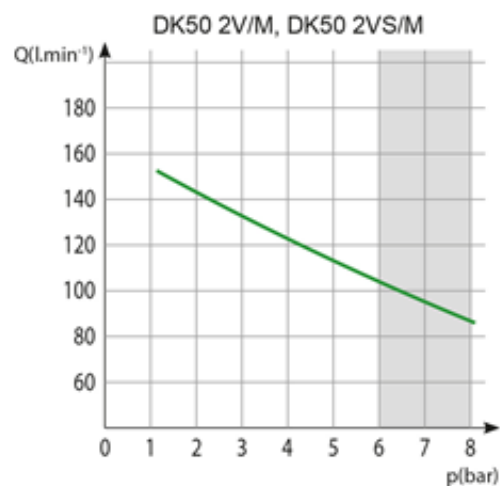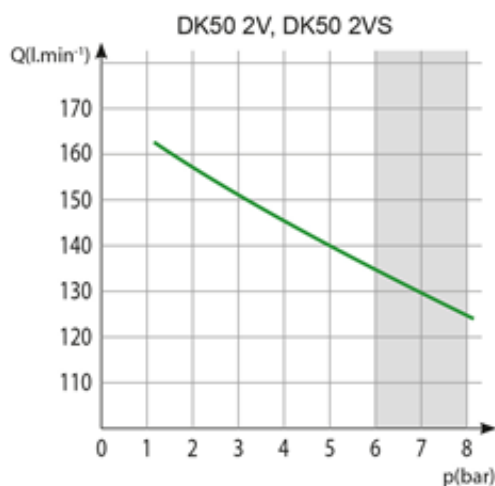

Working pressure 8 – 10 bar		DK50 2V		DK50 2V S		DK50 2V/M		DK50 2V S/M	
Nominal voltage Frequency ^{a)}	V, Hz	230,50 3x400, 50	230, 60 115, 60	230,50 3x400, 50	230,60 115,60	230,50 3x400, 50	230, 60 115, 60	230,50 3x400, 50	230, 60 115,60
Capacity at 8 bar (FAD)	l/min	110		110		87		87	
Working pressure ^{b)}	bar	8.0 – 10.0		8.0 – 10.0		8.0 – 10.0		8.0 – 10.0	
Max. current	A	8.4 4.6	9.3 18.6	8.6 4.8	9.5 18.8	8.5 4.7	9.4 18. 7	8.7 4.9	9.6 18.9
Motor power	kW	1.2 1.2	1.2 1.1	1.2 1.2	1.2 1.1	1.2 1.2	1.2 1.1	1.2 1.2	1.2 1.1
Air tank volume	l	25		25		25		25	
Air quality - filtration	µm	-		-		0.1		0.1	
Maximum operating pressure of safety valve	bar	11.0		11.0		11.0		11.0	
Noise level at 5 bar (L _{pA})	dB	≤67.0		≤51.5		≤69.0		≤55.0	
Operating mode	%	S1-100		S1-100		S1-100		S1-100	
PDP drying at 7 bar	°C	-		-		≤+3		≤+3	
Time to fill air tank from 0 to 7 bar	s	96		96		123		123	
Dimensions (net) w x d x h	mm	460x500x720		560x675x877		460x565x720		560x675x877	
Net weight	kg	58 56	58	96 94	96	63 61	63	101 99	101

- a) State the compressor variant in the order
- b) For other range of pressure consult with the supplier
- c) Weight value is only informative data and applies only to a product without any accessories

Dependence of compressor capacity on working pressure

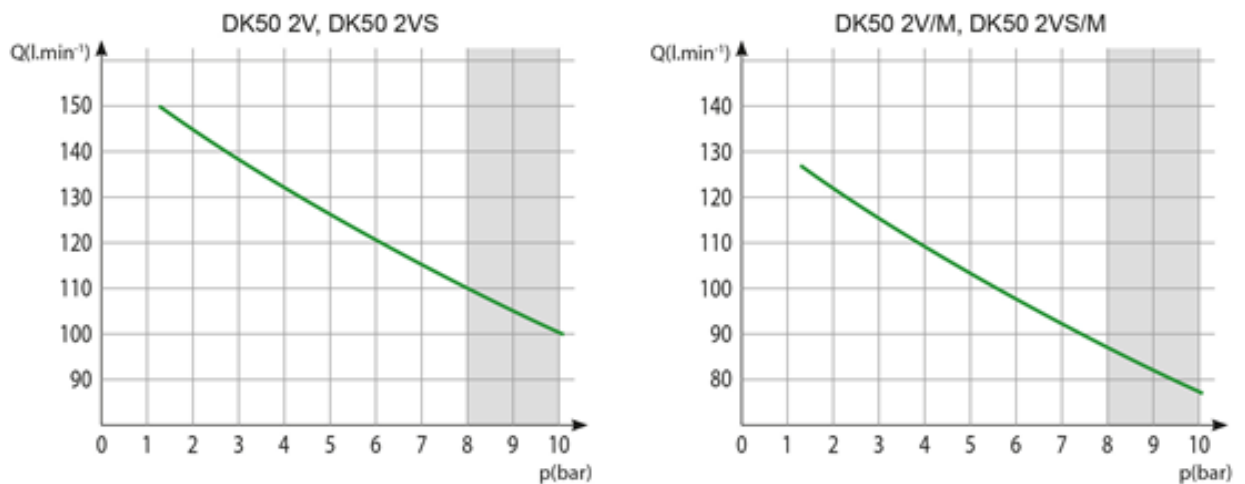FAD correction of capacity for altitude

Capacity given in the form of FAD („Free Air Delivery“) applies to the following conditions:

Altitude	0 m.n.m.	Temperature	20°C
Atmospheric pressure	101325 Pa	Relative humidity	0%

To calculate FAD compressor capacity in dependence on altitude, it is necessary to apply correction factor according to the following table:

Altitude [m.n.m.]	0 -1500	1501 - 2500	2501 - 3500	3501 - 4500
FAD correction factor	1	0,8	0,71	0,60

INSTALLATION

Risk of incorrect installation.

Only a qualified professional can install the compressor and place it into operation for the first time. His obligation is to train the operating personnel on the use and maintenance of the device. He shall confirm installation and training of operators by an entry into the installation record (see annex / anhang / príloha).

9. INSTALLATION CONDITIONS

- The compressor may only be installed and operated in dry, well-ventilated and dust-free rooms with conditions stated in Technical data.

Risk of damage to the device.

Do not operate the device in outdoor environment or in humid or wet rooms.

Explosion risk.

Do not install the device in any rooms, in which explosive gases, dusts or flammable liquids may be present.

- The compressor must be installed in such a way, that it is accessible at all times for operation and maintenance and label on the device is accessible.
- The compressor must stand on a flat, sufficiently stable base (be aware of the compressor weight, see Technical data).

You may notice a “new product” odour when you first place the product into service (for a short period of time). This odour is temporary and does not impede the normal use of the product. Ensure the space is properly ventilated after installation.

10. COMPRESSOR ASSEMBLY**10.1. Manipulation and fixation removal**

- Unpack the compressor from the packaging.
- Put the compressor at the place of operation (Fig. 4).


Use the handles on the compressor to move the compressor as needed. Do not use other parts of the compressor (the air pump, cooler, etc.) for grip

The number of persons required to handle the equipment must be matched to the weight of the device.

Fig. 4: Manipulation with the compressor

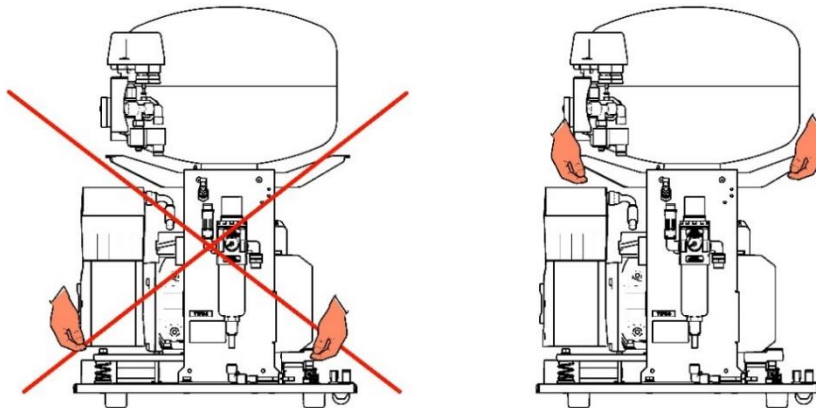
- Remove transport fixing elements from the air pumps (Fig. 5).

Prior to installation, ensure that the compressor is free of all transport packaging and stabilizers to avoid any risk of damage to the product.

Remove fixing elements of the air pumps once the compressor has been assembled and balanced at the final installation place.


Fig. 5: Releasing the pump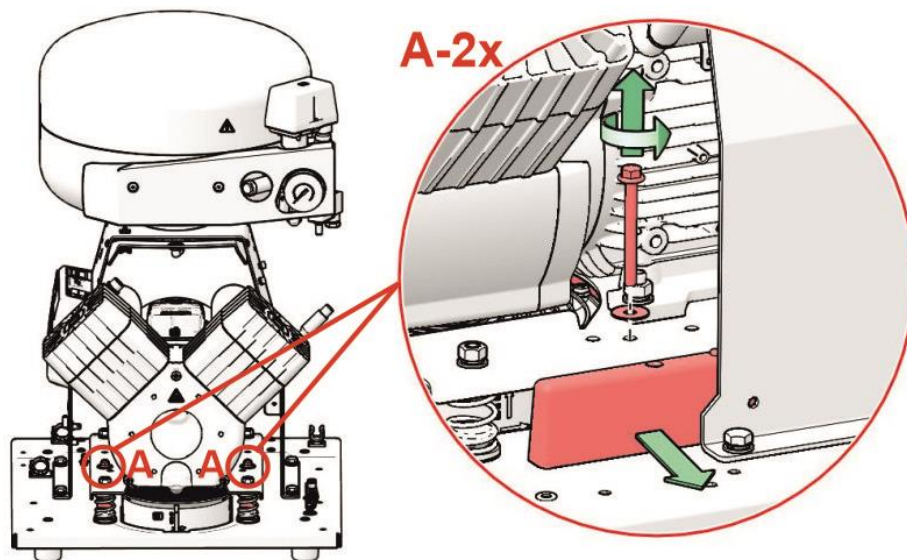

10.2. Changing door direction

- Disconnect the earthing conductor from the cabinet. (A)
- Remove the door, the rectifying screws on both sides of the cabinet (3) and the hinge (4) bracket (2). (B)
- Install the bracket for hinge (2) on the left side of the cabinet. (C)
- Rotate the door 180°. (B)
- Insert the spacer (5) between hinge (1) and the bottom of the door. (C)
- Install the door, the 2 rectification screws (3) and connect the earthing conductor. (C)
- Remove the door lock (6) and rotate 180°. (D)
- Remove the latch (7) and rotate 180°. (D)
- Install the lock. (D)

Fig. 6: Changing door direction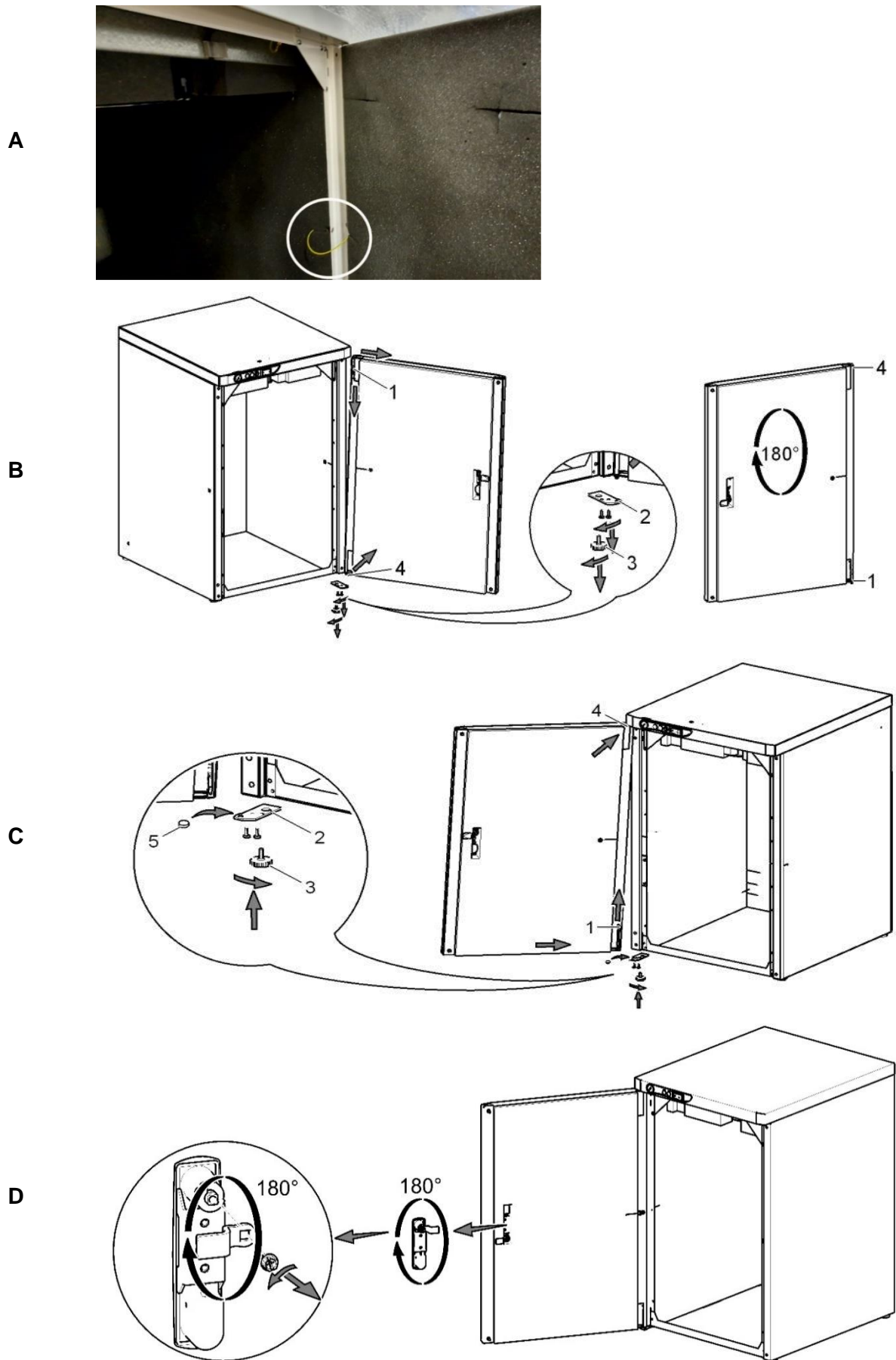


10.3. Adding sound-insulating material into the cabinet

A component from the sound-insulating material is to be installed in the opening of the cabinet of the compressor without a dryer.


This component is provided in the basic equipment delivered with the product.

Fig. 7: Adding sound-insulating material into the cabinet


A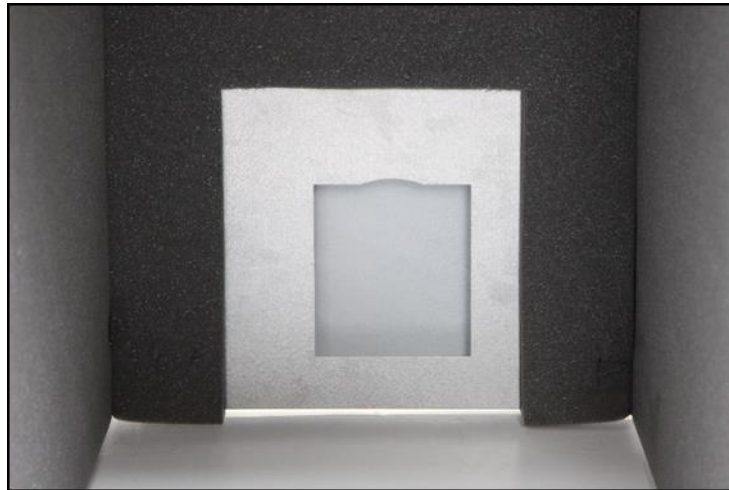

B

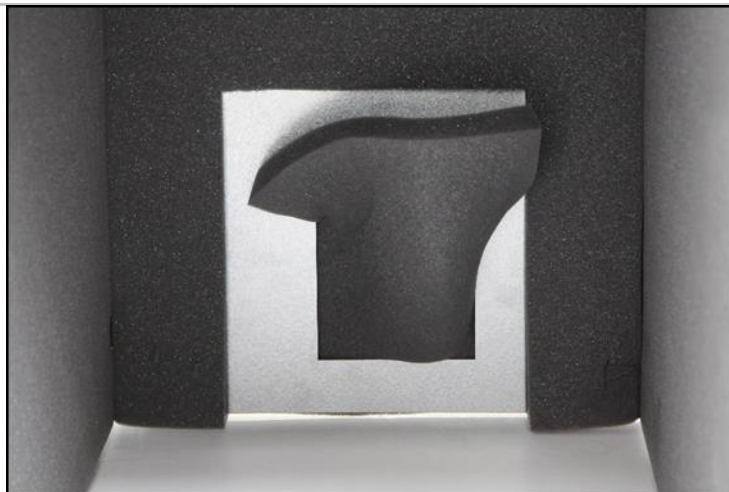C

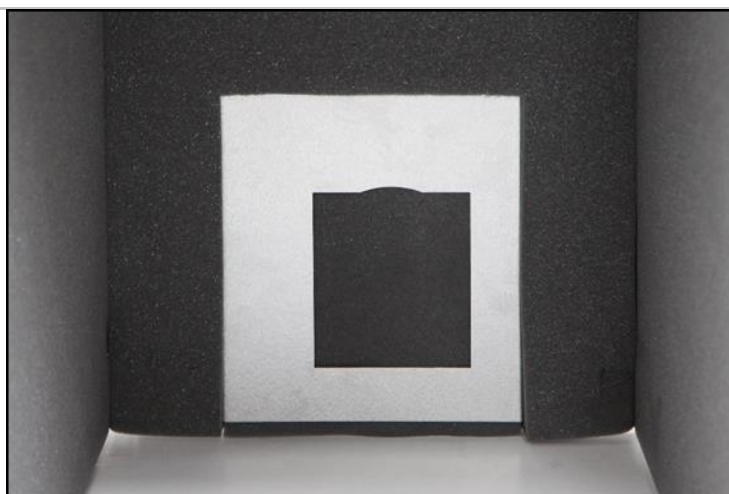11. PNEUMATIC CONNECTION

11.1. Connecting to the compressed air outlet

- The pressure line (2) is attached to the compressed air outlet (1) on the compressor.
- Route the pressure hose through the opening in the rear wall of the cabinet for cabinet-mounted compressors.

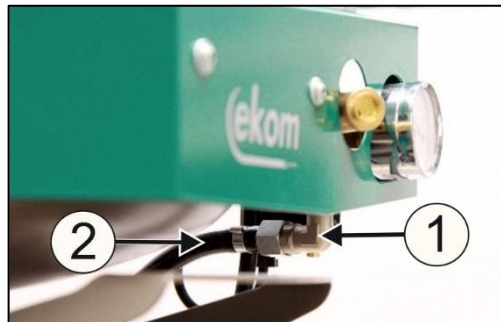
Fig. 8: Connecting to the compressed air outlet

11.2. Connecting the cabinet pressure gauge to the compressor

- Remove the plug (1) from the threads (2) on the pneumatic block of the compressor
- Connect the cabinet pressure gauge hose to the threaded fitting



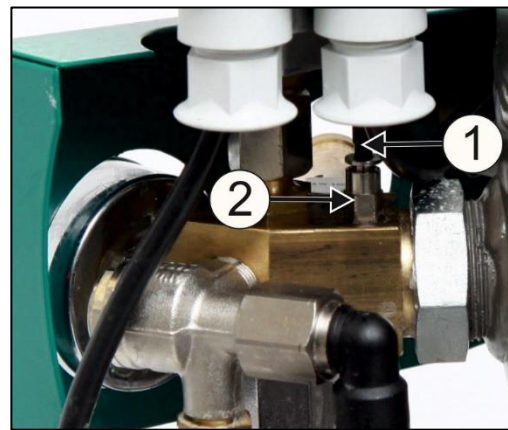
Fig. 9: Connecting the cabinet pressure gauge to the compressor

11.3. Connecting the condensate vessel

- Connect the condensate drain hose to the condensate collection vessel on compressors in cabinet.(Fig. 10)

Fig. 10: Connection of condensate collection vessel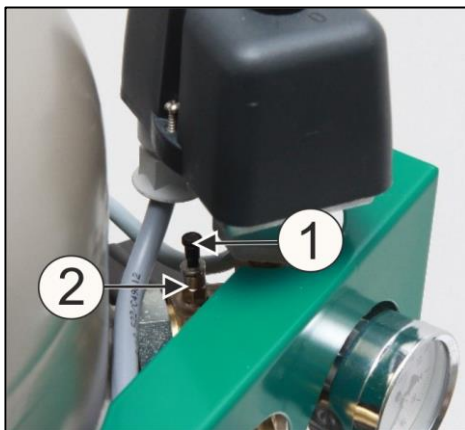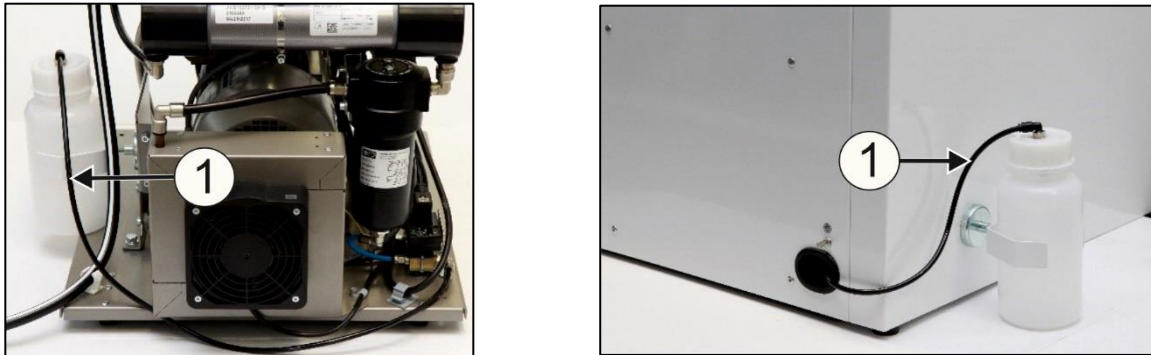

- Route the hose through the opening in the rear wall of the cabinet for compressors with dryers.


Risk of damage to pneumatic components.

Air hoses must not be broken.

11.4. Floor installation

Connect the compressor using the hoses prepared in advance in the floor, as per the installation plan.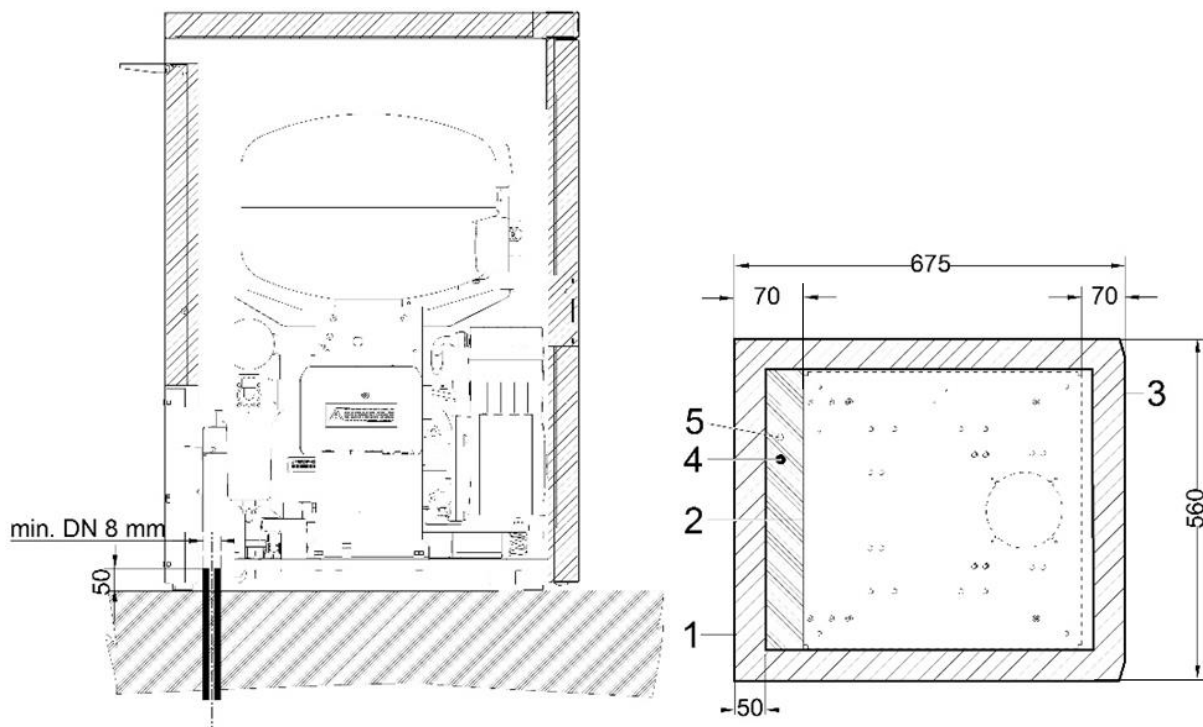

- 1 - Outline of the cabinet
- 2 - Outline of the base
- 3 - Front section - door
- 4 - Compressed air connection, G3/8"
- 5 - Power supply input:

230V/50(60)Hz	3Gx1.0x4000
3x400V/50Hz	5Gx1.0x4000

- all dimensions are in millimetres
- distance from wall min. 100 mm

Fig. 11: Floor installation

12. ELECTRICAL CONNECTION

- The product is delivered with a cord equipped with a plug with earthing pin.
- Keep the socket easily accessible to ensure that the device can be safely disconnected from the mains.
- The corresponding circuit must be protected in the power distribution max. 16 A.

Risk of electric shock.

it is necessary to follow all local electrotechnical regulations. The mains voltage and frequency must comply with the data stated on the device label.

Risk of fire and electric shock.

Electrical cable must not be in contact with hot compressor components.

Risk of fire and electric shock.

Electrical cord must not be broken.

DK50 PLUS S/M, DK50 2V S/M compressor cabinet, variant 10 bar, is equipped with a cooling fan and a switch. The cabinet must be connected using the cord with a connector to a connector at the compressor electrical box.

- Connect the pin for equipotential bonding \varnothing 6 mm (1) to the electrical circuit using the defined method under valid electrical engineering regulations.
- The socket for equipotential bonding (2) is an accessory and is not provided in the basic package.

Fig. 12: Equipotential bonding

12.1. Connecting a compressor not installed in a cabinet

- Insert the mains plug into a rated mains socket.
- The compressor is ready for operation.

12.2. Connecting a compressor installed in a cabinet


- Route the mains plug through the opening in the rear wall of the cabinet for cabinet-mounted compressors.

Fig. 13: Connecting a compressor installed in a cabinet

- Connect the cabinet electrically to the compressor by inserting the provided power cord with connector (1) into an outlet. (Fig. 14)
- Disconnect the cabinet electrically from the compressor by pulling the connector (1) from the outlet while the latch is released.

Fig. 14: Connecting a compressor installed in a cabinet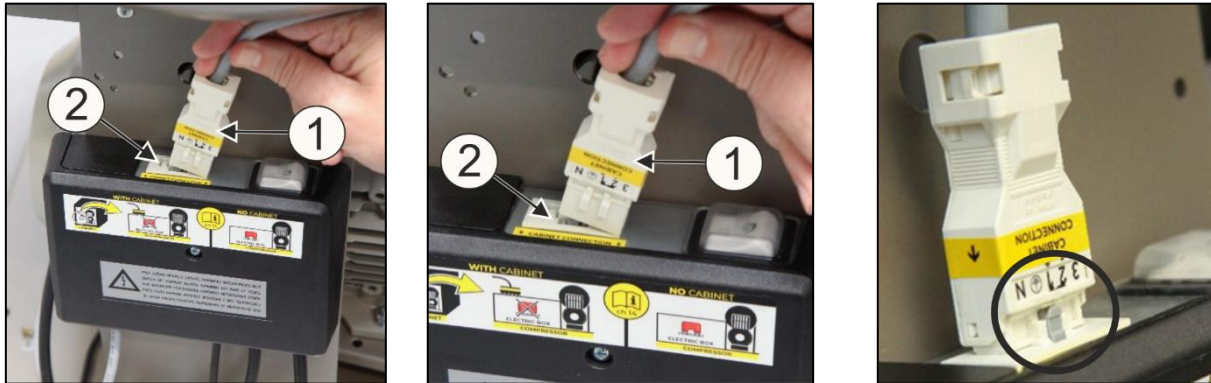

13. COMMISSIONING

- Check if all fixing elements used during transport have been removed.
- Check correct connection of compressed air (see chapter 11).
- Check correct connection to the mains (see chapter 12).
- Check position of the circuit breaker switch, it must be in position „I“. If it is in position „0“, turn the switch (4) to position „I“. (Fig. 15).
- On DK50-10S/M (8 - 10bar) compressor, switch the switch (5) on the front side of the cabinet to position „I“, the green light indicates the device status in operation. (Fig. 15).
- Check, if compressor cabinet (only 8 - 10 bar) is connected with the compressor by a cable with a connector. (Fig. 14).
- Check the connection of the the cabinet pressure gauge hose to the compressor pneumatic block.


The compressor is not equipped with a backup power supply.


14. PNEUMATIC AND ELECTRICAL DIAGRAMS

14.1. Pneumatic diagram

DK50 PLUS, DK50 PLUS S, DK50 2V, DK50 2V S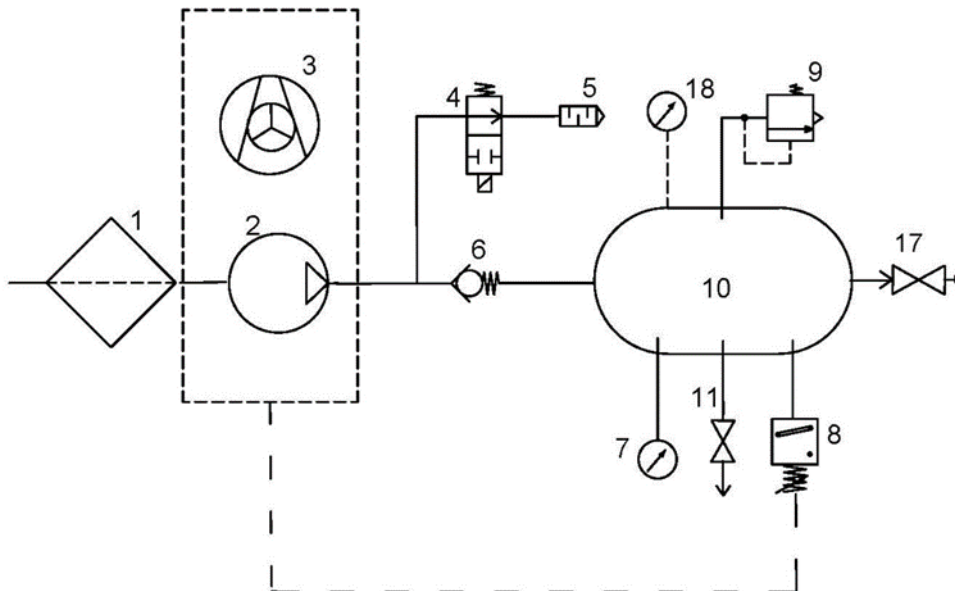

DK50 PLUS/M, DK50 PLUS S/M, DK50 2V/M, DK50 2V S/M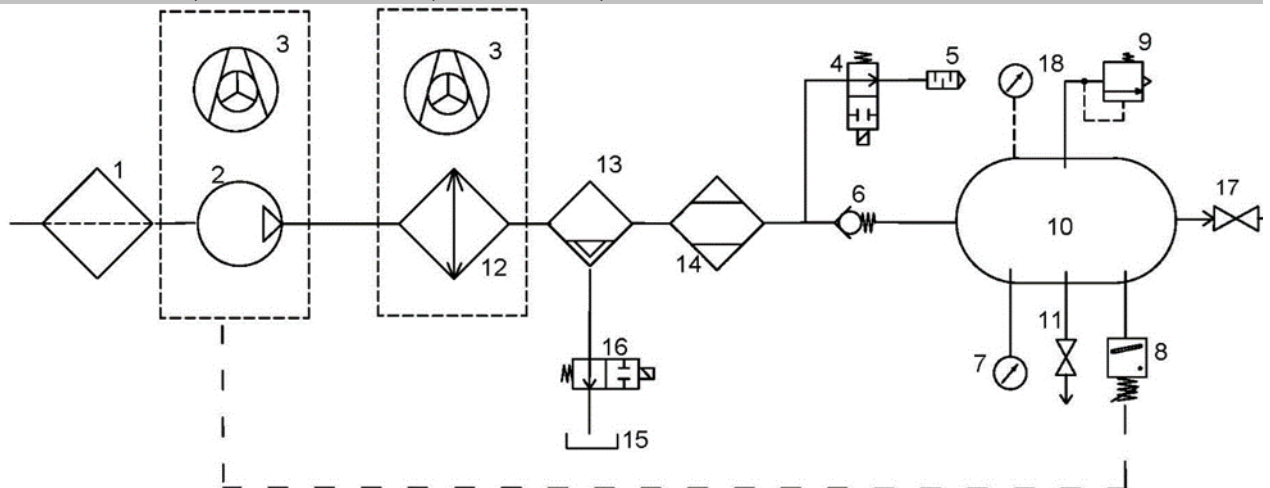

Description to pneumatic diagrams


- | | |
|--------------------|---------------------------------------|
| 1 Inlet filter | 10 Air tank |
| 2 Compressor | 11 Drain valve |
| 3 Fan | 12 Cooler |
| 4 Relief valve | 13 Coalescing filter |
| 5 Noise dampener | 14 Membrane dryer |
| 6 Non-return valve | 15 Condensate collection vessel |
| 7 Pressure gauge | 16 Solenoid valve of condensate drain |
| 8 Pressure switch | 17 Outlet valve |
| 9 Safety valve | 18 Cabinet pressure gauge |

14.2. Electrical diagrams

DK50 PLUS, DK50 PLUS S DK50 2V, DK50 2V S 5 - 7 bar, 6 - 8 bar, 8 - 10 bar

1/N/PE 230 V, 50/60 Hz
 230 V, 50/60 Hz***
 115 V, 60 Hz


ELECTRICAL OBJECT OF 1st. CAT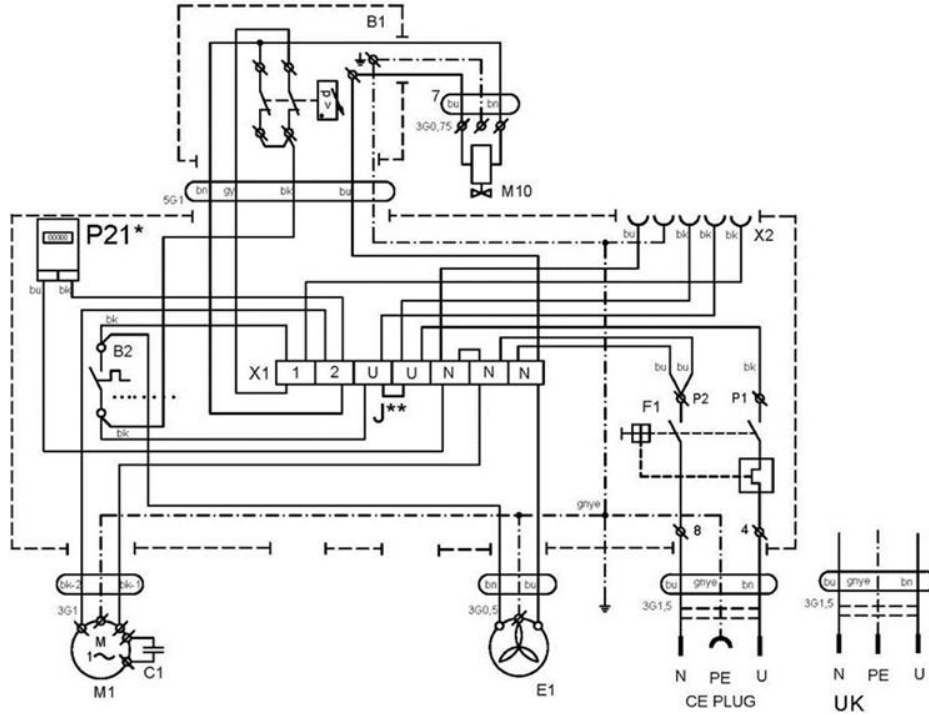

DK50 PLUS/M, DK50 PLUS S/M DK50 2V/M, DK50 2V S/M 5 - 7 bar, 6 - 8 bar, 8 - 10 bar

1/N/PE 230 V, 50/60 Hz
 115 V, 60 Hz

ELECTRICAL OBJECT OF 1st. CAT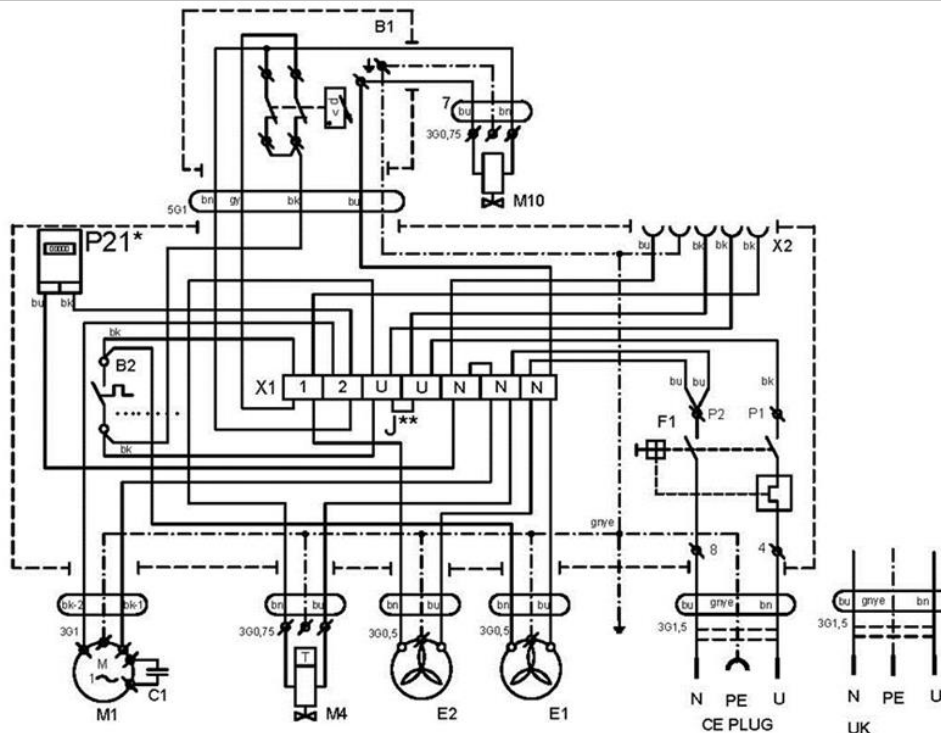


Compressor cabinet

1/N/PE 230V, 50 Hz

230 V, 50 Hz, 230 V, 60 Hz

ELECTRICAL OBJECT OF 1st. CAT

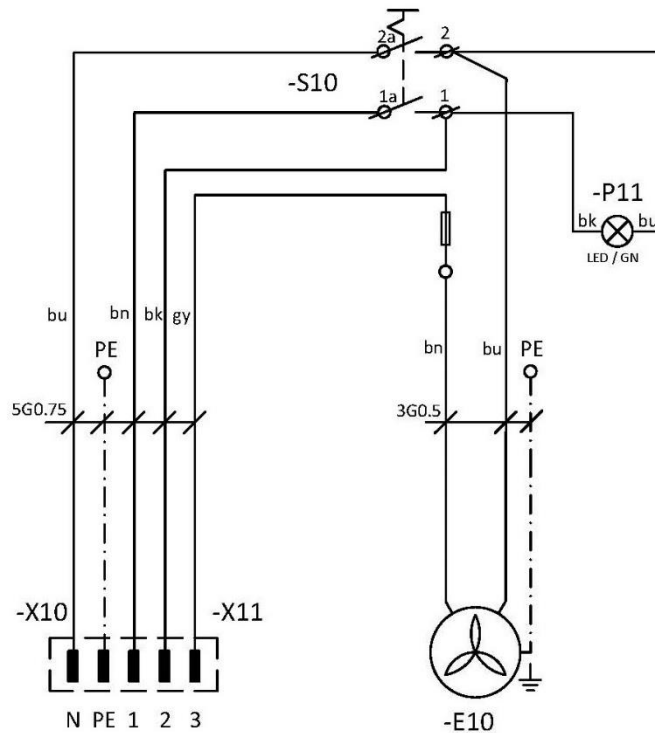Description to electrical diagrams

M1	Compressor motor	C1	Capacitor
E1	Compressor fan	B1	Pressure switch
E2	Dryer fan	X1	Terminal box
M10	Relief valve	F1	Circuit breaker switch
B2	Temperature switch	M4	Condensate drain valve
E10	Cabinet fan	P21	Hours counter
X10,X2	Connector	S10	Switch
F11	Breaker	P11	Indicator
Q11	Contacteur	X2	Socket

Note:

- J** - Only connect the jumper for compressors without cabinet (see chapter 17.10)
- P21* - The 8-10 bar compressor model is equipment with an hour counter
- *** - Configuration valid only for DK50 2V, DK50 2V S

OPERATION

THE EQUIPMENT MAY ONLY BE OPERATED BY THE TRAINED STAFF!

Risk of electric shock

In case of any danger, disconnect the compressor from the mains (pull out the mains plug).

Risk of burn or fire.

During compressor operation, the pump parts may heat to temperatures dangerous for contact with persons or materials.

Warning – compressor is controlled automatically.

Automatic start-up. When pressure in the pressure tank drops to switch-on pressure, the compressor automatically switches on. The compressor automatically switches off, when pressure in the air tank reaches the switch-off pressure.

Hazard of damage to pneumatic components.

Working pressures of the pressure switch set by the manufacturer must not be changed. If the compressor operates at higher working pressure than the switch-on pressure, it means that the air consumption is too high (see Chapter Malfunctions).

Hazard of damage to dryer.

When the dryer is operated at ambient temperature that is higher than maximum operating temperature stated in Technical data, the dryer can be damaged.

Required drying performance can only be achieved when the defined operating conditions are followed.

Drying performance decreases and the achieved dew point drops if the dryer is operated at any pressure below the minimum working pressure.

During longer operation of the compressor, the temperature in the surrounding of the compressor increases above 40 °C and the cooling fan switches on automatically. After cooling the space below 32 °C, the fan switches off.

15. SWITCHING ON THE COMPRESSOR


Start the compressor (without a cabinet) at the pressure switch (1) by turning the switch (2) to position "I." This starts the compressor and fills the tank to the switching off pressure, which then shuts off the compressor.

Start the compressor (with a cabinet) at the switch (5) on the front side of the cabinet, the indicator will turn green. This starts the compressor and fills the tank to the switching off pressure, which then shuts off the compressor.

The compressor operates in automatic mode, switched on and off by the pressure switch, depending on compressed air consumption.

Check the values of the switch-on and switch-off pressure using the pressure gauge (3). A tolerance of $\pm 10\%$ is acceptable. The pressure in the air tank must not exceed the permitted operating pressure.

Fig. 15: Switching on the compressor

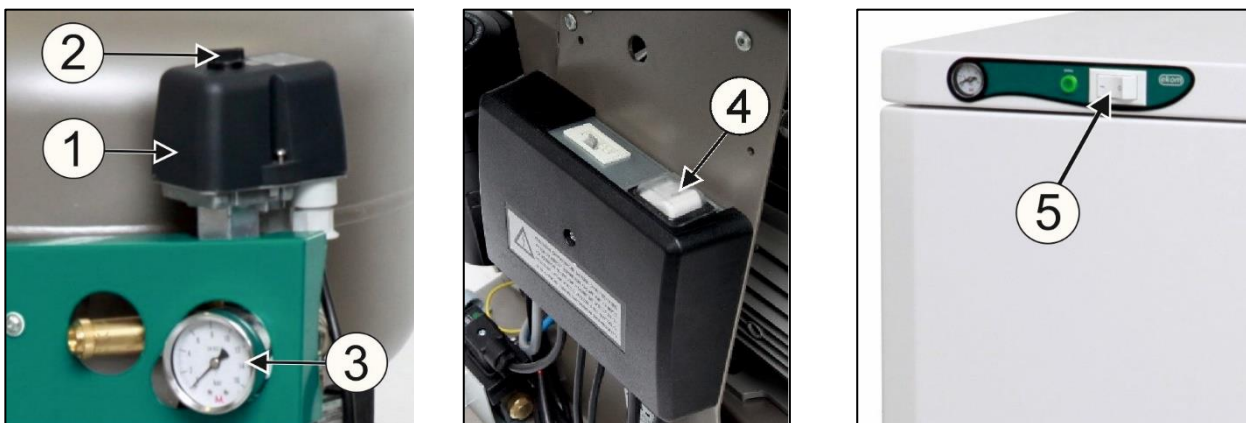- **Compressor (without dryer)** – when first started and put into operation, the compressor fills the air tank to the switch-off pressure and the pressure switch turns off the compressor. Then the compressor operates in automatic mode, switches on and off depending on compressed air consumption.
- **Compressor with dryer** – the compressor operates in the same manner as above, only the compressed air passes through the cooler that removes moisture from the compressed air.

Risk of damage to pneumatic parts.

The pressure switch (1) was set up by the manufacturer and any other changes to its settings may be performed only by a qualified technician trained by the manufacturer.

16. SWITCHING OFF THE COMPRESSOR

- Switching off the compressor due to service or any other reason shall be performed using the pressure switch (1) by turning the switch (2) to position „0“ and pulling out the mains plug from the socket. This disconnects the compressor from the mains supply.(Fig. 15)
- Vent the pressure in the air tank to zero by opening the drain valve (Fig. 16).

PRODUCT MAINTENANCE**17. PRODUCT MAINTENANCE**

The operator should carry out device checks regularly in the intervals defined by applicable regulations. Test results must be recorded.

The device has been designed and manufactured to keep its maintenance to a minimum. For correct and reliable operation of the compressor perform the following operations:

Risk of servicing by persons without required qualification.

Repair works beyond standard maintenance (see chapter.17.1) may only be performed by a qualified technician (organization authorized by the manufacturer) or manufacturer's customer service.

Standard maintenance works (see chapter 17.1) may only be carried out by a trained operator.

Only use spare parts and accessories approved by the manufacturer.

Risk of injury and damage to the device.

Before any maintenance work, it is necessary to:

- **check, if it is possible to disconnect compressor from the supplied equipment, so that there is no risk of damage to a person using the given appliance or any other material damages**
- **switch off the compressor**
- **disconnect it from the mains (pull out the mains plug)**
- **vent the compressed air from the air tank**

Risk of injury when venting the compressed air.

Protect eyesight – wear goggles when venting the compressed air from the compressor pneumatic system (air tank).

Risk of burn.

Pump components (head, cylinder, pressure hose) have high temperature during and shortly after compressor operation – do not touch these components!

Let the device cool before any product maintenance, service or connection/disconnection of pressurized air!

The removed grounding conductor during service must be connected back to the original position after completing the service.

- For any maintenance or repair work, the compressor may be removed from the cabinet by pulling it from the cabinet (on rolling casters as much as the cord between the compressor and the cabinet allows). See also chap. 17). The required maintenance or repair work may then be performed.
- If the cord between the compressor and the cabinet becomes disconnected (by pulling the mains plug, chap. 12.2), the compressor shall not be functional. In order to bring the compressor into function again, please follow chap. 12.2 or 17.10 or 17.11.

17.1. Maintenance intervals

Time interval ^{c)}	50 Hz		60 z		once a day	once a week	one a year	once every 2 years	2000 hours	4000 hours	6000 hours	8000 hours	10000 hours	12000 hours	Chap.	Spare parts set	Performed by			
	operator	operator	operator	operator													qualified technician	qualified technician	qualified technician	qualified technician
Check of product operation	x														17.2	-				
Drain condensate from air tank ^{b)} - at high humidity	x														17.5	-				
Drain condensate from the air tank ^{b)} - at normal humidity		x													17.5	-				
Check of product function			x												8	-				
Check of pneumatic connections leakage and device inspection							x								17.3	-				
Check of electrical connection							x								17.4	-				
Check of fan and cooler							x								17.9	-				
Replacement of filter element in dryer filter ^{d)}							x								17.8	025200304-000				
Check of safety valve							x								17.6	-				
Replacement of pump inlet filter and pre-filter DK50 PLUS ^{a)}							x				x				17.7	025200126-000				
Replacement of pump inlet filter and pre-filter DK50 2V ^{a)}							x				x				17.7	025200139-000 025200150-000				

^{a)} data is in hours, if not available, data is in years

^{b)} applies to compressor without dryer

^{c)} time interval reduces by 20 % for compressor variants 60 Hz

(2000h./1600h., 4000h./3200h., 6000h./4800h., 8000h./6400h., 10000h./8000h., 12000h./9600h.)

^{d)} only for compressors with dryers

17.2. Check of product operation

- Check pump condition – the pumps must operate evenly without excessive vibrations or noise. If there is any negative result, find the cause of the given condition or call service engineers.
- Check fans operation (visually) – the fans must be operating when the fans are running. If there is a negative result, look for the cause of the given condition or call service engineers.
- Check if the power cord and pneumatic hoses are not damaged. Replace the damaged parts or call service engineers.
- Check ambient temperature – ambient temperature must be below temperature limit (40°C). Improve cooling in the room if the temperature is higher.
- Compressor with air dryer - open the plug on the condensate collecting bottle and drain the condensate,

17.3. Check of pneumatic connections leakage and device inspection

Leakage check

- Check pneumatic connections of the compressor when the compressor is operating – pressurizing.
- Use a leakage analyser or soapy water to check all connections for leaks. Tighten or release the connection where leaks are found.

Device inspection

- Check condition of the compressor pump – even operation, appropriate noise level.
- Check fans operation – fans must be running in the defined cycles of the compressor operation.
- Check function of the temperature switch (B2) – heat the temperature switch to temperature above 40°C (e.g. using a heat gun – Be careful, do not operate with high temperature on plastic components in the surrounding, it can cause deformation of plastic materials). Fan E1, E3 starts up once the temperature reaches 40°C – the compressor must be under voltage.
- Check filters condition – filters must be undamaged and sufficiently clean.
- Check condition of the pump itself, check if there is no dirt inside the crankcase or clearance in the crankshaft.
- Replace any defective parts if there are any failures.

17.4. Check of electrical connections

Risk of electric shock.

Check electrical connections on the device disconnected from the mains.

- Check mechanical function of the main switch.
- Check if the power cable and conductors are not damaged.
- Visually check if cables are connected to the terminal box.
- Check all screw connections of the green-yellow PE grounding conductor.

17.5. Condensate drain

Risk of slipping on a wet floor in case of the collecting vessel overflow.

Switch off the compressor every time before emptying the vessel!

Open the drain valve carefully and slowly. If the drain valve is opened too much, it is accompanied by high noise level and uncontrolled jet of the accumulated condensate.

Compressors without a dryer

During regular operation, it is recommended to drain condensate from the pressure tank.

- Switch off the compressor from the mains and decrease air pressure in the device to max. 1 bar, e.g. by releasing air through the connected device.
- Release the drain valve from the clips on the compressor base. Place the open end of the hose into a suitable container and open the drain valve (1) to drain off the condensate from the container. In cabinet mounted compressor models, the condensate may be drained outside of the cabinet.

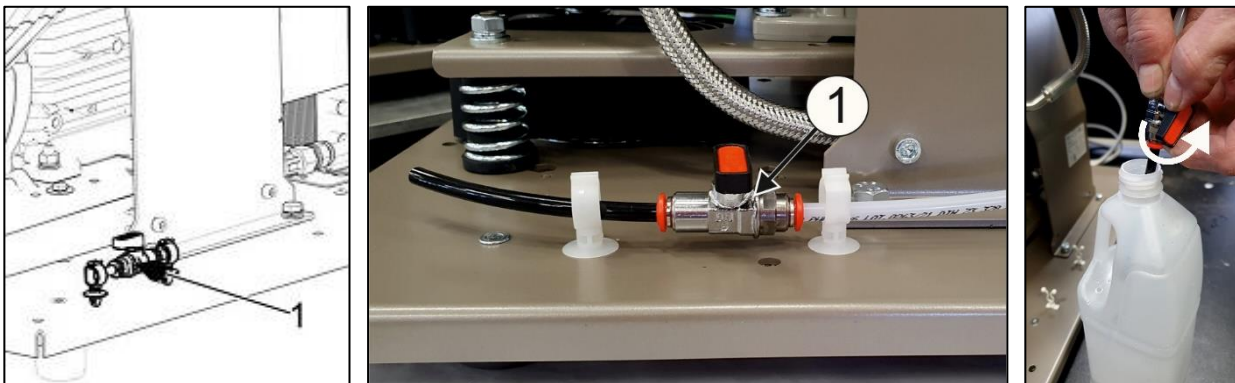
Fig. 16: Condensate drain from the pressure tank

- Compressors can be equipped with Automatic condensate drain (AOK), which provides draining of condensate from the air tank without the need of operator action (see chapter 7, Automatic condensate drain).

Compressors with air dryer

Condensate from compressors with air dryers is automatically drained into the condensate collection vessel.

- Monitor the level in the vessel using the 1 l or 2 l markings (depending on the volume of the vessel), and empty at least once a day.

Fig. 17: Check of condensate collection vessel

If necessary, it is possible to drain the air tank using the drain valve (1) from the lower side of the air tank (Fig. 18).

Switch off the compressor at the mains and reduce the air pressure in the device to a maximum of 1 bar, e.g. by venting air through the connected equipment.

Use a hose to drain with the free end placed in a suitable container and then open the valve to drain condensate from the tank.

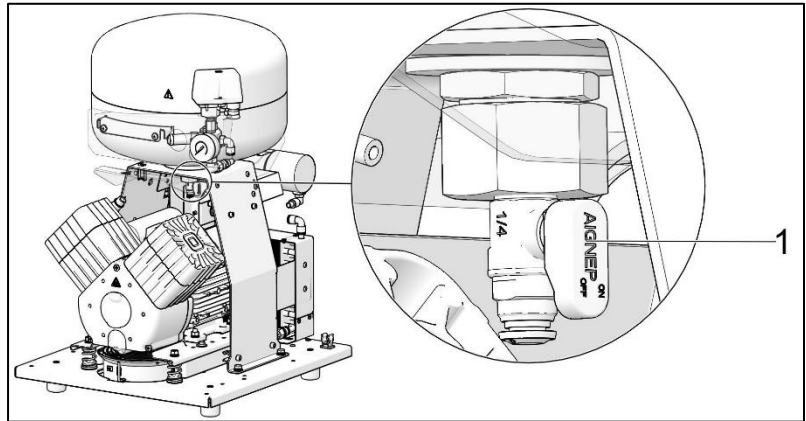
Fig. 18: Condensate drain

Before the following checks it is required:

- Compressor variants with cabinet DK50 PLUS S, DK50 PLUS S/M, DK50 2V/S, DK50 2V S/M – unlock the door lock and open the cabinet door.

17.6. Check of safety valve

- Turn the screw (2) on the safety valve (1) several times to the left until the safety valve releases the air.
- Let the safety valve vent for a few seconds.
- Turn the screw (2) fully to the right, the valve must be closed now.

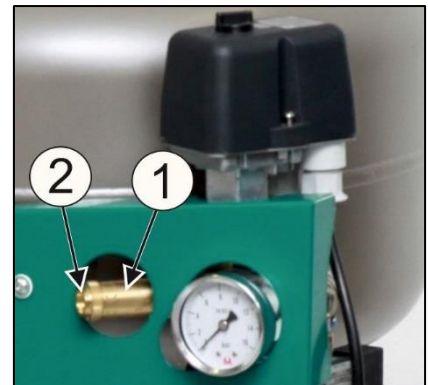
Fig. 19: Check of safety valve

Risk of pressure increase when safety valve is damaged.

Safety valve must not be used to depressurize the air tank. It could damage the safety valve function. The valve is set to the maximum allowed pressure, it is tested and marked.

Safety valve setting must not be adjusted.

Risk of injury when compressed air is vented.

Protect your eyesight when checking the safety valve – wear protective goggles.

17.7. Inlet filter and pre-filter replacement

The inlet filter (1) is located inside the compressor crankcase cover.

Inlet filter replacement:

- Pull out the rubber plug by hand (2).
- Remove the used filter (1).
- Insert a new filter and replace the rubber plug.

Pre-filter replacement:

- Pull out the pre-filter by hand (3).
- Replace the element and re-insert.



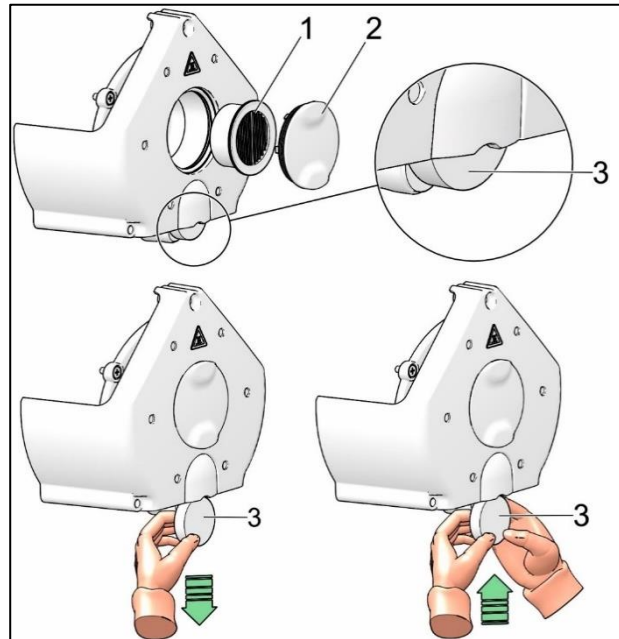
Fig. 20: Inlet filter and pre-filter replacement

17.8. Replacement of filter element

- Remove the tube (1) from the quick coupling.
- Loosen the filter bowl (3) using a wrench and dismantle.
- Dismantle the filter element (4) by pulling it downwards.
- Insert a new filter element.
- Mount the filter bowl.
- Slightly tighten the filter bowl using a wrench.
- Mount the tube back on the quick coupling.

Fig. 21: Replacement of filter element

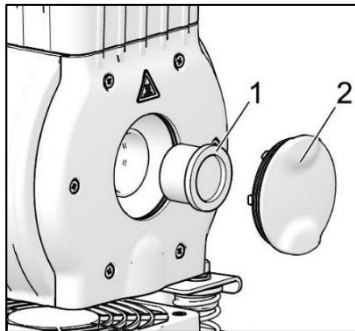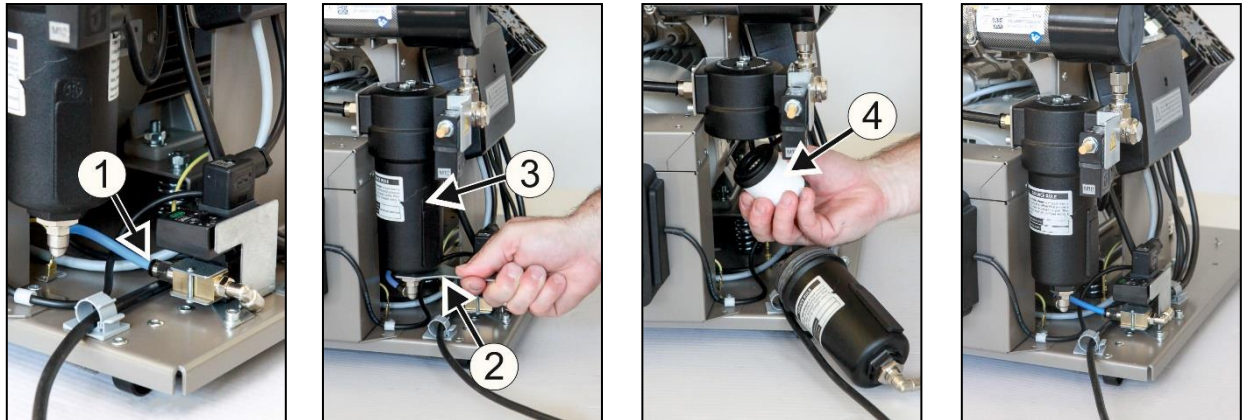17.9. Check of fan and cooler

The entire device and especially the compressor fan, cooler fan (21) and cooler (14) must be kept clean (Fig. 1, Fig. 2) – vacuum or blown down the cooling fins and fans with compressed air to remove any dust from the surface.

17.10. Procedure for connecting a compressor disconnected from the cabinet

Prior to any maintenance or repair work, switch off the compressor and disconnect it from the mains (pull out the mains plug).

The compressor (without cabinet) requires that the jumper is always mounted in the terminal strip for correct operation (Fig. 21, Fig. 22, Pos. B). It then replaces the switch on the cabinet.

If there is no jumper in the terminal strip, the compressor will not work!

If the cord between the compressor and the cabinet becomes disconnected (by pulling the mains plug) and the compressor is removed from the cabinet, the compressor will not be functional. Therefore, it is necessary to first create a connection to the terminal strip with a jumper (replacing the circuit breaker function) (Fig. 21, Fig. 22).



PROCESS:

Jumper installation (A-B process):

- Disconnect the product from the mains by pulling the plug out of the socket.
- Remove the cover on the electrical panel (on the compressor).
- The jumper is not in the terminal strip – A.
- Insert the jumper into the terminal strip – B.
- Reinstall the cover on the electrical panel.
- Connect the device to the electrical mains.
- Activate the compressor by turning on the switch on the pressure switch.



Compressor - when operating outside the cabinet, the electrical connection with an installed jumper **must** be created (Fig. 21, Fig. 22, Pos. B).

Fig. 22

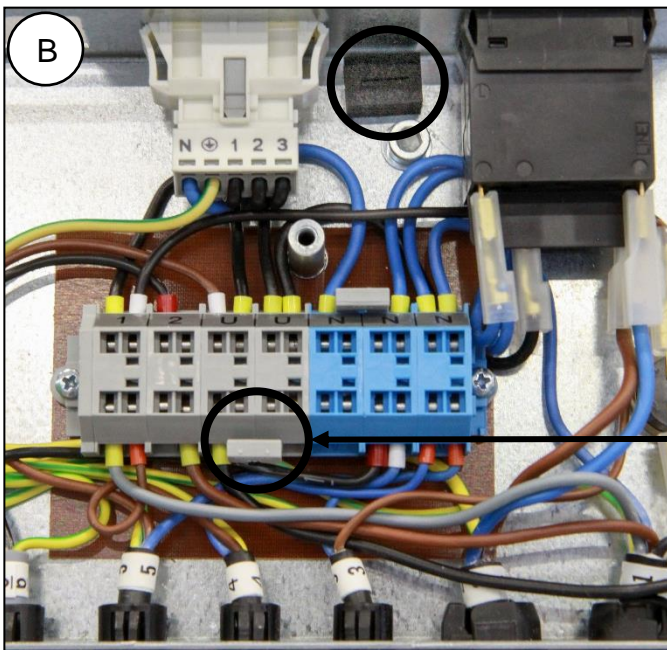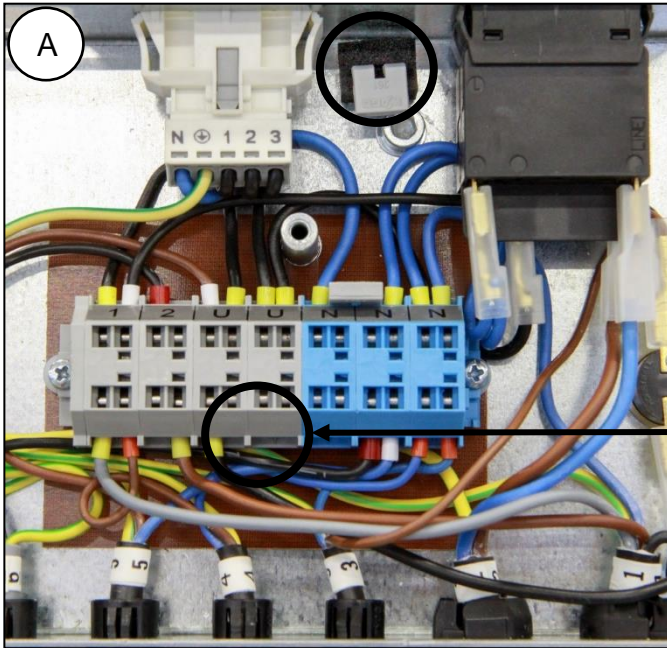230V

Fig. 23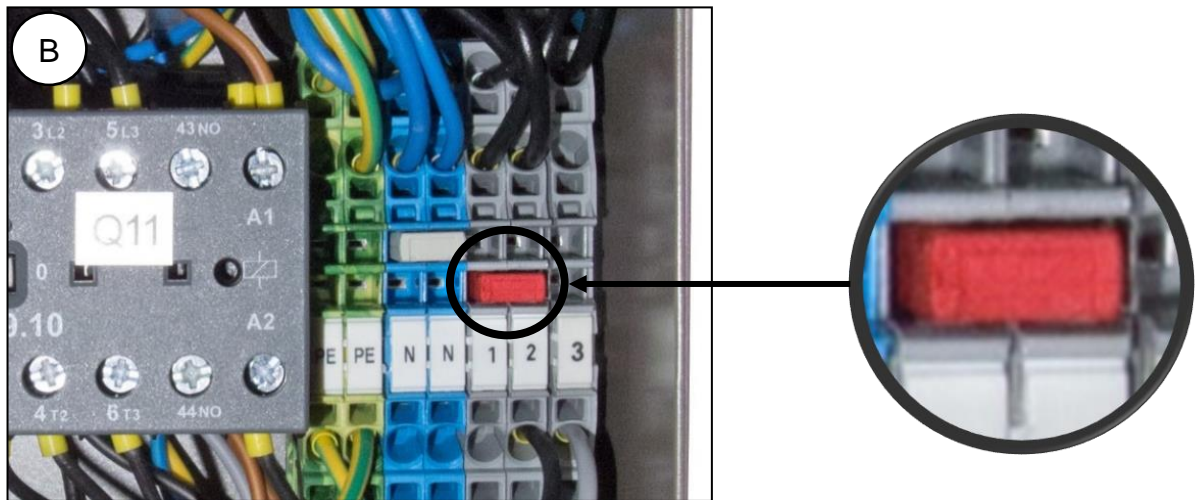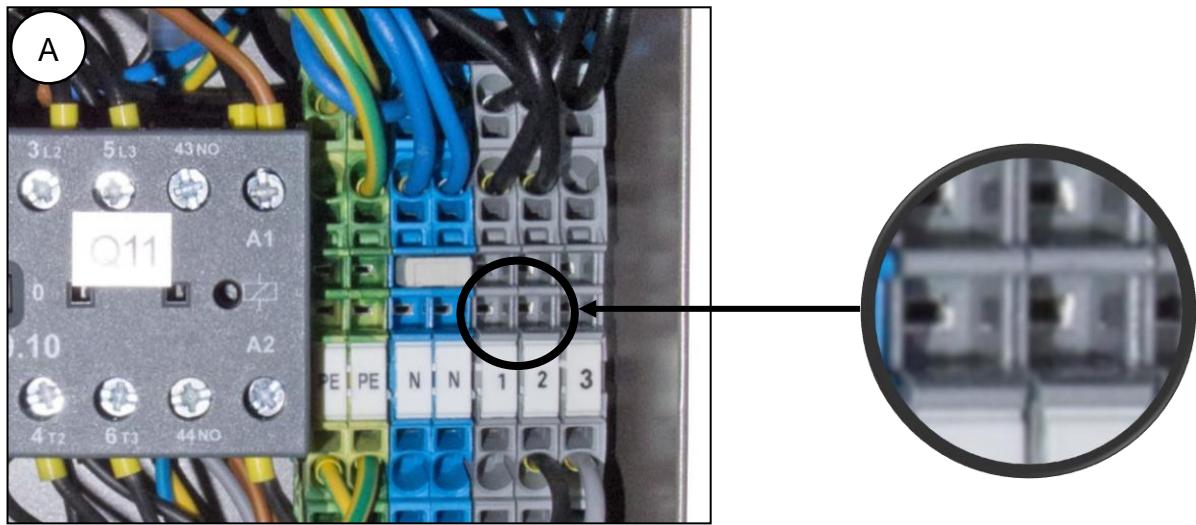

400V

17.11. Procedure for connecting a compressor to a new cabinet

Prior to any maintenance or repair work, switch off the compressor and disconnect it from the mains (pull out the mains plug).

The compressor in a cabinet requires that the jumper is not mounted in the terminal strip for correct operation (Fig. 21, Fig.22, Pos. A). The switch on the cabinet switches the entire product on and off, including the compressor.

If there is a jumper in the terminal strip, the switch on the compressor cabinet will not work!

To connect a new cabinet to a compressor that previously worked without a cabinet, it is necessary to cancel the connection in the compressor terminal strip by performing the procedure as follows. (Fig. 21, Fig. 22). See also chap. 12.1 and 12.2.

PROCESS:**Jumper removal (B-A process):**

- Disconnect the product from the mains by pulling the plug out of the socket.
- Remove the cover on the electrical panel (on the compressor).
- The jumper is in the terminal strip – B
- Remove the jumper from the terminal strip – A
- Reinstall the cover on the electrical panel.
- Insert the compressor into the cabinet.
- Connect the cabinet to the compressor by inserting the power cord with connector (1) into a power socket (2). (Fig. 14)
- Connect the device to the electrical mains
- Activate the compressor by turning on the switch on the pressure switch and the switch (5) on the cabinet. (Fig. 15)

Compressor with cabinet - when operating in the cabinet, there **must not** be an electrical connection. (Fig. 21, Fig. 22, Pos. A)

Note : **In case of using a compressor without a cabinet or in case of maintenance, a jumper which is not installed into the terminal strip should be kept** in a designated place next to the electrical panel.

17.12. Cleaning of the exterior surfaces of the product

Use neutral agents for cleaning of the external surfaces of the product.

The use of aggressive cleaning agents and comprising alcohol and chlorides may lead to the damage of the surface and the discoloration of the product.

18. LONG-TERM SHUTDOWN

If the compressor is not going to be used for long period, it is recommended to drain all condensate from the air tank and switch on the compressor for about 10 minutes, keeping the drain valve open (1) (Fig. 16). Thereafter switch off the compressor using the switch (2) on the pressure switch (1) (Fig. 15), close the drain valve and disconnect the device from the mains.

19. DISPOSAL OF DEVICE

- Disconnect the device from the mains.
- Release air pressure in the pressure tank by opening the drain valve (1) (Fig. 16).
- Dispose of the device according to the applicable local rules.
- Order a specialized organization to sort and dispose of waste.
- Product components after its operational lifetime have no negative environmental effect.

TROUBLESHOOTING**Risk of electric shock.****Before any of the following operations on the device, disconnect the device from the mains (pull out the mains plug).****Risk of injury during work with pneumatic components under pressure.****Before any of the following operations on the device, it is necessary to decrease pressure in the air tank and in the pneumatic system to zero.****Any operations concerning the troubleshooting may be performed only by a qualified technician.**

Malfunction	Possible cause	Solution
Compressor does not switch on	No voltage in the pressure switch	Check voltage in socket
		Check circuit breaker switch – switch to position switched-on „I“
		Loose conductor from terminal - repair
		Check electrical cord – replace defective cord
	Motor winding failure, damaged thermal protection	Replace motor or windings
	Capacitor failure	Replace capacitor
Cabinet switch does not switch off the compressor	Seized piston or other rotating part	Replace damaged components
	Pressure switch does not switch	Check function of pressure switch
	There is a compressor in the cabinet that previously worked separately (there is a jumper in the compressor terminal strip)	Remove the jumper from the terminal strip, see. Chap. 0
	Air leakage in pneumatic system	Check pneumatic system – seal loose connections
	Non-return valve (SV) leakage	Clean non-return valve, replace seals, replace non-return valve
Compressor often switches on	Large volume of condensed liquid in pressure tank	Drain condensed liquid
	Low compressor capacity	Check time of filling the air tank
	High air consumption of supplied equipment	Decrease air consumption Use compressor with higher capacity
	Leakage in pneumatic system	Check pneumatic system – seal loose connections
Low pressure in the air tank (compressor running constantly)	Low capacity of the pump	Clean / replace the pump
	Pump failure	Clean / replace the pump
	Dryer failure	Replace dryer
	Air leakage in pneumatic system	Check pneumatic system – seal loose connection
Prolonged operation of the		

compressor	Worn piston ring	Replace worn piston ring
	Dirty inlet filter	Replace dirty filter with new filter
	Incorrect function of solenoid valve	Repair or replace fan or coil
Compressor is noisy (knocking, metal noises)	Damaged piston bearing, piston rods, motor bearing	Replace damaged bearing
	Loose (cracked) dampening element (spring)	Replace damaged spring
Dryer does not dry (condensed water in the air) *	Defective cooler fan	Replace fan Check power source
	Damaged dryer	Replace dryer
	Defective automatic condensate drain	Clean / replace

)* Clean inside surfaces of the air tank thoroughly and remove all condensed liquid after dryer failure.

To protect the connected equipment from any damage, it is necessary to check humidity of the released air from the air tank (see chapter Technical data).

20. REPAIR SERVICE

Guarantee and post-guarantee repairs are provided by the manufacturer or organizations and technicians approved by the manufacturer.

Warning

The manufacturer reserves the right to make changes on the device, which will not significantly affect properties of the device.

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ANNEX / ANHANG / PRÍLOHA**21. INSTALLATION RECORD**

1. Product: (model) DK50 PLUS/M DK50 2V/M DK50 PLUS S/M DK50 2V S/M		2. Serial number:	
3.1. User's name:			
3.2. Address of installation:			
4. Equipment connected to the compressor:			
5. Installation / Commissioning:		6. Contents of operator training:	
Product completeness check **	Y	Description of the product and functions**	Y
	N		N
Documentation completeness check **	Y	Product operation: turning on/off, controls, control procedures, data on the display panel, alarms, operation in alarm conditions**	Y
	N		N
Installation/connection to equipment **	Y	Product maintenance: maintenance intervals, maintenance procedure, service intervals, operating activities**	Y
	N		N
Functional test **	Y	Safety measures, warnings – their meaning and compliance **	Y
	N		N
Notes:			
7. Operator instructed on safety measures, operations and maintenance:			
Name:		Signature:	
Name:		Signature:	
Name:		Signature:	
8. Installation and instruction performed by: First name/Last name		Signature:	
Company:		Address:	
Phone:			
Email:		Date:	
9. Distributor:			
Company:		Address:	
Contact person:			
Phone:		Email:	

** mark with an "X" in points 5 and 6 (Y - yes /N - no). Enter any observations from points 5 and 6 into the "Notes" section

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